





ottobock.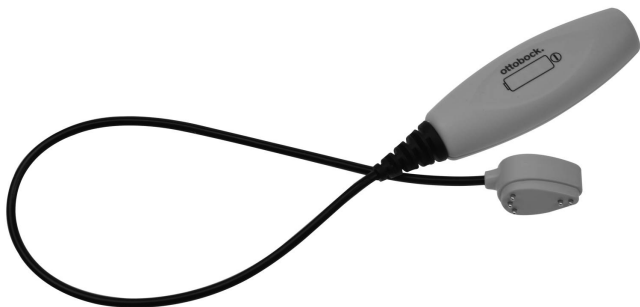

CE

757L35

DE Gebrauchsanweisung (Fachpersonal)	3
EN Instructions for use (qualified personnel)	10
FR Instructions d'utilisation (Personnel spécialisé)	18
IT Istruzioni per l'uso (Personale tecnico specializzato)	26
ES Instrucciones de uso (Personal técnico especializado)	33
PT Manual de utilização (Pessoal técnico)	41
NL Gebruiksaanwijzing (Vakmensen)	49
SV Bruksanvisning (Fackpersonal)	56
DA Brugsanvisning (Faguddannet personale)	64
NO Bruksanvisning (Fagpersonell)	71
TR Kullanma talimatı (Uzman personel)	78
EL Οδηγίες χρήσης (Τεχνικό προσωπικό)	85
RU Руководство по применению (Квалифицированный персонал) ...	93

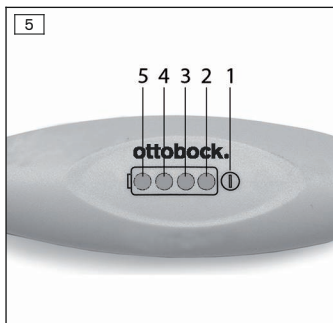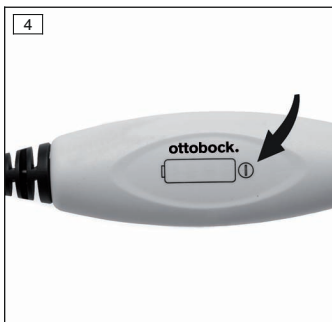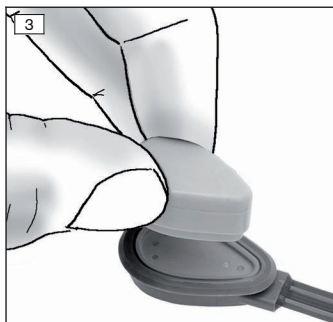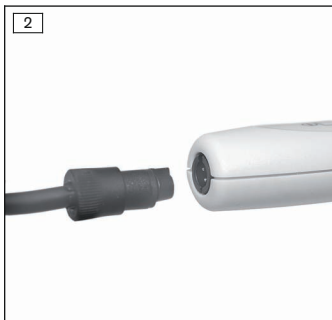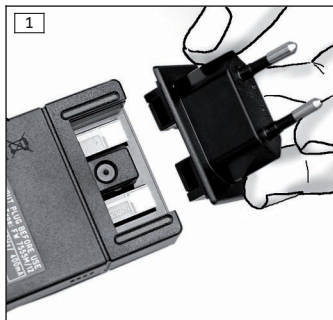INFORMATION

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INFORMATION

Last update: 2014-11-11

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.

► Please keep this document in a safe place.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Instruct the patient in the proper use and care of the product. The product may not be transferred to the patient without prior instruction.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product Description

The 757L35 MyoCharge Integral is used for charging the 757B25=* MyoEnergy Integral or 757B35=* MyoEnergy Integral, which are integrated in the socket.

For charging, the charging plug must be connected to the charging receptacle on the outside of the socket. The charging plug is secured to the charging receptacle by the integrated magnet. The special contour of the charging receptacle and charging plug ensures the quick, reliable positioning of the two components to each other.

LEDs indicate the readiness of the charger and the current battery charge level.

3 Application

3.1 Intended Use

The 757L35 MyoCharge Integral is intended solely for charging the 757B25=* MyoEnergy Integral or 757B35=* MyoEnergy Integral.

3.2 Area of Application

The 757L35 MyoCharge Integral can be used to charge unilateral or bilateral prosthetic fittings of the upper limbs.

3.3 Conditions of Use

The 757L35 MyoCharge Integral was developed for everyday activities.

3.4 Required Qualifications

The 757L35 MyoCharge Integral may only be used by persons who have previously been instructed in its proper use.

4 Safety

4.1 Explanation of Warning Symbols

WARNING

Warnings regarding possible risks of severe accident or injury.

CAUTION

Warnings regarding possible risks of accident or injury.

NOTICE

Warnings regarding possible technical damage.

5 Safety Instructions

5.1 Product-Specific Safety Instructions

NOTICE

Non-observance of safety instructions

Failure to observe the safety instructions may lead to malfunctions or defects of the product.

NOTICE

Improper use of the product

All types of improper use can lead to malfunctions or defects of the product. As a result, the unit may no longer be able to charge properly. Instruct the patient in the proper handling of the product according to the section "Patient-Specific Safety Instructions".

5.2 Patient-Specific Safety Instructions

NOTICE

Use in areas with strong electromagnetic radiation

Product malfunction

- Since certain ambient conditions (e.g. in the vicinity of high-voltage power lines, transformers, television and radio transmitters, metal detectors and security systems in department stores) can disrupt the wireless connection, which may delay or prevent stimulation from being triggered or interfere with the remote control, the recommended safe distances from the following typical sources of electromagnetic radiation must be maintained: mobile telephone – 0.99 m, DECT telephone (including base station) – 0.35 m, WLAN – 0.22 m, Bluetooth units – 0.22 m.

NOTICE

Penetration of dirt and humidity

Penetration of dirt and humidity can lead to malfunction or defects of the product. As a result, the unit may no longer be able to charge properly. Ensure that neither solid particles nor liquids can penetrate inside the product.

NOTICE

Mechanical overload

External mechanical influences or strain, such as impacts and vibration, may lead to malfunctioning or defects of the product. As a result, the unit may no longer be able to charge properly.

The product should not be exposed to any mechanical vibrations or impacts.

Check the product for any visible damage before every use.

NOTICE

Magnetic interference

The product can malfunction when near high-tension power lines, transmitters, transformers, or other sources of strong electromagnetic radiation (such as security systems for goods in department stores). As a result, the unit may no longer be able to charge properly.

Avoid proximity to sources of strong electromagnetic interference.

NOTICE

Manipulation of system components

Any changes or modifications you make to system components on your own initiative may result in malfunctions or defects of the product. As a result, the unit may no longer be able to charge properly.

The product and any damaged components may only be opened and/or repaired by certified Ottobock Myo-Service technicians.

NOTICE

Handling the electrical contacts

If the electrical contacts become contaminated or are damaged, the unit may no longer be able to charge properly.

Make sure that the contacts are always clean, free of grease and undamaged. Clean the electrical contacts of the battery charger and charging receptacle regularly using cotton swabs and a mild soap solution. Avoid damaging the contact surface coating with pointed or sharp objects.

NOTICE

Data loss of magnetic data media

Do not place the charging plug on credit cards, diskettes, audio and video cassettes or similar magnetic data carriers. The data carriers could be erased as a result.

INFORMATION

Proper charging functionality cannot be assured in case of exposure to extreme temperatures.

INFORMATION

If the charging plug has been connected to the charging receptacle, use of the prosthesis is not possible as the power supply of the prosthesis is turned off during the charging procedure.

6 Delivery/Storage

6.1 Scope of Delivery

757L35 MyoCharge Integral		
1 pc.	MyoCharge Integral	757L35
1 pc.	Power Supply 12 V	757L16=*
1 pc.	Instructions for Use	647G534

6.2 Storage

Please see the specifications in the "Technical Data" section.

7 Preparation for Use

7.1 Use/Operation

- 1) Insert the European power supply adapter or the US power supply adapter into the power supply until it snaps into place (Fig. 1).
- 2) Connect the power supply or the optionally available 4X74 Car Charging Cable to the MyoCharge Integral. Ensure correct polarity (Fig. 2).
- 3) Plug the power supply into a wall socket.
- 4) Connect the charging plug to the charging receptacle and allow it to snap into place (Fig. 3).
- 5) The charging process starts automatically if LED 1 shows blue light (Fig. 4, arrow) and the beeper of the charging receptacle sounds.
- 6) The current battery charge level is indicated by LEDs 2-5 lighting up in sequence (Fig. 5).
- 7) Once charging is complete, the charging plug can be disconnected. If the charging plug remains connected, the battery charger will switch to permanent charge (trickle charge). This will not damage the battery or charger.

INFORMATION: The charging plug may be disconnected at any time. Partially charging the battery does not shorten its lifespan (no memory effect).

8 Troubleshooting

8.1 Charging Indicators

Explanation of symbols	
LED illuminated	
LED flashing	

Battery charge level	LED 1	LED 2	LED 3	LED 4	LED 5
Battery empty					
Battery 25% charged					
Battery 50% charged					
Battery 75% charged					
Battery 100% charged					

8.2 Causes of Malfunctions and Correction

Cause of malfunction	Correction
If LED 1 lights up red, charging is not taking place. This signal may have the following causes:	
The charging plug is not properly connected to the charging receptacle	Disconnect the charging plug and connect it again
The charging plug or charging receptacle contacts are dirty	Use a cotton swab and a mild soap solution to clean the contacts of both components
The battery temperature is outside the permissible temperature range	Bring the battery within the operating temperature range (see the section "Technical Data"); then the charging process starts automatically

9 Disposal

This product must not be disposed of with regular domestic waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

10 Legal Information

10.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

10.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

10.3 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

11 Appendices

11.1 Technical Data

Service life	5 years
Max. charging current	1 A
Max. charging voltage	10 V
Operating temperature	0°C to +60°C (+32°F to +140°F)

Storage temperature	-20°C to +40°C (-4°F to +104°F)
Transport and storage temperature with and without packaging	-20°C to +70°C (-4°F to +158°F)
Battery charger operating voltage	12 V
Power supply operating voltage	100 V – 240 V
Power supply operating frequency	50-60 Hz

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Otto Bock Healthcare Products GmbH