## 453A2, 453A3, 453A4, 454A7, 454A8

<b>DE</b> Gebrauchsanweisung.....	2
<b>EN</b> Instructions for Use .....	4
<b>FR</b> Instructions d'utilisation .....	7
<b>IT</b> Istruzioni per l'uso.....	10
<b>ES</b> Instrucciones de uso .....	13
<b>PT</b> Manual de utilização .....	15
<b>NL</b> Gebruiksaanwijzing .....	18
<b>SV</b> Bruksanvisning .....	21
<b>DA</b> Brugsanvisning .....	23
<b>NO</b> Bruksanvisning .....	26
<b>PL</b> Instrukcja użytkowania .....	28
<b>HU</b> Használati utasítás.....	31
<b>CS</b> Návod k použití .....	33
<b>HR</b> Upute za uporabu.....	36
<b>EL</b> Οδηγίες χρήσης.....	38
<b>RU</b> Руководство по применению.....	41

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English

Last update: 2015-03-10

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

## 1 Application

All Ottobock Knee Sleeves are to be used for prosthetic fittings of the lower extremity only. Purpose of the Knee Sleeves is to give suspension to the prosthesis and/or to seal the socket for the use with valves or the Harmony® system.

## 2 General Safety Instructions

### **Warning!**

Patients with severe diabetes or blood circulation problems have to contact a physician before using a Knee Sleeve!

Keep the product out of reach of children – Risk of Suffocation!

Knee Sleeves should not be worn on injured skin!

## 3 Service Life

The product is a wear part, which means it is subject to normal wear and tear.

## 4 Description

### **4.1 Copolymer (TPE) Knee Sleeves**

The Knee Sleeves Derma Protection and Derma ProFlex are made from a Copolymer gel. These Knee Sleeves provide increased suspension between socket and skin while their elasticity allows a good range of motion at the knee joint. They can be used as primary suspension or with a valve or the Harmony® system. The Copolymer gel is molded with a medical mineral oil, which has a beneficial effect on the skin. During wear the oil slowly diffuses from the gel, thus lubricating and moisturizing the skin.

- Derma Protection Knee Sleeve (453A2): basic, cylindrical shape for flexible positioning; available in 4 sizes
- Derma ProFlex Knee Sleeve (453A3, 453A4): anatomical and conical shape with 15° pre-flexion for improved range of motion in the knee joint; available in 3 sizes

### **4.2 Polyurethane (PUR) Knee Sleeves**

The Knee Sleeves Harmony® and Harmony® conical are made from a Polyurethane gel. The main characteristic of the polyurethane gel is its unique yielding (flowing) effect. The Polyurethane Knee Sleeves are designed for the use with valves or the Harmony® system. They should always be used in connection with the Gaiter 454A11 to extend their longevity and retain their sealing properties (also see chapter 4.1). The Gaiter is included in the delivery.

- Harmony Knee Sleeve (454A7): slight conical shape, flexibility for positioning and a special durable textile cover; available in 5 sizes
- Harmony conical Knee Sleeve (454A8): very conical shape for improved fit with patients that have a large thigh circumference; available in 7 sizes

## 5 Instructions for Use

### **5.1 Gaiter 454A11 (only with Polyurethane Knee Sleeves)**

The Gaiter is designed to prolong the PUR Knee Sleeve longevity and is included in the delivery. The interior has a smooth Lycra® surface that will glide over the socket brim while the exterior has an airtight film surface that will seal against the Knee Sleeve. Please install the Gaiter before using the Knee Sleeve according to the following guidelines:

#### **5.1.1 Installation of the Gaiter**

- Apply the Gaiter on the exterior proximal section of the socket
- The distal Gaiter edge should be 1" (2,5cm) below the posterior socket trim line
- Tape the Gaiter to the socket with vinyl or polyethylene tape

### 5.1.2 Size Combinations Gaiter & Knee Sleeves

Size	Art.-No.	Fits PUR Knee Sleeve
Small	454A11=S	Size 1
Medium	454A11=M	Size 2 & 3
Large	454A11=L	Size 4 & 5
X-Large	454A11=XL	Size 6
XX-Large	454A11=XXL	Size 7

### 5.2 Installation and Handling of the Knee Sleeve

- Take the packaging foam out of the TPE Knee Sleeve and keep it for use when storing the sleeve
- Turn the distal half of the Knee Sleeve inside out so that the gel is on the outside
- Put it on top of the prosthesis and roll the outside turned gel down on the socket
- To don the prosthesis roll down the top half of the Knee Sleeve

Without Gaiter (TPE):	With Gaiter (PUR):
Step into the prosthesis with the liner (and a sock if needed/used)	Roll down upper part of the Gaiter over the socket brim  Step into the prosthesis with the liner (and a sock if needed/used)  Roll up the Gaiter wrinkle free

- Pull the Knee Sleeve wrinkle free over the residual limb
- To remove the prosthesis follow the steps in reverse order

## 6 Care Instructions

After use, hand wash the Knee Sleeve with Derma Clean 453H10 (or ph-neutral soap) and warm water (approx. 30°C/86°F). Do not wring! Rinse thoroughly under clear, running water. Blot the Knee Sleeve with a towel. Let it air dry completely on a rack or flat on a towel.

**Notice:** The Knee Sleeves should not be worn while wet!

For better hygiene it is recommended the Knee Sleeve be removed for a few hours each day, thus providing the skin with fresh air.

When not using the TPE suspension sleeve, insert the packaging foam back into the Knee Sleeve to allow for wrinkle free storage.

## 7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

## 7.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

## 7.3 Warranty

The manufacturer warrants this device from the date of fitting. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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Otto Bock HealthCare GmbH