1E90 Sprinter

DE Gebrauchsanweisung	2
EN Instructions for use	9
FR Instructions d'utilisation	16
IT Istruzioni per l'uso	23
ES Instrucciones de uso	31
PT Manual de utilização	38
NL Gebruiksaanwijzing	45
SV Bruksanvisning	52
DA Brugsanvisning	59
NO Bruksanvisning	66
FI Käyttöohje	72
HR Upute za uporabu	79
JA 取扱説明書	86

INFORMATION

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INFORMATION

Last update: 2015-03-25

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

The 1E90 Sprinter prosthetic foot is intended for use in a sport prosthesis. It is distinguished by its low weight. The spring contour ensures a high driving force and a low resistance.

1.2 Combination Possibilities

INFORMATION

The prosthetic socket must be able to withstand the higher loading of the sport.

	Designation	Reference number
Outer soles	Outer sole without spikes	2Z500
	Outer sole with spikes	2Z501
TF fitting for a body weight of up to 100 kg	TF test sport foot adapter	4R206
	TF definitive sport foot adapter	4R204
TT fitting for a body weight of up to 100 kg	TT test sport foot adapter	4R210
	TT definitive sport foot adapter	4R208
TT fitting for a body weight of up to 125 kg	Posterior connection plate (set)	4R420
	T-adapter for TT	2R176
	5° L-adapter	2R177=5
	18° L-adapter	2R177=18

For additional combination possibilities please consult the manufacturer.

2 Application

2.1 Intended Use

The product is intended solely for lower limb prosthetic fittings.
The product is not suitable for use in everyday prostheses.

2.2 Area of Application

Maximum permissible body weight, see table:

Stiffness version	SPR-1	SPR-2	SPR-3	SPR-4	SPR-5	SPR-6
Max. body weight [kg]	59	70	86	102	118	125
Max. body weight [lbs]	130	155	190	225	260	275

2.3 Environmental Conditions

Allowable Environmental Conditions
Temperature range for use: -10 °C to +60 °C (14 °F to 140 °F)
Allowable relative humidity 0 % to 90 %, non-condensing
Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e. g. talcum)

2.4 Service Life

A general service life for this prosthesis component as part of a sport prosthesis cannot be determined, since strain varies widely depending on the sport and intensity.

3 Safety

3.1 Explanation of Warning Symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

3.2 General Safety Instructions

 CAUTION
Exceeding the service life and reuse on another patient
Fall due to loss of functionality as well as damage to the product
▶ Ensure that the approved service life is not exceeded (see section "Service Life").
▶ Only use the product for a single patient.

► **Inform the patient.**

⚠ CAUTION

Excessive strain on the product

Fall due to breakage of load-bearing components

- Use the prosthetic components according to the classification (see section "Area of Application").
- **Inform the patient.**

⚠ CAUTION

Unallowable combination of prosthesis components

Fall due to breakage or deformation of the product

- Only combine the product with prosthesis components that are approved according to the section "Combination Possibilities".
- Based on the instructions for use of the prosthesis components, verify that they may be combined with each other.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage

- Use caution when working with the product.
- If the product is damaged, check it for proper function and readiness for use.
- In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).
- **Inform the patient.**

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- Do not expose the product to unallowable environmental conditions (see the section "Environmental Conditions").
- If the product has been exposed to unallowable environmental conditions, check it for damage.

- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e. g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).
- ▶ **Inform the patient.**

Signs of changes in or loss of functionality during use

Reduced spring effect or decreased forefoot resistance are indications of loss of functionality.

4 Scope of Delivery

The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering according to their respective symbols as single components (■), single components with minimum order quantities (▲) or single-component packs (●):

1E90 Sprinter					
Fig.	Item no.		Quantity	Designation	Reference number
-	-	■	1	Instructions for use	647G849
1	1	-	1	Prosthetic foot	-

5 Preparation for Use

CAUTION

Incorrect alignment or assembly

Injuries due to fall resulting from damaged prosthesis components

- ▶ Observe the alignment and assembly instructions.

5.1 Selection of the Stiffness Version

The stiffness of the prosthetic foot is selected depending upon the body weight and the preferred running style (short distance running or sprint/long distance running). The recommended body weight ranges for each stiffness version are listed according to running style in the following table.

Stiffness version		Short-distance running	Long-distance running
		Body weight	Body weight
SPR-1	[kg]	40 to 52	40 to 59
	[lbs]	90 to 115	90 to 130
SPR-2	[kg]	53 to 63	60 to 70
	[lbs]	116 to 140	131 to 155

Stiffness version		Short-distance running	Long-distance running
		Body weight	Body weight
SPR-3	[kg]	64 to 79	71 to 86
	[lbs]	141 to 175	156 to 190
SPR-4	[kg]	80 to 95	87 to 102
	[lbs]	176 to 210	191 to 225
SPR-5	[kg]	96 to 111	103 to 118
	[lbs]	211 to 246	226 to 260
SPR-6	[kg]	112 to 125	119 to 125
	[lbs]	247 to 275	261 to 275

5.2 Mounting the Outer Sole

CAUTION

Incorrect use of the prosthetic foot and the outer sole

Risk of falling due to lack of surface grip and damage to the product

- ▶ Only use the prosthetic foot with an outer sole.
- ▶ Only use the outer sole on the surface it was designed for.
- ▶ **Inform the patient.**

- ▶ Mount the chosen outer sole according to the corresponding instructions for use.

Connecting component	Instructions for use reference number
2Z500 Outer Sole without Spikes	647G848
2Z501 Outer Sole with Spikes	

5.3 TF Prosthesis Alignment Recommendations

For a TF fitting the prosthetic foot is connected to a sport knee joint by means of the TF test sport foot adapter or the TF definitive sport foot adapter.

INFORMATION

When using the 3S80 Sport Prosthetic Knee Joint:

- ▶ Observe the 647G813 Instructions for Use for the sport prosthetic knee joint.
- ▶ Observe the 647H537 Quick Guide for the sport prosthetic knee joint.
- ▶ Assemble the prosthetic foot according to the instructions for use for the adapter used.

Connecting component	Instructions for use reference number
4R206 TF Test Sport Foot Adapter	647G839
4R204 TF Definitive Sport Foot Adapter	

5.4 TT Alignment Recommendations

For a TT fitting the prosthetic foot is connected to the prosthetic socket by means of the suitable adapter.

- ▶ Assemble the prosthetic foot according to the mounting instructions for the selected adapter.

Connecting component	Mounting instructions
Posterior connection	
4R210 TT test sport foot adapter	Please note the 647G840 instructions for use.
4R208 TT definitive sport foot adapter	
4R420 posterior connection plate (set)	The mounting instructions are enclosed with the product.
Distal connection	
2R176 T-adapter	The mounting instructions are enclosed with the respective product.
2R177=5 5° L-adapter	
2R177=18 18° L-adapter	

6 Cleaning and Care

- 1) Clean with a damp, soft cloth and soap when needed.
- 2) Rinse the prosthesis component with clear fresh water.
- 3) Dry the prosthesis component with a soft cloth.

7 Maintenance

CAUTION

Failure to follow the maintenance instructions

Injuries due to changes in or loss of functionality and damage to the product

- ▶ Observe the maintenance instructions.
- ▶ **Inform the patient.**

- During the consultation, listen for unusual noises and inspect the entire prosthesis for damage, wear and functional changes.
- Arrange regular maintenance intervals with the patient depending on the level of use.

Direct the patient to observe the following maintenance instructions:

- Check the entire prosthesis for damage after each use.
- In case of changes in or loss of functionality and damage, do not continue using the prosthesis and have it checked by qualified personnel.
- Comply with the maintenance intervals.

8 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return and collection processes.

9 Legal Information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Otto Bock HealthCare GmbH