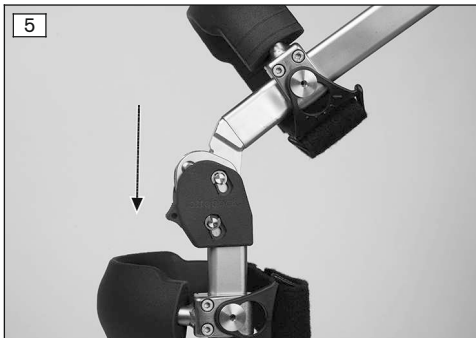
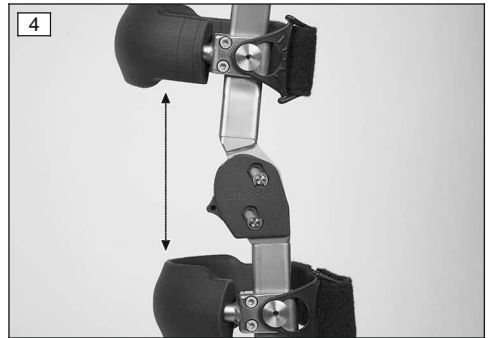
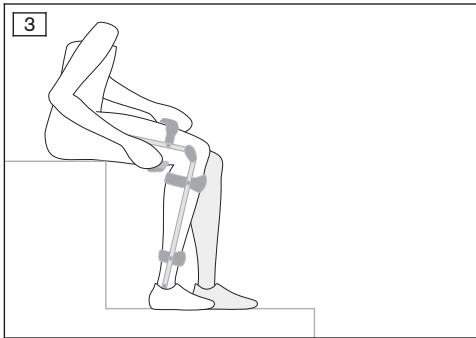
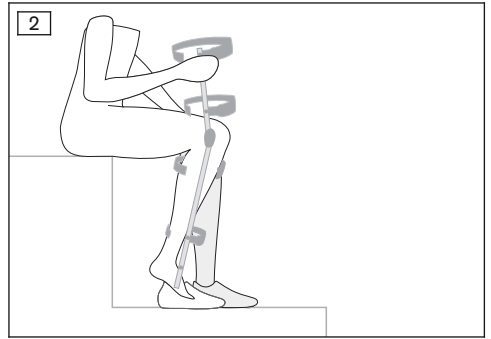
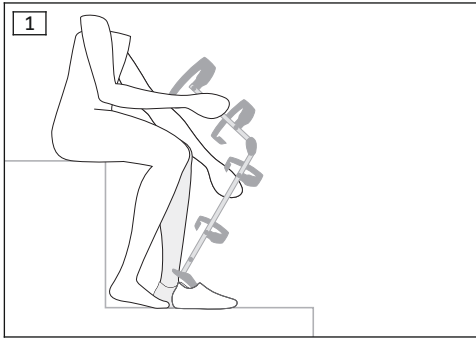
170K1=80 Free Walk, 170K1=120 Free Walk

DE Gebrauchsanweisung (Benutzer)	3
EN Instructions for use (user)	6
FR Instructions d'utilisation (Utilisateur)	10
IT Istruzioni per l'uso (Utilizzatore)	14
ES Instrucciones de uso (Usuario)	18
PT Manual de utilização (Usuário)	22
NL Gebruiksaanwijzing (Gebruiker)	26
SV Bruksanvisning (Användare)	30
DA Brugsanvisning (Bruger)	33
NO Bruksanvisning (Bruker)	37
FI Käyttöohje (Käyttäjä)	41
TR Kullanma talimatı (Kullanıcı)	45

INFORMATION

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INFORMATION

Last update: 2015-09-15

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Have yourself instructed by qualified personnel in the proper and safe use of the product.
- Please keep this document in a safe place.

These instructions for use provide you with important information on the use of the 170K1=80 and 170K1=120 Free Walk orthosis system.

2 Product Description

2.1 Design

The Ottobock Free Walk orthoses include an orthosis knee joint with lock. 3 modes can be selected on the orthosis knee joint:

- **Free Walk mode:** The knee joint locks in the stance phase and unlocks in the swing phase.
- **Permanent lock:** The knee joint is permanently locked.
- **Free motion mode:** The knee joint is permanently unlocked.

3 Application

3.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the lower limbs and **exclusively** for contact with intact skin.

4 Safety

4.1 Explanation of Warning Symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

4.2 General Safety Instructions

Operation of motor vehicles

Risk of accidents due to restricted body function

- ▶ Observe the applicable legal and insurance regulations for the operation of motor vehicles and have your driving ability examined and certified by an authorised agency.

Contact with heat, embers or fire

Injuries (e.g. burns) caused by melting of the material

- ▶ Keep the product away from open flame, embers and other sources of heat.

Excessive strain on load-bearing components

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service etc.).

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.

- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

⚠ CAUTION

Use on more than one patient

Skin irritation, formation of eczema or infections

- ▶ Only use the orthosis on one patient.

⚠ CAUTION

Improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Clean the product regularly.
- ▶ **Inform the patient.**

⚠ CAUTION

Improper handling

Risk of falling due to accidentally opening the lock

- ▶ Prior to handing over the orthosis, ensure that the locks are engaged securely and in parallel.
- ▶ Avoid accidentally opening the locks.
- ▶ Instruct the patient in the correct usage of the system joints.

⚠ CAUTION

Incorrect or excessively tight application

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the orthosis is applied properly and fits correctly.

⚠ CAUTION

Improper moulding or application

Injuries or damage to the orthosis due to overloading of the material and improper fit of the orthosis due to breakage of load-bearing components

- ▶ Do not make any improper changes to the orthosis.
- ▶ Always apply the orthosis according to the information in the instructions.

⚠ CAUTION

Using orthosis without padding

Risk of local pressure points, superficial injuries or skin irritation due to skin contact with rigid or sharp orthosis components

- ▶ Only use the orthosis with undamaged padding.

⚠ CAUTION

Improper handling

Risk of falling due to accidentally opening or engaging the lock

- ▶ Avoid accidentally opening or engaging the lock.
- ▶ Instruct the patient in the correct handling of the orthosis joints.

NOTICE

Contact with saltwater or water containing chlorine/soap, fresh water or abrasive substances (e.g. sand)

Damage and premature product wear

- ▶ Following any contact with the substances identified above, promptly clean the product in accordance with the section "Cleaning".

NOTICE

Contact with salves, lotions or other products that contain oils or acids

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to salves, lotions or other products that contain oils or acids.

NOTICE

Exposure of the product to unsuitable environmental conditions

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below $-10\text{ }^{\circ}\text{C}$ ($14\text{ }^{\circ}\text{F}$) or above $+60\text{ }^{\circ}\text{C}$ ($140\text{ }^{\circ}\text{F}$) (e.g. sauna, excessive sunlight, drying on a radiator).

5 Use

5.1 Application

- > To apply the orthosis, the user must be sitting down.
- > The orthosis is in Free Walk mode (see Page 9).
- 1) Slide the foot component of the orthosis into the shoe (see fig. 1).
- 2) Put on the shoe (see fig. 2).
- 3) Slide the orthosis over the leg from the front.
- 4) If required, adjust the width of the quick-release closures. Adjust the hook-and-loop closures to do so.
- 5) Fasten the quick-release closures (see fig. 3).

5.2 Removal

- > To remove the orthosis, the user must be sitting down.
- > The orthosis is in Free Walk mode (see Page 9).
- 1) Open the quick-release closures on the orthosis and pull it forward off the leg.
- 2) Remove the shoe.
- 3) Pull the foot component of the orthosis out of the shoe.

5.3 Selecting the function modes

Selecting Free Walk mode

- ▶ Slide the 3-phase switch to the upper position (see fig. 4).

Selecting permanent locking

- 1) Slide the 3-phase switch to the lower position (see fig. 4).
- 2) Push the orthosis ankle joint towards the body and extend the orthosis knee joint.
→ The knee joint is permanently locked.

Selecting free motion mode

- 1) Slide the 3-phase switch to the upper position (see fig. 4).
- 2) Push the orthosis ankle joint towards the body and extend the orthosis knee joint.

- 3) Flex the orthosis knee joint.
- 4) Slide the 3-phase switch to the lower position (see fig. 5).
→ The orthosis knee joint can be flexed freely at all times.

6 Cleaning

Clean the orthosis regularly:

- 1) Remove the straps from the orthosis.
- 2) Fasten all hook-and-loop closures.
- 3) Hand-wash the straps and pads in warm water at **30 °C (86 °F)** using neutral detergent. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

7 Legal Information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

7.2 CE Conformity

This custom product meets the requirements of the European Directive 93/42/EEC for medical devices according to Annex VIII. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The product does not bear the CE marking since it is a custom fabrication.

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Otto Bock HealthCare GmbH