

## 17B23, 17B45, 17B92, 17B33, 17B91, 17B23=K


DE	Gebrauchsanweisung .....	4
EN	Instructions for use .....	9
FR	Instructions d'utilisation .....	15
IT	Istruzioni per l'uso .....	21
ES	Instrucciones de uso .....	26
PT	Manual de utilização .....	32
NL	Gebruiksaanwijzing .....	38
SV	Bruksanvisning .....	44
DA	Brugsanvisning .....	49
NO	Bruksanvisning .....	54
FI	Käyttöohje .....	60
PL	Instrukcja użytkowania .....	65
HU	Használati utasítás .....	71
CS	Návod k použití .....	77
RO	Instrucțiuni de utilizare .....	82
HR	Upute za uporabu .....	88
SL	Navodila za uporabo .....	93
SK	Návod na používanie .....	98
BG	Инструкция за употреба .....	104
TR	Kullanma talimatı .....	110
EL	Οδηγίες χρήσης .....	115
RU	Руководство по применению .....	122
JA	取扱説明書 .....	128
ZH	使用说明书 .....	133
KO	사용 설명서 .....	138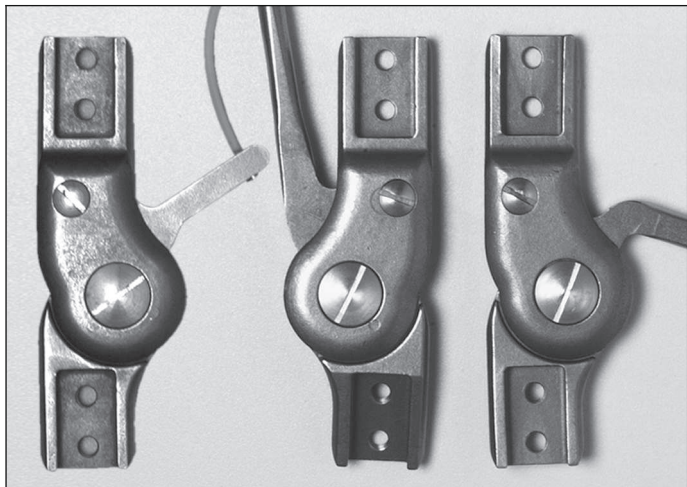


1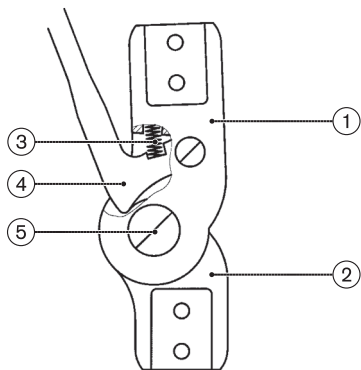

2

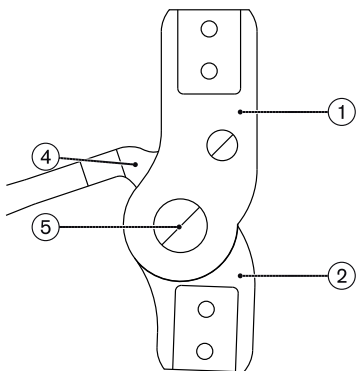3
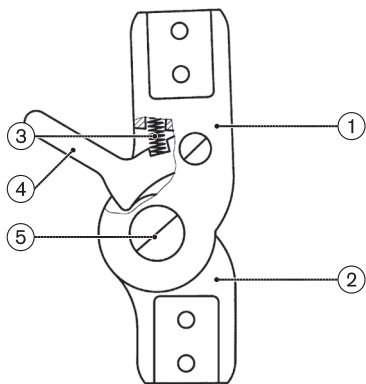
## INFORMATION

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## INFORMATION

Last update: 2016-03-23

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

These instructions for use provide you with important information on the installation and processing of the 17B23, 17B23=K, 17B33, 17B45, 17B91 and 17B92 system knee joints with Swiss locking mechanism.

## 2 Product description

### 2.1 Components/design

	Component	17B23, 17B45, 17B92 (see fig. 1)	17B33, 17B91 (see fig. 2)	17B23=K (see fig. 3)
		Reference number		
1	Upper joint section	30X59	30X59	30X59
2	Lower joint section	30U88=Z	30U88=Z	30U88
3	Compression spring	513D19=3.8x16	-	513D19=3.8x16
4	Lock lever	17Y20	17Y34	17Y104
5	Bearing nut/ joint screw	17Y93=9x9.2xM6/ 501S32=M6x14x14	17Y93=9x9.2xM6/ 501S32=M6x14x14	17Y93=9x9.2xM6/ 501S32=M6x14x14

### 2.2 Function

#### 17B23, 17B33, 17B45, 17B91, 17B92

The locking mechanism consists of a centred lock lever (4) and a centred lower joint section (2). The lock lever for system knee joints with a concealed locking mechanism and compression spring (17B23, 17B45, 17B92) points up. The lock lever for system knee joints without compression spring (17B33, 17B91) points down.

#### 17B23=K

The locking mechanism consists of an eccentric lock lever (4) and an eccentric lower joint section (2). The short lock lever has a stepped bore for the cable pull release.

## 3 Application

### 3.1 Indications for use

The system knee joints are to be used **exclusively** in pairs for the orthotic fitting of the lower limbs on a patient.

#### INFORMATION

##### Increased wear

The system knee joints are intended only for use in combination with free-motion or foot lifting system ankle joints. In a combination with system ankle joints with a dorsal stop, the components are subject to more rapid wear.

## 3.2 Indications

Partial or total paralysis of the leg muscles.

Indications must be determined by the physician.

## 4 Safety

### 4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

### 4.2 Safety instructions

 **CAUTION**

#### **Excessive strain on load-bearing components**

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service etc.).

 **CAUTION**

#### **Incorrect alignment or assembly**

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

 **CAUTION**

#### **Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

 **CAUTION**

#### **Improper handling**

Risk of falling due to accidentally opening the lock

- ▶ Prior to handing over the orthosis, ensure that the locks are engaged securely and in parallel.
- ▶ Avoid accidentally opening the locks.
- ▶ Instruct the patient in the correct usage of the system joints.

#### **⚠ CAUTION**

##### **Excessive strain due to use on more than one patient**

Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

#### **NOTICE**

##### **Thermal overloading of the product**

Damage due to improper thermal treatment

- ▶ Do not carry out any heat treatment at a temperature above 300 °C (570 °F).

#### **NOTICE**

##### **Exposure of the product to unsuitable environmental conditions**

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C (14 °F) or above +60 °C (140 °F) (e.g. sauna, excessive sunlight, drying on a radiator).

## **5 Preparation for use**

### **5.1 Assembly**

#### **Gluing the system side bars**

#### **INFORMATION**

##### **Corrosion protection**

The system side bars and system knee joints are resistant to corrosion. Finish sanding and polishing increase the protection against corrosion. Use coating powder (e.g. 618T40) for coating.

The following connecting elements are to be used for the system connection plugs:

System side bar set	17B38, 17B39, 17B40, 17B41, 17B84, 17B85
System lamination joint bars	17Y105, 17Y107

After adapting the connecting elements, the side bars must be glued into the system connection plugs for reasons of stability:

- 1) Degrease the joining surfaces with 634A1 thinner and solvent.
- 2) Mix the 636W28 special adhesive and hardener.
- 3) Apply the mixture inside the system connection plugs.
- 4) Insert the system side bar.
- 5) Fasten with 501T7=7.5x9xM5 oval head screws.
- 6) Allow to cure for at least **3-4 hours**.

**INFORMATION: Final bonding strength will be reached after 16 hours.**

### Inserting the compression spring

- ▶ Only for 17B23, 17B45, 17B92, 17B23=K: Insert the compression spring (3) into the upper joint section.

### Converting to cable pull release

- 1) Only for 17B45, 17B92, 17B33, 17B91: Replace the 30U88 lower joint section (2) with the 30U88-Z and the 17Y104 lock lever (4) with the 17Y34/17Y20.
- 2) The 17Y53 connection tube must be used for lock lever welds and solder joints.
- 3) Make sure that the locking mechanism is triggered simultaneously.

### Converting to the concealed lock

- ▶ Only for 17B23=K: Replace the 30U88 lower joint section (2) with the 30U88-Z and the 17Y104 lock lever (4) with the 17Y34/17Y20.

### Reworking the lock lever

- 1) Only for 17B23=K: If necessary, rework the lock lever (4) on the front until an engagement buffer of about 0.5 mm is achieved.  
Do **not** rework the locking surfaces of the lower joint section (2) and the lock lever (4).
- 2) Check the engagement buffer after approximately 4 weeks and rework it if necessary.

### Using a protective device

- ▶ Protect the product against dirt: Use suitable protective devices (e.g. joint protectors).

## 6 Cleaning

After contact with water containing salt, chlorine or soap, or when they get dirty, the system joints must be cleaned promptly.

- 1) Rinse the system joints with clean fresh water.
- 2) Dry the system joints with a cloth or allow to air dry. Avoid exposure to direct heat (e.g. from an oven or radiator).

## 7 Maintenance

### CAUTION

#### **Use of an incorrect lock**

Injuries from falling due to an incorrect lock being installed

- ▶ Only use locks of the same size when reinstalling or replacing.

### INFORMATION

#### **The product may be exposed to increased loads by the patient.**

- ▶ Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Only use 633F7 special lubricant.

## 8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Otto Bock HealthCare GmbH