ottobock.

Liner

DE	Gebrauchsanweisung	2
EN	Instructions for use	8
FR	Instructions d'utilisation	13
IT	Istruzioni per l'uso	20
ES	Instrucciones de uso	26
PT	Manual de utilização	32
NL	Gebruiksaanwijzing	38
SV	Bruksanvisning	44
DA	Brugsanvisning	50
NO	Bruksanvisning	55
FI	Käyttöohje	61
PL	Instrukcja użytkowania	67
HU	Használati utasítás	73
CS	Návod k použití	79
RO	Instrucțiuni de utilizare	85
HR	Upute za uporabu	91
SL	Navodila za uporabo	96
SK	Návod na používanie	102
BG	Инструкция за употреба	108
TR	Kullanma talimatı	114
EL	Οδηγίες χρήσης	120
RU	Руководство по применению	126
JA	取扱説明書	133
ZH	使用说明书	139
KO	사용 설명서	144

INFORMATION

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INFORMATION

Last update: 2016-11-21

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

INFORMATION

- ▶ The patient must be fitted with the liner no later than the date printed on the inside of the packaging or on the label (MHD/BBD).

1.1 Function

In combination with an appropriate suspension solution, liners increase wearer comfort and improve the suspension of the prosthesis. Together, they reduce the forces that affect the residual limb.

These instructions for use apply to the following liners:

TF fitting	
Material	Reference number
Polyurethane	6Y430, 6Y540

TT fitting	
Material	Reference number
Polyurethane	6Y400, 6Y416, 6Y510, 6Y512, 6Y520, 6Y521, 6Y522, 6Y523

Polyurethane (Uneo liners)

Uneo liners are made of a polyurethane gel. The material is viscoelastic. It adapts very well to the form and structure of the residual limb and protects very well against impact forces.

Ottobock recommends a total surface weight-bearing socket (the "Harmony" principle) with a harmony system or an exhaust valve.

1.2 Combination possibilities

This prosthetic component is part of the Ottobock modular system and can be combined with other products of this modular system.

2 Intended use

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

2.2 Service Life

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

 **WARNING** Warning regarding possible serious risks of accident or injury.

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

3.2 General safety instructions

 **WARNING**

Product is pulled over mouth and nose

Danger of suffocation due to lack of oxygen supply

- ▶ Store the product away from the reach of children.

 **CAUTION**

Contact with cold, heat, embers or fire

Injuries (e.g. burns) caused by melting of the material and damage to the product

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to temperatures above +80 °C or below -20 °C.

 **CAUTION**

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

⚠ CAUTION**Incorrect or excessively tight application of the product to the body**

Risk of pressure points and constriction of blood vessels and nerves due to improper application

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ Do not continue to use the product if there are problems with the correct fit.

⚠ CAUTION**Use of the product on damaged skin**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Only use the product on healthy skin.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Liner	–

5 Preparation for use

The following technical information is available if needed:

Polyurethane liners	646T2=1.8
---------------------	-----------

5.1 Adapting the liner

- > **Recommended tools for liners with textile cover:** 719S7 trimming scissors, 719S20 scissors for synthetic fibres or conventional straight edge scissors
- > **Recommended tools for liners without textile cover:** 756L9 trimming device, 756L10 liner trimmer or conventional straight edge scissors
- ▶ **NOTICE! Do not use wave edge scissors.**
Cut the liner horizontally at the required length. Ensure to obtain a smooth cutting edge to prevent the material from tearing.

5.2 Optional: attaching the pin

INFORMATION

The shape of the prosthetic socket must allow load on the end of the residual limb.

The pin to connect the liner to the shuttle lock is screwed into the thread at the distal end of the liner.

> **Required tools and materials:**

Torque wrench (adjustable to 3 Nm), 636K13 Loctite®

- 1) Apply Loctite® to the thread.
- 2) **NOTICE! Do not shorten flexible pins and pins with exhaust valve.**
Screw the pin into the thread and tighten it (installation torque: **3 Nm**).

6 Use

CAUTION

Use of powder (e.g. baby powder or talcum)

Skin irritation on the residual limb and loss of function of prosthetic components due to clogging with particles or deprivation of lubricant

- ▶ Keep the product away from powders.

NOTICE

Mechanical overload

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

INFORMATION

Some liners are treated with the Skinguard Technology additive. Skinguard Technology has an antibacterial effect and prevents the formation of unpleasant odours.

Substances in sweat containing sulphur may cause discolouration. This discolouration has no effect on the function or durability of the liner but cannot be removed. Sulphur may enter sweat through food, for example, protein products, cabbage, nuts, legumes and sulphured products (e.g. dried fruit).

Discolouration of the liner does not justify customer complaint claims.

INFORMATION

- ▶ Liners that have already been worn generally feel softer than new liners.

- ▶ Because of the relatively long drying times of polyurethane liners, a spare liner is recommended.
- ▶ Clean new liners before using for the first time.

INFORMATION

Derma Prevent may damage the outer and inner coating of polyurethane liners.

6.1 Putting on and taking off the liner

Donning

- 1) Check the liner for damage. If a pin is attached, also check the pin for damage.
- 2) Roll up the liner and place it on the end of the residual limb.
NOTICE! Align a distal connecting element (e.g. pin, metal cap) with the longitudinal axis of the residual limb.
- 3) Unroll the liner onto the residual limb so that it has no wrinkles or air inclusions, without shifting the soft tissue.
- 4) Check the fit and the alignment of the liner.
INFORMATION: Do not wrap the liner over the prosthetic socket (exception: 6Y100).

Doffing

- 1) Roll up the liner from the open end to the closed end and remove it from the residual limb.
- 2) **Recommendation:** clean the liner according to the "Cleaning and care" section.

6.2 Cleaning and Care

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

INFORMATION

Clean the liner and skin more frequently in case of increased perspiration. Wearing a liner requires daily intensive skin care. Ottobock recommends the use of the Derma skin care system.

- 1) Turn the liner inside out.

- 2) Hand wash the liner in warm water (**30 °C**) using neutral detergent (e. g. Ottobock 453H10=1 Derma Clean).
- 3) Thoroughly rinse the liner with clear water.
- 4) Put a hand towel into the liner and leave the liner to dry lying flat on a drying rack. Alternatively, use a drying stand (TT: 5Y100=1 TF: 5Y100=2).

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

7.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

7.3 Warranty

The manufacturer warrants this device from the date of fitting. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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Otto Bock HealthCare GmbH