3F1=1/3F1=2

DE Gebrauchsanweisung (Fachpersonal)	8
EN Instructions for use (qualified personnel)	24
FR Instructions d'utilisation (Personnel spécialisé)	39
IT Istruzioni per l'uso (Personale tecnico specializzato)	54
ES Instrucciones de uso (Personal técnico especializado)	70
PT Manual de utilização (Pessoal técnico)	86
NL Gebruiksaanwijzing (Vakmensen)	102
SV Bruksanvisning (Fackpersonal)	118
DA Brugsanvisning (Faguddannet personale)	133
NO Bruksanvisning (Fagpersonell)	148
FI Käyttöohje (Ammattihenkilöstö)	163
PL Instrukcja użytkowania (Personel fachowy)	178
HU Használati utasítás (szakszemélyzet)	195
CS Návod k použití (Odborný personál)	210
HR Upute za uporabu (Stručno osoblje)	225
SL Navodila za uporabo (Strokovno osebje)	240
SK Návod na používanie (Odborný personál)	255
TR Kullanma talimatı (Uzman personel)	270
RU Руководство по применению (Квалифицированный персонал)	285
JA 取扱説明書 (有資格担当者)	302
ZH 使用说明书 (专业人员)	316

Quick Reference Guide

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INFORMATION

Date of last update: 2017-08-11

- ▶ Please read this document carefully before using the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.
- ▶ Please keep this document in a safe place.

The product "3F1=1 functional cosmesis C-Leg / 3F1=2 functional cosmesis Genium" is called the product/functional cosmesis in the following.

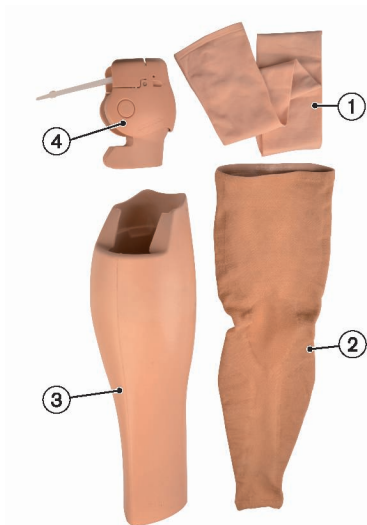
These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product description

2.1 Design

The functional cosmesis consists of the following components (example 3F1=1):

1. Sheath
2. Functional stocking (not included in the scope of delivery)
3. Functional shank C-Leg / functional shank Genium (referred to as "functional shank" in the following)
4. Functional cosmesis C-Leg (knee part) / functional cosmesis Genium (knee part) (referred to as "functional knee part" in the following)

2.2 Function

The product helps create the most natural possible appearance. The product forms the exterior finish of a prosthesis. The product is not required for the proper and safe functioning of the prosthesis.

The functional shank is shaped by the O&P professional to create the most natural possible appearance. Streaks, colour differences and small air pockets (up to 1 cm) in the functional shank are concealed by the functional stocking. These differences, which are due to production

reasons, do not affect the function of the product and therefore do not constitute grounds for complaints.

The functional stocking finishes the outside of the functional cosmesis. Various colours and sizes are available.

Use the table in the quick reference guide to determine the size.

INFORMATION

Size L*

Size S of the functional stocking cannot be used due to the body height. Additional socket volume has to be created for size L so the suspension of the functional stocking on the socket can be ensured.

The sheath improves the sliding capability of the functional stocking.

2.3 Combination possibilities

This product can be combined with the following Ottobock components:

Knee Joint Combination Possibilities

- 3C98-3, 3C88-3 C-Leg
- 3B1-2, 3B1-2=ST Genium

Prosthetic Foot Combination Possibilities

- 1C60 Triton
- 1C61 Triton Vertical Shock
- 1C63 Triton Low Profile
- 1C64 Triton Heavy Duty
- 1C30 Trias
- 1D10 Dynamic foot
- 1D11 Dynamic foot
- 1M10 Adjust
- Terion K2: 1C11
- 1A30 Greissinger plus
- 1D35 Dynamic Motion
- 1C40 C-Walk
- 1E56 Axtion
- 1E57 Lo Rider
- 1B1 Meridium
- 1C66 Triton smart ankle
- 1C10 Terion

Adapter combination possibilities

- 4R57, 4R57=ST rotation adapter

Limited compatibility:

C-Leg: Collision with functional knee part from a tilt angle of 4.3° in the direction of the rotation adapter button

Genium: Collision with functional knee part from a tilt angle of 3.2° in the direction of the rotation adapter button

3 Application

3.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

The product serves to finish the prosthesis in the sense of restoring a natural appearance that is as inconspicuous as possible.

3.2 Area of application

The product is intended exclusively for wearers of the 3C98-3/3C88-3 C-Leg and 3B1-2/3B1-2=ST Genium.

The product can be used with a knee axis-floor measurement of up to 560 mm. The minimum knee axis-floor measurement that can be fitted depends on the foot that is used with the tube adapter at maximum insertion.

3.3 Conditions of use

The product is intended **exclusively** for use on **one** user. Use of the product by another person is not approved by the manufacturer.

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

3.4 Indications

- For patients with knee disarticulation and transfemoral amputation
- The patient must be able to understand usage and safety messages and put them into practice.

3.5 Qualification

The fitting of patients with the product may only be carried out by qualified personnel (e.g. a prosthetist).

4 Safety

4.1 Explanation of warning symbols

 Warning regarding possible risks of accident or injury.

 Warning regarding possible technical damage.

4.2 Structure of the safety instructions

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General safety instructions

Non-observance of safety instructions

- > Injury due to faulty control or malfunction of the prosthesis.
- > Destruction of prosthesis.
- ▶ Observe the safety instructions and the stated precautions in this accompanying document.

Incorrect protective equipment

Injury due to defective or lack of protective equipment.

- ▶ Wear protective goggles and a dust mask during grinding.

⚠ CAUTION

Contact of the product with embers or fire

Injuries (e.g. burns) and damage to the product.

- ▶ Keep the product away from embers and open flames.

⚠ CAUTION

Wearing the product without a functional stocking

> Injuries due to pinching of body parts.

> Damage due to pinching of garments.

> Damage to the product.

> Loss of functionality due to the penetration of moisture into the foam.

- ▶ The product may only be worn with the functional stocking.

⚠ CAUTION

Improper product care

Injury due to the use of improper cleaning agents.

- ▶ Clean the product only as described in the section "Cleaning and care" (Cleaning and care).

⚠ CAUTION

Failure to observe warnings/error signals

Falling due to unexpected behaviour of the prosthesis because of changed damping behaviour.

- ▶ Observe the warnings/error signals and corresponding change in damping settings.

⚠ CAUTION

Skin irritation due to irregular care of the product

> Skin irritation due to contact with a soiled functional shank.

> Skin irritation due to contact with a soiled sheath.

- ▶ Clean the product only as described in the section "Cleaning and care" (Cleaning and care).

- ▶ The product may only be worn with the functional stocking.

⚠ CAUTION

Thermal overload due to increased heat accumulation

Falling due to altered damping behaviour as a result of earlier switching to overheating mode.

- ▶ Observe the warnings/error signals of the prosthesis (overheating mode).

⚠ CAUTION

Impaired perception of the feedback signals

Falling due to impaired perception of the feedback signals resulting from changed damping behaviour.

- ▶ The audibility of the feedback signals must be checked with the user and the corresponding volume has to be set.

⚠ CAUTION

Use of unsuitable prosthesis components

Falling due to unexpected behaviour of the product or breakage of load-bearing components.

- ▶ Use the product only in combination with components listed in the section "Combination possibilities" (see Page 25).

NOTICE

Independent user changes or modifications made to the product

Damage to the product.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage.

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

INFORMATION

The penetration of dirt and humidity into the product leads to a loss of comfort. Observe the instructions for use for the prosthesis (647G890 / 647G1148).

INFORMATION

The product can limit the range of the Bluetooth signal. Observe the instructions for use for the prosthesis (647G890 / 647G1148).

5 Scope of Delivery and Accessories

5.1 Scope of delivery

- 1 pc. Functional shank
- 1 pc. Functional knee part
- 1 pc. Sheath
- 1 pc. Installation ring for 4X259 inductive charger (only for 3F1=2)
- 1 pc. Screwdriver with wing handle
- 4 pc. Cable ties
- 1 pc. 647G1290 instructions for use (user)
- 1 pc. 647G1289 instructions for use (qualified personnel)
- 1 pc. 647G1294 additional documentation

5.2 Accessories

The following components are not included in the scope of delivery and must be ordered separately:

- 1 pc. Functional stocking

Accessories for the 3F1=1 functional cosmesis C-Leg

- 1 pc. 4X157 charger extension cable – knee
- 4H105 knee extender

6 Preparation for use

6.1 Required tools

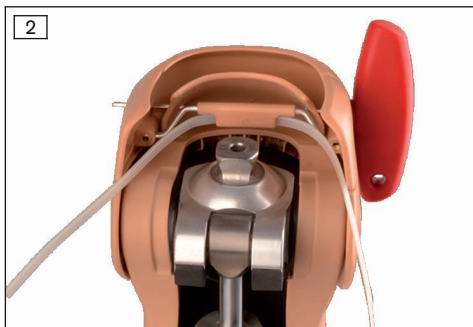
- Adhesive tape
- 636N9 contact adhesive
- Measurement equipment: folding rule
- Scissors
- Scriber
- 4X258 installation tool for inductive charger
- 710D4 torque wrench (up to 30 Nm)
- Allen key size 2
- Allen key size 4
- Double-sided adhesive tape
- Protective goggles
- Dust mask
- Pedilen or Plastazote (thickness 5 mm)

6.2 Preparation of the functional knee part


Complete the following steps to put on the functional knee part:

- 1) Slightly expand the functional knee part and pull it over the frame of the joint.

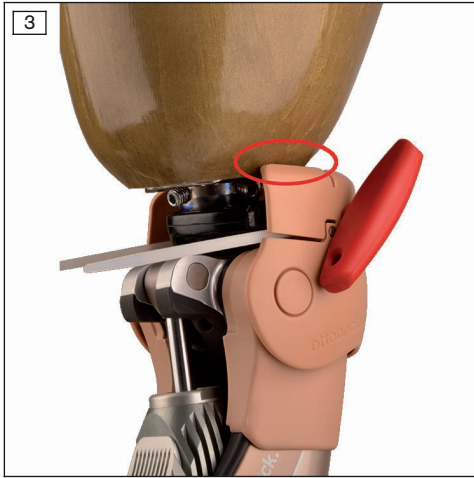- 2) Secure the position of the upper knee part with the Allen key size 2.

- 3) Mount the knee joint with adapter on the socket.
- 4) Fully extend the knee joint.
INFORMATION: Use the 4H105 knee extender for the C-Leg.
- 5) Check the upper knee part for collisions with the socket.
- 6) Mark the collision points on the upper knee part.

6.3 Adapting the functional knee part

⚠ CAUTION

Incorrect protective equipment

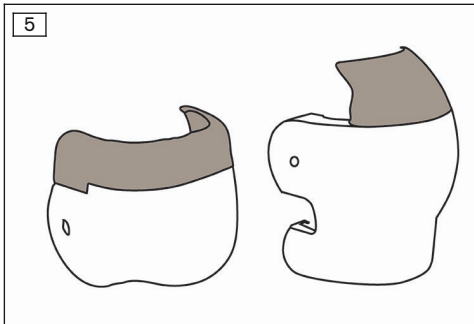
Injury due to defective or lack of protective equipment.

- ▶ Wear protective goggles and a dust mask during grinding.


Complete the following steps to adapt the functional knee part:

- 1) Press the clips out of the functional cosmesis (knee part) and remove the upper knee part from the lower knee part.

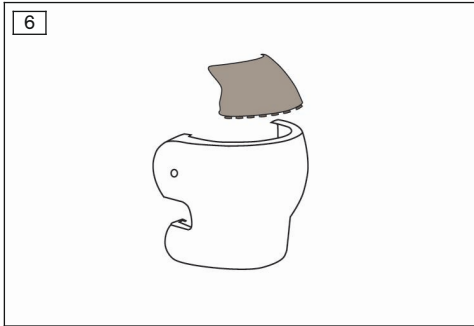- 2) Sand the upper knee part in the required area (see socket collision marking).
INFORMATION: Sanding the white area is not permitted.
→ If the functional knee part and socket continue to collide, the upper knee part can be removed together with the clips.


→ If the socket and lower knee part continue to collide, the ball top can be removed (see fig. 6).

- 3) Carefully deburr the sanded areas.

6.4 Assembling the functional knee part

Complete the following steps to assemble the functional knee part:

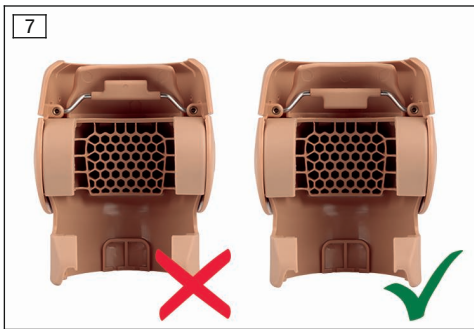- 1) Loosen the screws on the functional knee part.

- 2) Check the alignment of the actuator stirrup.

INFORMATION: The stirrup points up, the wing points down.

- 3) Install the charger extension cable (see the 647G1146 instructions for use).
- 4) Assemble the charger extension cable and O-rings so that the ribs of the functional knee part do not press on the cable and O-rings.
INFORMATION: Note the recess on the functional knee part.
- 5) Assemble the functional knee part on the joint.
INFORMATION: Verify the form fit of the functional knee part on the joint.
- 6) Secure the position of the functional knee part with the Allen key size 2.
- 7) Bring the joint into an extended position.
INFORMATION: Use the knee extender for the C-Leg.
- 8) Fasten the cable tie and tighten the adapter mounted directly on the knee.
- 9) Verify the proper fit of the cable tie.
- 10) Tighten the two screws on the functional knee part using the supplied screwdriver with wing handle.
- 11) Remove the Allen key.

6.5 Finishing the functional shank

⚠ CAUTION

Incorrect protective equipment

Injury due to defective or lack of protective equipment.

- ▶ Wear protective goggles and a dust mask during grinding.

Shortening the functional shank

Complete the following steps to shorten the functional shank:

- 1) Bring the prosthesis to full extension.
- 2) Measure from the bottom edge of the upper knee part to the top edge of the foot connection plate.
- 3) Add 15 mm to the measured length and transfer this to the functional shank.
INFORMATION: On the functional shank, measure from the top to the bottom.
- 4) Cut off the functional shank at the marked location.
INFORMATION: Note the foot position for an optimum fitting result.

Adaptation for a fitting with the Triton Vertical Shock

For a fitting with a Triton Vertical Shock, the functional shank has to be adapted accordingly on the lower side.

The following steps have to be completed to adapt the functional shank:

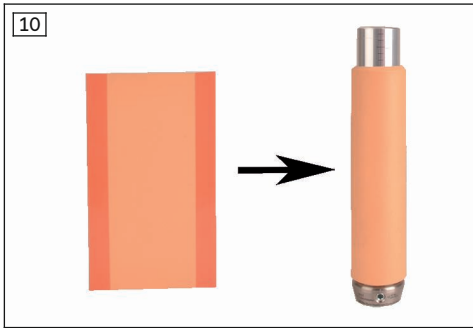
- Enlarge the opening on the lower side of the functional shank.

Adaptation for fitting feet with a small ankle circumference (optional)

Material can be applied to the tube adapter for fittings with feet that have a low structural height (such as the 1D35, Triton LP).

Complete the following steps to adapt the tube adapter:

- 1) Measure the length of the tube adapter (from the bottom edge of the joint to the start of the pyramid receiver of the tube adapter).
- 2) Cut a rectangle of the measured length and a width of 122 mm out of the material Pedilen or Plastazote (5 mm thickness).

- 3) Attach the material to the tube adapter with double-sided adhesive tape.

INFORMATION: Do NOT glue the material to the functional shank.

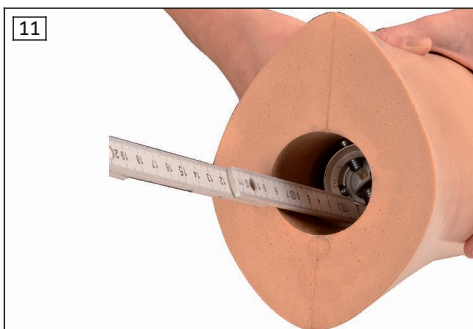
Exposing the tube adapter screws for feet with a high structural height

Complete the following steps to expose the tube adapter screws:

- 1) Remove the prosthetic foot.

INFORMATION: Using two Allen keys size 4 for the disassembly and assembly of the prosthetic foot is recommended.

- 2) Secure the screws to prevent changing the foot position.
- 3) Turn the two opened screws completely into the tube adapter.
- 4) Insert the knee joint with tube adapter into the functional shank.

- 5) Measure the distance between the bottom edge of the functional shank and the tube adapter screws.

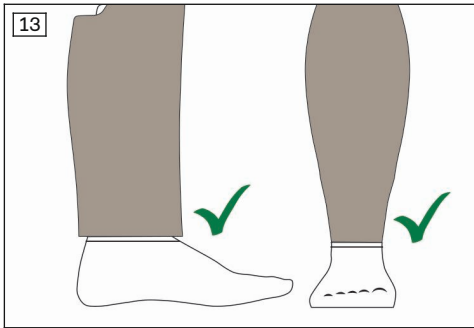
- 6) Mark the measured distance to the 4 screws on the outside of the functional shank.
- 7) Remove the knee joint and tube adapter.
- 8) With a pointed needle or a metal pin heated with a blow dryer, drill a hole in the functional shank at the marked locations.
- 9) Insert the knee joint with the tube adapter into the functional shank.
- 10) Assemble the prosthetic foot through the previously drilled holes.

6.6 Sanding the functional shank

Sand the functional shank according to the usual standards.


The following steps should be observed for sanding the functional shank:

- 1) Protect the footshell with adhesive tape.

- 2) In the area of the ankle, do not sand the functional shank flush with the foot cosmesis.

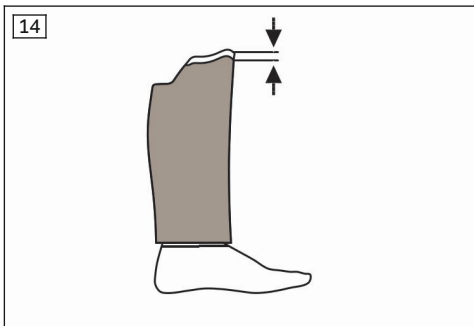INFORMATION: Leave more material at the front and rear. This is compressed in the ankle area by the functional stocking.


- 3) Sand the functional shank.
INFORMATION: Do not sand the transition (area marked in white).
- 4) **For the 3F1=2 functional cosmesis Genium:** In the calf area, sand the leg contour so that it is flush with the installation ring for the inductive charger (4X259).

6.7 Assembling the sheath


Complete the following steps to assemble the sheath:

- 1) Pull the sheath over the functional shank.

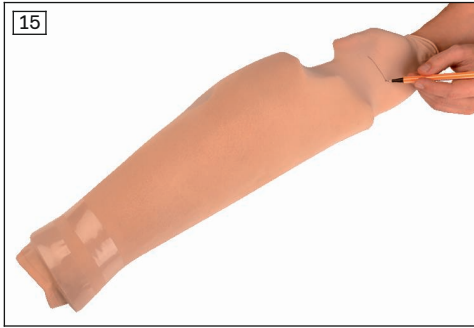INFORMATION: The sheath has to fit over the full length with no longitudinal tension on the foam.

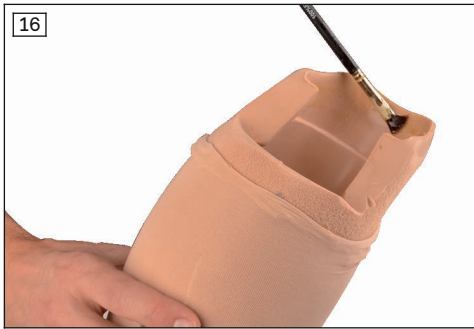- 2) Cut off the sheath at the top with an overhang of 6 cm.

- 3) Secure the sheath on the inside of the foam with contact adhesive.

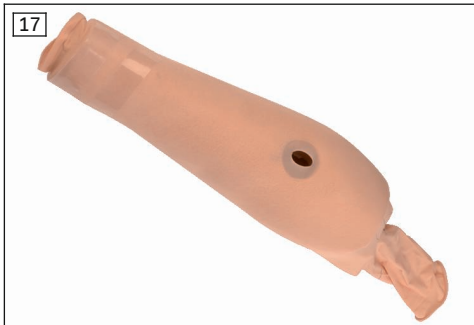
INFORMATION: Note the setting time of the contact adhesive depending on the application thickness and temperature.

- 4) Leave the sheath hanging in the ankle area. The sheath must not be glued or cut off in the ankle area.
- 5) Punch the 4 holes for the tube adapter screws in the sheath as well.

INFORMATION: Apply contact adhesive to the holes to prevent runs in the sheath.


Preparing the charging unit for the Genium

Complete the following steps to assemble the charging unit for the Genium:

- 1) Make a longitudinal cut in the sheath.

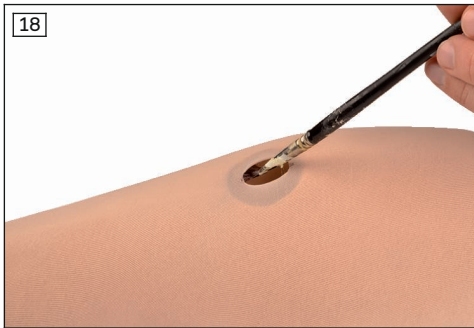INFORMATION: The slit has to be above the opening in the foam.

- 2) Secure the sheath on the inside of the opening with contact adhesive.

- 3) Press the installation ring for the inductive charger (4X259) into the opening so it is flush with the foam and sheath.
INFORMATION: Please note the 647G1148 instructions for use.
- 4) Assemble the charging unit flush on the outside of the foam.
INFORMATION: Please note the 647G1148 instructions for use.

6.8 Finishing the functional cosmesis

Complete the following steps to finish the functional cosmesis:

- 1) Assemble the functional shank, knee joint and prosthetic foot.

- 2) Pull the functional stocking over the assembly.
INFORMATION: The functional stocking should extend 1 cm beyond the foot cosmesis.

INFORMATION

The inductive charger can be used with the 3F1=2 functional cosmesis Genium after putting on the functional stocking. Exposing the charging unit is not necessary.

Putting on the functional stocking

Complete the following steps to put on the functional stocking:

21

- 1) Fold over the coated ankle part of the functional stocking (coating faces out).
- 2) Pull the functional stocking up over the functional cosmesis.

INFORMATION: Note the alignment of the functional stocking (see fig. 21).

7 Cleaning and Care

7.1 Cleaning the functional cosmesis

- ▶ Clean the product with a damp, soft cloth and then let it dry.

7.2 Cleaning the functional stocking

- ▶ Wash the product at 30 °C/86 °F in a washing machine (setting: easy care or delicates).

INFORMATION

Note the washing symbols on the label.

7.3 Care of the functional stocking

- Wipe the silicone band on the inside of the functional stocking with a damp cloth every three days.

8 Legal information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

8.3 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9 Technical data

Environmental conditions	
Storage and transport in original packaging	-25 °C/-13 °F to +70 °C/+158 °F
Storage and transport without packaging	-25 °C/-13 °F to +70 °C/+158 °F Max. 93% relative humidity, non-condensing
Operation	-10 °C/+14 °F to +60 °C/+140 °F 15% to 93% relative humidity, non-condensing

General information	
Reference number	3F1=1 / 3F1=2
Service life of the functional cosmesis	2 years The service life diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximum service life, in dependence of the level of use.
Service life of the functional stocking	6 months Wear part, subject to natural wear and tear. The service life diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximum service life, in dependence of the level of use.
Weight of the functional cosmesis (without adaptations)	910 g

10 Appendix

10.1 Symbols Used

Legal manufacturer

LOT:PPPP YYYY WW

Lot number

Declaration of conformity according to the applicable European directives

Do not dry clean

Iron at medium temperature, max. 150°

Do not tumble dry

Gentle cycle at 30°

Do not bleach

Not made with
natural rubber latex.

Natural rubber latex was not used as a material

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Otto Bock Healthcare Products GmbH