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1.0 Quick Release Mounting Hardware

Stealth's User Manual and Maintenance Guide for ADI's Quick Release Mounting Hardware

1.1 Customer Satisfaction

Stealth Products strives for 100% customer satisfaction. Your complete satisfaction is important. Please contact us with feedback or suggested changes that will help improve the quality and usability of our products. You may reach us at:

Stealth Products, LLC

104 John Kelly Dr.

Burnet, TX 78611

Phone: (512) 715-9995

Toll Free: 1 (800) 965-9229

Fax: (512) 715-9954

Toll Free: 1 (800) 806-1225

info@stealthproducts.com

www.stealthproducts.com

MDSS GmbH

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30175 Hannover, Germany

General

Read and understand all instructions prior to the use of the product. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse due to failure of the following instructions will void the warranty.

Immediately discontinue use if any function is compromised, parts are missing, loose, or show signs of excessive wear. Consult with your supplier for repair, adjustment, or replacement.

If this document contains information you do not understand, or there are concerns about safety or operation, contact your supplier.

1.2 Important Information

Important Information!

All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must:

- Read and understand all instructions and warnings.
- Maintain our products according to our instructions on care and maintenance.
- Devices should be installed and adjusted by a trained technician.

Supplier Reference

Supplier: _____

Telephone: _____

Address: _____

Purchase Date: _____

Model: _____

1.3 Introduction

Before you install or begin using this product, it is important that you read and understand the content of these operating instructions. These instructions will guide you through the options and possibilities with the Quick Release Mounting Hardware.

Instructions are written with the expressed intent of use with standard configurations. They also contain important safety and maintenance information, as well as describe possible problems that can arise during use. For further assistance or more advanced applications, please contact your supplier or Stealth Products at (512) 715-9995 or toll free at 1-800-965-9229.

Always keep the operating instructions in a safe place so they may be referenced as necessary.

All information, pictures, illustrations, and specifications are based on the product information that was available at the time of printing. Pictures and illustrations shown in these instructions are representative examples and are not intended to be exact depictions of the various parts of the product.

Ordering Documentation

You can download additional copies of this user manual on the Stealth website:

<http://www.stealthproducts.com/gui/docs/?type=usr>

Or search:

Quick Release Mounting Hardware User Manual in the search bar at the top of the page.

1.4 Warranty

Our products are designed, manufactured, and produced to the highest of standards. If any defect in material or workmanship is found, Stealth Products will repair or replace the product at our discretion. Any implied warranty, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of this warranty. Stealth Products, LLC does not warrant damage due to, but not limited to:

- Misuse, abuse, or misapplication of products.
- Modification of product without written approval from Stealth Products, LLC.
- Any alteration or lack of serial number, where applicable, will automatically void this warranty.
- Stealth Products, LLC is liable for replacement parts only.
- Stealth Products, LLC is not liable for any incurred labor costs.

No person is authorized to alter, extend, or waive the warranties of Stealth Products, LLC.

Stealth Products warrants against failure due to defective materials or workmanship:

Covers: 180 days

Hardware: 5 years

Electronics: 3 years

In Case of Product Failure

In the event of product failure covered by our warranty, please follow the procedures outlined below:

1. Call Stealth at +1 (512) 715-9995 or toll free +1-800-965-9229.
2. Request the Returns Department or obtain an RA from the Returns Department and follow department or documentation instructions.

1.0 Quick Release Hardware

1.1 Customer Satisfaction	20
1.2 Important Information.....	20
1.3 Introduction	21
1.4 Warranty	21
1.5 Table of Contents.....	22
1.6 Warning Labels.....	24
1.6.1 Warning Labels.....	24
1.6.2 Limited Liability.....	24
1.6.3 Testing	24
1.7 Design and Function.....	25
1.7.1 Intended Use.....	25
1.7.2 Features.....	25
1.7.3 Mounting.....	25
1.8 Parts and Accessories	25
1.8.1 Quick Release Hardware Package.....	25
1.8.2 Quick Release Replacement Parts.....	26
1.9 Features.....	27
1.9.1 Quick Release Hardware Features	27
1.10 Installation Instructions	29
1.10.1 Tools.....	29
1.10.2 Installation Plan.....	29
1.10.3 Installation.....	29

2.0 WC20 Approved Quick Release Mounting Hardware

2.1 Testing	31
2.1.1 WC20 Quick Release Hardware	31
2.2 WC20 Transit Information	31
2.2.1 General Guidelines.....	31
2.2.2 Before Transit	31
2.3 WC20 Safety	33
2.3.1 User Safety.....	33
2.4 WC20 Hardware Installation	34
2.4.1 WC20 Hardware Installation	34
2.4.2 WC20 Label.....	34
2.4.3 Removing the WC20 Pin.....	35
2.5 Maintenance	35
2.5.1 Cleaning.....	35
2.5.2 Maintenance	35
2.6 First Time Use	36
2.6.1 Dealer Assistance	36
2.6.2 User Testing	36
2.6.3 Conditions of Use.....	36

1.6.1 Warning Labels

Warnings are included for the safety of the user, client, operator and property. Please read and understand what the signal words **SAFETY**, **NOTICE**, **CAUTION**, **WARNING** and **DANGER** mean, how they could affect the user, those around the user, and property.

⚠ DANGER	Identifies an imminent situation which (if not avoided) will result in severe injury, death, and property damage .
WARNING	Identifies a potential situation which (if not avoided) will result in severe injury, death, and property damage .
⚠ CAUTION	Identifies a potential situation which (if not avoided) will result in minor to moderate injury, and property damage .
NOTICE	Identifies important information not related to injury, but possible property damage .
SAFETY	Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

1.6.2 Limited Liability

Stealth Products, LLC accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

1.6.3 Testing

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

The Quick Release hardware should always be tested without any person sitting in the wheelchair until every alteration of the physical installation or adjustment is complete.

1.7.1 Intended Use

ADI's Quick Release mounting hardware is designed to allow wheelchair users and assistants the ability to easily remove the back support with one hand. In addition, the Quick Release mounting hardware offers angle, depth, and width adjustability.

1.7.2 Features

The Quick Release Mounting Hardware:

- Angle adjustment without the loss of seat depth.
- Depth and width adjustment.
- Compatible with aluminum and carbon fiber backs.
- Compatible with all contours.

1.7.3 Mounting

The Quick Release hardware is designed to be removable and have additional width, angle and depth adjustments if needed. This hardware has also been WC20 tested and is approved to be used in transit ONLY if it is installed on a WC20 approved seat and WC19 approved chair frame.


1.8.1 Quick Release Hardware Package

The following hardware is included in a Quick Release package:

Part Number	Product Description	
BR1102	Quick Release Complete Mounting Kit	

1.8.2 Quick Release Replacement Parts

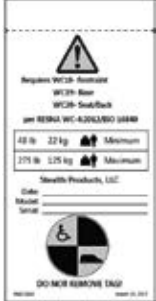
Part Number	Product Description	
BR1100	Quick Release Back Support Clamp	
BR1101	Quick Release Latch Assembly	
BR1102 with BR6005	Quick Release Latch Assembly with Pin* *Hardware is WC20 compliant with Pin	
BR6005	Pin Package	 

1.9.1 Quick Release Hardware Features

The Quick Release hardware includes features such as angle, width, and depth adjustments.


Angle Adjustment

- Approximately 20° of adjustment.
- The loss of seat depth does not occur with the angle adjustment.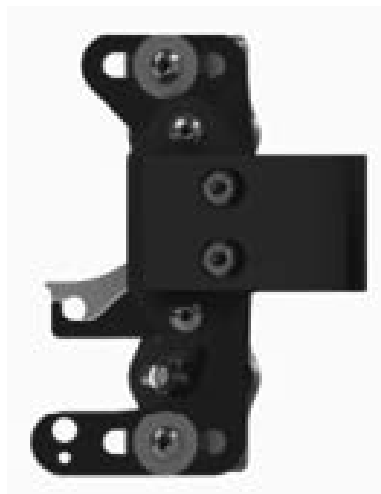

Width Adjustment

- Approximately ±1in. (2.54cm) of width adjustment is available.
- By loosening the screw on the latch assembly, the hardware can then be adjusted in an inward or outward direction from the chair.

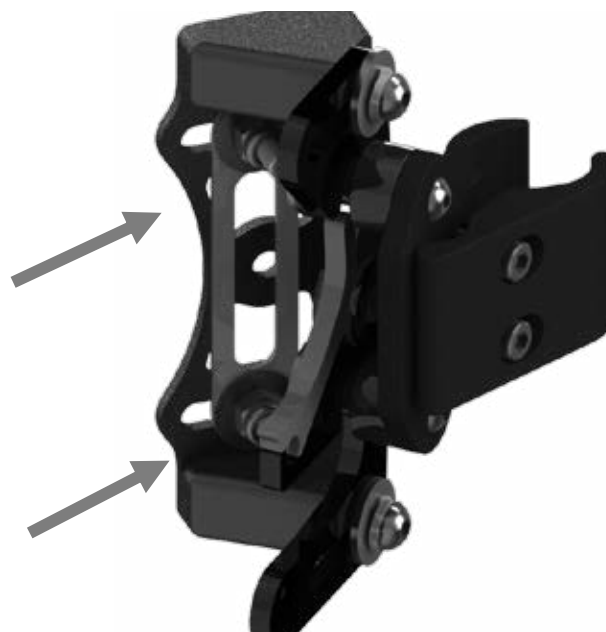Depth Adjustment

- Approximately 1^{1/4}" (3.18cm) of depth adjustment.
- Center mounting of hardware positions the back cushion valley behind the back posts.
- Forward mounting of hardware positions the back cushion forward 5/8".
- After mounting of the hardware positions the back cushion rearward 5/8".

* All Quick Release Hardware shown without WC20 pin package.

Cane Compatibility

Quick Release hardware mounts to multiple diameter cane clamps without the use of spacers:

- Hardware is compatible with 1", 3/4", and 7/8" cane diameters.

1" Clamp

3/4" Clamp

7/8" Clamp

WARNING

Only a qualified service technician may install the Quick Release Hardware.

1.10.1 Tools

- 10mm wrench
- 4mm Allen wrench/T-Handle
- Torque Wrench

1.10.2 Installation Plan


Set up an installation plan before installing the quick release hardware onto the back. Based on the users' needs, this plan should specify:

- Where the clamps should be located on the canes.
- The proper height of the hardware for the user.
- The correct angles and depths of the hardware.

1.10.3 Installation

- Install the stud plate and u-block onto the back shell along the pre-cut slots.
- Attach the quick release latch assembly to the u-block and secure the screws. Refer to the installation manual for the correct torque specifications required for tightening the hardware.
- Determine the height of the hardware on the wheelchair back posts. Attach and secure the clamp assembly at the desired height.
- Determine the desired width, angle and depth to which the hardware should be placed. Make the appropriate adjustments and tighten to secure.
- To attach the back clamps to the quick release hardware, lower the left and right bottom hooks onto the receiving posts.
- When the receiving post is attached, manually press the left and right locking levers to secure the hardware.

If using the WC20 Quick Release hardware, insert the locking pin into the latch assembly lock piece as the final step in installation.

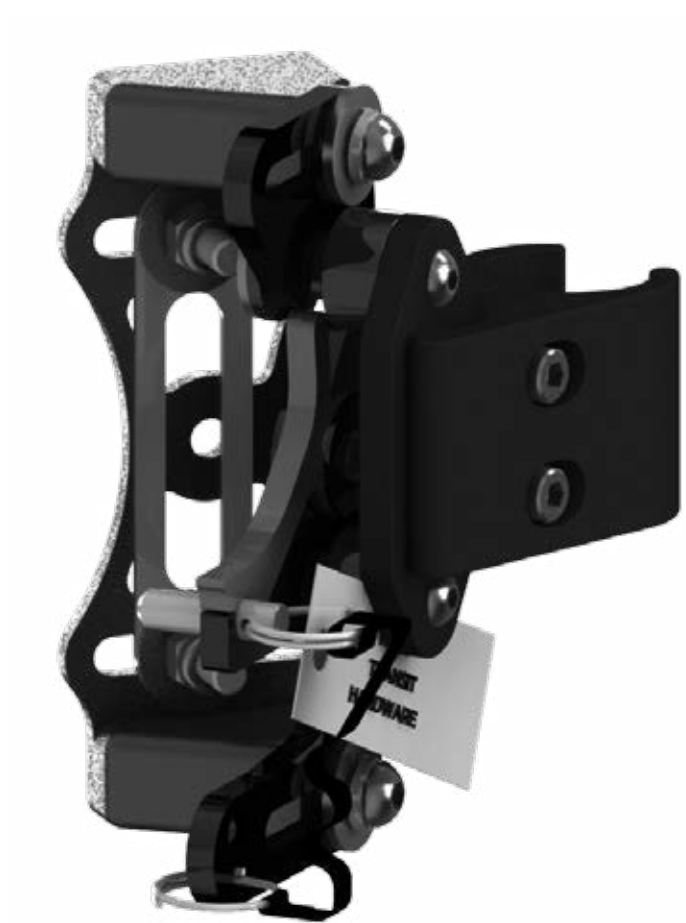2.0 WC20 Approved Quick Release Mounting Hardware

Stealth's User Manual and Maintenance Guide for ADI's Quick Release Mounting Hardware

2.1.1 WC20 Quick Release Hardware

The Quick Release mounting hardware has met WC20 and ISO16840-4 requirements and provisions. The hardware can be ordered as a WC20 approved item with transit pins.

Transit pins were added to the hardware to ensure secured points in the event that the Quick Release hardware could possibly become loose during transit and detach from the ADI back.

NOTICE

Keep these pins attached to the chair at all times.

WC20 establishes design and performance requirements and test methods for complete seating systems that consist of a seat, back support, and attachment hardware.

⚠ DANGER

The WC20 approved Quick Release Hardware and AL/AFT backs are designed **ONLY** for use with wheelchair frames that have been successfully crash tested to the frontal-impact requirements of RESNA WC-4:2012, Section 19.

2.2 WC20 Transit Information

2.2.1 General Guidelines

- Attach the tie-down straps only to the designated labeled transit security points.
- Tighten the straps sufficiently to remove all slack. Never attach tie-downs to adjustable, moving, or removable parts of the wheelchair.
- Position the anchor points for the rear tie-down straps directly behind the rear securement points on the wheelchair.
- The front tie-down straps should anchor to floor points that are spaced wider than the wheelchair to provide increased lateral stability.

⚠ CAUTION

The WC20 approved Quick Release Hardware and AL/AFT backs should only be used as described in the manufacturer's instructions.

2.2.2 Before Transit


- Client's who use wheelchairs should transfer to the vehicle seat and use the vehicle installed occupant-restraint systems.
- Prior to transporting, always double check the tie-downs and hardware.
- On tie-downs and restraints, look for tears or worn webbing straps. Replace as necessary.
- Look for damaged hooks or loose tie-down hardware.
- On the seating system, check for loose screws attaching the hardware to the back. Check for loose screws attaching the hardware to the seat. These should be snug and not overtightened.
- The Quick Release hardware should be securely attached to the back, and the transit pins should be properly inserted into the hardware.

⚠ DANGER

The WC20 approved Quick Release hardware and the AL/AFT backs are designed specifically for occupancy when used on wheelchairs that are facing forward in motor vehicles.

Belt Placement

- The pelvic belt restraint should be worn low, across the front of the pelvis near the junctions of the pelvis and the thighs.
- The belt restraint buckle of three-point belt restraints should be placed in contact with the occupant's body and away from wheelchair components.
- The junction of the shoulder belt and pelvic belt of three-point-belt restraints should be located near the hip opposite the shoulder over which the diagonal belt crosses and not near the midline of the occupant.

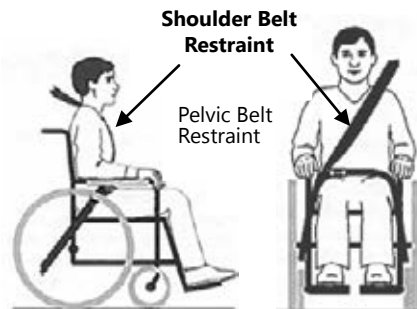Proper Positioning of Belts on Occupant

Upper torso belt restraints should fit directly over, and in contact with, the middle of the shoulder.

⚠ CAUTION

Belt restraints should be adjusted to fit as snugly as possible, consistent with user comfort.

SAFETY

Belt restraints should be positioned on the wheelchair occupant in accordance with WTORS manufacturer's instructions.


Improper Positioning of Belts on Occupant

- Belt restraints should not be routed outside of the wheelchair wheels or over the wheelchair arm supports and should not be held away from the body by wheelchair components or parts.

SAFETY

Belt restraints should not be worn twisted in a manner that reduces the area of contact of the belt webbing with the occupant.

Both pelvic-belt and shoulder-belt restraints that comply with RESNA WC-4:2012, Section 18 and/or 4.6, 5.2, 5.3 of RESNA WC-4:2012, Section 19 should be used to limit occupant movement in a crash and reduce the likelihood of occupant ejection from the vehicle and/or injurious contact of the occupant with the vehicle interior. The pelvic belt should be angled between 30° to 75° horizontally when viewed from the side, and ideally between 45° to 75° horizontally, as shown in the figure below.

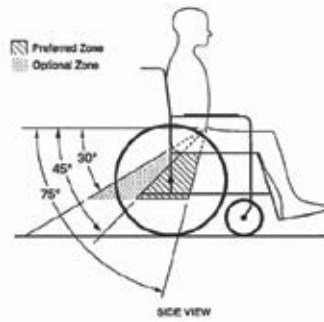**⚠ WARNING**

Back supports with adjustable recline angles should not be tilted back more than 30° to the vertical during travel in vehicles unless necessary for the medical and postural needs of the occupant.

⚠ WARNING

If the back support must be adjusted to an angle that is greater than 30° to the vertical during travel, the upper shoulder-belt anchor point should be adjusted rearward to maintain belt contact with the wheelchair occupant's shoulder and chest.

Trays

Wheelchair-mounted rigid trays that are not specifically designed for use during travel in motor vehicles should be:

- Removed and secured separately in the vehicle.
- Secured to the wheelchair so they will not break free in a crash.
- Be positioned with a gap of at least 7mm (3in.) between the back edge of the tray and the wheelchair occupant's abdomen and/or chest so as not to interfere with proper belt-restraint use.
- Have energy absorbing padding placed between the back edge of the tray and the wheelchair occupant.

2.3 WC20 Safety

2.3.1 User Safety

The use of a postural pelvic belt attached to the wheelchair base or seat frame is encouraged during travel, but these belts should be positioned so that they do not interfere with the proper positioning of crashworthy belt restraints, and they should not be relied on for occupant protection in crash situations unless the postural belt has been designed to comply with, and perform to, the requirements 4.6, 5.2, and 5.3 of RESNA WC-4:2012, Section 19.

DANGER

The WC20 approved Quick Release hardware and AL/AFT backs have been designed to accommodate the proper use of vehicle-anchored belt restraints and the ratings of the seating systems with regard to ease of properly positioning vehicle-anchored belt restraints on the wheelchair passenger, and the degree to which proper positioning of belt restraints can be achieved.

DANGER

Alterations or substitutions should not be made to the seating system or to its parts and components without consulting the manufacturer.


NOTICE

ADI's CF series backs are NOT WC20 approved.

2.3 WC20 Safety

Sufficient forward and rearward clear space should be provided around the wheelchair occupant.

The forward clear-space zone (FXZ) needs to be larger when a shoulder-belt restraint is not used.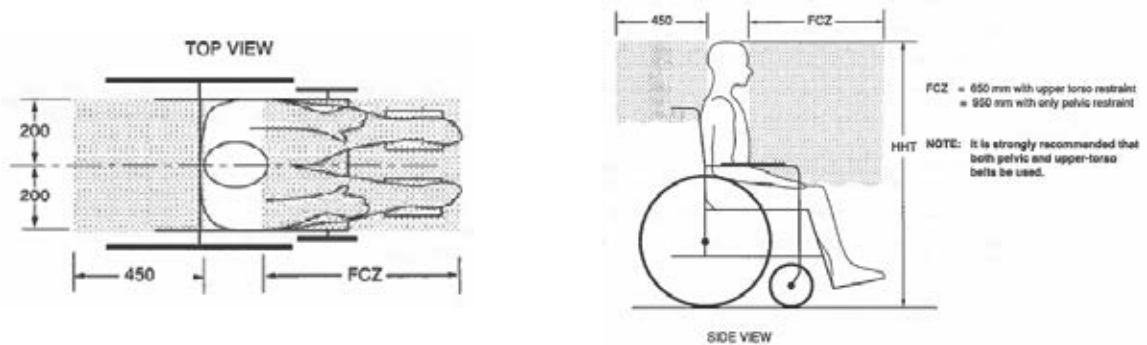

2.4 WC20 Hardware Installation

2.4.1 WC20 Hardware Installation


After the hardware has been installed, adjusted to the users' needs and securely fastened, insert the transit pins into the locking levers.

- The key ring, connected to the pin, will need to be attached to the hardware. A small, predrilled hole near the base of the hardware is designated for the key ring.
- When key ring is installed, slide pin through latch assembly.
- See pages 4 and 5 for hardware adjustment capabilities.

2.4.2 WC20 Label

The WC20 label will need to be secured to the cover of the ADI back cover if WC20 pins are ordered separately from the back and hardware. The label must be sewn to cover in order to be WC20 compliant.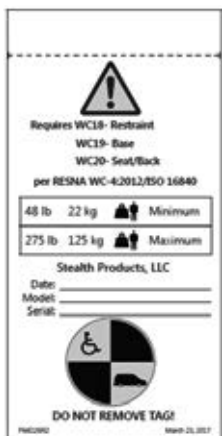

Warning!

You must comply with the following for proper transit use!
Failure to comply may result in serious injury or death to the user!

Minimum: _____

Maximum: _____

Stealth Products, LLC

Date: _____

Model: _____

Serial: _____

This seating system has passed crash testing with a midsize male ATD in accordance to **RESNA WC-4:2012/ISO16840** using methods described in Annex A of WC-4:2012, Sec.20, and has met requirements when attached to a wheelchair equipped with tie downs specified in WC-4:2012, Sec. 20.

DO NOT REMOVE TAG!

2.4.3 Removing the WC20 Pin

When the chair is not in transit, the pins for the Quick Release Hardware can be removed and placed in a holding spot on the hardware.

Image shows BR1102 & BR6005 with pin installed in latch assembly, ready for transit

Image shows BR1102 & BR6005 with pin placed in predrilled hole, not transit ready

SAFETY

Pin should always be placed in the designated predrilled holes near the base of the hardware. Letting the pin hang freely could cause damage to the chair or injury to the user.

2.5 Maintenance

2.5.1 Cleaning

- Use a soft cloth to clean the back shell.
- Ensure all cleaners are approved for finished steel, aluminum, plastic, and upholstered surfaces.

2.5.2 Maintenance

- Check and retighten all fasteners to the proper torque specifications on a regular basis. Reference the installation manual to find proper torque specs.
- Repair or replace parts as needed.

CAUTION

Always check all mounting hardware, making sure each fastener is properly tightened before using the back support.

CAUTION

250lbs (113.40 kg) user weight limit.

NOTICE

Altering an ADI back in any way and/or improper installation will void the warranty.

2.6.1 Dealer Assistance

During first time use by the client, it is advised that the dealer or service technician assists and explains the set-up to the customer (the user and/ or the attendant). If needed, the dealer can make final adjustments.

⚠ CAUTION

Do not attach the back with the user in the chair.

SAFETY

If using the non WC20 approved hardware, do NOT attach the ADI backs with the user in the chair or use them during transportation. Use the WC20 approved hardware with pin ONLY for transportation.

2.6.2 User Testing

It is important that the customer is fully aware of the installation, how the hardware works, and what can be adjusted. As a dealer, proceed as follows:

- Explain and show the customer how you have executed the installation, and explain the function of the hardware.
- If needed, adjust the hardware to the proper position.
- Explain to the customer possible problems and how to address them.

2.6.3 Conditions of Use

The ADI hardware is intended for use as installed by the dealer, in accordance to the installation instructions in this manual.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the usage conditions change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear or unintended damage.

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Stealth Products, LLC. • info@stealthproducts.com • www.stealthproducts.com