## 6Y200=\* Varos Liner

<b>DE</b> Gebrauchsanweisung (Benutzer) .....	2
<b>EN</b> Instructions for use (user) .....	10
<b>FR</b> Instructions d'utilisation (Utilisateur) .....	18
<b>IT</b> Istruzioni per l'uso (Utilizzatore) .....	26
<b>ES</b> Instrucciones de uso (Usuario) .....	34
<b>PT</b> Manual de utilização (Usuário) .....	42
<b>NL</b> Gebruiksaanwijzing (Gebruiker) .....	50
<b>JA</b> 取扱説明書(ユーザー用) .....	59

## INFORMATION

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### INFORMATION

Last update: 2018-04-04

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Have yourself instructed by qualified personnel in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

The product "6Y200=\* Varos liner" is referred to simply as the product/liner below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

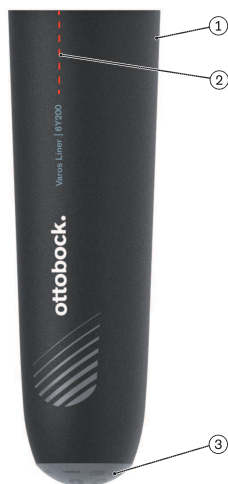
Only put the product into use in accordance with the information contained in the accompanying documents supplied.

## 2 Product description

### 2.1 Design

The product consists of the following components:

1. Silicone body with textile cover
2. Orientation thread (integrated on the silicone inner side)
3. Magnetic cap

### 2.2 Function

The liner is part of the suspension mechanism between the residual limb and prosthetic socket, and improves wearer comfort. The liner reduces the forces acting on the residual limb.

### 3 Application

#### 3.1 Indications for use

The product is to be used **solely** for lower limb exoprosthetic fittings.

#### 3.2 Conditions of use

The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, extreme sports (free climbing, parachuting, paragliding, etc.).

### 4 Safety

#### 4.1 Explanation of warning symbols

 <b>WARNING</b>	Warning regarding possible serious risks of accident or injury.
 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

#### 4.2 General safety instructions

 <b>WARNING</b>
<b>Risk of suffocation</b> Risk of suffocation due to lack of oxygen supply if the product is pulled over the nose and mouth. ▶ Store the product away from the reach of children.

 <b>CAUTION</b>
<b>Incorrect or excessively tight application of the product to the body</b> Risk of pressure points and constriction of blood vessels and nerves due to improper application. ▶ Ensure that the product is applied properly and fits correctly. Check skin areas daily. ▶ Consult a physician or O&P professional in case of any signs of skin damage, even if they are only minor. ▶ Do not continue using the product if there are problems with the fit, pain, loss of sensation or skin reddening.

 <b>CAUTION</b>
<b>Contact of the product with embers or fire</b> Injuries due to contact with molten material. ▶ Keep the product away from embers and open flames.

 <b>CAUTION</b>
<b>Use of the product on damaged skin</b> Skin irritation, formation of eczema or infections due to contamination with germs. ▶ Only use the product on healthy skin. ▶ Consult your attending physician.

**⚠ CAUTION****Use of powder (e.g. baby powder or talcum) as a donning aid**

Skin irritation on the residual limb.

- ▶ Verify that the use of powder does not cause skin irritation.

**NOTICE****Mechanical overloading of the product**

Damage to the product.

- ▶ Check the product for visible damage before each use.

**NOTICE****Remaining in areas outside the allowable temperature range**

Damage to the product.

- ▶ Avoid remaining in areas with temperatures outside of the permissible range (see Page 16).

**⚠ CAUTION****Improper care/disinfection of the product**

Skin irritation, formation of eczema or infections due to contamination with germs.

- ▶ Clean the product regularly with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).
- ▶ Disinfect the inner side of the product when needed with a colourless, conventional, medical disinfectant.
- ▶ For hygienic reasons, only use the product on yourself.

**INFORMATION****Limited Bluetooth range of a mechatronic knee joint**

The product contains permanent magnets. Note that the Bluetooth range of the mechatronic knee joint may be limited.

**4.3 Potential hazard due to magnets**

The product contains magnets for the distal coupling to the 5A60=\* VAROS socket.

**4.3.1 Effect on humans**

The "International Commission on non ionizing radiation protection (ICNIRP)" in the 2009 "Guidelines on limits of exposure to static magnetic fields" recommended adhering to an upper limit of 400 mT for the magnetic flux density for the exposure of any body parts in order to exclude possible health hazards. The magnetic flux density acting on the residual limb is far lower when wearing the product so that, according to the current state of knowledge, no impairment of health is expected.

**4.3.2 Pacemakers and implants****⚠ WARNING****Use of the product near active implanted systems**

The product's magnetic field can interfere with active implanted systems (e.g. pacemakers, defibrillators etc.).

- ▶ When using the product in the immediate vicinity of active implanted systems, ensure that the minimum distances stipulated by the manufacturer of the implant are observed.

- ▶ Make sure to observe any operating conditions and safety instructions stipulated by the manufacturer of the implant.

#### 4.3.3 Crushing

##### **⚠ CAUTION**

##### **Risk of pinching when handling the product**

Injury due to powerful magnetic attraction between the magnetic cap of the product and other metal parts.

- ▶ When inserting the product into the socket or handling the product without the socket, ensure that no fingers/body parts are between the product and other metal parts.
- ▶ Remove the product before entering a room or area with a strong magnetic field (e.g. MRT (MRI) equipment).

#### 4.3.4 Damage to data carriers and other devices

##### **⚠ WARNING**

##### **Insufficient distance of the liner's magnetic cap to other products**

Damage to data carriers and interference with products due to the liner's strong magnetic field when a connection to the socket is not established.

- ▶ Maintaining the following minimum distances between the liner's magnetic cap and other products is therefore recommended:
  - Pacemakers, defibrillators, implants: 10 cm
  - Mechatronic knee joints: 5 cm
  - High-quality magnetic card (credit card, EC card, bank card): 2 cm
  - Audio/video cassette: 4 cm
  - Simple magnetic card (parking, trade show entrance): 4 cm
  - Mechanical watch, anti-magnetic: 3 cm
  - Mechanical watch, not anti-magnetic: 15 cm
  - Hearing aid: 2 cm
  - The following are not affected: vehicle key, USB storage device, memory card, CD, DVD, Blu-Ray disc, camera, mobile phone.
- ▶ Note the information on the strength of the magnetic field in the section "Technical data".

##### **NOTICE**

##### **Shipment of magnets**

Damage to goods.

Interference with sorting systems of the shipper.

- ▶ Consult the shipper regarding shipping options.
- ▶ Use the original packaging or package the product so that the minimum distance between the magnetic cap and outside of the packaging is approx. 5 cm.
- ▶ Verify proper packaging with the paper clip test (paper clip falls off the packaging when the packaging surface is tilted 45°).

## 5 Scope of Delivery and Accessories

### Scope of Delivery

- 1 pc. 6Y200=\* Varos liner
- 1 pc. 646D0840 instructions for use (user)

## Accessories

The following components are not included in the scope of delivery and must be ordered separately:

- 1 pc. 5A60=\* Varos (socket)

## 6 Use

### CAUTION

#### **Metallic objects or particles adhering to the magnetic cap**

- > Falling due to lack of suspension on the prosthetic socket.
- > Damage to the product.

- ▶ Before putting on the liner, check for metallic objects or particles on the magnetic cap and remove any that are present.

### 6.1 Putting on the liner and socket

#### Putting on the liner

- 1) Check the liner and magnetic cap for damage.
- 2) Turn the liner inside out and apply it to the end of the residual limb.
- 3) Apply the liner to the end of the residual limb and check the alignment of the liner at the same time.

**INFORMATION: The orientation thread on the inner side of the liner has to be centred on the front of the transfemoral residual limb. The liner's two seams have to be positioned on the sides of the transfemoral residual limb.**

- 4) Unroll the liner onto the residual limb so that it has no wrinkles or air pockets, without shifting the soft tissue.
- 5) Check the fit and the alignment of the liner.
- 6) In a standing position (or position with the residual limb nearly extended), unroll the liner part-way and roll it back up again to release possible skin tension.

#### Putting on the socket

- 1) To open, pull the Boa dials away from the prosthetic socket.
- 2) Open the socket by pulling the shells apart.
- 3) Sit on the front edge of a chair.
- 4) Pull the socket over the residual limb.

**INFORMATION: In doing so, note the alignment of the toes and the position of the lower leg so the desired level of exterior foot rotation is given while standing and walking. If the exterior foot rotation is not correct, take the socket off again (at least, the magnetic cap has to be released from the bottom of the socket) and put it back on in a changed position.**

- The magnetic bottom shell of the socket and the magnetic cap of the liner attract each other when they are brought close together. A clicking sound is heard, indicating the proper connection of the socket and liner.
- 5) Push the Boa dials towards the socket.
  - 6) Turn the Boa dials clockwise.

→ The Boa cables are tightened and the socket is closed.

**INFORMATION: Position the outer socket shell (lateral) under the inner socket shell (medial).**

- 7) Tighten the Boa dials as needed.

### 6.2 Taking off the socket and liner

- 1) Pull the Boa dials away from the socket.

- 2) Pull the outer socket shell (lateral) to the outside.  
If plastic adhesive strips are used in the lateral socket shell, pull until there is no longer any contact between the plastic adhesive strips and textile cover of the liner.
- 3) Push the socket off the residual limb.  
In doing so, preferably grasp the socket brim of the lateral socket shell with one hand and the socket brim of the medial socket shell with the other hand.
- 4) Unroll and remove the liner from the residual limb.

## 7 Cleaning and care

### 7.1 Cleaning

#### NOTICE

##### **Improper product care**

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Only clean the product with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).

##### **Cleaning the silicone inner side**

Daily cleaning of the silicone inner side is recommended.

- 1) Turn the liner inside out.
- 2) Hand wash the liner in warm water (30°C–40°C) using mild detergent (e.g. 453H10=1 Ottobock Derma Clean).
- 3) Thoroughly rinse the liner with clear water and dab it dry with a lint-free towel.
- 4) Turn the liner so the silicone side is on the inside again.
- 5) Put a hand towel into the liner and leave the liner to dry lying flat on a drying rack. The liner can also be hung up to dry with the open side facing down with the magnetic cap on a metallic object.

##### **Cleaning the textile outer side**

The textile side of the liner should be cleaned at regular intervals (approx. every 1-2 weeks) or when it gets dirty.

- 1) Hand wash the textile side of the liner in warm water (30°C–40°C) using mild detergent (e.g. 453H10=1 Ottobock Derma Clean).
- 2) Thoroughly rinse the liner with clear water and dab it dry with a towel.
- 3) Leave the liner to dry lying flat on a drying rack. The liner can also be hung up to dry with the open side facing down with the magnetic cap on a metallic object.

##### **Cleaning the magnetic cap**

The magnetic cap can be cleaned with a dry or damp cloth if it gets dirty.

## 8 Legal information

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as class I accessories according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

### 8.3 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

### 8.4 Shipment

There must not be any significant magnetic effect on the outside of the packaging when the product is shipped. The provisions of the shipper must be observed.

## 9 Technical data

<b>Ambient conditions</b>	
Storage and transport in original packaging	-25°C/-13°F to +70°C/+158°F
Storage and transport without packaging	-25°C/-13°F to +70°C/+158°F Max. 90% relative humidity, non-condensing
Operation	-10°C/+14°F to +40°C/+104°F Max. 90% relative humidity, non-condensing

<b>Liner</b>	<b>Size M</b>	
Reference number	6Y200=M-1	6Y200=M-2
Service life	6 months Wear part, subject to natural wear and tear. The service life diminishes or increases depending on the individual level of use. Only when the instructions for use are observed can the maximum service life be reached, depending on the level of use.	
Length (not shortened, outside dimension)	450 mm	450 mm
Weight	650 g	680 g
Circumference (on the inside of the liner edge)	440 mm	470 mm
Circumference (on the inside, 50 mm from the bottom of the liner)	330 mm	360 mm
Coupling force to 5A60=* Varos	150 N (typical)	

<b>No connection of the liner to the socket</b>	<b>Magnetic flux density</b>
Inside of the liner in the area of the magnetic cap	< 3mT
Inside of the liner in the area of the magnetic cap at a distance of 10 cm	<0.05mT
Outside of the liner in the area of the magnetic cap	<400mT
Outside of the liner in the area of the magnetic cap at a distance of 10 cm	<0.2mT

<b>Liner connected to the socket</b>	<b>Magnetic flux density</b>
Inside of the liner in the area of the magnetic cap	< 15mT
Inside of the liner in the area of the magnetic cap at a distance of 10 cm	<0.05mT
Outside of the socket in the area of the connection to the prosthetic components	<10mT
Outside of the socket in the area of the connection to the prosthetic components at a distance of 10 cm	<0.05mT

## 10 Appendix

### 10.1 Symbols Used

Legal manufacturer

Declaration of conformity according to the applicable European directives

LOT PPPP YYYY WW

Lot number

Not made with natural rubber latex.

Natural rubber latex was not used as a material

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