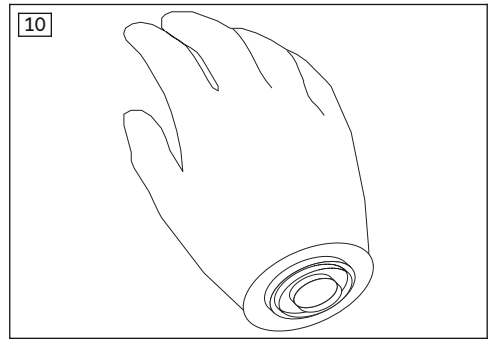
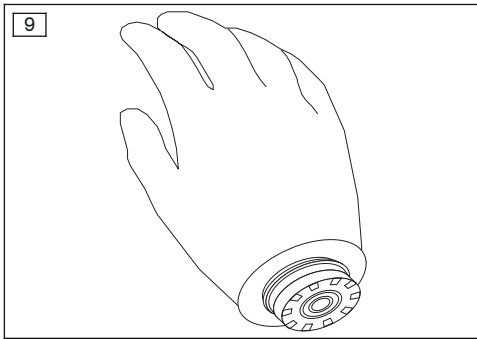
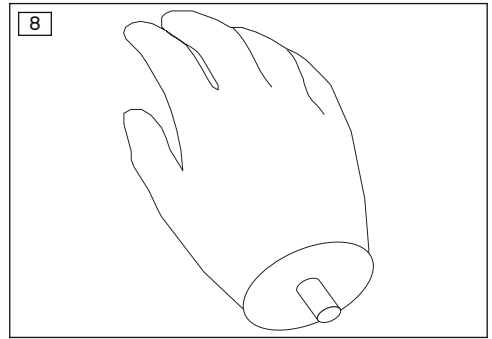
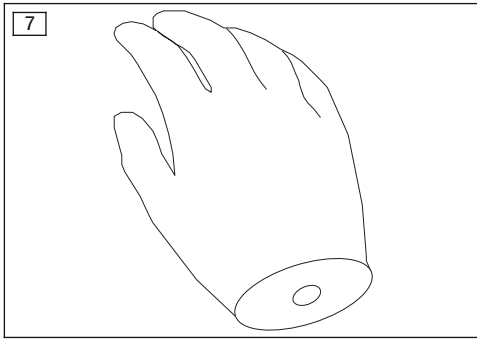
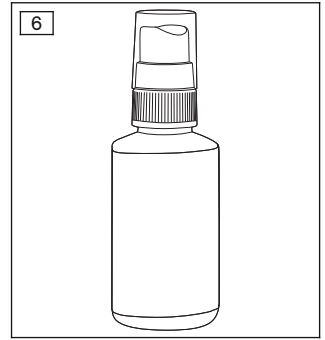
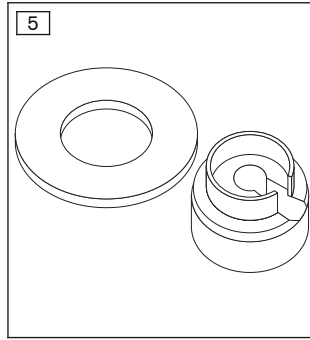
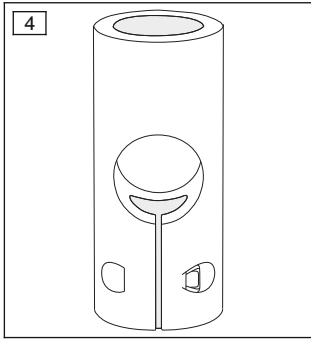
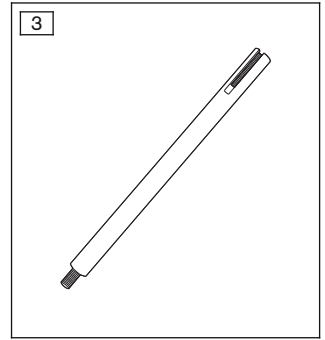
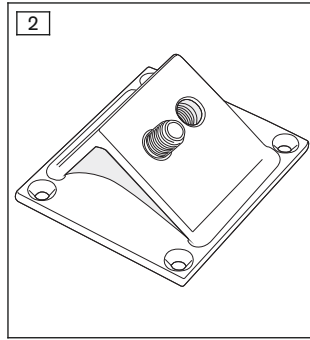
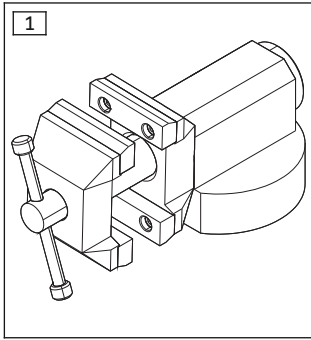
8S4N=\*, 8S5N=\*, 8S6N=\*, 8S11N=\*, 8S12N=\*,  
8S13N=\*, 8S20N=\*

|  |    |
|--|----|
| <b>DE</b> Gebrauchsanweisung (Fachpersonal) .....                      | 3  |
| <b>EN</b> Instructions for use (qualified personnel) .....             | 9  |
| <b>FR</b> Instructions d'utilisation (Personnel spécialisé) .....      | 15 |
| <b>IT</b> Istruzioni per l'uso (Personale tecnico specializzato) ..... | 21 |
| <b>ES</b> Instrucciones de uso (Personal técnico especializado) .....  | 27 |
| <b>PT</b> Manual de utilização (Pessoal técnico) .....                 | 32 |
| <b>NL</b> Gebruiksaanwijzing (Vakmensen) .....                         | 38 |
| <b>SV</b> Bruksanvisning (Fackpersonal) .....                          | 44 |
| <b>DA</b> Brugsanvisning (Faguddannet personale) .....                 | 50 |
| <b>NO</b> Bruksanvisning (Fagpersonell) .....                          | 56 |
| <b>EL</b> Οδηγίες χρήσης (Τεχνικό προσωπικό) .....                     | 61 |

## INFORMATION

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### INFORMATION

Last update: 2018-04-06

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

### 1.1 Construction and Function

The prosthetic glove is used as a protective glove for a prosthetic hand.

The PVC material is characterised by robustness and durability.

Thanks to the exterior coating, the Skin Natural is less susceptible to dirt and easier to clean than standard prosthetic gloves.

Its appearance is based on a real hand in order to achieve a high degree of naturalness.

The prosthetic glove is coloured by hand, giving rise to possible variations in appearance, which are not a product defect.

The fingernails can be painted with nail varnish.

The following prosthetic gloves are suitable for men:

- 8S4N=\*, 8S11N=\*

The following prosthetic gloves are suitable for women:

- 8S5N=\*, 8S12N=\*, 8S13N=\*

The following prosthetic gloves are suitable for children:

- 8S6N=\*, 8S20N=\*

### 1.2 Combination possibilities

For combination possibilities, please see the catalogue 646K6\* or consult the manufacturer.

## 2 Intended use

### 2.1 Indications for use

The product is to be used solely for upper limb prosthetic fittings.

### 2.2 Area of application

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

### 2.3 Environmental conditions

|   |
|---|
| <b>Allowable environmental conditions</b>   |
| Allowable operating temperature: <b>-10 °C to +60 °C (+14 °F to +140 °F)</b>                      |
| Allowable storage temperature: <b>-20 °C to +40 °C (-4 °F to +104 °F)</b>                         |
| Allowable transport temperature: <b>-20 °C to +60 °C (-4 °F to +140 °F)</b>                       |
| Allowable relative humidity: <b>max. 80%</b>  |
| Cleaning agents: warm water and soap, 640F12 Special Cleaner, nail varnish remover (acetone-free) |

### Unallowable environmental conditions

Mechanical vibrations or impacts, touching of sharp-edged or pointed objects

Contact with acids, unallowable cleaning agents, abrasive media (e.g. sand), damp soil, dust, dirt, highly staining materials (e.g. printing ink, pen ink, toner, new textiles), solvents (e.g. thinner, acetone, petrol), fire, embers and unallowable heat (> +60 °C/> +140 °F), silicone spray

## 2.4 Service life

The product is a wear part, which means it is subject to natural wear and tear. The service life diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximal service life, in dependence of the level of use.

## 3 Safety

### 3.1 Explanation of warning symbols

**CAUTION** Warning regarding possible risks of accident or injury.

**NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

#### **CAUTION**

#### **Using the product without following the instructions for use**

Injuries caused by damage to the product or malfunction of the product

- ▶ Only use the product by following the instructions for use.
- ▶ Make the patient aware of all safety instructions marked with "**Inform the patient**".

#### **CAUTION**

#### **Contact with heat, embers or fire**

Injuries (e.g. burns) and damage to the product due to unallowable environmental conditions

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ **Inform the patient.**

#### **NOTICE**

#### **Unallowable combination of prosthetic components**

Product damage due to malfunctions

- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination Possibilities".
- ▶ Verify that the prosthetic components may be combined with each other by consulting their instructions for use.

#### **NOTICE**

#### **Non-observance of allowable environmental conditions**

Damage to product and soiling due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for soiling and damage.
- ▶ Clean the product immediately if it gets dirty, following the instructions in the section "Cleaning", in order to prevent permanent soiling.

- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ **Inform the patient.**

#### NOTICE

#### Careless use of the product

Damage to the product due to contact with pointed or sharp-edged objects

- ▶ Do not allow the product to come into contact with the aforementioned objects.

## 4 Scope of Delivery and Accessories

The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering as single components (■), single components with minimum order quantities (▲) or single-component packs (●):

| 8S4N=*, 8S5N=*, 8S6N=*, 8S11N=*, 8S12N=*, 8S13N=*, 8S20N=* |          |   |          |  |                  |
|--|----------|---|----------|--|------------------|
| Fig.   | Item no. |   | Quantity | Designation                                | Reference number |
| -  | -        | ■ | 1        | Instructions for Use (qualified personnel) | 647G571          |
| -  | -        | ■ | 1        | Instructions for Use (Patient)             | 646D423          |
| -  | -        | - | 1        | Prosthetic Glove                           | -                |

#### Accessories

| Fig. | Item no. |   | Quantity | Designation                   | Reference number |
|------|----------|---|----------|-------------------------------|------------------|
| 2    | -        | ■ | 0        | Mounting Plate                | 711M2            |
| 3    | -        | ■ | 0        | Mounting Tool (Alignment Rod) | 711M1            |
| 4    | -        | ■ | 0        | Mounting Tool                 | 711M3            |
| 5    | -        | ■ | 0        | Mounting Aid                  | 711M7            |
| 6    | -        | ■ | 0        | Donning spray                 | 640F18           |
| -    | -        | ■ | 0        | Special Cleaner               | 640F12           |
| -    | -        | ■ | 0        | Pump Sprayer                  | 640F13           |

## 5 Preparation for use

#### NOTICE

#### Incorrect donning and removal of the prosthetic glove

Damage to the prosthetic hand and the prosthetic glove due to non-observance of instructions for use

- ▶ Observe all instructions for donning and removal. The prosthetic hand must always be mounted on the donning tool.
- ▶ Only use the components listed (equipment, tools, materials) and have them close to hand.
- ▶ Avoid damage that can be caused to the prosthetic hand and the prosthetic glove, e.g. from tools and excessive force (in particular on the thumb).

#### INFORMATION

It is not possible to colour the prosthetic glove with the 635P15=\* Colour Pens.

## 5.1 Preparations for donning and removal

### Prosthetic Hand with M12x1.5 Threaded Bore (see fig. 7)

#### > Tools required:

- 711M1 Mounting Tool (Alignment Rod) (see fig. 3), 711M2 Mounting Plate (see fig. 2) or vice (see fig. 1)
- ▶ Screw the alignment rod into the thread on the prosthetic hand.
- ▶ Attach the alignment rod to the mounting plate or clamp it in a vice.

### Prosthetic Hand with M12x1.5 Threaded Stud (see fig. 8)

#### > Tools required:

- 711M1 Mounting Tool (Alignment Rod), 711M2 Mounting Plate or vice
- ▶ Screw the alignment rod onto the thread of the prosthetic hand.
- ▶ Attach the alignment rod to the mounting plate or clamp it in a vice.

### Prosthetic Hand with Quick-Disconnect Wrist Unit (see fig. 9)

#### > Tools required:

- 711M3 Mounting Tool (see fig. 4), 711M1 Mounting Tool (Alignment Rod), 711M2 Mounting Plate or vice
- ▶ Install the prosthetic hand on the mounting tool.
- ▶ Install the mounting tool on the alignment rod on the side with the interior thread.
- ▶ Attach the alignment rod to the mounting plate or clamp it in a vice.

### 8E39 Prosthetic Hand (see fig. 10)

#### > Tools required:

- 711M7 Mounting Aid (see fig. 5), 711M1 Mounting Tool (Alignment Rod), 711M2 Mounting Plate or vice
- ▶ Screw the mounting aid (screw-in part with ring) to the prosthetic hand.
- ▶ Screw the mounting rod into the mounting aid.
- ▶ Attach the alignment rod to the mounting plate or clamp it in a vice.

### Prosthetic Hand with Other Wrist Connection

- ▶ Use a holding device that firmly secures the prosthetic hand during pulling on and pulling off.

## 5.2 Don the prosthetic glove on the prosthetic hand

### INFORMATION

A crucial factor when donning the prosthetic glove is that it is warm during the stretching, donning and adjustment process.

- ▶ Work quickly, adhering to the sequence of the instructions provided.

#### ▶ When using a convection oven:

- Preheat the oven (**60 °C**).
- ▶ Shorten the prosthetic glove at the point with the largest diameter (at the height of the elbow).
- ▶ Place the prosthetic glove into the oven (**10 – 15 min/60 °C**).
- ▶ Remove the prosthetic glove from the oven and spray the inside with donning spray.

#### > Required materials:

- 640F18 Donning Spray (see fig. 6)
- ▶ Spray the inner hand of the prosthetic hand (with the Electric Hand 2000, the inner fingers and the finger pad) with the donning spray.
- ▶ Position the prosthetic glove on the prosthetic hand for donning.
- ▶ Hold the prosthetic glove at the sleeve end and pull it over the prosthetic hand in one movement.
- ▶ Adjust the prosthetic glove to fit the prosthetic hand properly.

- ▶ **CAUTION!** In order to prevent skin irritation, shorten the prosthetic glove so that it does not lie on the skin when wearing the prosthesis.

#### Increasing flexibility and function for active prostheses

- ▶ **NOTICE!** Take care not to damage the prosthetic glove when warming the prosthetic hand (e.g. by exceeding the maximum allowable temperature).

Warm the palm of the prosthetic glove.

- ▶ Slowly open the prosthetic hand.
- ▶ With the prosthetic hand open, allow the prosthetic glove to cool.

### 5.3 Removal from the prosthetic hand

#### Removal with re-use of the prosthetic glove

- ▶ Moisten the prosthetic glove with donning spray.
- ▶ **NOTICE!** Avoid all excessive force on the fingers during the removal process.  
Hold the prosthetic glove at the sleeve end and remove.

#### Removal without re-use of the prosthetic glove

- ▶ **NOTICE!** Avoid damaging the prosthetic hand during cutting.  
Cut open the prosthetic glove on the inside.
- ▶ **NOTICE!** Avoid all excessive force on the fingers (especially the thumb) during the removal process.  
Pull the sleeve as far apart as possible and remove the prosthetic glove.

## 6 Use

### 6.1 Instructing the patient in the use of the product

#### INFORMATION

- ▶ Inform the patient about all the important points relevant to them that are contained in these instructions for use.
- ▶ Instruct the patient in the correct use of the product in accordance with the **Instructions for use (patient)** included in the scope of delivery.
- ▶ Hand over the Instructions for use (patient) to the patient when the briefing has been completed.

### 6.2 Cleaning

#### NOTICE

#### Non-observance of cleaning instructions

Damage to product and permanent soiling due to use of improper cleaning agents

- ▶ Only clean the product according to the cleaning instructions provided in the instructions for use.
- ▶ **Inform the patient.**

#### Light soiling

##### > **Materials required:**

warm water, soap, cloth, lint-free cloth

- 1) Clean the product with warm water, soap and a cloth.
- 2) Rinse off soap residues with water.
- 3) Dry the product with a lint-free cloth and allow it to air dry fully.

## Heavy soiling

### > **Materials required:**

640F12 Special Cleaner, 640F13 Pump Sprayer, clean, undyed cloth (e.g. cellulose, linen), water, lint-free cloth

### 1) **NOTICE! Clean the product promptly to avoid heavy dirt from permanently contaminating the material.**

Spray the special cleaner on to the soiled area of the product and leave to take effect for a short time.

2) Wipe off the dirt from the product with a clean, undyed cloth using light pressure.

3) Rinse off residual cleaner and dirt with water.

4) Dry the product with a lint-free cloth and allow it to air dry fully.

## Removal of nail varnish

### > **Materials required:**

Nail varnish remover (acetone-free), clean, undyed cloth (e.g. cellulose, linen), water, lint-free cloth

1) Remove the nail varnish from the product with nail varnish remover and a clean, undyed cloth, using light pressure.

2) Rinse off residual cleaner and dirt with water.

3) Dry the product with a lint-free cloth and allow it to air dry fully.

## 7 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

## 8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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