





# ottobock.

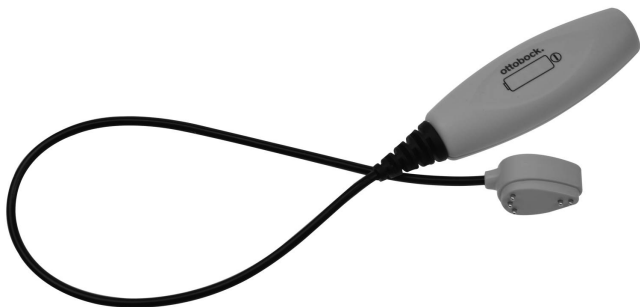CE

## 757L35

<b>DE</b>	Gebrauchsanweisung (Fachpersonal) .....	3
<b>EN</b>	Instructions for use (qualified personnel) .....	11
<b>FR</b>	Instructions d'utilisation (Personnel spécialisé) .....	19
<b>IT</b>	Istruzioni per l'uso (Personale tecnico specializzato) .....	28
<b>ES</b>	Instrucciones de uso (Personal técnico especializado) .....	36
<b>PT</b>	Manual de utilização (Pessoal técnico) .....	45
<b>NL</b>	Gebruiksaanwijzing (Vakmensen) .....	54
<b>SV</b>	Bruksanvisning (Fackpersonal) .....	62
<b>DA</b>	Brugsanvisning (Faguddannet personale) .....	70
<b>NO</b>	Bruksanvisning (Fagpersonell) .....	78
<b>SK</b>	Návod na používanie (Odborný personál) .....	86
<b>TR</b>	Kullanma talimatı (Uzman personel) .....	94
<b>EL</b>	Οδηγίες χρήσης (Τεχνικό προσωπικό) .....	102
<b>RU</b>	Руководство по применению (Квалифицированный персонал) ...	110
<b>JA</b>	取扱説明書 (有資格担当者) .....	120

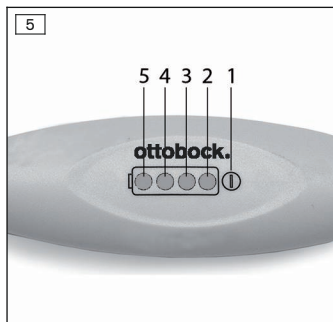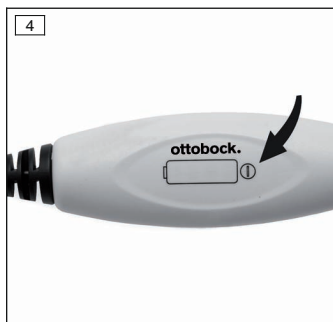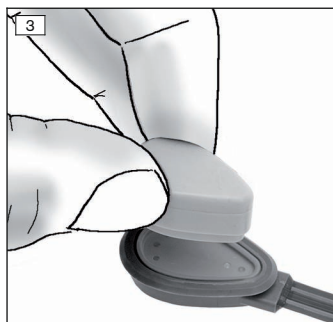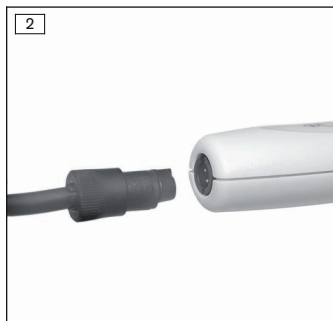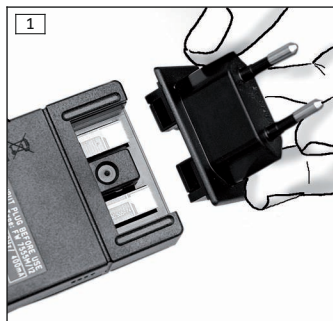## INFORMATION

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### INFORMATION

Date of last update: 2018-09-24

- ▶ Please read this document carefully before using the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.
- ▶ Please keep this document in a safe place.

In the following, the "MyoCharge Integral" is referred to simply as the battery charger/product.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

The product may not be transferred to the patient without prior instruction.

## 2 Product description

### 2.1 Function

The product is intended for charging the batteries integrated in the socket. For charging, the charging plug must be connected to the charging receptacle on the outside of the socket. The charging plug is secured to the charging receptacle by the integrated magnet. The special contour of the charging receptacle and charging plug ensures the quick, reliable positioning of the two components to each other.

During the charging process the current battery charge level is indicated by the LEDs on the battery charger lighting up in sequence (Status signals).

## 3 Application

### 3.1 Indications for use

The product to be used **exclusively** to supply energy to myoelectric upper limb prosthesis components.

### 3.2 Area of application

The product is intended **exclusively** for use with 757B35=\* batteries.

### 3.3 Conditions of use

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

## 4 Safety

### 4.1 Explanation of warning symbols

 <b>WARNING</b>	Warning regarding possible serious risks of accident or injury.
 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
<b>NOTICE</b>	Warning regarding possible technical damage.

### 4.2 Structure of the safety instructions

 <b>CAUTION</b>
<b>The heading describes the source and/or the type of hazard</b>
The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:
> E.g.: Consequence 1 in case of failure to observe the hazard

- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

### 4.3 General safety instructions

#### **WARNING**

##### **Non-observance of safety notices**

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

#### **WARNING**

##### **Use of damaged power supply unit, adapter plug or battery charger**

Risk of electric shock due to contact with exposed, live components.

- ▶ Do not open the power supply unit, adapter plug or battery charger.
- ▶ Do not expose the power supply unit, adapter plug or battery charger to extreme loading conditions.
- ▶ Immediately replace damaged power supply units, adapter plugs or battery chargers.

#### **WARNING**

##### **Operating the product near active implanted systems**

Interference with active implantable systems (e.g., pacemaker, defibrillator, etc.) due to electromagnetic interference of the product.

- ▶ When operating the product in the immediate vicinity of active implantable systems, ensure that the minimum distances stipulated by the manufacturer of the implant are observed.
- ▶ Make sure to observe any operating conditions and safety instructions stipulated by the manufacturer of the implant.

#### **NOTICE**

##### **Penetration of dirt and humidity into the product**

Lack of proper charging functionality due to malfunction.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.

**NOTICE****Mechanical stress on the power supply/battery charger**

Lack of proper charging functionality due to malfunction.

- ▶ Do not subject the power supply/battery charger to mechanical vibrations or impacts.
- ▶ Check the power supply/battery charger for visible damage before each use.

**NOTICE****Operating near sources of strong magnetic or electrical interference (e.g. theft prevention systems, metal detectors)**

Lack of proper charging functionality due to malfunction.

- ▶ Avoid operating in the vicinity of visible or concealed theft prevention systems at the entrance/exit of shops, metal detectors/body scanners for persons (e.g. in airports) or other sources of strong magnetic and electrical interference (e.g. high-voltage lines, transmitters, transformer stations, computer tomographs, magnetic resonance tomographs, etc.).

**NOTICE****Improper product care**

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Clean the product with a damp cloth and mild soap only.

**NOTICE****Contact of the charging plug with magnetic data carriers**

Wiping of the data carrier.

- ▶ Do not place the charging plug on credit cards, diskettes, audio or video cassettes.

**NOTICE****Independent changes or modifications carried out to the battery charger**

Lack of proper charging functionality due to malfunction.

- ▶ Have any changes or modifications carried out only by Ottobock authorised, qualified personnel.

#### NOTICE

##### **Using the product with soiled or damaged contacts**

Lack of proper charging functionality.

- ▶ Make sure that the contacts are always clean and free of grease.
- ▶ Clean the electrical contacts regularly using cotton swabs and a mild soap solution.
- ▶ Take care to avoid damaging the contacts with pointed or sharp objects.

#### NOTICE

##### **Operating the product outside of the permissible temperature range**

Lack of proper charging functionality due to malfunction.

- ▶ Avoid operating in areas outside the permissible temperature range. The section "Technical Data" contains information on the permissible temperature range.

#### NOTICE

##### **Distance to HF communication devices is too short (e.g. mobile phones, Bluetooth devices, WiFi devices)**

Lack of proper charging functionality due to malfunction.

- ▶ Therefore, keeping the following minimum distances to these HF communication devices is recommended:
  - Mobile phone GSM 850 / GSM 900: 0.99 m
  - Mobile phone GSM 1800 / GSM 1900 / UMTS: 0.7 m
  - DECT cordless phones incl. base station: 0.35 m
  - WiFi (routers, access points ...): 0.22 m
  - Bluetooth devices (third-party products not approved by Ottobock): 0.22 m

## **5 Scope of delivery**

- 1 pc. 757L35 MyoCharge Integral
- 1 pc. 757L16\* power supply
- 1 pc. 647G534 instructions for use (qualified personnel)

## **6 Preparation for use**

- 1) Slide the country-specific plug adapter onto the power supply until it locks into place (see fig. 1).

- 2) Connect the round plug of the power supply to the receptacle on the battery charger so that the plug locks into place.

**INFORMATION: Ensure correct polarity (guide lug). Do not use force when connecting the cable plug to the battery charger (see fig. 2).**

- 3) Plug the power supply unit into the wall socket.

→ The green LED on the back of the power supply lights up.

## 7 Handling

We recommend charging the battery once a day when using the prosthesis on a daily basis.

- ▶ Connect the charging plug to the charging receptacle and allow it to snap into place (see fig. 3).

→ LED 1 lights up blue (see fig. 4, arrow) and the beeper sounds.

→ The charging process starts.

During the charging process, the current battery charge level (Status signals) is indicated by the LEDs on the battery charger lighting up in sequence (see fig. 5).

Once charging is complete, the charging plug can be disconnected. If the charging plug remains connected, the battery is trickled charged. This will not damage the battery or the battery charger.

### INFORMATION

The charging plug may be disconnected at any time. Partially charging the battery does not shorten its lifespan.

### INFORMATION

As long as the charging plug is connected to the charging receptacle, the prosthesis cannot be used. The power supply of the prosthesis is turned off during the charging procedure.

## 8 Disposal

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

## 9 Legal information

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

### 9.3 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

The product meets the requirements under the RoHS Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

## 10 Appendices

### 10.1 Symbols Used

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

Declaration of conformity according to the applicable European directives

**LOT**

Lot number

## 10.2 Operating states/error signals

### 10.2.1 Status signals

#### Battery charge level

Battery charger	Description
The blue LED lights up	Power supply and battery charger operational
LEDs light up/4 blue LEDs flash	Battery is charging
LEDs light up/3 blue LEDs flash	Battery is charging, battery 25% charged
LEDs light up/2 blue LEDs flash	Battery is charging, battery 50% charged
LEDs light up/1 blue LED flashes	Battery is charging, battery 75% charged
The blue LEDs light up	Battery is fully charged, battery 100% charged

### 10.2.2 Warnings/error signals

#### Error while charging the product

Battery charger	Explanation/correction
LED lights up/flashes red	<b>The charging plug is not properly connected to the charging receptacle</b> <ul style="list-style-type: none"><li>• Disconnect the charging plug and connect it again</li></ul>
	<b>The charging plug or charging receptacle contacts are dirty</b> <ul style="list-style-type: none"><li>• Use a cotton swab and a mild soap solution to clean the contacts of both components</li></ul>
LED lights up/flashes red	<b>The battery is outside the permissible temperature range</b>

Battery charger	Explanation/correction
	<ul style="list-style-type: none"> <li>Bring the battery into the permissible temperature range (Technical data); the charging process resumes automatically</li> </ul>

### 10.3 Technical data

Environmental conditions	
Storage and transport in original packaging	-20 °C/-4 °F to +70 °C/+158 °F
Storage and transport without packaging	-20 °C/-4 °F to +70 °C/+158 °F
Storage	-20 °C/-4 °F to +40 °C/+104 °F
Operation	0 °C/+32 °F to +60 °C/+140 °F

General	
Reference number	757L35
Product service life	5 years
Max. charging current	1 A
Max. charging voltage	10 V
Battery charger operating voltage	12 V
Power supply operating voltage	100 V - 240 V
Power supply operating frequency	50 Hz - 60 Hz

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Otto Bock Healthcare Products GmbH