3F1=1/3F1=2

DE Gebrauchsanweisung (Benutzer)	2
EN Instructions for use (user)	9
FR Instructions d'utilisation (Utilisateur)	15
IT Istruzioni per l'uso (Utilizzatore)	23
ES Instrucciones de uso (Usuario)	30
PT Manual de utilização (Usuário)	37
NL Gebruiksaanwijzing (Gebruiker)	44
SV Bruksanvisning (Användare)	51
DA Brugsanvisning (Bruger)	58
NO Bruksanvisning (Bruker)	65
FI Käyttöohje (Käyttäjää)	72
PL Instrukcja użytkowania (Użytkownik)	78
HU Használati utasítás (használó)	86
CS Návod k použití (Uživatel)	93
HR Upute za uporabu (Korisnik)	100
SL Navodila za uporabo (Uporabnik)	106
SK Návod na používanie (Užívateľ)	114
TR Kullanma talimatı (Kullanıcı)	120
RU Руководство по применению (Пользователь)	128
JA 取扱説明書(ユーザー用)	135
ZH 使用说明书(用户)	141

INFORMATION

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INFORMATION

Date of last update: 2019-02-05

- ▶ Please read this document carefully before using the product.
- ▶ Have yourself instructed by qualified personnel in the proper and safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). The contact information for the manufacturer is found on the back page.
- ▶ Please keep this document in a safe place.

The product "3F1=1 functional cosmesis C-Leg / 3F1=2 functional cosmesis Genium" is called the product/functional cosmesis in the following.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Application

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

The product serves to finish the prosthesis in the sense of restoring a natural appearance that is as inconspicuous as possible.

2.2 Area of application

The product is intended exclusively for wearers of the 3C98-3/3C88-3 C-Leg, 3B1-2/3B1-2=ST Genium and 3B1-3/3B1-3=ST Genium.

2.3 Conditions of use

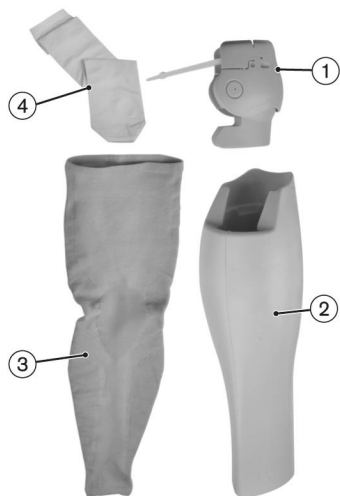
The product is intended **exclusively** for use on **one** user. Use of the product by another person is not approved by the manufacturer.

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

3 Product description

3.1 Design

The functional cosmesis consists of the following components:

1. Functional cosmesis C-Leg (knee part) / functional cosmesis Genium (knee part) (referred to as "functional knee part" in the following)
2. Functional shank C-Leg / functional shank Genium (referred to as "functional shank" in the following)
3. Functional stocking
4. Sheath

3.2 Function

The product helps create the most natural possible appearance. The product forms the exterior finish of a prosthesis. The product is not required for the proper and safe functioning of the prosthesis.

The functional shank is shaped by the O&P professional to create the most natural possible appearance. Streaks, colour differences and small air pockets (up to 1 cm) in the functional shank are concealed by the functional stocking. These differences, which are due to production reasons, do not affect the function of the product and therefore do not constitute grounds for complaints.

4 Safety

4.1 Explanation of warning symbols

CAUTION	Warning regarding possible risks of accident or injury.
----------------	---

NOTICE	Warning regarding possible technical damage.
---------------	--

4.2 Structure of the safety instructions

CAUTION

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General safety instructions

CAUTION

Non-observance of safety instructions

- > Injury due to faulty control or malfunction of the prosthesis.
- > Destruction of prosthesis.
- ▶ Observe the safety instructions and the stated precautions in this accompanying document.

CAUTION

Contact of the product with embers or fire

Injuries (e.g. burns) and damage to the product.

- ▶ Keep the product away from embers and open flames.

CAUTION

Wearing the product without a functional stocking

- > Injuries due to pinching of body parts.
- > Damage due to pinching of garments.
- > Damage to the product.
- > Loss of functionality due to the penetration of moisture into the foam.
- ▶ The product may only be worn with the functional stocking.

CAUTION

Skin irritation due to irregular care of the product

- > Skin irritation due to contact with a soiled functional shank.
- > Skin irritation due to contact with a soiled sheath.
- ▶ Clean the product only as described in the section "Cleaning and care" (see Page 13).
- ▶ Only wear the product with the functional stocking.

CAUTION

Thermal overload due to increased heat accumulation

Falling due to altered damping behaviour as a result of earlier switching to overheating mode.

- ▶ Observe the warnings/error signals of the prosthesis (overheating mode).

CAUTION

Influence of the product on the behaviour of the prosthesis

Change in prosthesis functionality

- ▶ Observe the safety instructions for your prosthesis.
- ▶ Have the settings of your prosthesis in combination with the product checked by the O&P professional.

CAUTION

Impaired perception of the feedback signals

Falling due to impaired perception of the feedback signals resulting from changed damping behaviour.

- ▶ The audibility of the feedback signals must be checked and the corresponding volume has to be set by the O&P professional.

NOTICE

Independent user changes or modifications made to the product

Damage to the product.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage.

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

NOTICE

Improper product care

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Clean the product only as described in the section "Cleaning and care" (see Page 13).

INFORMATION

The penetration of dirt and humidity into the product leads to a loss of comfort. Observe the safety information in the instructions for use for your prosthesis.

INFORMATION

The product can limit the range of the Bluetooth signal. Observe the safety information in the instructions for use for your prosthesis.

5 Scope of delivery

- 1 pc. 3F1=1 functional cosmesis C-Leg/3F1=2 functional cosmesis Genium
- 1 pc. 647G1290 instructions for use (user)
- 1 pc. 99B120=* functional stocking

6 Handling

6.1 Putting on and taking off

Putting on the functional stocking

INFORMATION

Note the alignment of the functional stocking (zones of the functional stocking). The label in the functional stocking identifies the back of the stocking.

Complete the following steps to put on the functional stocking:

- 1) Fold over the coated ankle part of the functional stocking (coating faces out).
- 2) The functional stocking should extend approx. 1 cm beyond the footshell.
- 3) Pull the functional stocking up.

INFORMATION: Adjust how far you pull the functional stocking up according to your needs, so that your use of the joint is not noticeably impaired. To accomplish this, secure the functional stocking to the prosthetic socket, for example while the joint is flexed.

- 4) For the best possible adhesion, position the end of the functional stocking directly on the socket surface if possible.

Taking off the functional stocking

Complete the following steps to take off the functional stocking:

- 1) Fold over the coated socket part of the functional stocking (coating faces out).
- 2) Pull the functional stocking down over the functional cosmesis.

INFORMATION: The functional stocking is turned inside-out.

6.2 Charging the prosthesis battery

Charging the battery with the 3F1=1 functional cosmesis C-Leg

- 1) Push the functional stocking down until the charging receptacle on the rear of the prosthesis is visible.
- 2) Open the charging receptacle cover.
- 3) Connect the charging plug to the charging receptacle of the product.

INFORMATION: Observe the instructions for use for the prosthesis.

Charging the battery with the 3F1=2 functional cosmesis Genium

INFORMATION

The inductive charger for Genium can be used after putting on the functional stocking. Exposing the charging unit is not necessary.

- ▶ Connect the inductive charger to the receiver of the charging unit on the rear of the product.

INFORMATION: Observe the instructions for use for the prosthesis.

7 Cleaning and care

7.1 Cleaning the functional cosmesis

- ▶ Clean the product with a damp, soft cloth and then let it dry.

7.2 Cleaning the functional stocking

- ▶ Wash the product at 30°C/86°F in a washing machine
Setting: gentle cycle (for delicates).

INFORMATION

Note the washing symbols on the label.

INFORMATION

Drying may take between 12 and 24 hours. A second functional stocking for changing is recommended.

7.3 Care of the functional stocking

- Wipe the silicone coatings on the inside of the functional stocking with a damp cloth every three days.

8 Legal information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

8.3 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9 Technical data

Environmental conditions	
Storage and transport in original packaging	-25 °C/-13 °F to +70 °C/+158 °F
Storage and transport without packaging	-25 °C/-13 °F to +70 °C/+158 °F Max. 93% relative humidity, non-condensing
Operation	-10 °C/+14 °F to +60 °C/+140 °F 15% to 93% relative humidity, non-condensing

General information	
Reference number	3F1=1 / 3F1=2
Service life of the functional cosmesis	2 years The service life diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximum service life, in dependence of the level of use.
Service life of the functional stocking	6 months Wear part, subject to natural wear and tear. The service life diminishes or increases depending on the individual level of use. Only

General information	
	when the instructions for use are observed it is possible to achieve the maximum service life, in dependence of the level of use.
Weight of the functional cosmesis (without adaptations)	910 g

10 Appendix

10.1 Symbols Used

Manufacturer

Lot number

Declaration of conformity according to the applicable European directives

Do not dry clean

Iron at medium temperature, max. 150°

Do not tumble dry

Gentle cycle at 30°

Do not bleach

Not made with natural rubber latex.

Natural rubber latex was not used as a material

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Otto Bock Healthcare Products GmbH