## 756G9

<b>DE</b> Gebrauchsanleitung (Benutzer / Techniker) .....	3
<b>EN</b> Instructions for use (user / technician) .....	15
<b>FR</b> Mode d'emploi (Utilisateur / Technicien) .....	27
<b>IT</b> Istruzioni d'uso (utilizzatore / tecnici) .....	39
<b>ES</b> Instrucciones de uso (usuario / técnico) .....	51
<b>ZH</b> 使用说明书 (用户/技术人员) .....	63

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# Table of contents

EN

<b>1</b>	<b>Notes regarding the document .....</b>	<b>16</b>
1.1	Explanation of warning symbols .....	16
1.2	Meanings of pictograms in the illustration .....	16
<b>2</b>	<b>Product description .....</b>	<b>17</b>
<b>3</b>	<b>Scope of delivery .....</b>	<b>19</b>
<b>4</b>	<b>Use .....</b>	<b>20</b>
<b>5</b>	<b>Cleaning .....</b>	<b>26</b>
<b>6</b>	<b>Disposal.....</b>	<b>26</b>
<b>7</b>	<b>Legal information .....</b>	<b>26</b>
7.1	Liability .....	26

# 1 Notes regarding the document

## INFORMATION

Date of last update: 2019-03-05

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Please keep this document in a safe place.

When a text refers to a figure, this is pointed out by a reference to the figure [example: (1)].

This document and the product are intended exclusively for O&P professionals who possess technical expertise for fabricating the following products:


- Prostheses (lower limb)
- Prostheses (upper limb)
- Orthoses (lower limb)

### 1.1 Explanation of warning symbols

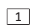 **CAUTION** Warning regarding possible risks of accident or injury.


 **NOTICE** Warning regarding possible technical damage.

### 1.2 Meanings of pictograms in the illustration

	Numbering for the illustrations		The instructions accompanying the illustrations are relevant to safety
	Numbering for a defined sequence		Numbering for the parts of an illustration
	Acceptable		Unacceptable
	Closed		Open


## 2 Product description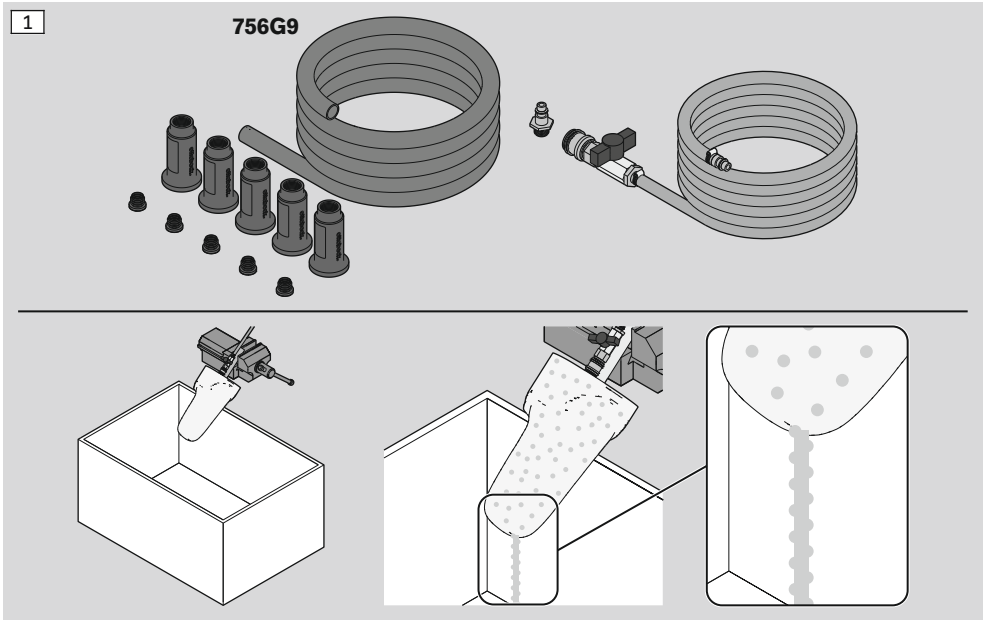

The 756G9 plaster compressed air set is designed to help you dry plaster models more quickly. Compressed air presses the water out of the plaster. This considerably reduces the time required to dry a plaster model. The components are heat-resistant to 130 °C, which means the plaster model can later be used in the plaster drying oven or convection and prepreg ovens for further drying and for processing carbon and prepreg materials.

## Acceptable plaster types

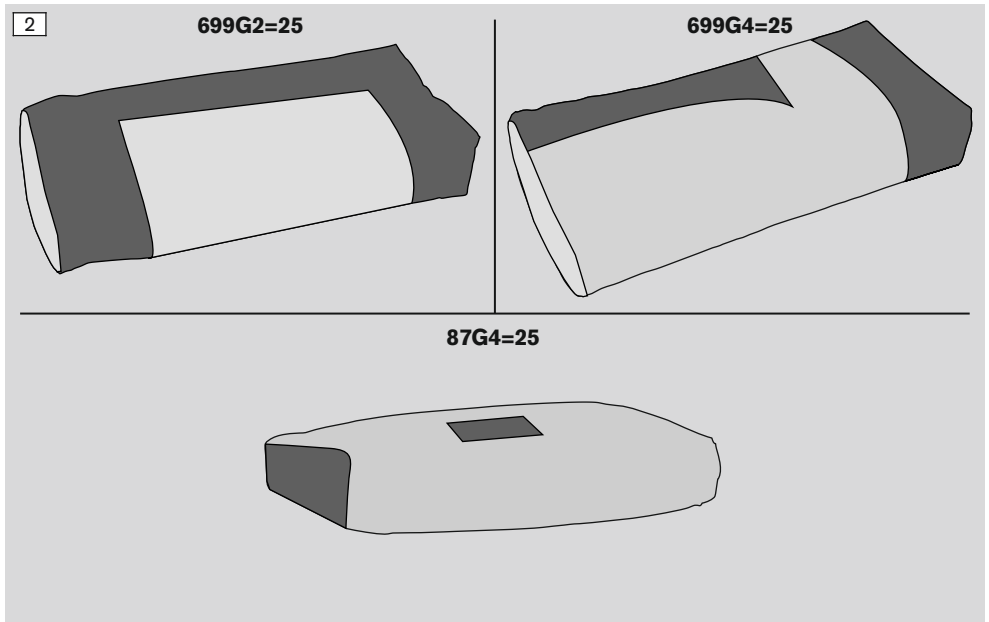- 699G2=25 Stucco plaster
- 699G4=25 Alabaster modelling plaster
- 87G4=25 Porous plaster

Plaster from other suppliers with the same material properties can be used in addition to the Ottobock plaster types shown here.


### Unacceptable plaster types

Any plaster types that include additives to reduce their porosity and thereby make drying with compressed air impossible are not permitted.

The following Ottobock plaster types are not permitted:

- 87G1=40 Synthetic plaster

### 3 Scope of delivery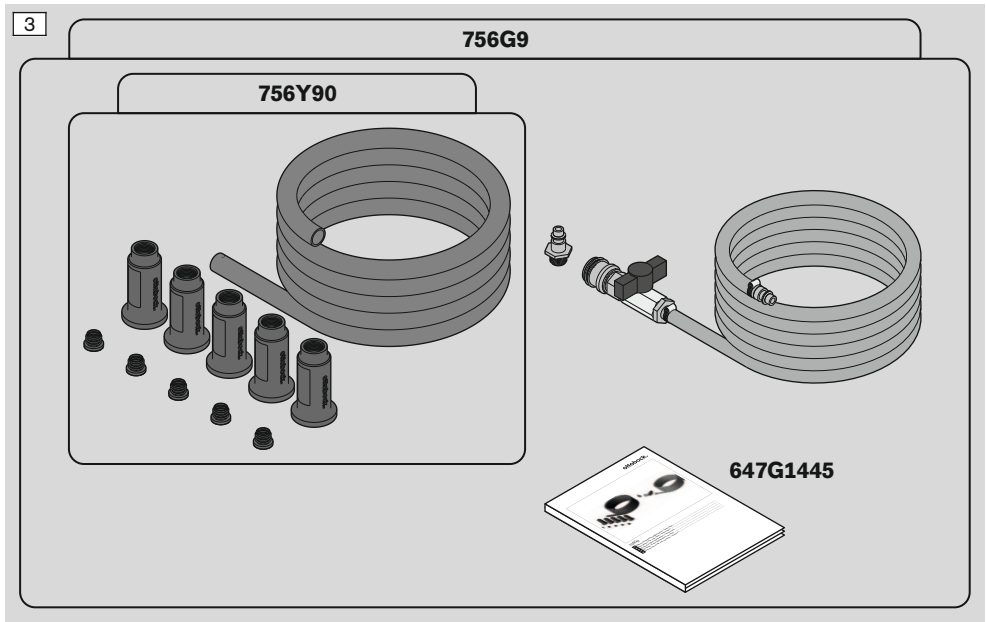

## 4 Use

### **⚠ CAUTION**

#### **Product damage**

Risk of injury due to product damage

- ▶ Use caution when working with the product.
- ▶ Check the product for damage before each use.
- ▶ Do not continue using the product in case of product damage relevant to safety.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

### **INFORMATION**

The illustrations below demonstrate the process for drying plaster with compressed air. The actual processing steps for the plaster positive during the fabrication of the product remain unchanged. Only the drying time in the oven is reduced.

#### **▶ CAUTION!**

Use the required protective clothing during work (hearing protection, safety goggles, chemical protection gloves).

#### **▶ NOTICE!**

Only uncoiled compressed air may be used.

Figure [7](#): Note the distance measurements when positioning the porous tube in the plaster negative. Position the porous tube as close to the centre as possible so the compressed air is applied evenly to the plaster and does not destroy it by overloading it on one side.


Observe the manufacturer's processing instructions when mixing the plaster (e.g. mixing ratio and processing time).

Figure [9](#): Wait the prescribed time before drying with compressed air so the plaster reaches the required strength and is not destroyed by the pressure.


Figure [10](#) (~ 20° rotation): To avoid destroying the plaster, increase the flow of compressed air very gradually in several small increments up to the maximum specified set value for the ball valve.

Figure [12](#): In conjunction with drying in the oven, check the degree of drying prior to removal (e.g. with a measuring device).


### Connecting to compressed air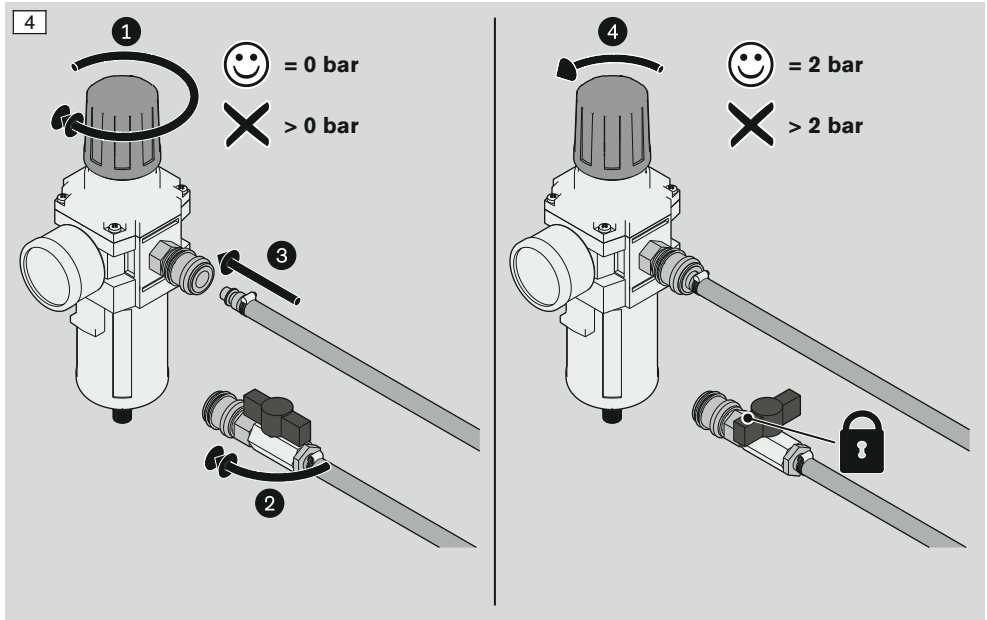


### Shortening the porous tube

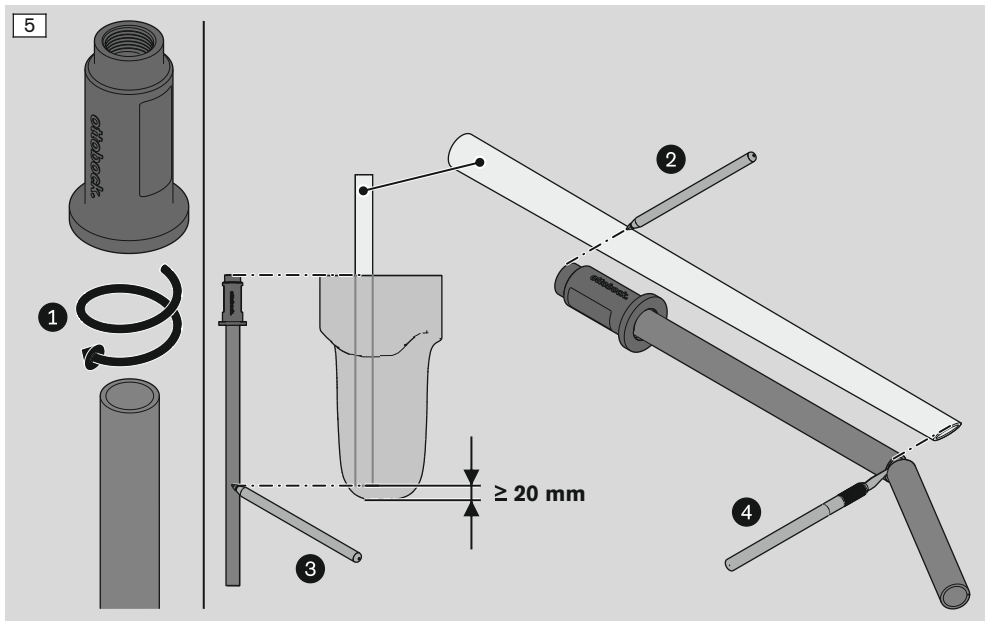
## Attaching the adapter and plug to the porous tube

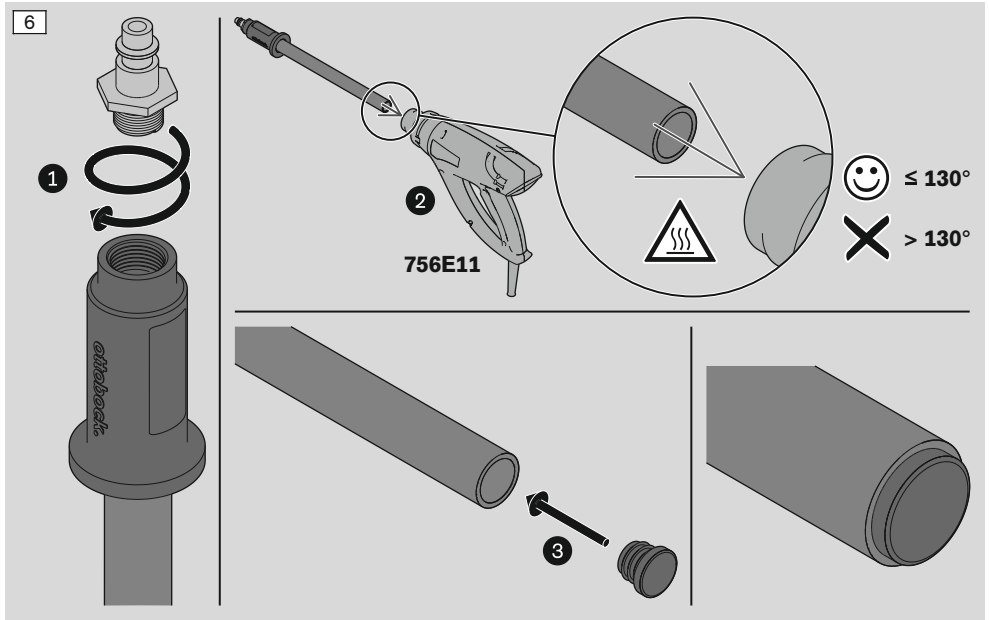
## Positioning the plaster negative

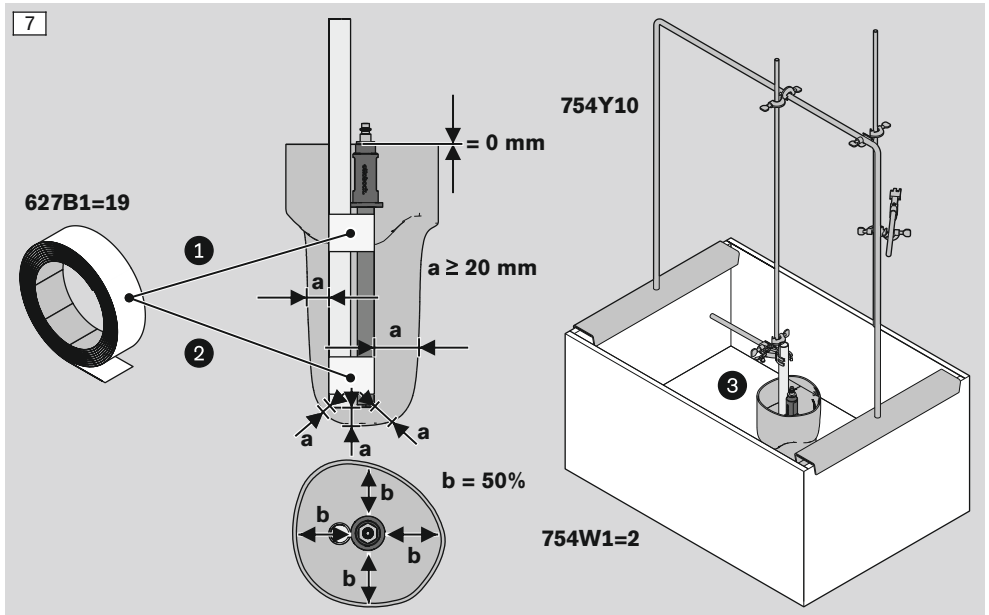
## Casting the plaster positive

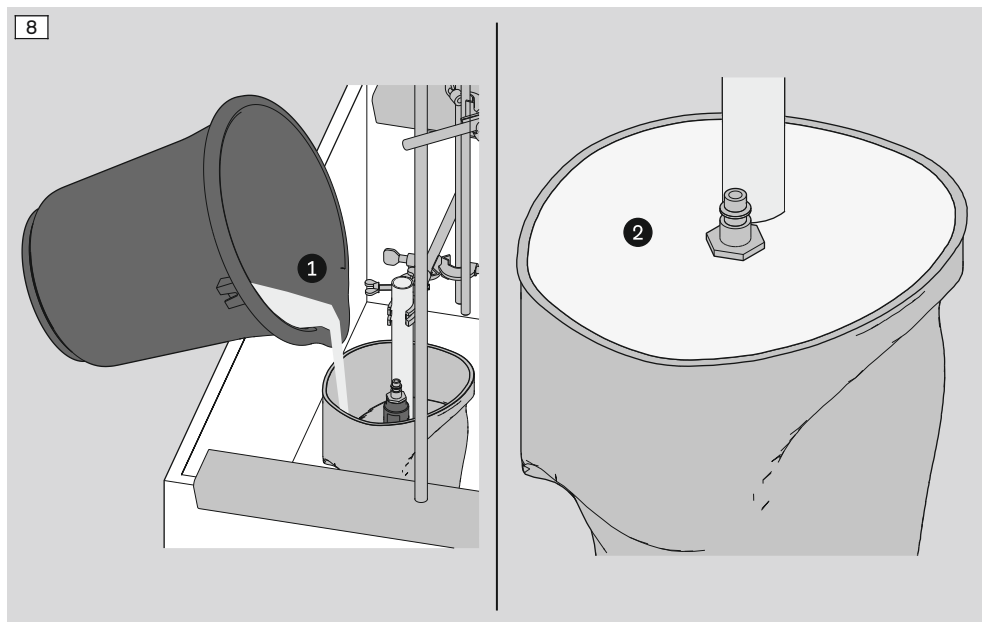
## Preparing the plaster positive for drying with compressed air
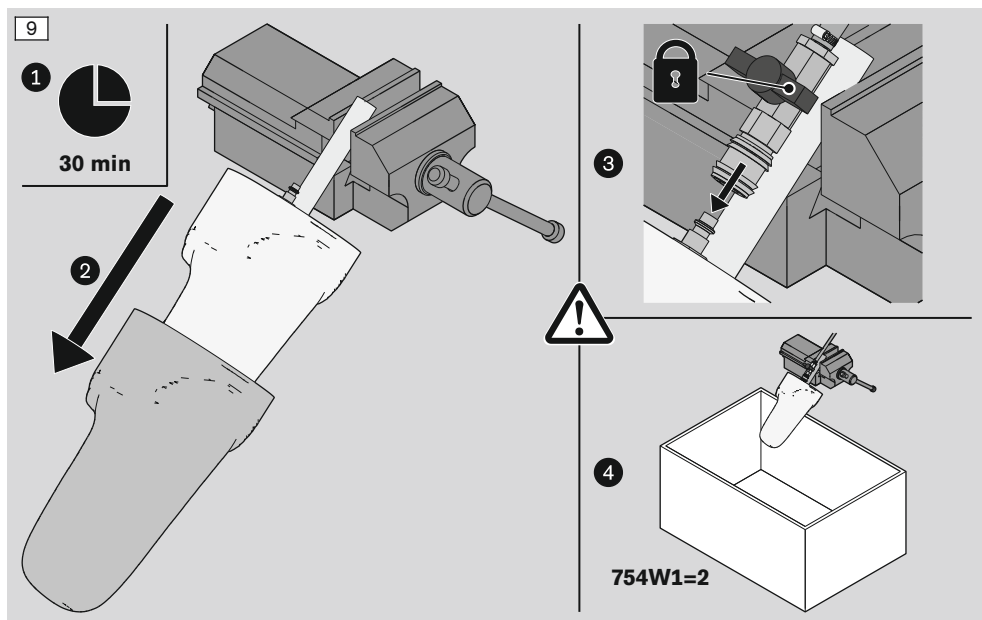
## Drying the plaster positive with compressed air
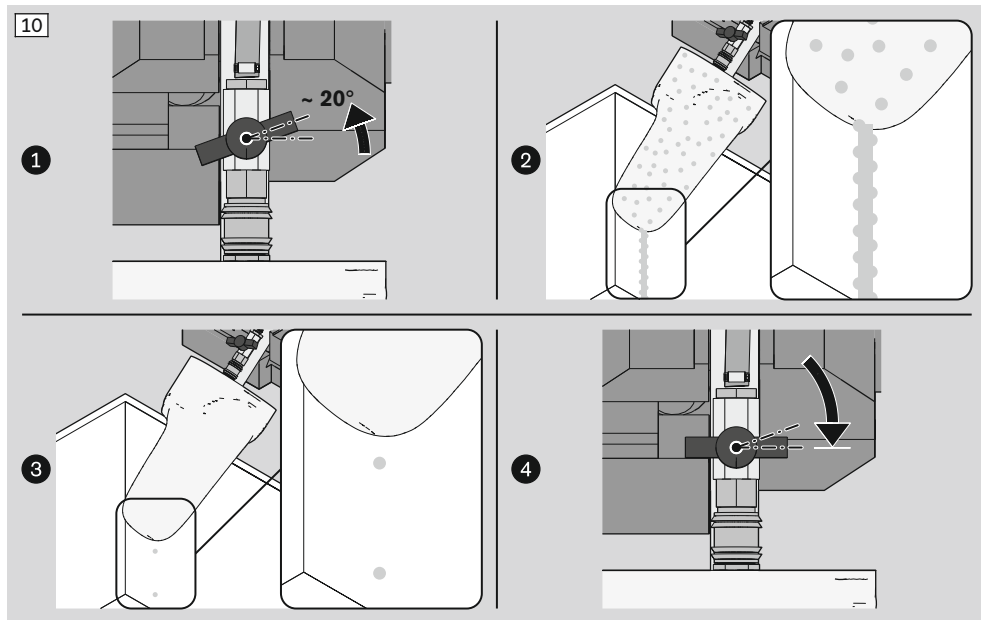
## Clamping the plaster positive

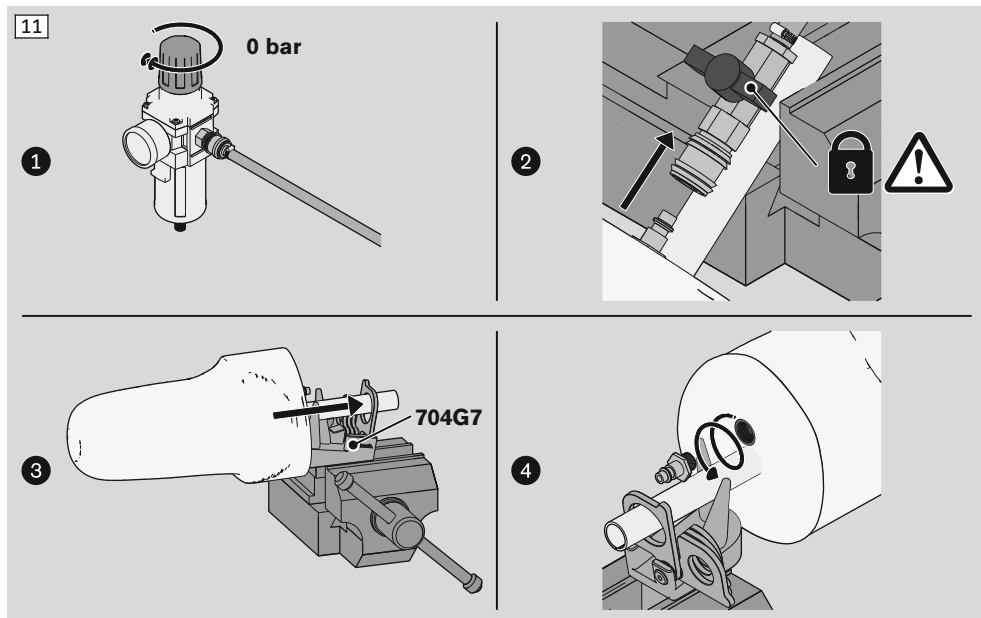
## Further processing of the plaster positive

12


1

716G1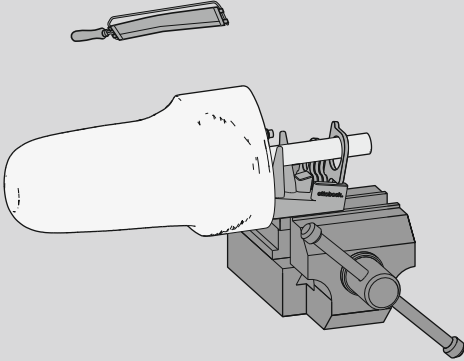

2

701E30

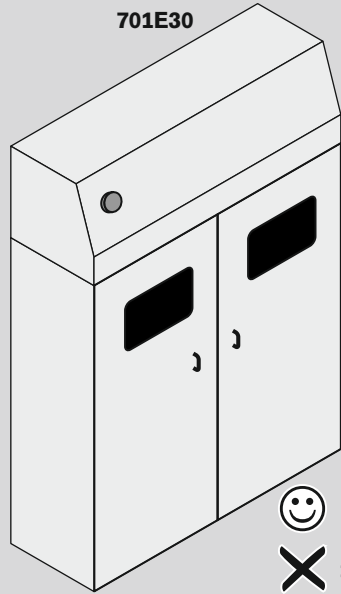## 5 Cleaning

### CAUTION

#### **Failure to follow the instructions for disinfection or cleaning**

Health hazard due to contact with pathogens and damage to the product

- ▶ Observe all instructions in this document.

#### **Cleaning off plaster residue**

- ▶ Remove plaster residue with plaster solvent and a brush.

#### **Cleaning light soiling**

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

#### **Cleaning heavier soiling**

- > **Required materials:** cleaning cloth, 634A58 isopropyl alcohol, soft cloth
- ▶ **NOTICE!** Verify material compatibility.  
Clean the product with a cleaning cloth and isopropyl alcohol.
- ▶ Dry the product with a cloth.

## 6 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

## 7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

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