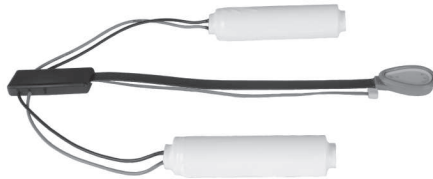
757B35=0

757B35=1/757B35=3











757B35=5

757B35=0, 757B35=1, 757B35=3, 757B35=5

DE Gebrauchsanweisung (Fachpersonal)	4
EN Instructions for use (qualified personnel)	13
FR Instructions d'utilisation (Personnel spécialisé)	22
IT Istruzioni per l'uso (Personale tecnico specializzato)	32
ES Instrucciones de uso (Personal técnico especializado)	42
PT Manual de utilização (Pessoal técnico)	52
NL Gebruiksaanwijzing (Vakmensen)	61
SV Bruksanvisning (Fackpersonal)	71
DA Brugsanvisning (Faguddannet personale)	80
NO Bruksanvisning (Fagpersonell)	90
FI Käyttöohje (Ammattihenkilöstö)	99
SK Návod na používanie (Odborný personál)	108
TR Kullanma talimatı (Uzman personel)	117
EL Οδηγίες χρήσης (Τεχνικό προσωπικό)	126
RU Руководство по применению (Квалифицированный персонал)	136
JA 取扱説明書 (有資格担当者)	147

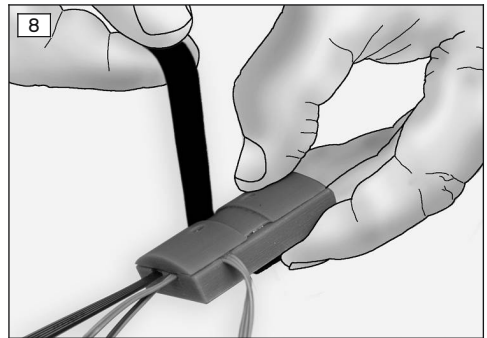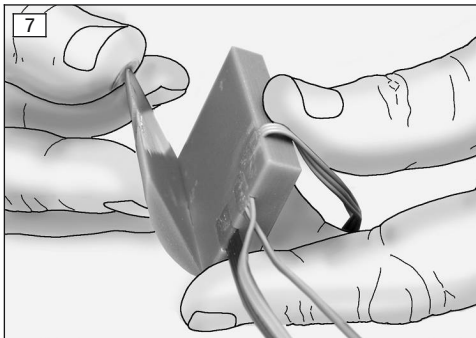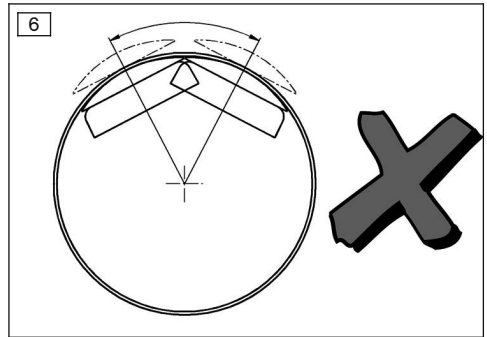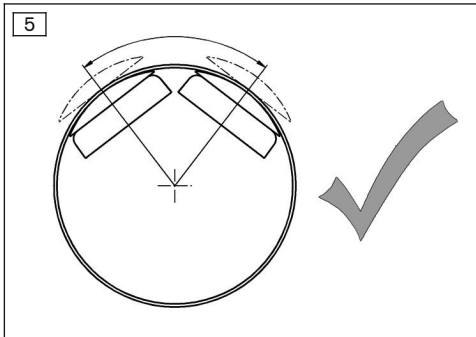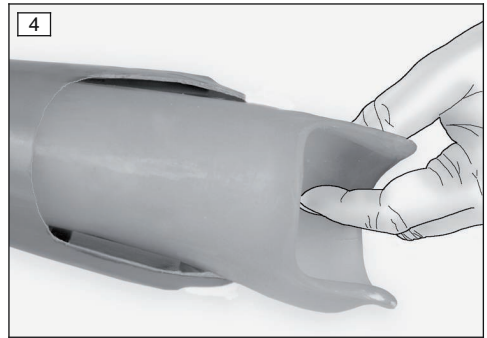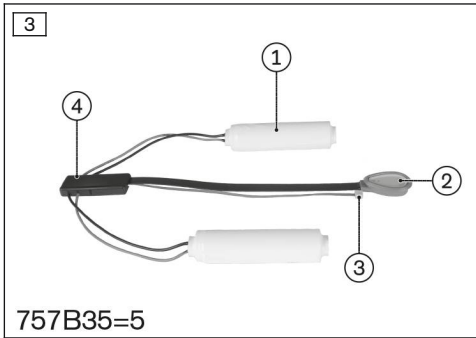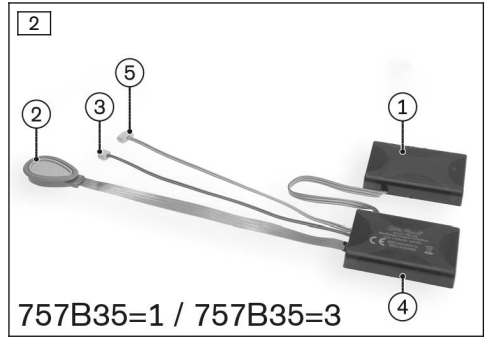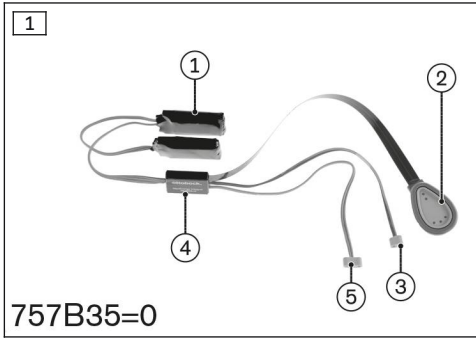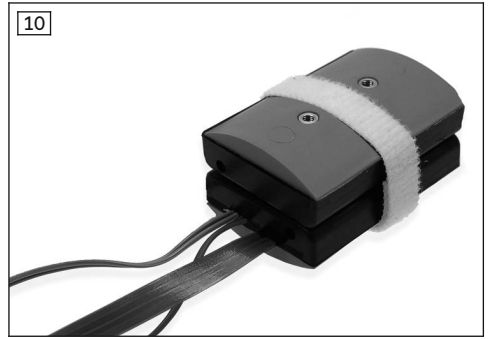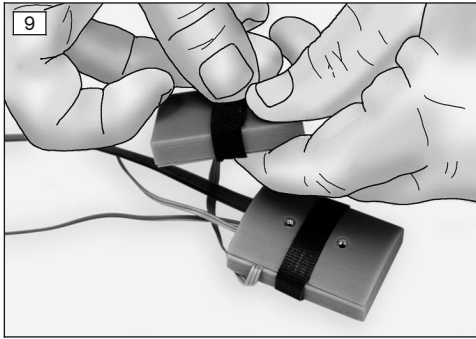INFORMATION

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INFORMATION

Date of last update: 2019-08-19

- ▶ Please read this document carefully before using the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.
- ▶ Please keep this document in a safe place.

The 757B35=* MyoEnergy Integral is referred to simply as the product below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

The product may not be transferred to the patient without prior instruction.

2 Product description

2.1 Construction and Function

The product serves to provide power to a prosthesis.

The product consists of several components. These components are inseparable and ready for mounting.

2.1.1 Components

The product consists of the following components:

Rechargeable battery

The battery (see fig. 1, see fig. 2 and see fig. 3 , item 1) consists of two cells.

Charging receptacle

The charging receptacle (see fig. 1, see fig. 2 and see fig. 3, item 2) has the following functions:

- Contact for charging the battery
- LED to indicate the current charge level
- LED for providing feedback on operating states
- Button for turning the prosthesis on and off
- Beeper for providing feedback on operating states

Supply cable

The supply cable (see fig. 1, see fig. 2 and see fig. 3, item 3) establishes the connection between the product and the respective prosthetic component.

Electronics

The electronics (see fig. 1, see fig. 2 and see fig. 3, item 4) protect the battery against short circuits, overvoltage, undervoltage and charging outside the allowable temperature range.

Communication cable

The communication cable (see fig. 1 and see fig. 2, item 5) is used to exchange data and connects the respective prosthetic components to the product.

INFORMATION

The communication cable is included only with the products 757B35=0, 757B35=1 and 757B35=3.

3 Application

3.1 Indications for use

The product to be used **exclusively** to supply energy to myoelectric upper limb prosthesis components.

3.2 Conditions of use

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

The product is intended **exclusively** for use on **one** patient. Use of the product by another person is not approved by the manufacturer.

Permissible ambient conditions are described in the technical data (see Page 20).

3.3 Qualification

The fitting of a patient with the product may only be carried out by O&P professionals who have been authorised with the corresponding Ottobock training.

4 Safety

4.1 Explanation of warning symbols

 **WARNING** Warning regarding possible serious risks of accident or injury.

 **CAUTION** Warning regarding possible risks of accident or injury.

NOTICE

Warning regarding possible technical damage.

4.2 Structure of the safety instructions

⚠ WARNING**The heading describes the source and/or the type of hazard**

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in the event of failure to observe the hazard
- > E.g.: Consequence 2 in the event of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General safety instructions

⚠ WARNING**Non-observance of safety notices**

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

⚠ CAUTION**Changing prosthetic components when switched on**

Injury due to faulty control or malfunction of the prosthetic components.

- ▶ Before changing prosthetic components (e.g. terminal device) switch the product off.

⚠ CAUTION**Overloading due to unusual activities**

Injury due to unexpected product behaviour as a result of a malfunction.

- ▶ The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports with excessive strain and/or shocks to the wrist joint (pushups, downhill mountain biking, ...) or extreme sports (free climbing, paragliding, etc.).
- ▶ Careful handling of the product and its components not only increases their service life but, above all, ensures the patient's personal safety!
- ▶ If the product and its components have been subjected to extreme loads (e.g. due to a fall, etc.), then the product must be inspected for damage immediately. If necessary, forward the product to an authorised Ottobock Service Centre.

⚠ CAUTION**Penetration of dirt and humidity into the product**

Injury due to unexpected prosthesis behaviour.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.
- ▶ Do not use the product for bathing prostheses.

⚠ CAUTION**Penetration of perspiration into the product**

Injury and chemical burns due to leakage of battery fluids.

- ▶ Wear protective clothing when removing or replacing the battery (disposable gloves and protective goggles at a minimum).

▶ Avoid direct skin contact!

⚠ CAUTION

Using the product with pointed or sharp objects

Destruction or damage to the product during installation.

▶ Do not use any pointed or sharp objects when installing the product.

⚠ CAUTION

Damage to the product due to improper installation

Injury due to product damage.

▶ Install the battery according to the steps in the "Laminating instructions" section (see fig. 5).

NOTICE

Improper care of the housing

Damage to the casing through the use of acetone, white spirit or similar solvents.

▶ Only clean the housing with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).

NOTICE

Use of incorrect power supply unit/battery charger

Damage to product due to incorrect voltage, current or polarity.

▶ Use only power supply units/battery chargers approved for this product by Ottobock (see instructions for use and catalogues).

INFORMATION

Prior to initial operation of the product, charge the supplied battery.

4.4 Patient information

⚠ WARNING

Operating the product near active implanted systems

Interference with active implantable systems (e.g., pacemaker, defibrillator, etc.) due to electromagnetic interference of the product.

- ▶ When operating the product in the immediate vicinity of active implantable systems, ensure that the minimum distances stipulated by the manufacturer of the implant are observed.
- ▶ Make sure to observe any operating conditions and safety instructions stipulated by the manufacturer of the implant.

⚠ CAUTION

Mechanical stress on the product

Injury due to faulty control or malfunction of the product components.

- ▶ Do not subject the product to mechanical vibrations or impacts.
- ▶ Check the product for visible damage before each use.

⚠ CAUTION

Remaining, operating and charging in areas outside the allowable temperature range

Injury due to faulty control or malfunction of the product.

- ▶ Avoid remaining, operating and charging in areas outside the allowable temperature range (see Page 20).

⚠ CAUTION

Proximity to sources of strong magnetic or electrical interference (e.g. theft prevention systems, metal detectors)

Injury due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Avoid remaining in the vicinity of visible or concealed theft prevention systems at the entrance/exit of stores, metal detectors/body scanners for persons (e.g. in airports) or other sources of strong magnetic and electrical interference (e.g. high-voltage lines, transmitters, transformer stations, computer tomographs, magnetic resonance tomographs, etc.).
- ▶ When walking through theft prevention systems, body scanners or metal detectors, watch for unexpected behaviour of the product.

⚠ CAUTION

Independent manipulation of the product

Injury due to malfunction and resulting unexpected prosthesis actions.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.

⚠ CAUTION

Charging the product with soiled or damaged contacts

Injury due to unexpected behaviour of the product caused by insufficient charging.

- ▶ Make sure that the contacts are always clean and free of grease.
- ▶ Clean the electrical contacts of the charging plug and charging receptacle regularly using cotton swabs and a mild soap solution.
- ▶ Take care to avoid damaging the contacts with pointed or sharp objects.

⚠ CAUTION

Distance to HF communication devices is too small (e.g. mobile phones, Bluetooth devices, WiFi devices)

Injury due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Therefore, keeping the following minimum distances to these HF communication devices is recommended:
 - Mobile phone GSM 850/GSM 900: 0.99 m
 - Mobile phone GSM 1800/GSM 1900/UMTS: 0.7 m
 - DECT cordless phones incl. base station: 0.35 m
 - WiFi (routers, access points,...): 0.22 m
 - Bluetooth devices (third-party products not approved by Ottobock): 0.22 m

5 Scope of Delivery and Accessories

5.1 Scope of delivery

- 1 pc. 757B35=* MyoEnergy Integral
- 1 pc. 647G1402 Instructions for use (qualified personnel)

5.2 Accessories

- 2 pc. lamination dummy
- 1 pc. lamination dummy – charging receptacle
- 1 pc. drilling template for charging receptacle

Only included with 757B35=0 and 757B35=5 MyoEnergy Integral

- 1 pc. electronics lamination dummy

Only included with 757B35=1 and 757B35=3 MyoEnergy Integral

- 2 pc. battery installation aid
- 2 pc. Hook and loop closure strap (only included in the scope of delivery for the 757B35=3 MyoEnergy Integral)
- 4 pc. countersunk head screws

Required accessories for operation

- 757L35 MyoCharge Integral

6 Preparation for use

6.1 Laminating instructions

The product must be properly installed in the socket. Please note the corresponding technical information.

6.2 Fastening the MyoEnergy Integral in the outer socket

6.2.1 Installing the 757B35=0 / 757B35=5 battery

- 1) Fasten the batteries and electronics in the outer socket with double-sided adhesive tape.
- 2) Slide the inner socket into the outer socket and screw them together (see fig. 4).

6.2.2 Installing the 757B35=1 / 757B35=3 battery

Assembly of the separate battery units

- 1) Determine the battery positions using the installation aids.

INFORMATION: Maintain sufficient distance between the installation aids/batteries (see fig. 5).

- 2) Mark the hole positions on the socket and drill them with the drill bit.
- 3) Pull the double-sided adhesive tape off the installation aids and apply it to the battery (see fig. 7).
- 4) Secure the installation aid and battery with the hook-and-loop closure strap (757B35=3) or adhesive tape (757B35=1).

INFORMATION: Lay the hook-and-loop closure strap into the groove on the installation aid (see fig. 8). The overlapping ends of the hook-and-loop closure strap have to be up against the battery.

- 5) Use the countersunk head screws to screw the batteries to the socket.

Installation of the assembled battery units

- 1) Fasten the batteries and installation aid with the double-sided adhesive tape (see fig. 10).
- 2) Wrap the installation aid and batteries with the hook and loop closure strap (757B35=3) or adhesive tape (757B35=1).

INFORMATION: Lay the hook and loop closure strap into the groove on the installation aid.

7 Handling

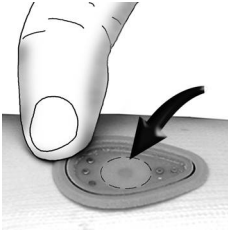
7.1 Switching the terminal device on and off

NOTICE

Signs of wear and tear on the product components

Injury due to faulty control or malfunction of the product.

- ▶ Excessive pressure on the charging receptacle can lead to increased wear and tear on the button.

- ▶ 1) Push and hold the charging receptacle button until a confirmation sound is heard (at least 1 second).
→ The prosthesis and prosthetic components are switched on. Repeat this process to switch the prosthesis and prosthetic components off.

7.2 Charging the battery

We recommend charging on a daily basis. If the device is not used for an extended period, the battery charge level should be checked regularly and the battery recharged.

- 1) Connect the charging plug to the charging receptacle of the product.
 - A correct connection between the battery charger and the product is indicated by feedback (see Page 22).
 - The prosthesis is switched off automatically.
 - The charging process starts.
- 2) Disconnect the product after the charging process is complete.

7.3 Display of the current charge level

The battery charge level can be queried at any time.

- 1) With the prosthetic system switched on, press the charging receptacle button (Taste betätigen) and hold for less than one second.
- 2) The LED display on the charging receptacle provides information on the current battery charge level (Status signals).

7.4 Safety shutoff

The purpose of the safety shutoff is to protect the product; it is triggered in case of:

- Excessively high or low temperature in the battery's environment during the charging process
- Short circuit
- Overvoltage or undervoltage

INFORMATION

After a safety shutdown, the battery charger must be connected to reactivate the electronics.

8 Cleaning and Care

NOTICE

Improper care of the housing

Damage to the casing through the use of acetone, white spirit or similar solvents.

► Only clean the housing with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

9 Disposal

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

10 Legal information

10.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

10.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

10.3 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

The product meets the requirements of the RoHS Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic devices.

11 Technical data

Ambient conditions	
Storage (with and without packaging)	+5 °C/+41 °F to +40 °C/+104 °F Max. 85% relative humidity, non-condensing For reasons of quality (service life), it is recommended that the product be stored at low temperatures.
Transport (with and without packaging)	-20 °C/-4 °F to +60 °C/+140 °F Max. 90% relative humidity, non-condensing
Operation	-5 °C/+23 °F to +45 °C/+113 °F Max. 95% relative humidity, non-condensing
Charging the battery	+5 °C/+41 °F to +40 °C/+104 °F Max. 85% relative humidity, non-condensing

Battery of the product	757B35=0	757B35=1	757B35=3	757B35=5
Battery type	Li-Po	Li-Po	Li-Ion	Li-Ion
Output voltage	approx. 7.4 V	approx. 7.4 V	approx. 7.4 V	approx. 7.4 V
Charging voltage	approx. 8.2 V	approx. 8.2 V	approx. 8.2 V	approx. 8.2 V
Capacity	300 mAh	600 mAh	1,150 mAh	3,450 mAh
Dimensions	35 x 20 x 6.5 mm/battery cell 22 x 17 x 5 mm/electronics	52 x 25 x 9 mm/battery cell	52 x 36 x 9 mm/battery cell	79.5 x 20 mm/battery cell 43 x 19 x 7 mm/electronics
Weight	25 g	45 g	70 g	120 g
Operating time of prosthesis	approx. 1,000 grip cycles for MyoBock adult prostheses approx. 2,500 grip cycles for MyoBock prostheses for children	approx. 1,500–2,000 grip cycles for MyoBock adult prostheses approx. 4,000 grip cycles for MyoBock prostheses for children	approx. 4,000 grip cycles for MyoBock adult prostheses approx. 8 hours for the bebionic hand	approx. 24 hours for MyoBock adult prostheses approx. 24 hours for the bebionic hand
Battery service life	2 years	2 years	2 years	2 years
Charging time until battery is fully charged	2.0 h	2.5 h	2.5 h	3.0 h

12 Appendices

12.1 Symbols Used

Declaration of conformity according to the applicable European directives

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

Lot number

Legal manufacturer

Only included with 757B35=0 MyoEnergy Integral:

IP33

Protection against penetration of solid foreign objects with a diameter > 2.5 mm, protection against water spray

12.2 Operating states/error signals

12.2.1 Status signals

Charging receptacle	Event
LED shows green light	Battery fully charged
LED shows yellow light	Battery 50% charged
LED shows orange light	Battery empty

12.2.2 Beep signals

The following beep signals are set by default:

Beep signal	Additional display	Event
1 x long	-	<ul style="list-style-type: none">Turn off prosthesis on charging receptacleCharging begins (charging plug connected to charging receptacle)Charging ends (charging plug disconnected from charging receptacle)
2 x short	LED on charging receptacle lights up briefly	Switch on prosthesis
3 x short	-	Battery voltage too low, prosthesis shuts off automatically

The following beep signals exist only in conjunction with the 7in1 Controller (9E420=*):

Beep signal	Status
1 x long	Disconnect the MyoLino Link: Prosthesis was previously switched off
2 x short	Connect the MyoLino Link
2 x short	Disconnect the MyoLino Link: Prosthesis was previously switched on
Continuous pulses (as long as the button is being pressed)	Parental access switch

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Otto Bock Healthcare Products GmbH