Liner

DE Gebrauchsanweisung	2	CS Návod k použití	61
EN Instructions for use	6	RO Instrucțiuni de utilizare.....	65
FR Instructions d'utilisation.....	11	HR Upute za uporabu	70
IT Istruzioni per l'uso	16	SL Navodila za uporabo	74
ES Instrucciones de uso	20	SK Návod na používanie	79
PT Manual de utilização.....	25	BG Инструкция за употреба.....	83
NL Gebruiksaanwijzing.....	29	TR Kullanma talimatı	88
SV Bruksanvisning	34	EL Οδηγίες χρήσης	92
DA Brugsanvisning	38	RU Руководство по применению	97
NO Bruksanvisning.....	43	JA 取扱説明書.....	103
FI Käyttöohje.....	47	ZH 使用说明书.....	107
PL Instrukcja użytkowania.....	52	KO 사용 설명서.....	111
HU Használati utasítás	56		

1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2020-02-18

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

INFORMATION

- ▶ The patient must be fitted with the liner no later than the date printed on the inside of the packaging or on the label (MHD/BBDD).

1.1 Function

In combination with an appropriate suspension solution, liners increase wearer comfort and improve the suspension of the prosthesis. Together, they reduce the forces that affect the residual limb.

These instructions for use apply to the following liners:

Name	Reference number
Skeo Unique	6Y700, 6Y702, 6Y704, 6Y716, 6Y717, 6Y718, 6Y730, 6Y732, 6Y734
Uneo standard	6Y510, 6Y512, 6Y520, 6Y522, 6Y523, 6Y540

Name	Reference number
Uneo Unique	6Y400, 6Y400=0_C, 6Y402, 6Y404, 6Y416, 6Y417, 6Y418, 6Y430, 6Y432, 6Y434, 6Y510, 6Y512, 6Y520, 6Y522, 6Y523, 6Y540

Skeo liners

Skeo liners improve shock absorption and wearer comfort.

Ottobock recommends a specific weight-bearing prosthetic socket with distal end contact and the use of a shuttle lock for TT prostheses.

Ottobock recommends using a shuttle lock, KISS lanyard system or suitable elevated vacuum socket for TF prostheses.

Uneo liners

Uneo liners are made of a viscoelastic material. It adapts very well to the form and structure of the residual limb and protects very well against impact forces.

Ottobock recommends a total surface weight-bearing socket (the "Harmony" principle) with a suitable vacuum system.

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

- **Skeo Unique liner with a metal cap at the distal end:** The metal cap is for connection to the 4R220* Dynamic Vacuum System. **The liner is intended exclusively for use in combination with the 4R220* Dynamic Vacuum System.**

2 Intended use

2.1 Indications for use

- **6Y400, 6Y402, 6Y404, 6Y416, 6Y417, 6Y418, 6Y430, 6Y432, 6Y434, 6Y700, 6Y702, 6Y704, 6Y716, 6Y717, 6Y718, 6Y730, 6Y732, 6Y734:** The product is to be used exclusively for prosthetic fittings of either the upper or the lower limbs.
- **6Y510, 6Y512, 6Y520, 6Y522, 6Y523, 6Y540:** The product is to be used exclusively for prosthetic fittings of the lower limbs.

2.2 Service life

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

 **WARNING** Warning regarding possible serious risks of accident or injury.

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

3.2 General safety instructions

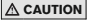
 **WARNING**
Product is pulled over mouth and nose

Danger of suffocation due to lack of oxygen supply

- ▶ Store the product away from the reach of children.

 **CAUTION**
Contact with cold, heat, embers or fire
Injuries (e.g. burns) caused by melting of the material and damage to the product

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to temperatures above +80 °C or below -20 °C.

 **CAUTION**
Reuse on other persons and improper cleaning
Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

⚠ CAUTION

Incorrect or excessively tight application of the product to the body

Risk of pressure points and constriction of blood vessels and nerves due to improper application

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ Do not continue to use the product if there are problems with the correct fit.

⚠ CAUTION

Use of the product on damaged skin

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Only use the product on healthy skin.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Liner	–

5 Preparation for use

The following technical information is available if needed:

Uneo liner	646T2=1.8
------------	-----------

5.1 Size selection (standard liner only)

INFORMATION

Be sure to select the correct liner size. Possible problems are listed below.

- ▶ **Liner too small:** Feelings of numbness, formation of blisters, pistoning in the prosthetic socket
- ▶ **Liner too large:** Increased perspiration, rotation of the residual limb in the liner, formation of blisters, skin rash

- 1) **TF liner:** Measure the residual limb circumference **5 cm** proximal of the residual limb end.
TT liner: Measure the residual limb circumference **4 cm** proximal of the residual limb end.
- 2) **Only with 6Y510, 6Y512:** Measure the residual limb length from the MPT point (mid-patella-tendon) to the end of the residual limb.
- 3) **CAUTION! Reduce by a maximum of 1 liner size to avoid excessive compression.**
Select the liner size that corresponds to the resulting measurement. Alternatively the next smaller size can be chosen.
- 4) Check the fit of the liner.

5.2 Adapting the liner

- > **Recommended tools for liners with textile cover:** 719S7 trimming scissors, 719S20 scissors for synthetic fibres or conventional straight edge scissors
- > **Recommended tools for liners without textile cover:** 756L9 trimming device, 756L10 liner trimmer or conventional straight edge scissors
- ▶ **NOTICE! Do not use wave edge scissors.**
TT liner: Cut the liner horizontally at the required length. Make sure a smooth cutting edge is obtained to prevent the material from tearing.
TF liner: Cut the liner according to the trim line. The liner always has to end outside the prosthetic socket.

5.3 Optional: attaching the pin

INFORMATION

The shape of the prosthetic socket must allow load on the end of the residual limb.

The pin to connect the liner to the shuttle lock is screwed into the thread at the distal end of the liner.

> **Required materials:** Torque wrench (adjustable to 3 Nm), 636K13 Loctite

- 1) Apply Loctite to the thread.
- 2) **NOTICE! Do not shorten flexible pins and pins with one-way valve.**
Screw the pin into the thread and tighten it (installation torque: **3 Nm**).

6 Use

CAUTION

Use of powder (e.g. baby powder or talcum)

Skin irritation on the residual limb and loss of function of prosthetic components due to clogging with particles or deprivation of lubricant

- ▶ Keep the product away from powders.

NOTICE

Mechanical overload

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

INFORMATION

Some liners are treated with the Skinguard Technology additive. Skinguard Technology has an antibacterial effect and prevents the formation of unpleasant odours.

Substances in sweat containing sulphur may cause discolouration. This discolouration has no effect on the function or durability of the liner but cannot be removed. Sulphur may enter sweat through food, for example, protein products, cabbage, nuts, legumes and sulphured products (e.g. dried fruit).

Discolouration of the liner does not justify customer complaint claims.

INFORMATION

- ▶ Liners that have already been worn generally feel softer than new liners.
- ▶ Because of the relatively long drying times of Uneo liners, a spare liner is recommended.
- ▶ Clean new liners before using for the first time.

INFORMATION

Derma Prevent can damage the outer and inner coating of Uneo liners and the outer coating of Skeo liners.

6.1 Putting on and taking off the liner

Donning

INFORMATION

Wear the liner directly on the skin. Fabrics between the skin and the liner can cause skin irritation.

If any allergic reaction occurs, immediately discontinue use and contact your medical professional.

- 1) Check the liner for damage. If a pin is attached, also check the pin for damage.

- 2) Roll up the liner and place it on the end of the residual limb.
NOTICE! Align a distal connecting element (e.g. pin, metal cap) with the longitudinal axis of the residual limb.
- 3) Unroll the liner onto the residual limb so that it has no wrinkles or air inclusions, without shifting the soft tissue.
- 4) Check the fit and the alignment of the liner.
INFORMATION: TF liners are not permitted to end inside the prosthetic socket.

Doffing

- 1) Roll up the liner and remove it from the residual limb.
- 2) **Recommendation:** Clean the liner (see Page 10).

6.2 Cleaning and Care

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- Only clean the product with the approved cleaning agents.

INFORMATION

Clean the liner and skin more frequently in case of increased perspiration. Wearing a liner requires daily intensive skin care. Ottobock recommends the use of the Derma skin care system.

- 1) Turn the liner inside out.
- 2) Hand wash the liner in warm water (**30 °C**) using neutral detergent (e.g. Ottobock 453H10=1 Derma Clean).
- 3) Thoroughly rinse the liner with clear water until all detergent residues have been removed.
- 4) Turn the liner again so the outer side/textile material is on the outside.

- 5) Put a hand towel into the liner and leave the liner to dry lying flat on a drying rack. Alternatively, use a drying stand (TT: 5Y100=1 TF: 5Y100=2).

NOTICE! Liner with textile cover: To avoid damaging the liner, dry it with the textile material on the outside.

7 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

8.3 Warranty

The manufacturer warrants this device from the date of fitting. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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