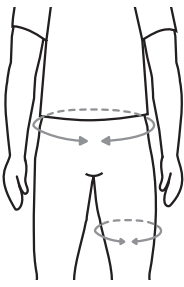
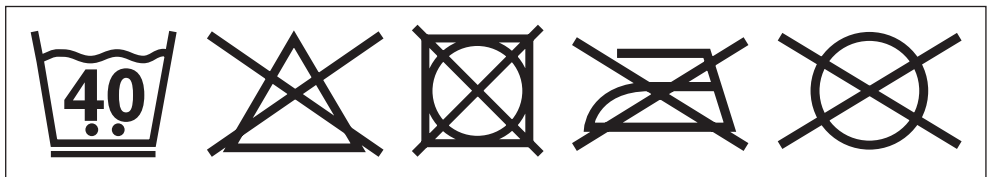
28LT1 Fit Kit

DE Gebrauchsanleitung (Benutzer / Techniker)	3
EN Instructions for use (user / technician)	5
FR Mode d'emploi (Utilisateur / Technicien)	7
IT Istruzioni d'uso (utilizzatore / tecnici)	10
ES Instrucciones de uso (usuario / técnico)	12
PT Manual de instruções (Usuário / Técnico)	14
NL Gebruiksaanwijzing (Gebruiker/Technicus)	16
SV Bruksanvisning (användare/tekniker)	19
DA Brugsvejledning (bruger/tekniker)	21
NO Bruksanvisning (bruker/teknikker)	23
FI Käyttöohje (käyttäjät/teknikko)	25
PL Instrukcja użytkowania (Użytkownik / Technik)	28
HU Használati utasítás (felhasználó / műszerész)	30
CS Návod k použití (pro uživatele / techniky)	32
HR Upute za uporabu (korisnik/tehničar)	35
SK Návod na použitie (používateľ / technik)	37
TR Kullanım kılavuzu (Kullanıcı / Teknisyen)	39
RU Руководство по применению (для пользователей / технического персонала)	41
ZH 使用说明书 (用户/技术人员)	44

	Artikelnummer	Konfektionsgröße	Hüftumfang (cm)	Polsternummer	Oberschenkelumfang (cm)
	28L100=80	74-86	57-59	1*	26-32
	28L100=92	86-98	59-62		
	28L100=104	98-110	62-64	2*	32-36
	28L100=116	110-122	64-68		
	28L101=128	122-134	68-72	3*	36-44
	28L101=140	134-146	72-78		
	28L101=152	146-158	78-86	4*	44-54
	28L101=S	158-170	86-96		
		♀ 36-38 ♂ 44-46			
28L101=M	♀ 40-42 ♂ 48-50	96-105	5*	54-62	
	28L101=L	♀ 44-46 ♂ 52-54			105-109

Material	PA, EA, PU Schaum / PU foam
-----------------	-----------------------------

INFORMATION

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INFORMATION

Date of last update: 2020-03-31

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information for using the test pants of the 28LT1 Fit Kit for the 28L100 Cosa Junior and 28L101 Cosa Active abduction pants. Further information about the handling of the 28L100 and 28L101 is found in the 647G1115 instructions for use.

2 Product description

Quantity	Designation	Reference number
1	Fit Kit	28LT1
	Consisting of:	
10	Test pants in 10 sizes	
5	Test pads in 5 sizes	
4	Suspender clips with connecting strap	
1	Carrying bag	

3 Intended use

3.1 Indications for use

The test pants have the same sizes as the original pants and have no pockets for the pads. The test pants are intended solely for selecting the right size. They are used to assess whether changes have to be made to the original pants and the original pads in order to improve the fit.

3.2 Area of application

The product is used to test and evaluate whether treatment with the 28L100 Cosa Junior or 28L101 Cosa Active is suitable.

4 Safety

4.1 Explanation of warning symbols

NOTICE

Warning regarding possible technical damage.

4.2 General safety instructions

NOTICE**Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

5 Handling

INFORMATION**Using the product for an extended period of time**

- ▶ The product is not suitable for ongoing use.

INFORMATION**Unsupervised use of the product**

- ▶ Only use the product if a specialist instructed and trained for the product is available to you on site.

INFORMATION**Improper cleaning, skin infection due to germs**

- ▶ For hygiene reasons, the product always has to be worn over clothing.

Selection according to clothes size

- ▶ Select the orthosis size based on the clothing size (see sizing table).

Selection according to circumferences

- 1) Determine the size of the pants (see size chart).
- 2) Determine the size of the pads (see size chart).

Application

- ▶ Use pants with a belt for the functional test.
- 1) Put the test pants on the patient and check the chosen size.
 - 2) Take off the test pants when the correct size is found.
 - 3) Attach the suspender clips to the belt on the pants.
 - 4) Apply the test pads to the thigh according to the patient and engage the connecting straps with the help of the hook-and-loop closures.
 - 5) **NOTICE! Do not adapt the test pads by sanding or gluing.**
Determine the required change in the abduction angle.

Functional test

- ▶ Verify the fit and functionality of the test pants and test pads.

5.1 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Cleaning the test pants, connecting straps and test pads

- 1) Close the hook-and-loop closures on the straps.
- 2) Wash the pants and/or straps in warm water at **40 °C** with standard mild detergent. Do not use fabric softener. Rinse thoroughly. **Recommendation:** Use a laundry bag or net.
- 3) Wash the pads in clear water using a cloth. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

6 Disposal

Dispose of the product in accordance with national regulations.

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

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