**453A2, 453A3, 453A4, 453A30,
453A40, 454A7, 454A8****CE**

[DE] Gebrauchsanweisung	2
[EN] Instructions for use	5
[FR] Instructions d'utilisation	9
[IT] Istruzioni per l'uso	13
[ES] Instrucciones de uso	17
[PT] Manual de utilização	21
[NL] Gebruiksaanwijzing	24
[SV] Bruksanvisning	28
[DA] Brugsanvisning	32
[NO] Bruksanvisning	35
[PL] Instrukcja użytkowania	39
[HU] Használati utasítás	43
[CS] Návod k použití	46
[HR] Upute za uporabu	50
[EL] Οδηγίες χρήσης	54
[RU] Руководство по применению	58

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English

Date of last update: 2020-04-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

1 Application

All Ottobock sealing sleeves are to be used exclusively as prosthetic devices for the lower limbs in order to provide the prosthesis with additional suspension and/or to provide an airtight seal for the inner socket (e.g. when using passive or active elevated vacuum sockets).

2 Safety

2.1 Explanation of warning symbols

⚠ WARNING

Warning regarding possible serious risks of accident or injury.

⚠ CAUTION

Warning regarding possible risks of accident or injury.

NOTICE

Warning regarding possible technical damage.

2.2 General safety instructions

⚠ WARNING

Product is pulled over mouth and nose

Danger of suffocation due to lack of oxygen supply

1. Store the product away from the reach of children.

⚠ CAUTION

Reuse on other patients and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

1. Only use the product for a single patient.
2. Clean the product regularly.

⚠ CAUTION

Incorrect or excessively tight application of the product to the body

Risk of pressure points and constriction of blood vessels and nerves due to improper application

1. Ensure that the product is applied properly and fits correctly.
2. Do not continue to use the product if there are problems with the correct fit.

⚠ CAUTION

Use of the product on damaged skin

Skin irritation, formation of eczema or infections due to contamination with germs

1. Only use the product on healthy skin.

⚠ CAUTION

Contact with heat, embers or fire

Injuries (e.g. burns) caused by melting of the material

1. Keep the product away from open flame, embers and other sources of heat.

3 Service life

The product is a wear part, which means it is subject to normal wear and tear.

4 Description

4.1 Copolymer (TPE) Knee Sleeves

The Derma Protection, Derma ProFlex and Derma ProFlex Plus sealing sleeves are coated with copolymer gel on the inside. These sealing sleeves provide optimum suspension on the socket and leg, while their elasticity allows a high degree of mobility for the knee. They can be used as the primary suspension system or together with elevated vacuum sockets. The Copolymer gel is molded with a medical mineral oil, which has a beneficial effect on the skin. During wear the oil slowly diffuses from the gel, thus lubricating and moisturizing the skin.

- Derma Protection Knee Sleeve (453A2): basic, cylindrical shape for flexible positioning; available in 4 sizes
- Derma ProFlex/Derma ProFlex Plus Knee Sleeve (453A3, 453A4, 453A30, 453A40): anatomical and conical shape with 15° pre-flexion for improved range of motion in the knee joint; available in 3 sizes

4.2 Polyurethane (PUR) Knee Sleeves

The Knee Sleeves Harmony® and Harmony® conical are made from a Polyurethane gel. The main characteristic of the polyurethane gel is its unique yielding (flowing) effect. The Polyurethane Knee Sleeves are designed for the use with valves or the Harmony® system. They should always be used in connection with the Gaiter 454A11 to extend their longevity and retain their sealing properties (also see chapter 5.1). The Gaiter is included in the delivery.

- Harmony Knee Sleeve (454A7): slight conical shape, flexibility for positioning and a special durable textile cover; available in 5 sizes
- Harmony conical Knee Sleeve (454A8): very conical shape for improved fit with patients that have a large thigh circumference; available in 7 sizes

5 Indications for use

5.1 Gaiter 454A11 (only with Polyurethane Knee Sleeves)

The Gaiter is designed to prolong the PUR Knee Sleeve longevity and is included in the delivery. The interior has a smooth Lycra® surface that will glide over the socket brim while the exterior has an airtight film surface that will seal against the Knee Sleeve. Please install the Gaiter before using the Knee Sleeve according the following guidelines:

5.1.1 Installation of the Gaiter

- Apply the Gaiter on the exterior proximal section of the socket
- The distal Gaiter edge should be 1" (2,5cm) below the posterior socket trim line
- Tape the Gaiter to the socket with vinyl or polyethylene tape

5.1.2 Size combinations Gaiter & Knee Sleeves

Size	Art.-No.	Fits PUR Knee Sleeve
Small	454A11=S	Size 1
Medium	454A11=M	Size 2 & 3
Large	454A11=L	Size 4 & 5
X-Large	454A11=XL	Size 6
XX-Large	454A11=XXL	Size 7

5.2 Installation and handling of the Knee Sleeve

- Take the packaging foam out of the TPE Knee Sleeve and keep it for use when storing the sleeve
- Turn the distal half of the Knee Sleeve inside out so that the gel is on the outside
- Put it on top of the prosthesis and roll the outside turned gel down on the socket
- To don the prosthesis roll down the top half of the Knee Sleeve

Without Gaiter (TPE):	With Gaiter (PUR):
Step into the prosthesis with the liner (and a sock if needed/used)	Roll down upper part of the Gaiter over the socket brim Step into the prosthesis with the liner (and a sock if needed/used) Roll up the Gaitor wrinkle free

- Pull the Knee Sleeve wrinkle free over the residual limb
- To remove the prosthesis follow the steps in reverse order

6 Care instructions

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

1. Only clean the product with the approved cleaning agents.

After use, hand wash the Knee Sleeve with Derma Clean 453H10 (or ph-neutral soap) and warm water (approx. 30°C / 86°F). Do not wring! Rinse thoroughly under clear, running water. Blot the Knee Sleeve with a towel. Let it air dry completely on a rack or flat on a towel.

For better hygiene it is recommended the Knee Sleeve be removed for a few hours each day, thus providing the skin with fresh air.

When not using the TPE suspension sleeve, insert the packaging foam back into the Knee Sleeve to allow for wrinkle free storage.

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

7.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

7.3 Warranty

The manufacturer warrants this device from the date of fitting. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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