# ottobock.

## 1E90\* Sprinter

<b>DE</b> Gebrauchsanweisung .....	2
<b>EN</b> Instructions for use .....	9
<b>FR</b> Instructions d'utilisation .....	15
<b>IT</b> Istruzioni per l'uso .....	23
<b>ES</b> Instrucciones de uso .....	30
<b>PT</b> Manual de utilização .....	37
<b>NL</b> Gebruiksaanwijzing .....	44
<b>SV</b> Bruksanvisning .....	52
<b>DA</b> Brugsanvisning .....	58
<b>NO</b> Bruksanvisning .....	65
<b>FI</b> Käyttöohje .....	72
<b>PL</b> Instrukcja użytkowania .....	79
<b>HU</b> Használati utasítás .....	86
<b>CS</b> Návod k použití .....	93
<b>RO</b> Instrucțiuni de utilizare .....	100
<b>HR</b> Upute za uporabu .....	107
<b>SL</b> Navodila za uporabo .....	114
<b>SK</b> Návod na používanie .....	120
<b>BG</b> Инструкция за употреба .....	127
<b>TR</b> Kullanma talimatı .....	135
<b>EL</b> Οδηγίες χρήσης .....	141
<b>RU</b> Руководство по применению .....	149
<b>JA</b> 取扱説明書 .....	157
<b>ZH</b> 使用说明书 .....	164
<b>KO</b> 사용 설명서 .....	169

## INFORMATION

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## INFORMATION

Date of last update: 2020-03-20

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

### 1.1 Construction and Function

The 1E90\* Sprinter prosthetic foot is intended for use in a sport prosthesis. It is distinguished by its low weight. The spring contour ensures a high driving force and a low resistance.

### 1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

## INFORMATION

The prosthetic socket must be able to withstand the higher loading of the sport.

### Limited combination options for Ottobock components

	Designation	Reference number
<b>Outer soles and safeguard sticker</b>	Outer sole without spikes	2Z500
	Outer sole with spikes	2Z501
	Safeguard sticker	2Z358
<b>TF fitting for a body weight of up to 100 kg</b>	TF test sport foot adapter	4R206
	TF definitive sport foot adapter	4R204
<b>TT fitting for a body weight of up to 100 kg</b>	TT test sport foot adapter	4R210, 4R212
	TT definitive sport foot adapter	4R208

	<b>Designation</b>	<b>Reference number</b>
<b>TT fitting for a body weight of up to 125 kg</b>	Posterior connection plate (set)	4R420
	T-adapter for TT	2R176
	5° L-adapter	2R177=5
	18° L-adapter	2R177=18

## 2 Intended use

### 2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings. The product is not suitable for use in everyday prostheses.

### 2.2 Area of application

- The maximum approved body weight is specified in the technical data (see Page 15).

### 2.3 Environmental conditions

<b>Allowable environmental conditions</b>
Temperature range for use: -10 °C to +60°C
Allowable relative humidity 0 % to 90 %, non-condensing
<b>Unallowable environmental conditions</b>
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e.g. talcum)

### 2.4 Service life

As part of a sport prosthesis, an exact service life cannot be determined for the product since strain varies widely depending on the sport and intensity of use.

## 3 Safety

### 3.1 Explanation of warning symbols

 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

## 3.2 General safety instructions

### CAUTION

#### **Exceeding the service life and reuse on another patient**

Risk of injury due to loss of functionality as well as damage to the product

- ▶ Ensure that the approved service life is not exceeded.
- ▶ Only use the product on a single patient.

### CAUTION

#### **Excessive strain on the product**

Risk of injury due to breakage of load-bearing components

- ▶ Use the product according to the specified area of application (see Page 10).

### CAUTION

#### **Unallowable combination of prosthetic components**

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

### NOTICE

#### **Mechanical damage to the product**

Change in or loss of functionality due to damage

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

### NOTICE

#### **Use under unallowable environmental conditions**

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.

- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

### Signs of changes in or loss of functionality during use

Decreased forefoot resistance or changes in roll-over behaviour are noticeable indications of loss of functionality.

## 4 Scope of delivery

1E90 Sprinter					
Fig.	Item no.		Quantity	Designation	Reference number
–	–	■	1	Instructions for use	647G849
1	1	–	1	Prosthetic foot	–

## 5 Preparing the product for use

### CAUTION

#### Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

### 5.1 Selection of the Stiffness Version

The stiffness of the prosthetic foot is selected depending upon the body weight and the preferred running style (short distance running or sprint/long distance running). The recommended body weight ranges for each stiffness version are listed according to running style in the following table.

Stiffness version		Short-distance running	Long-distance running
		Body weight	Body weight
SPR-1	[kg]	40 to 52	40 to 59
	[lbs]	90 to 115	90 to 130
SPR-2	[kg]	53 to 63	60 to 70
	[lbs]	116 to 140	131 to 155
SPR-3	[kg]	64 to 79	71 to 86
	[lbs]	141 to 175	156 to 190
SPR-4	[kg]	80 to 95	87 to 102
	[lbs]	176 to 210	191 to 225

Stiffness version		Short-distance running	Long-distance running
		Body weight	Body weight
SPR-5	[kg]	96 to 111	103 to 118
	[lbs]	211 to 246	226 to 260
SPR-6	[kg]	112 to 125	119 to 125
	[lbs]	247 to 275	261 to 275

## 5.2 Mounting the Outer Sole

### CAUTION

#### **Incorrect use of the prosthetic foot and the outer sole**

Risk of falling due to lack of surface grip and damage to the product

- ▶ Only use the prosthetic foot with an outer sole.
- ▶ Only use the outer sole on the surface it was designed for.
- ▶ **Inform the patient.**

- ▶ Mount the chosen outer sole according to the corresponding instructions for use.

Connecting component	Instructions for use reference number
2Z500 Outer Sole without Spikes	647G848
2Z501 Outer Sole with Spikes	

## 5.3 TF Prosthesis Alignment Recommendations

For a TF fitting the prosthetic foot is connected to a sport knee joint by means of the TF test sport foot adapter or the TF definitive sport foot adapter.

### **INFORMATION**

When using the 3S80 Sport Prosthetic Knee Joint:

- ▶ Observe the 647G813 Instructions for Use for the sport prosthetic knee joint.
- ▶ Observe the 647H537 Quick Guide for the sport prosthetic knee joint.
- ▶ Assemble the prosthetic foot according to the instructions for use for the adapter used.

Connecting component	Instructions for use reference number
4R206 TF Test Sport Foot Adapter	647G839
4R204 TF Definitive Sport Foot Adapter	

## 5.4 TT Alignment Recommendations

For a TT fitting the prosthetic foot is connected to the prosthetic socket by means of the suitable adapter.

- ▶ Assemble the prosthetic foot according to the mounting instructions for the selected adapter.

Connecting component	Assembly instructions
<b>Posterior connection</b>	
4R210 TT test sport foot adapter	Please note the 647G840 instructions for use.
4R208 TT definitive sport foot adapter	
4R212 TT test sport foot adapter	Please note the 647G981 instructions for use.
4R420 posterior connection plate (set)	The mounting instructions are enclosed with the product.
<b>Distal connection</b>	
2R176 T-adapter	The mounting instructions are enclosed with the respective product.
2R177=5 5° L-adapter	
2R177=18 18° L-adapter	

## 6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

## 7 Maintenance

### CAUTION

#### **Failure to follow the maintenance instructions**

Injuries due to changes in or loss of functionality and damage to the product

- ▶ Observe the maintenance instructions.
  - ▶ **Inform the patient.**
- During the consultation, listen for unusual noises and inspect the entire prosthesis for damage, wear and functional changes.
  - Arrange regular maintenance intervals with the patient depending on the level of use.
- Direct the patient to observe the following maintenance instructions:
- Check the entire prosthesis for damage after each use.
  - In case of changes in or loss of functionality and damage, do not continue using the prosthesis and have it checked by qualified personnel.
  - Comply with the maintenance intervals.

## 8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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