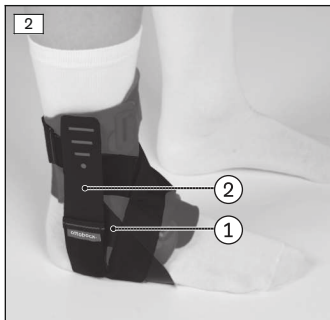
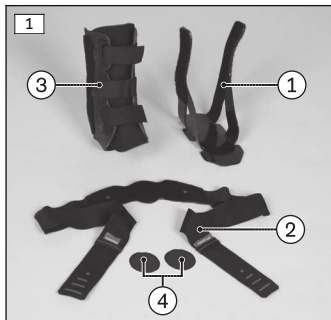
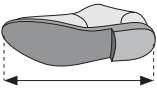
50S8 Malleo Tristep

DE Gebrauchsanweisung	5
EN Instructions for use	11
FR Instructions d'utilisation	18
IT Istruzioni per l'uso	25
ES Instrucciones de uso	31
PT Manual de utilização	38
NL Gebruiksaanwijzing	45
SV Bruksanvisning	51
DA Brugsanvisning	57
NO Bruksanvisning	64
FI Käyttöohje	70
PL Instrukcja użytkowania	76
HU Használati utasítás	83
CS Návod k použití	89
RO Instrucțiuni de utilizare	95
SK Návod na používanie	102
TR Kullanma talimatı	108
EL Οδηγίες χρήσης	114
RU Руководство по применению	121
ZH 使用说明书	129

	Artikelnummer/ Article Number	Größe/ Size	Schuhgröße/ Shoe Size
	50S8	=S	36-39
	50S8	=M	40-44
	50S8	=L	45-48

Material	PU, PA, PES, PVC
-----------------	------------------

INFORMATION

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INFORMATION

Date of last update: 2020-05-20

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.

- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the 50S8 Malleo Tristep immobilising orthosis.

2 Product description

2.1 Product overview

The product is delivered ready for trial fitting in phase 1 (see fig. 1).

Item	Component
1	Foot shell
2	Stabilisation strap
3	Orthosis
4	Padding set

2.2 Design

The individually adjustable hook and loop closures are arranged to close in opposite directions to prevent the orthosis from twisting.

3 Intended use

3.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the lower limbs and **exclusively** for contact with intact skin.

The orthosis must be used in accordance with the indications.

3.2 Indications

- Acute ankle injuries
- Acute and chronic capsular ligament instability
- Inflammatory joint diseases
- Degenerative ankle diseases
- Rehabilitation after ankle surgery
- Medial ligament ruptures

Indications must be determined by the physician.

3.3 Contraindications

3.3.1 Absolute Contraindications

- The orthosis is not suitable for use as a dorsiflexion-assist orthosis for paralysis of the peroneal nerve.

3.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb; pronounced varicose veins, especially with return flow impairment, lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the support will be applied; sensory and circulatory disorders in the foot area, e.g. in case of diabetic neuropathy.

3.4 Effects

The orthosis can be used in 3 phases. The orthosis can be worn in all 3 phases in a flat shoe that can be opened wide.

Phase I – Inflammatory Phase:

The ankle joint is immobilised and stabilised in the neutral position (90°) using the orthosis with the foot shell (see fig. 2, item 1) and the stabilisation strap (see fig. 2, item 2).

Phase II – Proliferation Phase:

The foot shell can be removed during the day. The orthosis is worn with the stabilisation strap to achieve stability (see fig. 3). At night, the foot shell is adapted to immobilise and stabilise the ankle joint. Removing the foot shell during the day enables targeted and secure mobilisation and loading of the ankle joint.

Phase III – Remodeling Phase:

Towards the end of the healing process the stabilisation strap is removed. The orthosis can be used during sports or other activities with a high rerupture rate to provide support against new injuries and as secondary prevention (see fig. 4). The stabilisation strap can be attached optionally.

4 Safety

4.1 Explanation of warning symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

4.2 General safety instructions

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

CAUTION

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

NOTICE

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

NOTICE

Exposure of the product to unsuitable environmental conditions

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C or above +50 °C (e.g. sauna, excessive sunlight, drying on a radiator).

5 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.

- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

5.1 Size Selection

- ▶ Select the orthosis size based on the shoe size (see sizing table).

5.2 Fitting and Application

CAUTION

Improper moulding or application

Injuries or damage to the orthosis due to overloading of the material and improper fit of the orthosis due to breakage of load-bearing components

- ▶ Do not make any improper changes to the orthosis.
- ▶ Always apply the orthosis according to the information in the instructions.

CAUTION

Direct skin contact with the product

Skin irritation due to chafing or sweat formation

- ▶ Do not wear this product directly on the skin.

CAUTION

Incorrect or excessively tight application

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

NOTICE

Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

Phase I – Inflammatory Phase:

- > All hook and loop closures on the orthosis are unfastened.
- > The patient is in a lying position.
- 1) Position the foot at a 90° angle in the orthosis (see fig. 5).
- 2) Fasten the centre hook and loop closure (marking I) of the orthosis (see fig. 6).
- 3) Fasten the lower hook and loop closure (marking II) of the orthosis.
- 4) Fasten the upper hook and loop closure (marking III) of the orthosis.
- 5) **Optional:** Shorten or move the hook and loop closures.
- 6) With the patient sitting, place the foot at a 90° angle sole-down on the floor.
- 7) Unfasten the medial and lateral straps of the foot shell and tighten them so that the foot is in the neutral position (90°) (see fig. 7).

INFORMATION: Wearing a sock under the orthosis and using the padding provided (see fig. 1, item 4) prevent skin irritation from the foot shell for patients sensitive to pressure.

- 8) Guide the lateral (outer) end of the stabilisation strap over the instep towards the medial side (inwards) (see fig. 8).
- 9) Pull the stabilisation strap laterally (outwards) underneath the foot shell (see fig. 9).
- 10) Fasten the stabilisation strap on the lateral (outer) side at the upper end of the foot shell (see fig. 10).
- 11) Guide the medial (inner) end of the stabilisation strap over the instep towards the lateral side (outwards) (see fig. 11).
- 12) Pull the stabilisation strap medially (inwards) underneath the foot shell (see fig. 12).
- 13) Fasten the stabilisation strap on the medial (inner) side at the upper end of the foot shell (see fig. 13).
- 14) **Optional:** Shorten the stabilisation strap.

Phase II – Proliferation Phase:

- 1) Unfasten the stabilisation strap.
- 2) Open and remove the orthosis.
- 3) Remove the foot shell from the orthosis (see fig. 14).
- 4) Put on the orthosis and fasten the hook and loop closures.
- 5) Refasten the stabilisation strap as described in phase 1.

Phase III – Remodeling Phase:

- 1) Unfasten and remove the stabilisation strap (see fig. 15).
- 2) The orthosis is worn without the stabilisation strap and foot shell in this phase.
- 3) **Optional:** Refasten the stabilisation strap as described in phase 1.

5.3 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Textile component:

- 1) Remove the foot shell from the orthosis.
- 2) Close all hook and loop closures.
- 3) Wash the orthosis in warm water at 30 °C with standard mild detergent. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

Foot shell:

- 1) If necessary, wipe down the foot shell excluding the hook and loop closures with a moist, lint-free cloth and neutral soap.
- 2) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

6 Disposal

Dispose of the product in accordance with national regulations.

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

7.2 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the

information in this document, particularly due to improper use or unauthorised modification of the product.

7.3 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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