

50R49 Dorso Carezza Posture

DE	Gebrauchsanweisung	3
EN	Instructions for use	7
FR	Instructions d'utilisation	11
IT	Istruzioni per l'uso	15
ES	Instrucciones de uso	19
PT	Manual de utilização	23
NL	Gebruiksaanwijzing	28
SV	Bruksanvisning	32
DA	Brugsanvisning	36
NO	Bruksanvisning	40
FI	Käyttöohje	43
IS	Notkunarleiðbeiningar	47
PL	Instrukcja użytkowania	51
HU	Használati utasítás	55
CS	Návod k použití	59
SL	Navodila za uporabo	63
SK	Návod na používanie	67
TR	Kullanma talimatı	71
ET	Kasutusjuhhis	75
RU	Руководство по применению	78
JA	取扱説明書	83
ZH	使用说明书	87

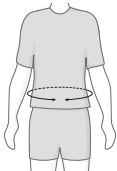	Größe / Size	Taillenumfang* / Waist circumference	
		cm	inch
	XXS	60 – 70	23.6 – 27.6
	XS	70 – 80	27.6 – 31.5
	S	80 – 90	31.5 – 35.4
	M	90 – 100	35.4 – 39.4

* ca. 2 cm oberhalb des Beckenkamms / approx. 2 cm/0.8 inch above the iliac crest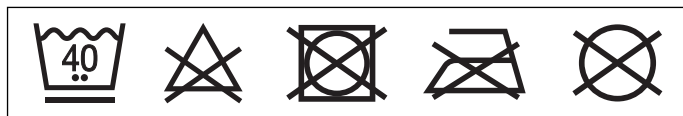

Material	Gewebe/Fabric: PA, EL, Lycra®
	Korsettstäbe/Corset Splints: POM

INFORMATION

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INFORMATION

Date of last update: 2020-05-25

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the 50R49 Dorso Carezza Posture thoracic spine brace.

2 Intended use

2.1 Indications for use

The brace is intended **exclusively** for orthotic fittings of the back and **exclusively** for contact with intact skin.

The brace must be used in accordance with the indications.

2.2 Indications

- Poor body posture (hunchback without fused vertebrae, Scheuermann's disease)
- Dorsalgia

The indication must be determined by the physician.

2.3 Contraindications

2.3.1 Absolute Contraindications

None known.

2.3.2 Relative Contraindications

The following conditions require consultation with a physician: skin diseases/injuries, inflammation, raised scars with swelling, reddening, and

warming in the area where the support is fitted, tumors, lymphatic congestion – including unclear soft tissue swelling distal to the support, and dysesthesia in the trunk.

2.4 Mechanism of Action

The brace supports muscular stabilization of the thoracic spine and straightens it actively.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

CAUTION

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

NOTICE

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

4 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

4.1 Selecting the Size

- 1) Measure the waist circumference approx. **2 cm** above the iliac crest.
- 2) Determine the brace size (see size chart).

4.2 Fitting and Application

⚠ CAUTION

Direct skin contact with the product

Skin irritation due to chafing or sweat formation

- ▶ Do not wear this product directly on the skin.

⚠ CAUTION

Incorrect or excessively tight application

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

NOTICE

Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

- 1) **Application** (see fig. 1): place the brace over the shoulders like a backpack.

- 2) **Hook-and-loop material:** route the ends of the lower strap to the front (see fig. 2) and secure the hook-and-loop closure (see fig. 3). Feed the ends of the shoulder strap to the front around the chest and secure the hook-and-loop closure below the chest (see fig. 4).
- 3) **Final check:** the correct fit and proper positioning of the brace must be checked during handover to the patient.
- 4) **Instruct the patient:** explain to the patient how to don the brace and ensure that it is seated correctly. Practice donning and doffing the brace with the patient.

4.3 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- Only clean the product with the approved cleaning agents.

Clean the brace regularly:

- 1) Close all hook-and-loop closures.
- 2) Recommendation: Use a laundry bag or net.
- 3) Wash the brace in warm water at **40 °C** with standard mild detergent. Do not use fabric softener. Rinse well.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

5 Disposal

Dispose of the product in accordance with national regulations.

6 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

6.1 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

6.2 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorized modification of the product.

6.3 CE Conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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