## Eco Buggy

<b>DE</b> Gebrauchsanweisung .....	3
<b>EN</b> Instructions for use .....	35
<b>FR</b> Instructions d'utilisation .....	65
<b>IT</b> Istruzioni per l'uso .....	97
<b>PT</b> Manual de utilização .....	127
<b>NL</b> Gebruiksaanwijzing .....	159
<b>PL</b> Instrukcja użytkowania .....	191
<b>HU</b> Használati utasítás .....	223
<b>CS</b> Návod k použití .....	253
<b>HR</b> Upute za uporabu .....	283
<b>SK</b> Návod na používanie .....	313
<b>RU</b> Руководство по применению .....	343

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# Table of contents

EN

<b>1</b>	<b>Foreword</b> .....	<b>37</b>
<b>2</b>	<b>Product description</b> .....	<b>37</b>
2.1	Function .....	37
2.2	Product overview .....	38
<b>3</b>	<b>Intended use</b> .....	<b>38</b>
3.1	Indications for use .....	38
3.2	Indications .....	39
3.3	Contraindications .....	39
3.3.1	Absolute Contraindications .....	39
3.3.2	Relative Contraindications .....	39
3.4	Qualification .....	39
<b>4</b>	<b>Safety</b> .....	<b>39</b>
4.1	Explanation of warning symbols .....	39
4.2	Safety instructions for assembly and adjustments .....	39
4.3	Safety instructions for use .....	39
4.4	Side effects .....	43
4.5	Further information .....	43
4.6	Nameplate .....	43
<b>5</b>	<b>Delivery</b> .....	<b>44</b>
5.1	Scope of delivery .....	44
5.2	Options .....	44
5.3	Storage .....	44
5.3.1	Storage during daily use .....	44
5.3.2	Storage during extended disuse .....	44
<b>6</b>	<b>Preparing the product for use</b> .....	<b>44</b>
6.1	Unfolding .....	44
<b>7</b>	<b>Delivery</b> .....	<b>46</b>
7.1	Transport to the customer .....	46
7.2	Handing over the product .....	46
<b>8</b>	<b>Use</b> .....	<b>46</b>
8.1	Instructions for use .....	46
8.2	Basic functions .....	46
8.2.1	Adjusting the back support angle .....	46
8.2.2	Adjusting the seat depth .....	47
8.2.3	Removing and fastening the calf strap .....	48
8.2.4	Adjusting the leg support .....	48
8.2.5	Using the swivel lock .....	48
8.2.6	Operating the wheel lock .....	49
8.2.7	Installing and removing wheels .....	49
8.3	Positioning belts .....	50
8.3.1	Safety instructions for qualified personnel .....	50
8.3.2	Safety instructions for the user .....	50
8.3.3	Lap belt .....	50
8.3.4	Five-point belt .....	51
8.4	Additional options .....	54
8.4.1	Storage bag .....	54
8.4.2	Canopy .....	55
8.4.3	Buggy board .....	56
8.4.4	Padding .....	57
8.5	Disassembly and transport .....	59
8.6	Use in vehicles for transporting persons with reduced mobility .....	60
8.7	Care .....	60
8.7.1	Cleaning .....	60
8.7.1.1	Cleaning by hand .....	60

8.7.1.2	Cleaning belts/straps .....	60
8.7.2	Disinfection .....	60
<b>9</b>	<b>Maintenance and repair .....</b>	<b>61</b>
9.1	Maintenance.....	61
9.1.1	Maintenance intervals .....	61
9.1.2	Maintenance tasks .....	61
9.2	Repair .....	62
<b>10</b>	<b>Disposal .....</b>	<b>62</b>
10.1	Disposal information .....	62
10.2	Information on re-use.....	62
<b>11</b>	<b>Legal information .....</b>	<b>62</b>
11.1	Liability .....	62
11.2	Warranty .....	62
11.3	Service life .....	62
<b>12</b>	<b>Technical data .....</b>	<b>62</b>
<b>13</b>	<b>Appendices .....</b>	<b>63</b>
13.1	Threshold values for rehab buggies transportable by train .....	63

# 1 Foreword

## INFORMATION

Date of last update: 2020-04-01

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

## INFORMATION

- ▶ New information regarding product safety and product recalls as well as the declaration of conformity can be obtained at [oa@ottobock.com](mailto:oa@ottobock.com) or from the manufacturer's service department (see inside or outside of back cover for addresses).
- ▶ You can request this document as a PDF file at [oa@ottobock.com](mailto:oa@ottobock.com) or from the manufacturer's service department (see inside or outside of back cover for addresses). The PDF file can also be displayed in a larger size.

You have received a product that is very versatile for everyday use at home and outdoors.

In order to exclude injuries of any type, familiarise yourself with the handling, functions and intended use of the product before using it. These instructions for use provide you with the related necessary information.

### **Please note the following in particular:**

- All users and/or their attendants must be trained by qualified personnel in the use of the product. In particular, users and/or attendants must be informed of the residual risks with the aid of the safety notices in the instructions for use (user).
- The product was adapted to the needs of the user. Subsequent changes may be made only by qualified personnel. We recommend checking the product settings **once per year** to ensure optimal treatment over the long term. Especially for users with a changing anatomy (for example body dimensions, weight), an adjustment at least **once every six months** is recommended.
- Your product may differ from the models shown. In particular, not all the options described in these instructions for use will be installed on your product.
- The manufacturer reserves the right to make technical changes to the model described in these instructions for use.

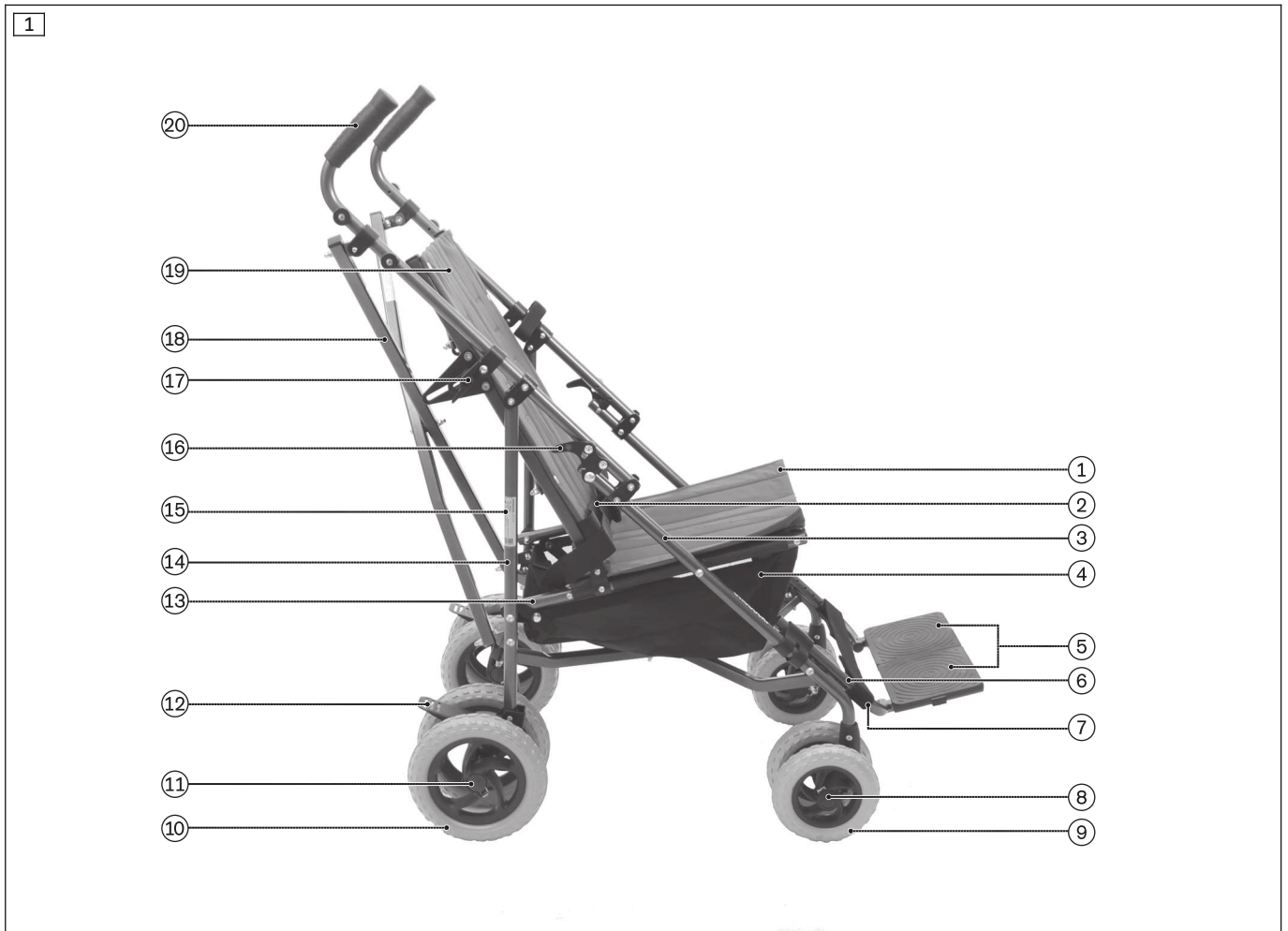
## 2 Product description

### 2.1 Function

The product is intended exclusively for transporting one child on the seat.

The product can be used on solid ground both indoors and outdoors.

## 2.2 Product overview

- |    |                                      |    |   |
|----|--------------------------------------|----|---|
| 1  | Seat                                 | 11 | Rear axle                                     |
| 2  | Lap belt                             | 12 | Wheel lock                                    |
| 3  | Front frame                          | 13 | Seat frame                                    |
| 4  | Storage bag                          | 14 | Rear frame                                    |
| 5  | Segmented footplate                  | 15 | Nameplate                                     |
| 6  | Leg support                          | 16 | Folding guard                                 |
| 7  | Calf band                            | 17 | Back support angle adjustment                 |
| 8  | Front axle                           | 18 | Folding scissor construction for back support |
| 9  | Front wheel (integrated swivel lock) | 19 | Back support                                  |
| 10 | Rear wheel                           | 20 | Push handle                                   |

## 3 Intended use

The safe use of the product can only be ensured in case of intended use in accordance with the information contained in these instructions for use. The user is ultimately responsible for accident-free operation.

### 3.1 Indications for use

The product is intended for everyday indoor and outdoor use, by an attendant, of children with temporary or permanent limitations of the ability to walk, inability to walk or difficulty standing up.

The product is suitable for users with intact skin whose anatomy (such as body dimensions and weight) permits the intended use of the product. It is operated by an attendant.

The product may only be used with the options that are listed on the product order form. Ottobock assumes no liability for combinations with medical devices and/or accessories from other manufacturers outside the modular system.

Combinations based on a combination agreement that have been evaluated for effectiveness and safety are an exception to this.

### 3.2 Indications

- Minor to pronounced or complete restrictions of mobility

### 3.3 Contraindications

#### 3.3.1 Absolute Contraindications

- None known

#### 3.3.2 Relative Contraindications

- Failure to meet physical or mental requirements

### 3.4 Qualification

Installation and adjustment tasks may only be carried out by qualified personnel. Compliance with all manufacturer specifications and all applicable legal provisions is required. Please contact the manufacturer's service department for further information (see inside or outside of rear cover for addresses).

## 4 Safety

### 4.1 Explanation of warning symbols

 <b>WARNING</b>	Warning regarding possible serious risks of accident or injury.
 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

### 4.2 Safety instructions for assembly and adjustments

<p> <b>WARNING</b></p> <p><b>Making incorrect adjustments</b></p> <p>Risk of falling, tipping over or malposition of the user due to incorrect settings</p> <ul style="list-style-type: none"> <li>▶ Settings may only be changed by qualified personnel unless otherwise prescribed.</li> <li>▶ Before testing setting changes with the user seated, all screw connections must be firmly tightened.</li> <li>▶ Check the product for safety and functionality after changing the settings.</li> </ul>
--

<p> <b>CAUTION</b></p> <p><b>Unsecured screw connections</b></p> <p>Pinching, crushing, tipping over, falling of user due to assembly errors</p> <ul style="list-style-type: none"> <li>▶ After all adjusting/readjusting work, retighten the attachment screws/nuts firmly.</li> <li>▶ Please note that all adjustment levers are tightened by hand without the help of tools.</li> </ul>
---

### 4.3 Safety instructions for use

#### Hazards during preparation for use

<p> <b>WARNING</b></p> <p><b>Independent modification of settings</b></p> <p>Serious injuries to the user due to improper changes to the product</p> <ul style="list-style-type: none"> <li>▶ Do not modify the settings established by the qualified personnel. Only the settings described in the section "Use" in these instructions for use may be adjusted independently.</li> <li>▶ In case of problems with the settings, please contact the qualified personnel who adjusted your product.</li> </ul>
--

**⚠ WARNING**

**Improper handling of packaging materials**

Risk of suffocation due to neglect of the duty to supervise

- ▶ Packaging materials must be kept out of the reach of children.

**Hazards when getting in**

**⚠ CAUTION**

**Wheel lock not engaged when getting in or out**

Falling, tipping over of the user due to user error

- ▶ Before the user gets in or out, always engage and check the wheel lock.
- ▶ Engage the wheel lock to prevent the rehab buggy from moving on uneven ground or during transfers (e.g. into a car).

**⚠ CAUTION**

**Improper use of leg support while getting in**

Tipping over, falling due to user error

- ▶ The leg support must not be used for getting in or out.
- ▶ In necessary, fold up the footplates before getting in.

**Hazards while driving**

**⚠ WARNING**

**Pushing too fast**

Tipping over, falling of the user due to caster wheel wobble

- ▶ The swivelling front wheels may begin to wobble at higher speeds and lead to an abrupt stop, causing the rehab buggy to tip over forwards.
- ▶ Therefore, push the rehab buggy only at normal walking speed or engage the swivel lock (see section "Use"). It is not permissible to let go of the push bar while pushing the buggy or to thrust it away from you.

**⚠ WARNING**

**Unsupervised parking**

Falling out, falling of the user due to neglect of the duty to supervise

- ▶ Never leave the user unattended, not even when the positioning belts or safety belts are secured and the wheel locks are activated.

**⚠ WARNING**

**Impermissible use**

Tipping over, rolling over of the user due to failure to observe specifications

- ▶ Using a rehab buggy beyond normal conditions may be dangerous.
- ▶ Please note that this product is not suitable for jogging, running, skating, etc.

**⚠ CAUTION**

**Lack of driving experience**

Tipping over, falling due to errors in handling the product

- ▶ Practice on level, open ground first.
- ▶ Learn with the support of an assistant how the rehab buggy reacts to changes in the centre of gravity, e.g. downward or upward slopes, inclines or when overcoming obstacles.
- ▶ Always activate the anti-tipper (if any).

**⚠ CAUTION****Incorrect centre of gravity**

Tipping over, overturning due to incorrect centre of gravity

- ▶ When reaching for objects, the user must not lean too far out of the seat.
- ▶ Put the back support into neutral position (0–5°) before negotiating slopes and obstacles on slopes or ramps.
- ▶ Always provide support to the rear when negotiating slopes and obstacles on slopes or ramps.
- ▶ Hold the product tightly by the push handles with both hands when driving on inclines and downgrades. Otherwise, the rehab buggy could tip over sideways when driving diagonally on a slope or roll away when driving on downgrades.
- ▶ Never park the product on slopes with the user in the seat.

**⚠ CAUTION****Improper loading**

Tipping over, falling of the user due to shift in centre of gravity

- ▶ Please note that any additional load may have a negative impact on the stability of the rehab buggy.
- ▶ Do not hang heavy bags or the like on the push bar/push handles.

**⚠ CAUTION****Incorrect centre of gravity setting**

Falling of the user due to extreme settings

- ▶ Check the standard adjustments of the rehab buggy for stability against tipping and the ergonomically correct sitting position of the user. Avoid any extreme settings.

**⚠ CAUTION****Risky operation**

Falling, tipping over backwards due to approaching obstacles incorrectly

- ▶ Push slowly when crossing obstacles (e.g. steps, curbs) and negotiating uphill or downhill slopes and inclines.
- ▶ Never cross obstacles at an angle. Always approach obstacles head on (at an angle of 90°).
- ▶ Raise the front wheels before crossing obstacles.
- ▶ Avoid collisions with obstacles and dropping off curbs/ledges.
- ▶ Avoid riding cross-country.

**Hazards when overcoming obstacles****⚠ WARNING****Overcoming steps and obstacles without assistance**

Tipping over, falling of the user due to failure to observe transportation instructions

- ▶ Always have accompanying persons help you negotiate steps and other obstacles.
- ▶ Use available facilities (e.g. access ramps or lifts).

**⚠ CAUTION****Improper lifting by attendants**

Tipping over, falling of the user due to lifting by removable components

- ▶ Attendants may lift the buggy only by welded or permanently attached components (e.g. front frame tube above the front wheels, rear axle).
- ▶ It is particularly dangerous to lift the buggy by the unfixed push handle/push bar.

### Hazard in case of broken skin

**⚠ CAUTION**

**Skin damage**

Skin damage or pressure points due to overloading

- ▶ Check your skin for intactness before and during use of the product.
- ▶ Pay attention to diligent skin care and pressure redistribution during interruptions in using the product.
- ▶ If skin damage or other problems occur during use, stop using the product. Consult the qualified personnel.

### Hazards due to fire, heat and cold

**⚠ WARNING**

**Seat cushions and back support pads may ignite**

Burns due to user error

- ▶ The seat and back support upholstery as well as seat cushions, padding and covers fulfil the normative requirements for flame resistance. However, they may still ignite if fire is handled improperly or negligently.
- ▶ Keep away from all ignition sources, especially lit cigarettes.

**⚠ CAUTION**

**Extreme temperatures**

Hypothermia or burns due to contact with components, failure of components

- ▶ Do not expose the product to any extreme temperatures (e.g. direct sunlight, sauna, extreme cold).
- ▶ Do not leave the product in the immediate vicinity of heaters.

### Hazards due to improper use of the product

**⚠ WARNING**

**Overloading**

Severe injuries if the product tips over due to overloading, damage to the product

- ▶ Do not exceed the maximum load capacity (see the nameplate and section "Technical data").
- ▶ Please note that certain accessories and add-on components will reduce the remaining load capacity.

**⚠ WARNING**

**Loss of stability due to overloading**

Falling, tipping over of the user due to changed centre of gravity

- ▶ Please note that any additional load may have a negative impact on the stability of the rehab buggy.
- ▶ Never hang heavy bags or the like on the push bar.

**⚠ WARNING**

**Exceeding the service life**

Serious injuries due to failure to observe the manufacturer's requirements

- ▶ Using the product beyond the specified expected service life leads to increased residual risk.
- ▶ Observe the specified service life.

**⚠ CAUTION**

**Use of the product during diagnostic examinations and therapeutic treatment**

Impairment of the examination results or the effectiveness of treatment due to interactions of the product with devices that are used

- ▶ Make sure that examinations and treatments are carried out exclusively under the prescribed conditions.

**CAUTION**

**Uncontrolled driving behaviour, unexpected sounds or odours**

Falling, tipping, collision with persons or nearby objects due to defects

- ▶ If any faults, defects or other hazards that can lead to personal injury are detected, the product must be taken out of service immediately. This includes uncontrolled movements as well as sounds that are unexpected or previously not noted or odours that deviate significantly from the state of the product at the time of delivery.
- ▶ Contact the qualified personnel.

**CAUTION**

**Lack of maintenance**

User falling or tipping over due to lack of maintenance

- ▶ Check that the product is working correctly before each use.
- ▶ Observe the specified maintenance intervals (see Page 61, "Maintenance" section).

**NOTICE**

**Use under incorrect environmental conditions**

Damage to product due to corrosion or abrasion

- ▶ Do not use the product in salt water.
- ▶ Make sure that the wheel bearings are not damaged by sand or other particles.

**4.4 Side effects**

The following side effects may occur during use of the product:

- Neck, muscle and joint pain
- Circulatory disorders, risk of pressure sores

Contact a doctor or therapist in case of problems.

**4.5 Further information**

**INFORMATION**

Even in the event of compliance with all applicable guidelines and standards, alarm systems (e.g. in department stores) may respond to your product. Should this happen, remove your product from the area where the alarm was triggered.

**INFORMATION**

Please note that all options and add-on components will reduce the remaining load capacity.

**INFORMATION**

The serial number required for enquiries and orders can be found on the nameplate. For explanations of the nameplate, see the section "Nameplate" (see Page 43).

**4.6 Nameplate**

The nameplates are found on the frame.

Label	Meaning
	<b>A</b> Manufacturer's product name
	<b>B</b> CE marking
	<b>C</b> Maximum user weight (see section "Technical data")
	<b>D</b> Manufacturer information/address
	<b>E</b> Serial number <sup>1)</sup>
	<b>F</b> Manufacturing date <sup>2)</sup>
	<b>G</b> Symbol for medical device
	<b>H</b> <b>WARNING!</b> Read the instructions for use before using the product. Observe important safety-related information (e.g. warnings, precautions).

Label	Meaning
	<b>I</b> The product has not been approved by the manufacturer for use as a seat in vehicles for transporting persons with reduced mobility
	<b>J</b> Manufacturer's reference number for the product variant
	<b>K</b> Serial number (PI) <sup>3),1)</sup>
	<b>L</b> Global Trade Item Number (DI) <sup>4)</sup>

1) YYYY = year of manufacture; WW = week of manufacture; PP = production site; XXXX = sequential production number

2) YYYY = year of manufacture; MM = month of manufacture; DD = day of manufacture

3) UDI-PI to GS1 standard; UDI = Unique Device Identifier, PI = Product Identifier

4) UDI-DI to GS1 standard; UDI = Unique Device Identifier, DI = Device Identifier

## 5 Delivery

### 5.1 Scope of delivery

- Rehab buggy, folded with included leg support and calf band
- Options according to the order
- Instructions for use and required tools
- Instructions for use for accessories (depending on equipment)

### 5.2 Options

The standard model can be fitted to the user's personal requirements thanks to a large range of options. For use of these options: see Page 46 ff.

### 5.3 Storage

#### 5.3.1 Storage during daily use

The product should always be stored so it is protected against external influences.

#### 5.3.2 Storage during extended disuse

The product must be stored in a dry place. Specific information for extended storage: see Page 62.

It is not necessary to disassemble the product.

Maintain sufficient clearance from sources of heat. If the product is parked for an extended period of time or the tyres overheat (e.g. in the vicinity of radiators or in case of exposure to strong sunlight behind glass), the tyres may become permanently deformed.

Direct exposure to sunlight or UV radiation causes the tyres to age more quickly. As a result, the tread surface hardens and corner pieces break out of the tread.

The tyres should be replaced every **2 years** regardless of wear and tear.

## 6 Preparing the product for use

### 6.1 Unfolding

#### CAUTION

#### Holding on to the product incorrectly while unfolding and folding

Pinching, crushing due to lack of caution in danger areas

- ▶ Hold the product only by the described parts.
- ▶ Never reach into the folding scissor mechanism.
- ▶ Avoid pinching yourself on movable components.

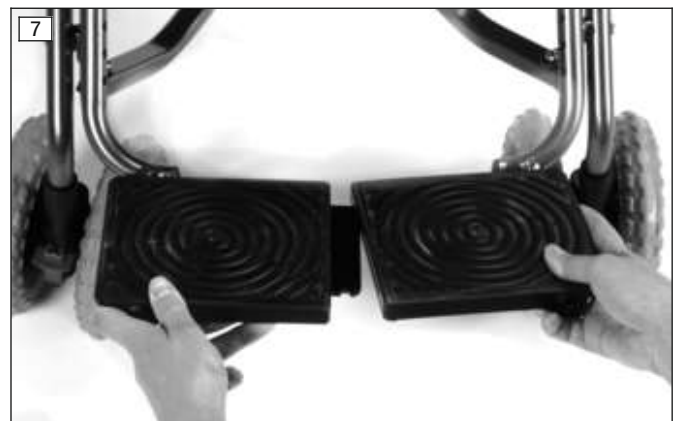
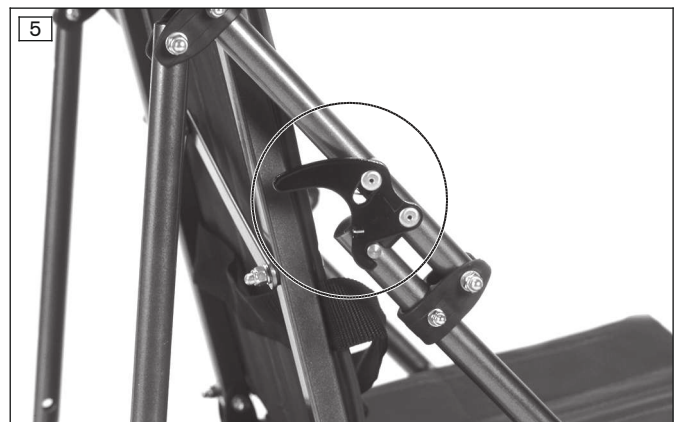
#### INFORMATION

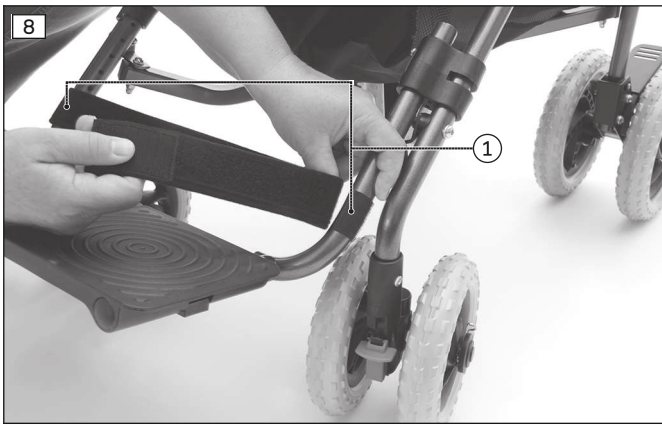
On the topic of disassembly/transport: see Page 59.

1) Remove the transport locks and packaging material.

2) Grasp the rehab buggy by the push handles (see fig. 2) and unfold it to the rear as far as possible (see fig. 3).

- 3) Allow the folding guards on both sides to engage. If necessary, grasp the rehab buggy by the push handle and front frame, and apply slight pressure to push it towards the floor until the folding guards engage (see fig. 4).  
**CAUTION! Make sure that both folding guards are engaged (see fig. 5).**
- 4) Press the folding lever for unfolding the scissor mechanism in to the stop (see fig. 6).
- 5) Swing the leg support brackets in from the outside and fold down the footplates.
- 6) Insert the retaining bracket into the slotted hole and then guide the footplates together sideways until you hear the locking pin engage (see fig. 7).  
**CAUTION! Before using the rehab buggy, make sure that the footplates of the leg support are firmly engaged and locked in place.**
- 7) Attach the self-adhesive hook-and-loop straps (included in the scope of delivery) at the same height on the lower section of the two leg support brackets (see fig. 8, item 1).  
**INFORMATION: Make sure the bores on the two leg support brackets are not covered when attaching the hook-and-loop straps.**
- 8) Attach the calf band (see Page 48).

## 7 Delivery

### 7.1 Transport to the customer

#### NOTICE

##### Use of unsuitable packaging

Damage to the product caused by transportation using incorrect packaging

- ▶ Use only the original packaging for delivery of the product.

### 7.2 Handing over the product

#### ⚠ WARNING

##### Lack of instruction

Tipping over, falling of the user due to lack of knowledge

- ▶ Instruct the user or the attendant in the proper use of the product when handing it over.

The following steps must be performed for the safe delivery of the product:

- A sitting test has to be conducted with the user of the product. Pay special attention to proper positioning according to medical considerations.
- The user and any attendants must be instructed in the safe use of the product. In particular, the enclosed instructions for use are to be used.
- The instructions for use must be given to the user/attendant upon delivery of the wheelchair.
- **Depending on equipment:** The supplied instructions for use for accessories have to be handed over in addition.

## 8 Use

### 8.1 Instructions for use

- Attaching loads (e.g. backpacks) can adversely affect stability. Therefore, suspending additional loads on the product is not permitted.
- The products in this series generally meet the minimum technical requirements for wheelchairs transportable by train. However, please note that it is possible that not every individual product will fulfil the minimum requirements due to the variation in designs (see Page 63).

### 8.2 Basic functions

#### 8.2.1 Adjusting the back support angle

#### ⚠ CAUTION

##### Change in centre of gravity following back angle adjustment

Falling over, falling out of the user due to adjustment errors

- ▶ After each modification of the back angle, check the tilt stability. To do so, hold the product with both hands by the push handle/push bar.

- 1) Loosen the wing nuts on both sides of the frame (see fig. 9).
- 2) Adjust the continuously variable back inclination.
- 3) Firmly retighten the wing nuts.

**CAUTION! Pull on the back support to check for tightness.**

### 8.2.2 Adjusting the seat depth

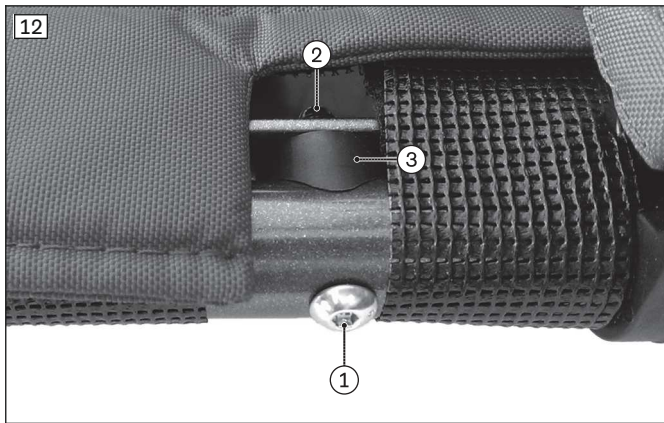
The seat depth is adjusted by changing the position of the clamp brackets on the seat tubes. Two positions are available:

- To select the longer seat depth, choose the rear position for the clamp bracket (see fig. 10, item 2).
- To select the shorter seat depth, choose the front position for the clamp bracket (see fig. 11, item 1).

The shorter seat depth is set at the factory. The process for changing from the long to the short seat depth is described below.

- 1) Loosen and remove the screw connection between the back tubes and the clamp brackets (see fig. 10, item 1). Make sure that the inserted bushings do not slide out of the back tube.
- 2) Loosen and remove the screw connection between the clamp brackets and the seat tubes at the rear position (see fig. 10, item 2).
- 3) Bend open the clamp brackets far enough so they can be pushed off the seat tubes.
- 4) Loosen and remove the screw connection at the front position between the clamp brackets and the saddle spacers (see fig. 12, item 1/2). Remove the saddle spacer between the round tube and long plate (see fig. 12, item 3).
- 5) Move the clamp brackets to the front position that is now free, and slide them back onto the seat tubes (see fig. 11, item 1).
- 6) Mount the clamp brackets on the seat tubes (see fig. 10, item 2). Retighten the screws.
- 7) Mount the back tubes on the clamp brackets (see fig. 10, item 1). In doing so, ensure that the bushings are inserted into the bores in the back tube. Retighten the screws.
- 8) Reinstall the saddle spacers:
  - Insert the screw at the rear position (see fig. 12, item 1).
  - Push the saddle spacer between the round tube and long plate from the inside (see fig. 12, item 3) and firmly screw it in place with the nut behind the long plate (see fig. 12, item 2).

### 8.2.3 Removing and fastening the calf strap

The calf strap offers additional support for the user's legs. It also prevents the feet from sliding off the foot plate into the danger area. The calf strap can be removed for cleaning.

#### Removing the calf band

- 1) Open all hook-and-loop closures on the calf band (see fig. 13, item 1).
- 2) Remove the calf band from both leg support brackets.

#### Fastening the calf band

- 1) Open all hook-and-loop closures on the calf band.
- 2) Route the calf band around both leg support brackets and fasten the hook-and-loop (see fig. 14).
- 3) Adjust the length and fasten the hook-and-loop closure.

### 8.2.4 Adjusting the leg support

- 1) Use a Phillips screwdriver (not included in the scope of delivery) to loosen the screws on the clamp brackets on both of the front frames (see fig. 15, item 1).
- 2) Slide the leg support brackets up or down in the clamp brackets according to the bores, by **20 mm** respectively.
- 3) Insert the screws on the clamp brackets and into the bores and firmly tighten them (see fig. 15, item 1).
- 4) Check that the leg support is securely mounted.

### 8.2.5 Using the swivel lock

The rehab buggy can be equipped with swivelling front wheels to make steering easier. If needed, the caster wheels can be locked with the integrated swivel lock. This function increases directional stability on uneven ground.

- 1) **Swivel lock activation:** Hold the front wheels straight and push the red locking slides up (see fig. 15, item 2).
- 2) **Swivel lock deactivation:** Push the red locking slides down.

## 8.2.6 Operating the wheel lock

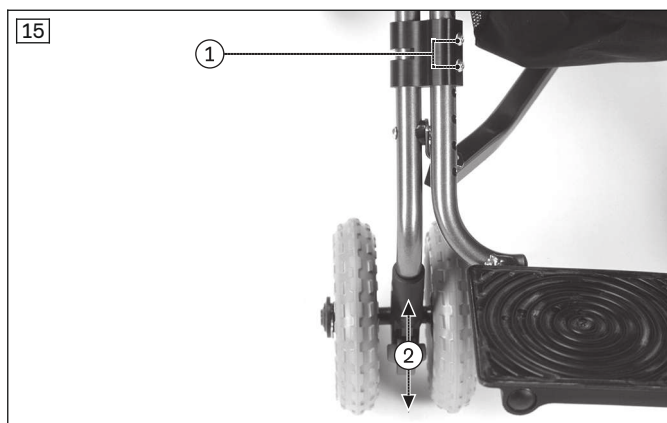
### **⚠ WARNING**

#### **Insufficient braking**

Accident, falling of the user due to loss of braking function

► Verify the functionality of the wheel lock prior to every use.

- 1) **Brake engagement:** Press the trip levers on the right and left down with your foot (see fig. 16).
- 2) **Brake release:** Push the trip levers on the right and left up with your foot.

## 8.2.7 Installing and removing wheels

### **⚠ WARNING**

#### **Improper assembly of removable wheels**

Tipping, falling over of the user due to wheels coming off

► After each assembly, verify the proper fit of the removable wheels. The quick-release axles must be firmly locked in the wheel attachment.

#### **Installation**

- 1) Slide the wheel onto the axle. In doing so, ensure that the locking spring engages audibly (see fig. 17).
- 2) If you do not hear the locking spring engage, manually pull it back into the locking position.

#### **Removal**

- 1) Disengage the locking spring on the wheel in the direction of the axle (see fig. 18).
- 2) Pull the wheel off the axle.

## 8.3 Positioning belts

### 8.3.1 Safety instructions for qualified personnel

#### CAUTION

##### **Incorrect approach to the adjustment process**

Injuries, malpositions, user discomfort due to adjustment errors

- ▶ The belt system may only be installed by qualified personnel.
- ▶ The qualified personnel is responsible for the individual positioning and fitting of the belt system.
- ▶ Adjusting the belt system too tightly may lead to unnecessary pain or user discomfort.
- ▶ Adjusting the belt system too loosely can cause the user to slide into a dangerous position. In addition, the fastening snaps could open unintentionally if they slide against hard parts of clothing (e.g. buttons).
- ▶ For proper installation and adjustment, follow the instructions for use included with the product.

#### CAUTION

##### **Lack of instruction**

Injuries, malpositions, illness of the user due to incorrect information

- ▶ The qualified personnel is responsible for making sure that the user and/or attendant/nursing staff has understood the proper adjustment, use, maintenance and care of the belt system.
- ▶ In particular, ensure that the user and/or attendant/care staff knows how to quickly loosen and open the product to avoid delays in case of emergency.

### 8.3.2 Safety instructions for the user

#### WARNING

##### **Incorrect application of the belt system/positioning aid**

Strangulation, suffocation due to incorrectly placed positioning belts/positioning aids

- ▶ Follow the instructions for proper application so that the user cannot slip into a position that would endanger him or her.
- ▶ Note that the user's back must be up against the back support pad when securing the belt system/positioning aid and during use of the product.
- ▶ Do not use positioning belts or positioning aids if the user is able to sit without them.

#### CAUTION

##### **Improper adjustments**

Injuries, malpositions, illness of the user due to adjustment changes

- ▶ The belt system is an important part of an individual seating unit/seating solution. Do not modify the installation position and basic settings established by the qualified personnel.
- ▶ In case of problems with these adjustments (such as an unsatisfactory sitting position), promptly contact the qualified personnel who fitted the product.
- ▶ **Immediately** consult the qualified personnel if you detect signs of discomfort or fear when using the product.
- ▶ Have the basic settings of the belt system checked regularly. Adjustments may be required due to the growth of the user or because of changes in the course of the disease.

#### CAUTION

##### **Improper use**

Falls, user falling out due to improper use

- ▶ The positioning system for the upper body must be put on when getting into the product and used in combination with a positioning lap belt at all times while using the product.
- ▶ Only open the positioning system for the upper body and the lap belt when the user is ready to get out of the product.
- ▶ Do not leave the user unsupervised if the cognitive abilities of the user could lead to unintentional opening of the positioning system.

### 8.3.3 Lap belt

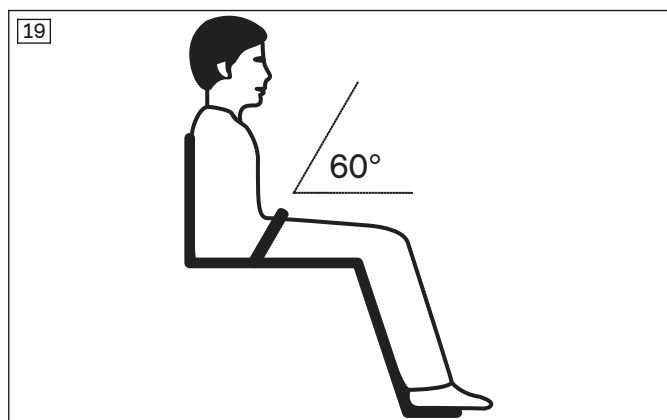
The standard equipment of the rehab buggy includes a simple lap belt. It provides additional support for the pelvis and keeps the user from sliding out of the seat.

### Operating instructions

- Place the user in an upright, **90°** seated position (if physiologically possible).
- Ensure that the back is up against the back support pad (if physiologically possible).
- The lap belt should run in the direction of the thighs in front of the pelvic bones if possible (not illustrated).
- The lap belt should be at an angle of about **60°** to the seat surface. The belt strap should run across the thighs in front of the pelvic bones (see fig. 19).

### Possible errors

- The lap belt is positioned above the user's pelvis in the area of the soft part of the stomach.
- The user does not sit upright in the seat.
- If the lap belt is too loose, the user can shift/slide out to the front.
- During the installation/adjustment, the lap belt is routed over parts of the seating system (e.g. over arm supports or seat pads). This causes the lap belt to lose its positioning and retaining function.

### Adjusting the belt

- ▶ Adjust the belt ends at the belt closure. Do so by loosening the belt ends in the buckle, adjusting the length and pulling to tighten (see fig. 20, see fig. 21).

### Using the belt

- 1) To open the belt, press the buckle from above and below. Pull the buckle out of the closure (see fig. 20).
- 2) Sit the user into the seat.
- 3) Fasten the belt using the buckle so you hear the closure engage (see fig. 21).

### 8.3.4 Five-point belt

The rehab buggy can be equipped with a five-point belt. It additionally stabilises the pelvis and shoulders and prevents the user from sliding out of the seat. The buckle makes it easy to get in and out.

The optional five-point belt consists of shoulder harnesses with shoulder pads, the lap belt piece and the centre strap.

### Operating instructions

- Place the user in an upright, 90° seated position (if physiologically possible).
- Ensure that the back is up against the back support pad (if physiologically possible).

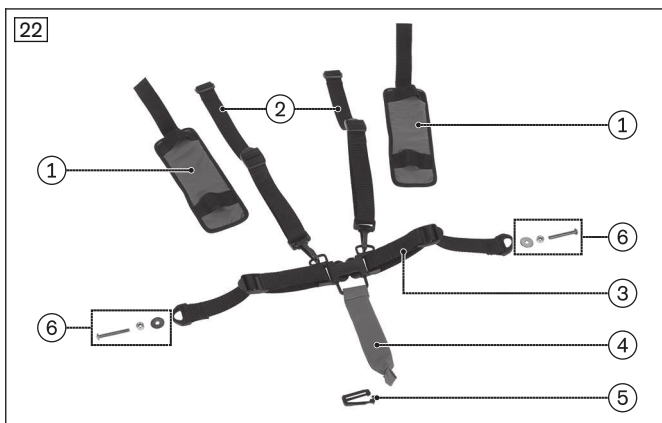
### Possible errors

- The user does not sit upright in the seat.
- If the positioning belt is too loose, the user can shift/slide out to the front.
- During the installation/adjustment, the positioning belt is routed over parts of the seating system (e.g. over arm supports or seat pads). This causes the lap belt to lose its retaining function.

### Installing the belt – checking the scope of delivery (see fig. 22)

The five-point belt is complete if the following parts are on hand:

2x shoulder pads (item 1), 2x shoulder belts (item 2), 1x lap belt section (item 3), 1x centre fixation (item 4), 1x attachment buckle (item 5), 2x M5x40 screws (item 6), 2x spacers (item 6), 2x washers (item 6)

### Removing the lap belt

- 1) Unthread the buckle from the belt (see fig. 23).
- 2) Pull the lap belt out of the back support upholstery.

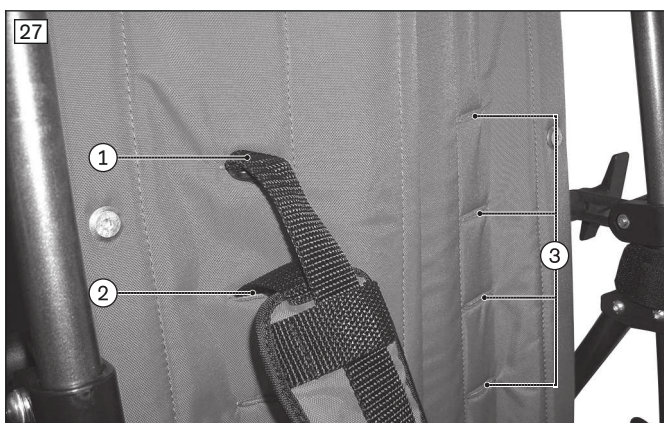
### Attaching the five-point belt in the pelvic region

- 1) Remove the M5x35 oval head screw (see fig. 24, item 1).
- 2) Attach the pelvic brackets of the five-point belt with the included M5x40 screws, washers and spacers.

### Attaching the five-point belt in the shoulder region

- 1) Thread the shoulder pads onto the shoulder belts (see fig. 25).
- 2) Hold the buckle (see fig. 26) and thread it through the back slit (see fig. 27, item 1). Use the upper 3 back slits (see fig. 27, item 3).
- 3) Put the belt flaps of the shoulder pads respectively through the back slit below (see fig. 27, item 2).
- 4) Fasten the shoulder belts on both sides behind the back support (see fig. 28).

### Attaching the five-point belt in the area of the seat

- 1) Thread the centre fixation tongue through the suitable slit in the seat surface from above (see fig. 31).
  - 2) Put the attachment buckle through the tongue and fasten it underneath the seat surface (see fig. 32).
- This completes the installation of the five-point belt (see fig. 33).

### Adjusting the belt

- 1) For subsequent length adjustment, use the four adjustment buckles of the lap belt section and the shoulder belts (see fig. 29).
  - 2) For optimum height adjustment of the shoulder belts, use the slots in the back support upholstery (see fig. 27, item 3). Release the shoulder belt attachment. To adjust the height, thread the belt ends around through two slots in the back support pad that are above each other and fasten the belt attachment again (see fig. 30).
  - 3) To adjust the centre fixation, open and remove the attachment buckle under the seat surface. Thread the centre fixation through the suitable slot and fasten the attachment buckle under the seat surface again (see fig. 31, see fig. 32).
- The five-point belt is adjusted correctly when the closed belt buckle lies against the child's abdomen.

### Using the belt

- 1) To open the five-point belt, press the snap closure of the lap belt section (see fig. 34).
- 2) Set the shoulder belts to the side.
- 3) Sit the user into the seat.
- 4) Place the belts over the shoulders.
- 5) Make sure the two belt tabs click into the buckle.

## 8.4 Additional options

### ⚠ CAUTION

#### Improper installation of options/accessories

Falling of the user due to incorrect installation, damage to option components

- ▶ All option/Accessory components must be installed by the qualified personnel unless they were pre-installed by the manufacturer. A few notes on installation are provided here for the benefit of the qualified personnel.
- ▶ Always install the option/Accessory components such that the stability of the product is maintained.
- ▶ Inform the attendant that installing some of the option components (e.g. storage bag, canopy) may restrict the ability to fold the product. These components should be removed prior to folding if necessary.

### 8.4.1 Storage bag

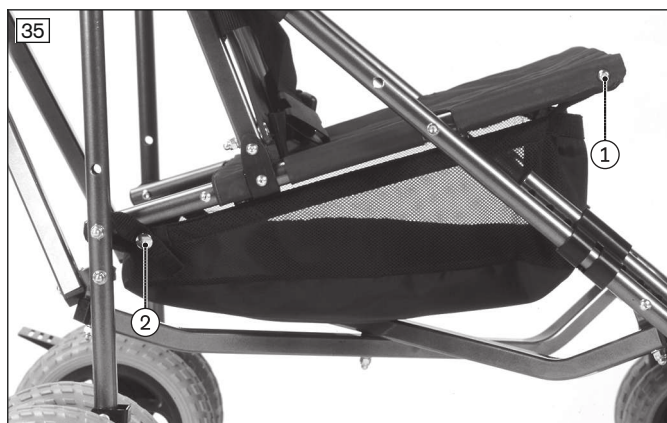
#### NOTICE

#### Incorrect attachment

Damage to product due to incorrect installation

- ▶ During the attachment, make sure that the product cannot grind against the wheels.

- 1) Screw the storage bag to the left and right of the front frame (see fig. 35, item 1).
- 2) Attach the storage bag to the rear frame with the existing snap fasteners (see fig. 35, item 2).

### 8.4.2 Canopy

#### ⚠ CAUTION

##### Incorrect use of the canopy

Falling, tipping of the user due to uncontrolled rolling away

- ▶ An unfolded canopy is a large surface that can be caught by the wind. Close the canopy every time you park the rehab buggy. Never leave the user unattended.

#### NOTICE

##### Improper use of the canopy

Damage to the fabric due to user error

- ▶ Always remove the canopy before folding the rehab buggy so that it does not get folded up as well.
- ▶ Remove the canopy from the rehab buggy if it is stored for a long period to protect it from damage.

The rehab buggy can be equipped with a canopy to protect the user from the sun. A mesh integrated into the canopy permits directly visual contact with the child.

##### Mounting the canopy

- 1) Move the back support to an upright position (see Page 46).
- 2) Pull the folding lever under the seat (see fig. 36) to release the tension from the frame.
- 3) On both sides of the canopy, insert first the upper and then the lower locking lugs of the canopy into the guide slots of the mounting points (see fig. 37).
- 4) Press down the locking lugs until they lock in place (see fig. 38).
- 5) Press the folding lever under the seat down to the stop (see fig. 39).

##### Removing the canopy

- 1) Move the back support to an upright position (see Page 46).
- 2) Pull the folding lever under the seat (see fig. 36).
- 3) Pull the locking lugs of the canopy out of the guide slots of the mounting points with slight pressure.
- 4) Remove the canopy.

### 8.4.3 Buggy board

#### ⚠ CAUTION

##### Improper use of the buggy board

Falling, tipping, injuries due to user error

- ▶ Do not exceed the max. load of the buggy board. This is **20 kg**.
- ▶ Do not use the buggy board as a scooter.

#### NOTICE

##### Damage during folding

Damage to the product due to user error

- ▶ Do not fold the rehab buggy up when the buggy board is installed.

#### INFORMATION

All information regarding the attachment, adjustment and use of the buggy board are contained in the manufacturer's instructions for use enclosed with the product.

The buggy board allows children to ride safely while standing on their own feet, has spring loaded casters and offers sufficient standing room. It is attached to the rear frame and can be flipped up or detached using the quick closures.

#### Mounting instructions

- 1) Mount couplers to the right and left at the position indicated (see fig. 40).  
**NOTICE! Make sure that there is no interference with the braking mechanism, wheels or folding mechanism.**
- 2) Turn the coupling screw clockwise and firmly tighten the plastic strap (see fig. 41).  
**INFORMATION: It is not necessary to trim the excess of the plastic strap as long as it does not interfere with the mechanism of the rehab buggy.**
- 3) Push the red lock button down (see fig. 42, item 1) and insert the arm of the buggy board with the locking pin into the coupler (see fig. 42, item 2). Release the red lock button.  
**NOTICE! Verify that the distance of the arms from the middle of the buggy board is the same and that the locking pins are engaged in the couplers.**

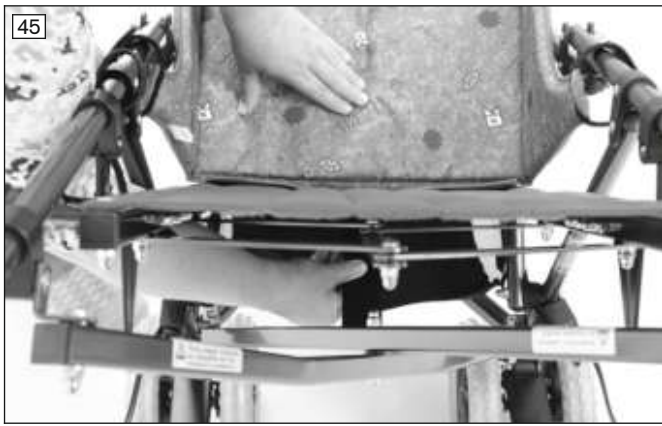
#### 8.4.4 Padding

The padding enhances the user's sitting comfort and also reduces the seat surface in the first months.

##### Attaching the simplified padding (HR32135300-343)

- 1) Place the seat pad onto the seat upholstery (see fig. 43).
- 2) Fasten the hook-and-loop closures on the frame (see fig. 44).
- 3) Push the tongue on the rear end of the seat pad back between the seat and back support upholstery, and attach it below the seat pad with the hook-and-loop fastener (see fig. 45).
- 4) Set the side pads of the seat pad on the right and left upright.
- 5) Put on the back support pad and, on the lower end, fasten the hook-and-loop fastener to the seat pad across the full width (see fig. 46).


### Attaching the padding (HR32070020-343)

The back support pad and the seat pad are connected to each other with snap fasteners.

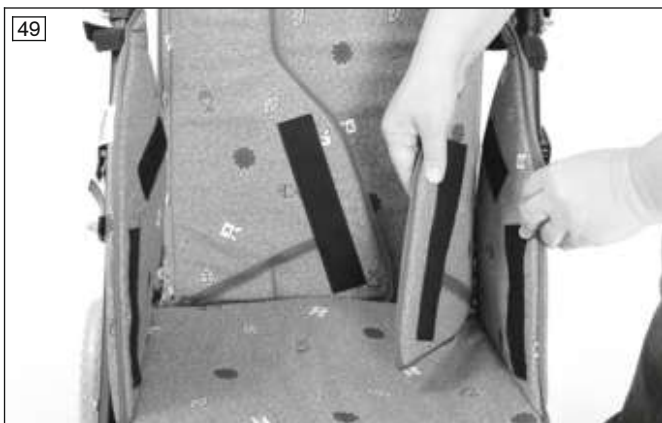- 1) Set the seat pad onto the seat upholstery and lean the back support pad against the back support upholstery (not illustrated).
- 2) Push the tongue on the rear end of the seat pad back between the seat and back support upholstery, and attach it below the seat pad with the hook-and-loop fastener (see fig. 47).
- 3) Set the side pads of the seat pad on the right and left upright and fasten them to the frame tube with the hook-and-loop closure (see fig. 48).
- 4) Apply the side pads of the back support pad to the side pads of the seat pad and press the hook-and-loop fasteners together (see fig. 49).

## 8.5 Disassembly and transport

### ⚠ CAUTION

#### Holding on to the product incorrectly while unfolding and folding

Pinching, crushing due to lack of caution in danger areas

- ▶ Hold the product only by the described parts.
- ▶ Never reach into the folding scissor mechanism.
- ▶ Avoid pinching yourself on movable components.

### NOTICE

#### Deformation while folded

Damage to the product, problems with unfolding due to improper loading

- ▶ Never place heavy objects onto the folded product.
- ▶ Make sure that objects placed in the storage bag cannot be crushed while folding.

- 1) Move the back support forward as far as possible (see Page 46).
- 2) Remove the calf band (see Page 48).
- 3) Release the lock on the bottom of the footplates, (see fig. 50) and flip up the footplates.
- 4) Swing the leg support brackets to the outside beside the front wheels (see fig. 51).
- 5) Pull up the folding lever under the seat (see fig. 52).
- 6) Open the folding guards on both sides of the front frame (see fig. 53).
- 7) Grasp the rehab buggy by the push handles and fold it together (see fig. 2).

## 8.6 Use in vehicles for transporting persons with reduced mobility

### ⚠ WARNING

#### Improper use in vehicles for transporting persons with reduced mobility

Risk of serious injury when using the product as a seat

- ▶ The product has **not** been approved by the manufacturer for use as a seat in vehicles for transporting persons with reduced mobility.
- ▶ In vehicles for transporting persons with reduced mobility, only use the seats installed in the vehicle with the corresponding personal restraint systems.
- ▶ Information on the current status of our measures is available from the qualified personnel.

The product Eco Buggy has not been approved by the manufacturer for use as a seat in a vehicle for transporting persons with reduced mobility.

## 8.7 Care

### ⚠ CAUTION

#### Lack of or improper cleaning

Health hazard due to infections, damage to the product due to user error

- ▶ Clean the product at regular intervals.
- ▶ Do not clean the product with a jet of water or a pressure washer. The penetration of water can cause corrosion and subsequent malfunctions.
- ▶ Check the driving behaviour of the product after cleaning it.

### 8.7.1 Cleaning

#### 8.7.1.1 Cleaning by hand

- 1) Remove the pads and covers (Detaching and attaching pads, Detaching/attaching the covers) and separate the covers from the pads.
- 2) Wipe the covers and pads with a damp cloth. Allow to dry.
- 3) **If needed:** Wash covers made from spacer fabrics at max. **40 °C**.  
**INFORMATION: The covers may also be washed in a washing machine if they are put in a washing bag or pillowcase.**

#### 8.7.1.2 Cleaning belts/straps

### INFORMATION

Follow the laundry recommendations on the product and the information in the instructions for use for the product concerned.

#### Straps with plastic buckles

If needed, the straps may be cleaned as follows:

- In the washing machine at **40 °C - 60 °C**. The manufacturer recommends using a washing bag and a mild detergent.
- By gently wiping with warm soapy water. Some disinfectant may also be added if necessary.
- By wiping with a clean, dry, absorbent cloth.

#### Additional cleaning instructions

- Allow the belts to air dry. Ensure that the belts and pads are completely dry before installation.
- Do not expose the belts to direct heat (e.g. sunshine, stove or radiator).
- Do not iron or bleach the belts.

### 8.7.2 Disinfection

- 1) Thoroughly clean the pads before disinfecting.
- 2) Wipe all parts of the product with a disinfectant.

#### Important information about disinfecting

- If the product is used by more than one person, using a conventional disinfectant is required.
- Only use colourless water-based disinfectants. Observe the instructions for use provided by the manufacturer.

## 9 Maintenance and repair

### 9.1 Maintenance

#### **⚠ WARNING**

#### **Insufficient maintenance**

Severe user injuries, damage to the product due to failure to observe maintenance intervals

- ▶ Only carry out the maintenance tasks described in this section. All other maintenance and service tasks may only be carried out by qualified personnel.
- ▶ The functionality and operating safety of the product must be verified and a service performed at least **once per year**.
- ▶ For users with a changing anatomy (for example body dimensions, weight) or users with a changing clinical picture, have the product inspected, adjusted and serviced at least **once every six months**.

- The function of the product should be checked **before each use**.
- The product may not be used if defects are noted. This applies in particular in case of instability of the product or altered driving characteristics as well as problems with the user's seating position or the stability of the seat. Inform the qualified personnel promptly for the rectification of defects.
- This also applies if loose, worn, bent or damaged components, cracks in the frame or broken frame components are identified.
- Some maintenance tasks can be carried out to a certain extent by the user at home (see the sections "Maintenance intervals" and "Maintenance tasks").
- Failure to maintain the product can lead to injuries for the user of the product.

#### **9.1.1 Maintenance intervals**

The functions described below must be checked by the user or an attendant at the specified intervals:

<b>Inspection task</b>	<b>Before each use</b>	<b>Monthly</b>	<b>Quarterly</b>
Functional test of the wheel locks	X		
Check of all screw connections		X	
Wear and tear of the folding mechanism		X	
Locking mechanism of the footplates		X	
Check belts for wear		X	
Check that padding and calf band are in proper condition			X
Visual inspection of wear and tear parts (e.g., tyres, bearings)			X
Soiling of bearings			X
Check the legibility of all labels and markings on the product			X

#### **9.1.2 Maintenance tasks**

To ensure smooth operation at all times, users or attendants with some technical skills can maintain some parts of the product:

- Particles of dirt and hair accumulate on the caster wheel axle and threaded axle of the caster fork over time. This makes the steering stiffer. Regularly remove dirt and oil the axles. See the section "Approach in case of stiffness".
- Dirt particles and hair accumulate on the axle over time. This causes the wheels to turn less easily. Regularly remove dirt from the axle to ensure wheels remain functional. Then lubricate the axle with a few drops of thin, resin-free oil (e.g. sewing machine oil).

**CAUTION: Do not use pointed or sharp objects that could damage the product when removing dirt. If the wheels still do not rotate smoothly, please contact the qualified personnel.**

- If the rehab buggy gets wet, it should be rubbed dry again.
- Screw connections must be periodically checked for tightness, especially during the initial period of use or after making adjustments to the rehab buggy. If a screw connection loosens repeatedly, contact the qualified personnel promptly.

## 9.2 Repair

### **⚠ WARNING**

#### **Prohibited repairs**

Severe user injuries, damage to the product due to adjustment and installation errors

- ▶ All repairs may only be carried out by qualified personnel.

## 10 Disposal

### 10.1 Disposal information

Return the product to the qualified personnel for disposal.

All components of the product must be disposed of properly in accordance with the respective national environmental regulations.

### 10.2 Information on re-use

### **⚠ CAUTION**

#### **Used seat padding**

Functional and/or hygienic risks due to re-use

- ▶ Replace the seat padding if the wheelchair is to be re-used.

The product is suitable for re-use.

Similar to second-hand machines or vehicles, products that are being re-used are subject to increased strain. Features and performance must not change in a way that could impair the safety of users or third parties during the period of use.

The relevant product must be thoroughly cleaned and disinfected before re-use. Then have the product inspected by qualified personnel with respect to its condition, wear and tear, and damage. Worn and damaged parts as well as components that do not fit or are unsuitable for the user must be replaced.

Detailed information on replacing components as well as information on the required tools can be found in the service manual.

## 11 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 11.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 11.2 Warranty

Further information on the warranty terms and conditions can be obtained from the qualified personnel that has fitted this product or the manufacturer's service (see inside back cover for addresses).

### 11.3 Service life

Expected lifetime: **4 years**

The design, manufacturing and requirements for the intended use of the product are based on the expected lifetime. These also include the requirements for maintenance, ensuring effectiveness and the safety of the product.

## 12 Technical data

### **INFORMATION**

- ▶ Much of the technical data below is given in mm. Please note that product settings – unless otherwise specified – cannot be adjusted in the mm range but only in increments of approx. **0.5 cm** or **1 cm**.
- ▶ Note that the values achieved during adjustment may deviate from the values specified below. The deviation can be **±10 mm** and **±2°**.

<b>Dimensions, angles, weights</b>	<b>Model 3210</b>
Overall length (with leg supports) [mm]	1,020
Overall width [mm]	545
Overall height [mm]	1,100
Folding size (L x W x H) [mm]	1,150 x 310 x 310
Seat depth [mm]	310 / 360
Seat width [mm]	350
Seat height (front edge to floor) [mm]	455
Seat bottom angle [°]	15
Back support height [mm]	550
Angle from back support to seat surface [°]	90–110
Lower leg length (distance from footplates to seat surface) [mm]	200–300
Angle from leg support to seat surface [°]	95
Horizontal axle position [mm]	200
Min. turning radius [mm]	880
Front wheel (polyurethane, puncture-proof) (L x W) [mm (")]	165 x 36 (6.5)
Rear wheel (polyurethane, puncture-proof) (L x W) [mm (")]	205 x 40 (8.0)
Max. inclination while driving [°]	7
Max. inclination with wheel lock engaged [°]	9
Max. user weight [kg]	50
Max. load for storage bag [kg]	5
Weight [kg]	9
Weight of the heaviest removable component:	
• Leg support [kg]	0.35

### Ambient conditions

<b>Temperatures and relative humidity</b>	
Temperature during use [°C (°F)]	-10 to +40 (14 to 104)
Transport and storage temperature [°C (°F)]	-10 to +40 (14 to 104)
Relative humidity [%]	10 to 85; non-condensing

## 13 Appendices

### 13.1 Threshold values for rehab buggies transportable by train

#### INFORMATION

- The products in this series fully satisfy the minimum technical requirements of regulation (EU) No. 1300/2014 regarding train accessibility for people with disabilities.

<b>Feature</b>	<b>Threshold value (according to Regulation (EU) No. 1300/2014)</b>
Length [mm]	1200 (plus 50 mm for the feet)
Width [mm]	700 (plus 50 mm on each side for the hands when moving)
Smallest wheels ["]	approx. 3 or greater (according to the regulation, the smallest wheel must be able to overcome a gap measuring 75 mm horizontally and 50 mm vertically)
Height [mm]	max. 1375; including a 1.84 m large male user (95th percentile)
Turning radius [mm]	1500
Maximum weight [kg]	200 (product with user, including luggage)

<b>Feature</b>	<b>Threshold value (according to Regulation (EU) No. 1300/2014)</b>
Maximum obstacle height that can be overcome [mm]	50
Ground clearance [mm]	60 (at an upward slope angle of 10°, ground clearance must measure at least 60 mm under the footrest for going forward at the end of the slope)
Maximum inclination angle on which the product will remain stable [°]	6 (dynamic stability in all directions) 9 (static stability in all directions, also when wheel lock engaged)

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Otto Bock Healthcare Products GmbH  
Brehmstraße 16 · 1110 Wien · Austria  
F +43 1 5267985  
service-admin.vienna@ottobock.com · www.ottobock.at

Otto Bock Adria d.o.o. Sarajevo  
Ramiza Salčina 85  
71000 Sarajevo · Bosnia-Herzegovina  
T +387 33 255-405 · F +387 33 255-401  
obadria@bih.net.ba · www.ottobockadria.com.ba

Otto Bock Bulgaria Ltd.  
41 Tzar Boris III<sup>rd</sup> Blvd. · 1612 Sofia · Bulgaria  
T +359 2 80 57 980 · F +359 2 80 57 982  
info@ottobock.bg · www.ottobock.bg

Otto Bock Suisse AG  
Luzerner Kantonsspital 10 · 6000 Luzern 16 · Suisse  
T +41 41 455 61 71 · F +41 41 455 61 70  
suisse@ottobock.com · www.ottobock.ch

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Otto Bock South Africa (Pty) Ltd  
Building 3 Thornhill Office Park · 94 Bekker Road  
Midrand · Johannesburg · South Africa  
T +27 11 564 9360  
info-southafrica@ottobock.co.za  
www.ottobock.co.za

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Otto Bock HealthCare Canada  
5470 Harvester Road  
Burlington, Ontario, L7L 5N5, Canada  
T +1 800 665 3327 · F +1 800 463 3659  
CACustomerService@ottobock.com  
www.ottobock.ca

Oficina Ottobock Habana  
Calle 3ra entre 78 y 80.  
Edificio Jerusalem · Oficina 112 · Calle 3ra.  
Playa, La Habana. Cuba  
T +53 720 430 69 · +53 720 430 81  
hector.corcho@ottobock.com.br  
www.ottobock.com.br

Otto Bock HealthCare Andina Ltda.  
Calle 138 No 53-38 · Bogotá · Colombia  
T +57 1 8619988 · F +57 1 8619977  
info@ottobock.com.co · www.ottobock.com.co

Otto Bock de Mexico S.A. de C.V.  
Prolongación Calle 18 No. 178-A  
Col. San Pedro de los Pinos  
C.P. 01180 México, D.F. · Mexico  
T +52 55 5575 0290 · F +52 55 5575 0234  
info@ottobock.com.mx · www.ottobock.com.mx

Otto Bock HealthCare LP  
11501 Alterra Parkway Suite 600  
Austin, TX 78758 · USA  
T +1 800 328 4058 · F +1 800 962 2549  
USCustomerService@ottobock.com  
www.ottobockus.com

## Asia/Pacific

Otto Bock Australia Pty. Ltd.  
Suite 1.01, Century Corporate Centre  
62 Norwest Boulevard  
Baulkham Hills NSW 2153 · Australia  
T +61 2 8818 2800 · F +61 2 8814 4500  
healthcare@ottobock.com.au · www.ottobock.com.au

Beijing Otto Bock Orthopaedic Industries Co., Ltd.  
B12E, Universal Business Park  
10 Jiuxianqiao Road, Chao Yang District  
Beijing, 100015, P.R. China  
T +8610 8598 6880 · F +8610 8598 0040  
news-service@ottobock.com.cn  
www.ottobock.com.cn

Otto Bock Asia Pacific Ltd.  
Unit 1004, 10/F, Greenfield Tower, Concordia Plaza  
1 Science Museum Road, Tsim Sha Tsui  
Kowloon, Hong Kong · China  
T +852 2598 9772 · F +852 2598 7886  
info@ottobock.com.hk · www.ottobock.com

Otto Bock HealthCare India Pvt. Ltd.  
20th Floor, Express Towers  
Nariman Point, Mumbai 400 021 · India  
T +91 22 2274 5500 / 5501 / 5502  
information@indiaottobock.com · www.ottobock.in

Otto Bock Japan K. K.  
Yokogawa Building 8F, 4-4-44 Shibaura  
Minato-ku, Tokyo, 108-0023 · Japan  
T +81 3 3798-2111 · F +81 3 3798-2112  
ottobock@ottobock.co.jp · www.ottobock.co.jp

Otto Bock Korea HealthCare Inc.  
4F Agaworld Building · 1357-74, Seocho-dong  
Seocho-ku, 137-070 Seoul · Korea  
T +82 2 577-3831 · F +82 2 577-3828  
info@ottobockkorea.com · www.ottobockkorea.com

Otto Bock South East Asia Co., Ltd.  
1741 Phaholyothin Road  
Kwaeng Chatuchark · Khet Chatuchark  
Bangkok 10900 · Thailand  
T +66 2 930 3030 · F +66 2 930 3311  
obsea@ottobock.co.th · www.ottobock.co.th

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Otto Bock Mobility Solutions GmbH