1B1-2=* ⓘ 647G1441, 647G1440=1, 647G1440=2
 ⓘ 647G1441=all_INT, 647H64-1=all_INT, 647H64-2=all_INT

1B1-2=* Meridium

DE Zusatzdokument	2
EN Additional documentation	2
FR Document supplémentaire	3
IT Documento aggiuntivo	4
ES Documento adicional	4
PT Documento adicional	5
NL Aanvullend document	6
SV Tilläggsdokument	6
DA Supplerende dokument	7
NO Tilleggsdokument	8
FI Lisäasiakirja	8
PL Dokument dodatkowy	9
HU Kiegészítő dokumentum	10
CS Doplňující dokument	10
HR Dodatni dokument	11
SK Dodatočný dokument	12
TR İlave doküman	12
RU Дополнительный документ	13
JA 追加資料	14
ZH 补充文件	14

1 Vorwort

INFORMATION

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INFORMATION

Date of last update: 2020-09-10

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

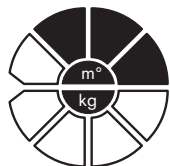
The content of this additional documentation extends the recommended MOBIS classification marked below under the conditions listed in the "Conditions for use" section.

This additional documentation is intended for qualified personnel that have been authorised by Ottobock through training.

The content of this additional documentation does not replace the instructions for use included with the respective products.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Conditions of use

Despite the recommended mobility grade classification for the product (mobility grades 2 and 3), this prosthesis is permitted for users with mobility grade 4 (unrestricted outdoor walker with particularly high demands) provided the following sections are observed:

Increased activity may trigger excessive temperature mode

In individual cases, the excessive temperature mode may occur more frequently because patients with mobility grade 4 have an increased activity level. The steps in the instructions for use on the cover sheet should be followed if the excessive temperature mode is triggered (brief vibration every five seconds).

Recommended trial fitting for patients with increased activity

We recommend carrying out a trial fitting with the 1B1-2=* prosthetic foot lasting at least one week for patients with increased activity. If the excessive temperature mode is activated during this period – triggered by increased activity levels – a different prosthetic foot would be more suitable for the patient.

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Otto Bock Healthcare Products GmbH