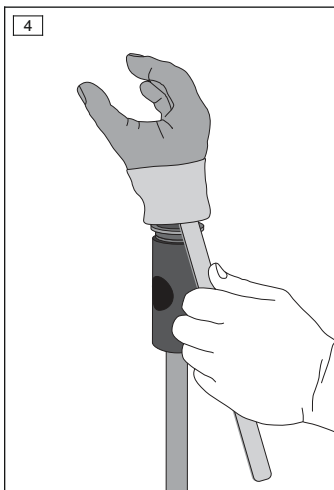
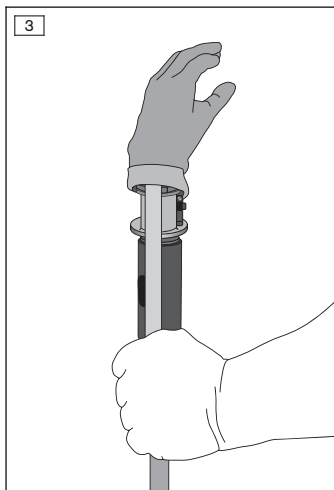
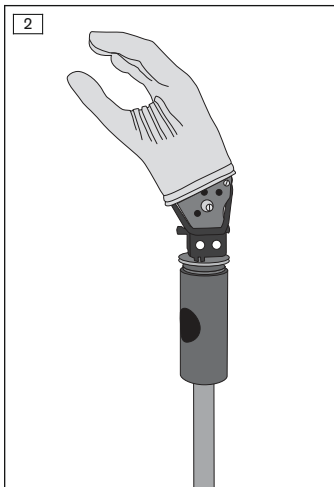
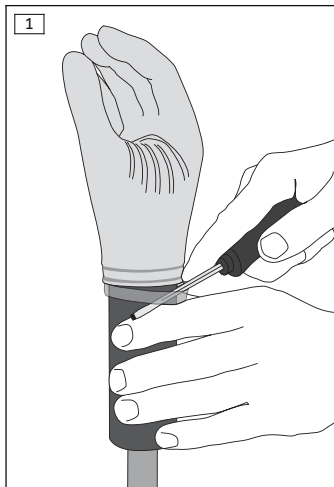
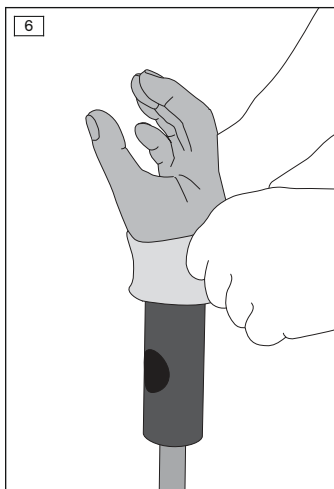
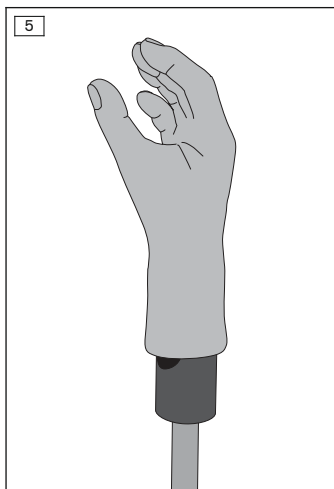
## 88A15=\* MySkin Myo

<b>DE</b> Gebrauchsanweisung .....	4
<b>EN</b> Instructions for use .....	10

## INFORMATION

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### INFORMATION

Date of last update: 2020-12-17

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The 88A15=\* MySkin Myo is a silicone cover for myoelectrically controlled prosthetic hands. It is an aesthetically pleasing and easy to care for alternative to standard prosthetic gloves.

The cover is pulled directly over the myoelectric prosthetic hand (without inner hand) and protects the components underneath against moisture and dirt.

The appearance is based on the contralateral side in order to achieve a high degree of naturalness.

The material is coloured by hand, giving rise to possible variations in appearance, which are not a product defect.

The fingernails **cannot** be painted.

## 2 Intended use

### 2.1 Indications for use

The product is to be used solely for upper limb prosthetic fittings.

## 2.2 Area of application

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

## 2.3 Combination possibilities

### Limited combination options for Ottobock components

Compatible prosthetic hands: 8E38\*, 8E39\*, 8E41\*

## 2.4 Environmental conditions

Allowable environmental conditions
Allowable operating temperature: <b>-10 °C to +60 °C (+14 °F to +140 °F)</b>
Allowable storage temperature: <b>-20 °C to +40 °C (-4 °F to +104 °F)</b>
Allowable transport temperature: <b>-20 °C to +60 °C (-4 °F to +140 °F)</b>
Allowable relative humidity: <b>max. 80%</b>
Cleaning agents: warm water and soap

Unallowable environmental conditions
Mechanical vibrations or impacts, touching of sharp-edged or pointed objects
Contact with acids, unallowable cleaning agents, abrasive media (e.g. sand), damp soil, dust, dirt, highly staining materials (e.g. printing ink, pen ink, toner, new textiles), solvents (e.g. thinner, acetone, petrol), fire, embers and unallowable heat (> <b>+60 °C / &gt; +140 °F</b> ), silicone spray

## 2.5 Service life

The product is a wear part, which means it is subject to natural wear and tear. The service life diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximal service life, in dependence of the level of use.

## 3 Safety

### 3.1 Explanation of warning symbols

 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

## 3.2 General safety instructions

### CAUTION

#### **Contact with cold, heat, embers or fire**

Injuries (e.g. burns) and damage to the product due to unallowable environmental conditions

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").

### NOTICE

#### **Use under unallowable environmental conditions**

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

### NOTICE

#### **Unallowable combination of prosthetic components**

Product damage due to malfunctions

- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination Possibilities".
- ▶ Verify that the prosthetic components may be combined with each other by consulting their instructions for use.

### NOTICE

#### **Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

**NOTICE****Careless use of the product**

Damage to the product due to contact with pointed or sharp-edged objects

- ▶ Do not allow the product to come into contact with the aforementioned objects.

## 4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	MySkin Myo prosthesis cover	88A15=*

## 5 Preparing the product for use

**NOTICE****Incorrect donning and removal of the prosthesis cover**

Mechanical damage to the prosthetic hand and prosthesis cover

- ▶ Observe all instructions for donning and removal. The prosthetic hand must always be mounted on the donning tool.
- ▶ Only use the materials listed.
- ▶ Avoid damage that can be caused to the prosthetic hand and the prosthesis cover, e.g. from tools and excessive force (in particular on the thumb).

### 5.1 Preparations for donning and removal

**Prosthetic hand with quick-disconnect wrist unit**

> **Required materials:** 711M3 mounting tool, 711M1 mounting tool (alignment rod), 711M2 mounting plate or vice

- 1) Install the prosthetic hand on the mounting tool.
- 2) Install the mounting tool on the alignment rod on the side with the interior thread.
- 3) Attach the alignment rod to the mounting plate or clamp it in a vice.

**Pulling off the inner hand**

Before pulling on the prosthesis cover, the inner hand has to be pulled off the prosthetic hand.

- 1) Remove the sealing lock ring from the prosthetic hand using a screwdriver (see fig. 1).
- 2) Carefully pull the inner hand off the prosthetic hand (see fig. 2).

## 5.2 Pulling on the prosthesis cover

- > **Required materials:** rounded tool (e.g. wood spatula), liquid soap
- > **Prerequisite:** The distance between the thumb and index finger of the prosthetic hand is approx. **4 cm**.
- 1) **NOTICE! Do not use silicone spray for pulling on the prosthesis cover. Do not glue the prosthesis cover to the prosthesis.**  
**Optional:** Apply liquid soap to the outside of the prosthesis cover to make it slide more easily.
- 2) Turn the prosthesis cover inside out past the wrist joint.
- 3) Make sure that the finger support between the ring finger and little finger has not shifted out of place.
- 4) Position the cover on the prosthesis, carefully stretch it with a rounded tool and pull it over the motor unit (see fig. 3). In doing so, ensure that the gripping fingers of the prosthetic hand are positioned in the openings provided in the prosthesis cover.
- 5) Position the cover in the retention grooves on the wrist joint (see fig. 4).
- 6) Pull the cover fully over the prosthesis and align it (see fig. 5).
- 7) **If liquid soap was used:** Remove all soap residue by carefully rinsing the prosthesis cover.

## 5.3 Pulling off the prosthesis cover

- 1) Apply liquid soap or donning spray to the prosthesis cover.  
**NOTICE! Avoid all excessive force on the fingers during the removal process.**
- 2) Carefully hold the prosthesis cover on the sleeve end and pull it off (see fig. 6).

## 6 Use

### INFORMATION

Minor damage to the product can be repaired. Contact the manufacturer to discuss the further procedure.

## 6.1 Cleaning

### NOTICE

#### **Non-observance of cleaning instructions**

Damage to product and permanent soiling due to use of improper cleaning agents

- ▶ Only clean the product according to the cleaning instructions provided in the instructions for use.

#### NOTICE

##### **Penetration of moisture**

Loss of or limited functionality of the prosthesis

- ▶ Make sure that no water gets into the prosthesis while cleaning the prosthesis cover. The prosthesis cover is not firmly secured to the socket, so that water can get into the prosthesis.

#### INFORMATION

The product must be cleaned and inspected for damage daily.

- > **Recommended cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)
- 1) Clean the product with clear fresh water and a pH neutral soap.
- 2) Rinse the soap away with clear fresh water.
- 3) Dry the product with a soft cloth.
- 4) Allow to air dry in order to remove residual moisture.

## 7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## 8 Legal information

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

Ottobock SE & Co. KGaA  
Max-Näder-Straße 15 · 37115 Duderstadt · Germany  
T +49 5527 848-0 · F +49 5527 848-3360  
healthcare@ottobock.de · [www.ottobock.com](http://www.ottobock.com)