RS2-K1 Runway[®] HX

DE Gebrauchsanweisung	3
EN Instructions for use	8
FR Instructions d'utilisation.....	12
ES Instrucciones de uso.....	17
PT Manual de utilização	21
NL Gebruiksaanwijzing	25

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Artikelnummer: RS2-K1

Produkt

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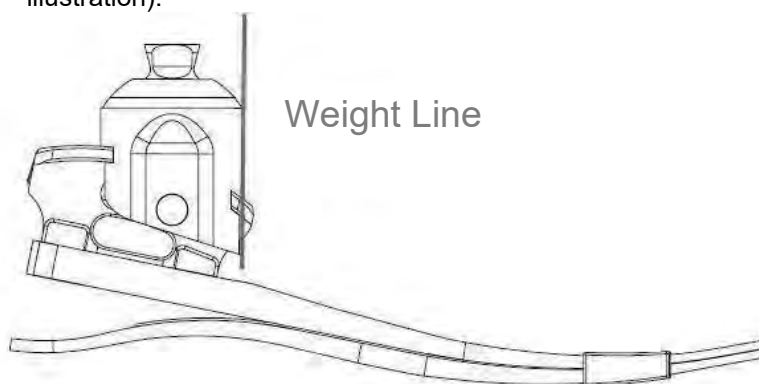
Runway® HX Instruction for Use

Product Number: RS2-K1

Product Description The Runway® HX is a carbon fiber adjustable heel height prosthetic foot. Its anatomical gliding ankle maintains appropriate foot alignment to provide knee stability and consistent performance at all heel heights. The Runway® carbon fiber material stores and returns energy so that users can walk farther, faster, longer. The Runway® fiber layers are tailored to an individual's body weight and impact level, providing a customized product for superior comfort.

Bench Alignment Prior to donning the prosthesis:

- Plantarflex/Dorsiflexion foot to match the shoe heel height. (See Operation Section)
- Adduct/Abduct socket to provide appropriate frontal plane angle.
- Flex/Extend socket to provide appropriate sagittal plane angle.
- Move the socket to ensure the weight line falls along the anterior edge of the pylon (see illustration).

Dynamic Alignment To optimize the heel to toe rollover motion, adjust the following variables:

- Anterior/posterior foot placement
- Dorsiflexion/plantarflexion
- Heel stiffness

Heel Stiffness	Heel too soft	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> • Foot flat occurs too rapidly • Toe feels excessively stiff • Knee hyperextension <p><i>Solutions</i></p> <ul style="list-style-type: none"> • Shift socket anteriorly in relation to the foot • Attach rubber <i>stiffening bumpers</i>
	Heel too hard	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> • Rapid knee flexion, instability • Heel to toe progression too rapid • Lack of energy return sensation <p><i>Solutions</i></p> <ul style="list-style-type: none"> • Shift socket posteriorly in relation to the foot • Verify appropriate foot module category
Toe Stiffness	Toe too stiff	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> • Flat spot in rollover motion at slow cadences <p><i>Solutions</i></p> <ul style="list-style-type: none"> • Consider a lower category foot module
	Toe too soft	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> • Clicking noise at <i>initial contact</i> • Excessive toe deflection with high impact activity <p><i>Solutions</i></p> <ul style="list-style-type: none"> • Consider a higher category foot module

Runway® HX Instruction for Use

Product Number: RS2-K1

Stiffening Bumpers	Rubber <i>stiffening bumpers</i> are included to adjust the heel stiffness during <i>loading response</i> . The bumpers may be temporarily attached between the heel lever and the keel using the pre-applied adhesive in the location indicated on the bumper package to increase heel stiffness one category. If the heel stiffness is too stiff, move the bumper posteriorly; too soft, move it anteriorly. For permanent placement, clean off the pre-applied adhesive with Acetone, and adhere bumpers using Super Glue (cyanoacrylate).
Spectra™ Sock	Replace Spectra™ socks at intervals appropriate to the user's activity level to protect the foot shell and minimize noise. Failure to inspect and replace the Spectra™ socks may prematurely wear the foot module.
Foot Shell	Use the Foot Shell Removal Tool (ACC-00-10300-00) to install or remove the foot shell to prevent damage to the foot module.
Runway®	Maximum user weight: 116 kg Available sizes: 22cm-28cm Heel height: Adjustable, 0-5cm

Operation (Prosthetist)

- Insert a 4mm hex key (provided) through the posterior foot shell into the ankle mechanism.
- To increase the foot height, turn the hex key clockwise.
- To decrease the foot height, turn the hex key counterclockwise.
- Remove the hex key prior to allowing ambulation.

Operation (User)

- Your prosthetist may let you use the prosthesis without a foot shell for a short period of time, allowing you to visually learn to adjust the ankle mechanism by setting the pylon in the correct vertical orientation.
- Inform your prosthetist if the toe and heel of the foot do not feel balanced.
- For best result, be seated in a chair to adjust the heel height.
- Do not attempt to adjust the ankle while standing without using a grab rail or other appropriate fixed object for support.
- Insert a 4mm hex key (provided) into the back of the foot shell.
- To increase the foot height, turn the hex key clockwise.
- To decrease the foot height, turn the hex key counterclockwise.
- Fourteen turns will change the heel height by one inch.
- The wrench must be removed from the ankle before placing weight on the foot.

Maintenance (Prosthetist)

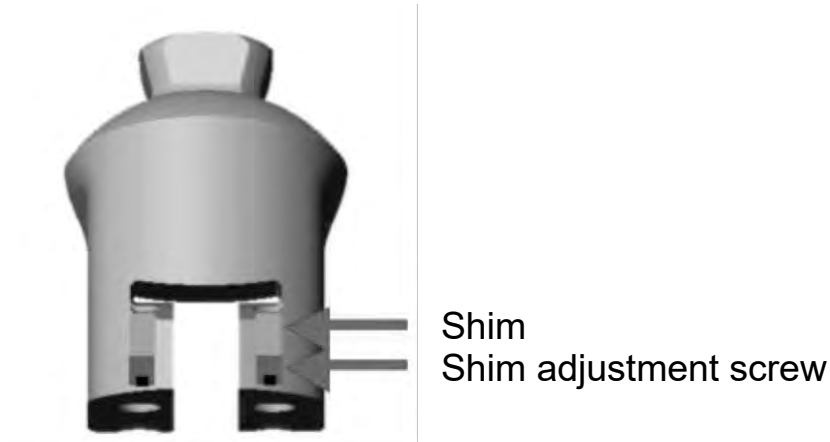
The ankle mechanism requires periodic maintenance.

- Inspect the foot every six months. For an active user, increased inspection frequency may be needed. Service as necessary. Replace Spectra™ sock and/or foot shell if worn to prevent damage to the foot module components.
- Wet or dirty environments may cause the ankle module to malfunction and make it difficult to operate.
 - If the ankle module is exposed to wet or dirty environments, it is recommended to periodically clean and lubricate the module in order to minimize impact on module performance.
 - Disassemble from shell, rinse with fresh water and thoroughly dry.
 - If corrosion or buildup has occurred, clean the ankle mechanism with solvent (i.e., acetone, WD-40) and compressed air.
 - Cycle the mechanism thru the entire range while cleaning.
 - Lubricate the slide with light grease, or spray lubricant and wipe off excess.
- If the ankle mechanism makes a clicking noise when tapping the toe or heel of the foot, ankle shims adjustment will be required by the prosthetist.

Runway® HX Instruction for Use

Product Number: RS2-K1

- Remove the foot shell and Spectra™ sock to adjust the ankle shims.
- Ensure ankle assembly is clean and appropriately lubricated before adjustment.
- Use a 1/8" hex wrench (provided) to tighten slide adjustment screws located on the underside of the pyramid slide.
- Tighten each screw until firm resistance is felt, then loosen until resistance to slide movement is minimal.
- Check slide action after each shim is adjusted.
- If shims are loose, the ankle may make noise when walked. If shims are too tight, the ankle mechanism will be difficult to adjust. (See Figure below)

Maintenance (User)

- Use soap and warm water to clean and/or disinfect the foot shell.
- Wet or dirty environments may cause the ankle module to malfunction and make it difficult to operate.
 - If the ankle module is exposed to wet or dirty environments, it is recommended to periodically clean and lubricate the module in order to minimize impact on module performance.
 - Disassemble from shell, rinse with fresh water and thoroughly dry.
 - If corrosion or buildup has occurred, clean the ankle mechanism with solvent (i.e., acetone, WD-40) and compressed air.
 - Cycle the mechanism thru the entire range while cleaning.
 - Lubricate the slide with light grease, or spray lubricant and wipe off excess.
- Have the foot module serviced at the intervals specified by the prosthetist.

Runway® HX Instruction for Use

Product Number: RS2-K1

Warnings

- Wet or dirty environments may cause the ankle module to malfunction and make it difficult to operate.
- Never allow aggregates such as sand to remain in the foot shell.
- If the ankle module is exposed to wet or dirty environments, it is recommended to periodically clean and lubricate the module in order to minimize impact on module performance.
 - Disassemble from shell, rinse with fresh water and thoroughly dry.
 - If corrosion or buildup has occurred, clean the ankle mechanism with solvent (i.e., acetone, WD-40) and compressed air.
 - Cycle the mechanism thru the entire range while cleaning.
 - Lubricate the slide with light grease, or spray lubricant and wipe off excess.
- Always use the foot with a foot shell. Failure to comply may cause premature wear, loss of function, and/or product failure.
- Always use the foot with a sock and shoe. Failure to comply may cause noise, premature wear, loss of function, and/or product failure.
- Do not attempt to adjust the ankle while standing, without a grab rail or other appropriate fixed object for support.
- Never attempt to loosen the bolts affixing the ankle mechanism to the keel.
- Discontinue use and consult prosthetist if prosthesis starts to make any noise.
- Do not share your prosthesis with others; it may cause damage to the device and/or injury.
- Inform your prosthetist if you lose or gain a significant amount of weight.

Instructions for use Heel Bumper

Test bumper placement

Temporarily attach at recommended location using pre-applied adhesive. Suggested placement will stiffen the heel by approximately one category. If necessary, adjust anterior/posterior placement to achieve desired stiffness.

Permanently bond

Lightly abrade carbon fiber with sandpaper, then clean with acetone. Remove temporary adhesive on bumpers using a cloth with acetone. Do not soak bumpers in acetone. Bond bumpers in pre-selected position using cyanoacrylate (Super Glue).

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Advena Limited, Tower Business Centre, 2nd Flr., Tower