743G15 Ipso Cast

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Table of contents

EN

1	Notes regarding the document	16
1.1	Explanation of warning symbols	16
1.2	Meanings of pictograms in the illustration	16
2	Product description	17
3	Scope of delivery	19
4	Preparation for use	19
5	Use	20
6	Cleaning	25
7	Disposal	25
8	Legal information	25
8.1	Liability	25

1 Notes regarding the document

INFORMATION

Date of last update: 2021-04-16

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Please keep this document in a safe place.

When a text refers to a figure, this is pointed out by a reference to the figure [example: (1)].

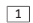
This document and the product are intended exclusively for the fabrication of a prosthesis by orthopaedic technicians with technical knowledge of lower limb prosthetic fittings.

1.1 Explanation of warning symbols


 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

1.2 Meanings of pictograms in the illustration

	Numbering for the illustrations		Observe the safety notice in the section
	Numbering for a defined sequence		Numbering for the parts of an illustration
	Acceptable		Unacceptable
	Locked		Unlocked

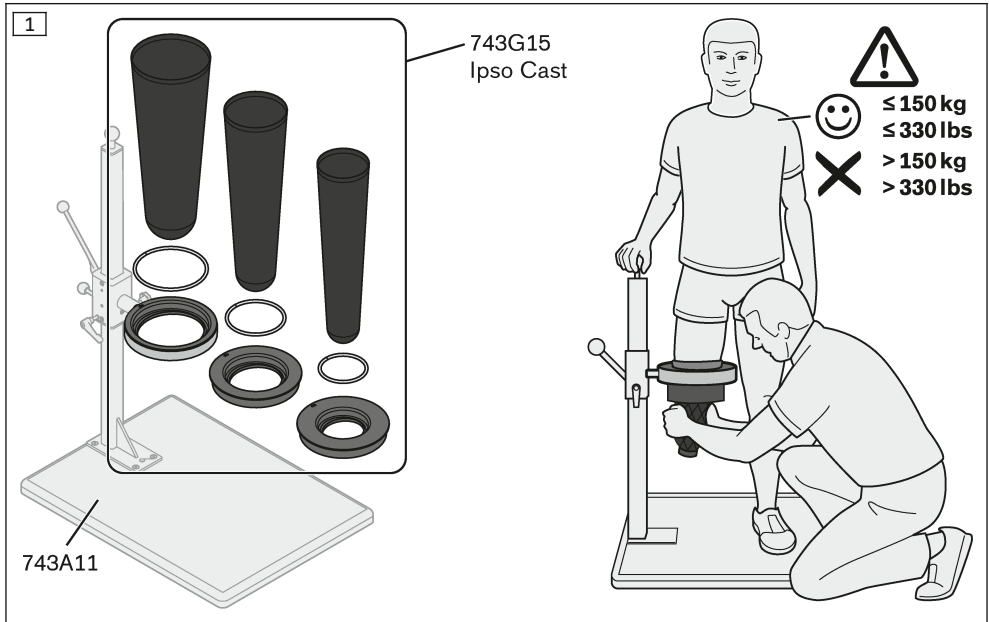2 Product description

The 743G15 Ipso Cast was designed for installation in the 743A11 Ottobock plaster device and for producing a plaster cast for transtibial residual limbs. The product's main components are an adapter as well as casting tubes, retaining rings and adapter rings in three different sizes. The product is intended exclusively for use in an orthopaedic workshop.

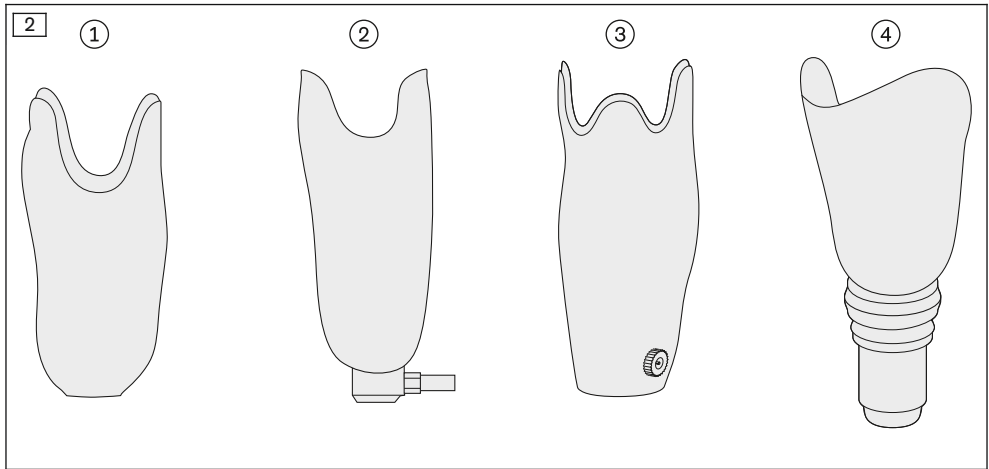
Allowable body weight (1)

CAUTION

Excessive strain on the product

Risk of injury due to breakage of load-bearing components and functional limitations

- ▶ Make sure that only the allowable body weight is applied to the product.

Plaster casts can be produced for various prosthetic sockets (e.g. ① standard socket, ② socket with shuttle lock, ③ socket with valve, ④ Harmony socket).

Unacceptable environmental conditions

NOTICE

Storage or use under unacceptable environmental conditions

Damage to product due to unacceptable environmental conditions

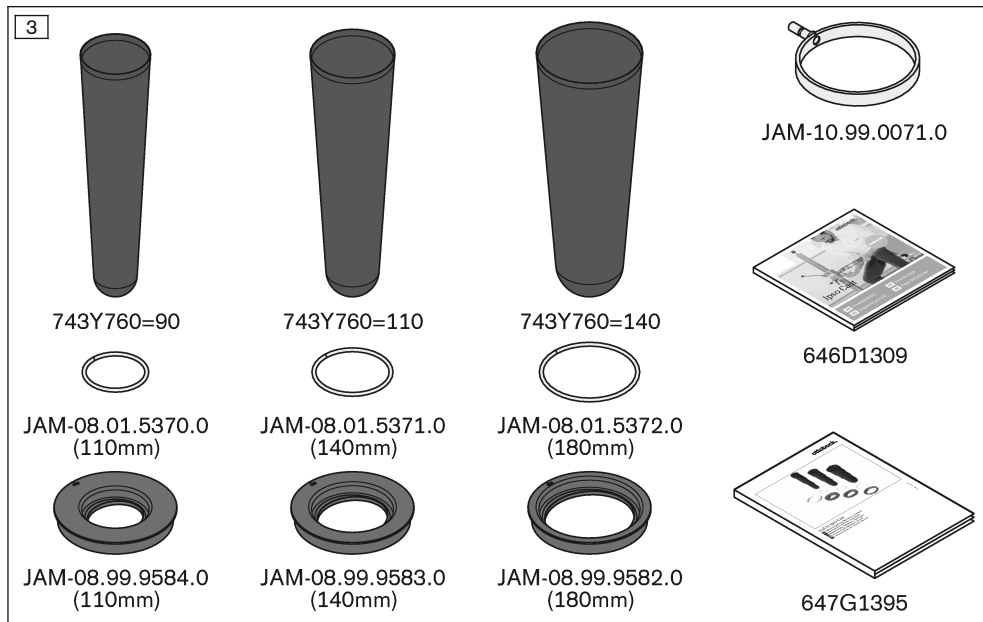
- ▶ Do not expose the product to unacceptable environmental conditions.
- ▶ If the product has been exposed to unacceptable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

Unacceptable environmental conditions


Unacceptable means of drying: heater, oven, hair dryer, hot air blower

Unacceptable substances: cleaning agents containing solvents, acids, urine

3 Scope of delivery

4 Preparation for use

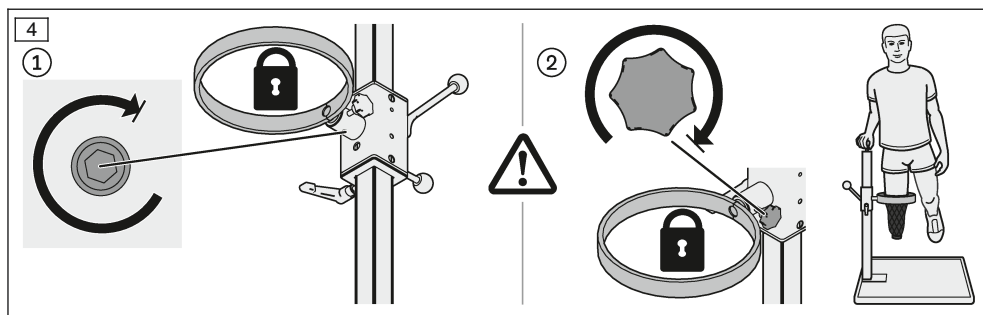CAUTION

Incorrect alignment or assembly

Risk of injury due to the adapter sliding out or turning inadvertently

- ▶ The adapter of the Ipsocast, which is inserted into the 743A11 Ottobock plaster device, has to be secured with the lock screw so it cannot slide out.
- ▶ When the patient only puts weight on the amputated side, the adapter has to be secured against rotation with the knurled screw.

5 Use

⚠ CAUTION

Product damage

Risk of injury due to product damage

- ▶ Use caution when working with the product.
- ▶ Check the product for damage before each use.
- ▶ Do not continue using the product in case of product damage relevant to safety.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

⚠ CAUTION

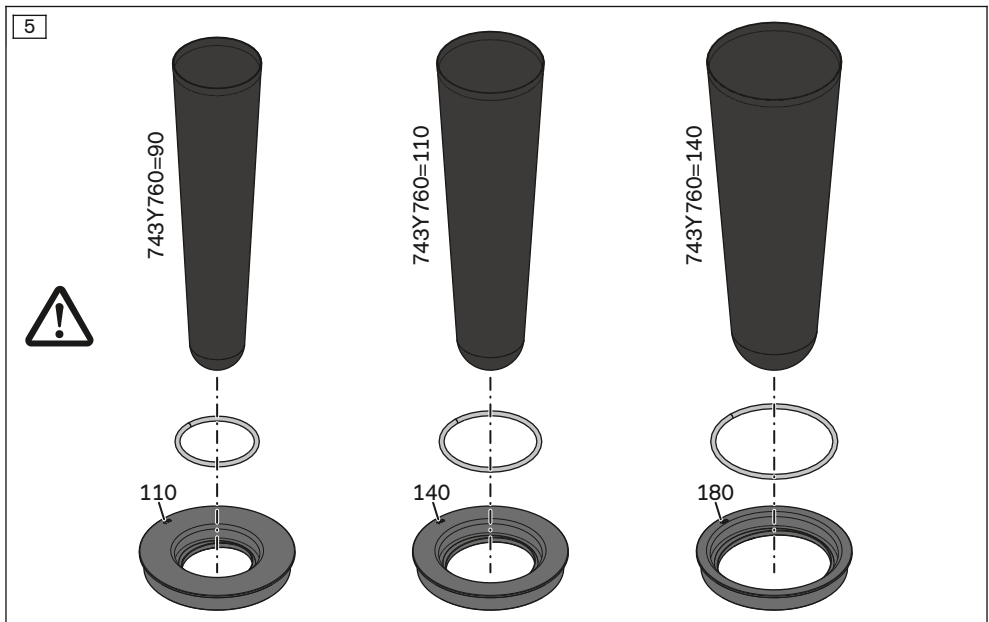
Exceeding the maximum number of plaster casts per casting tube

Risk of falling due to tearing caused by material fatigue

- ▶ Make sure that the maximum number of plaster casts per casting tube established in this section is not exceeded. Otherwise, the casting tube loses its structural strength and functionality.

- Maximum number of plaster casts per casting tube: **15**

Combining product components (5)

- ▶ **CAUTION!** Only combine the casting tubes with the retaining rings and adapter rings as shown in the figure.

Selecting the adapter ring (6)


<p>6</p> <p>~50 mm</p>	<p>110 mm 120 mm</p> <p>140 mm 150 mm</p> <p>180 mm 190 mm</p>	<p>m-l</p> <p>a-p</p> <p>a-p: ≤95 mm m-l: ≤85 mm</p> <p>a-p: 96 – 125 mm m-l: 86 – 115 mm</p> <p>a-p: 126 – 165 mm m-l: 116 – 155 mm</p>
------------------------	--	--

The appropriate adapter ring ensures that the casting tube evenly encompasses and compresses the remaining limb. The a-p and m-l measurements include the use of optional products (such as liners, bedding elements).

Positioning the adapter ring (7)


<p>7</p>	<p>😊</p> <p>140</p> <p>×</p>
----------	------------------------------

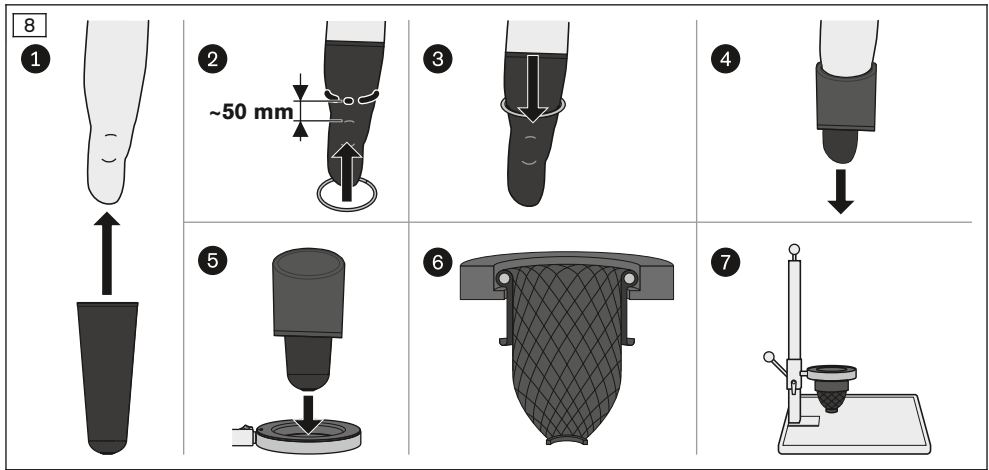Inserting the casting tube with retaining ring (8)

Due to the distance dimension (~50 mm) above the patella, sufficient space is available when taking impressions of the proximal area.

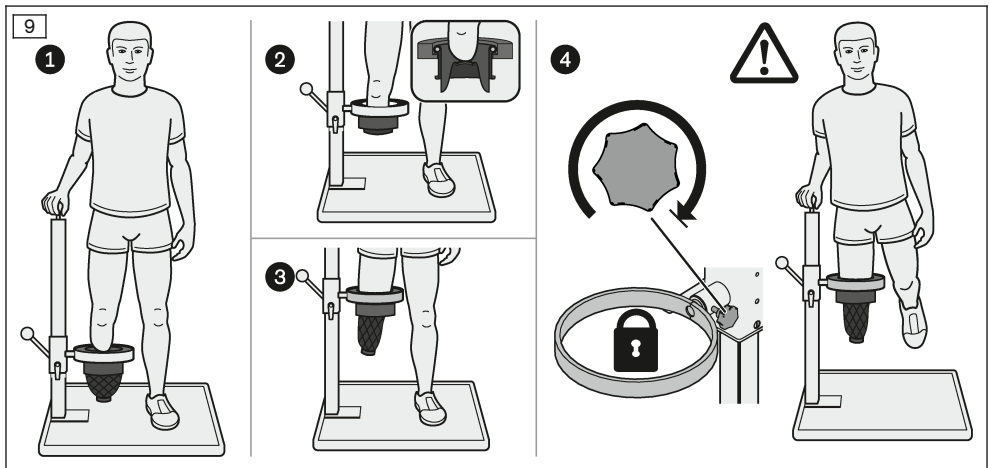
Optional: stepping into the casting tube on a trial basis (9)

By stepping in on a trial basis without plaster bandages, the correct height adjustment for the adapter ring can be determined and the length setting for the casting tube can be checked.


- ▶ ②: To achieve the best possible compression, press the distal end of the casting tube against the remaining limb during the entire process of stepping in.
- ▶ ④: **CAUTION!** Only with patients who are physically able, have the patient put all the weight on the remaining limb to check whether additional protection measures are needed (padding for example).

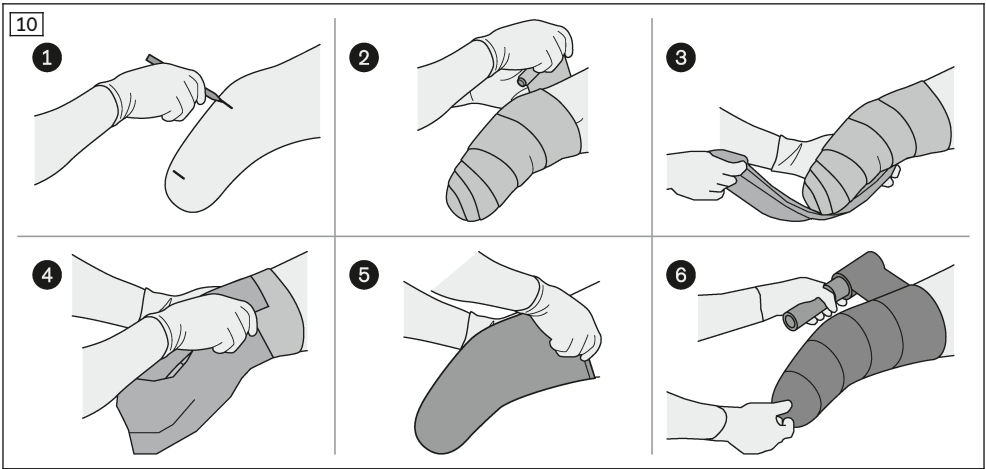Applying plaster bandages (10)


- ▶ **CAUTION!** Use an appropriate relief mechanism (such as a residual limb socket) on residual limbs that need to be distally protected against excessive compression.
- ▶ **NOTICE!** To protect against soiling and maintain the service life, protect the casting tube against contact with plaster (example: 5 pull on a tube sock, 6 wrap with stretch film).

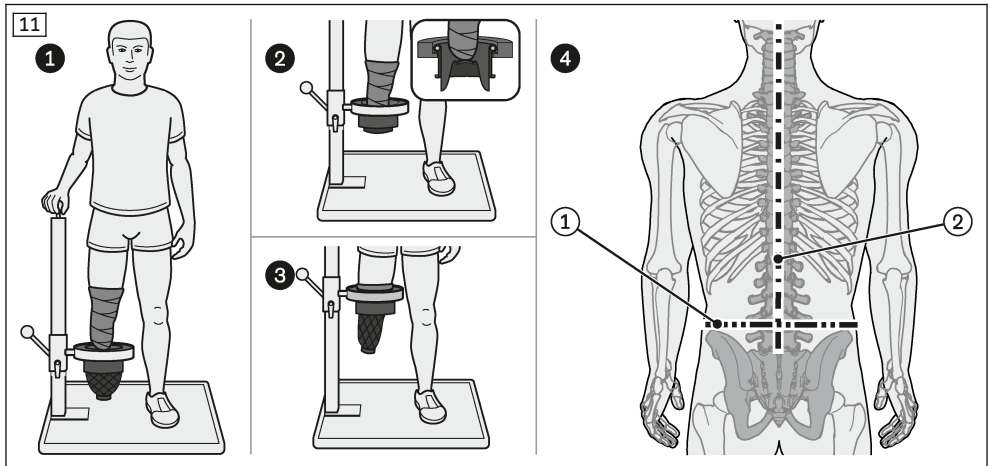Stepping into the casting tube (11)


- ▶ 2: To achieve the best possible compression, press the distal end of the casting tube against the remaining limb during the entire process of stepping in.
- ▶ 4: Check the correct position against the pelvis 1 and spine 2.

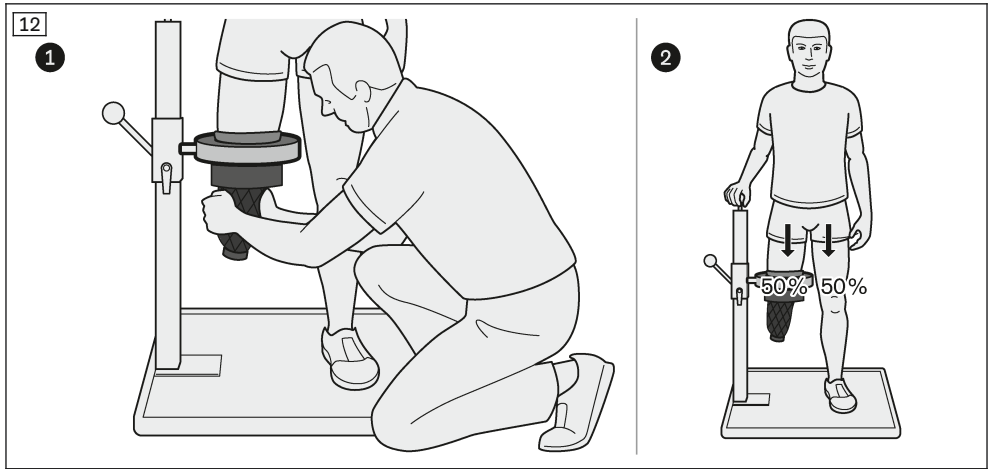Modelling and curing the plaster (12)

► 2: Ensure that the load is also even (50:50) during curing.

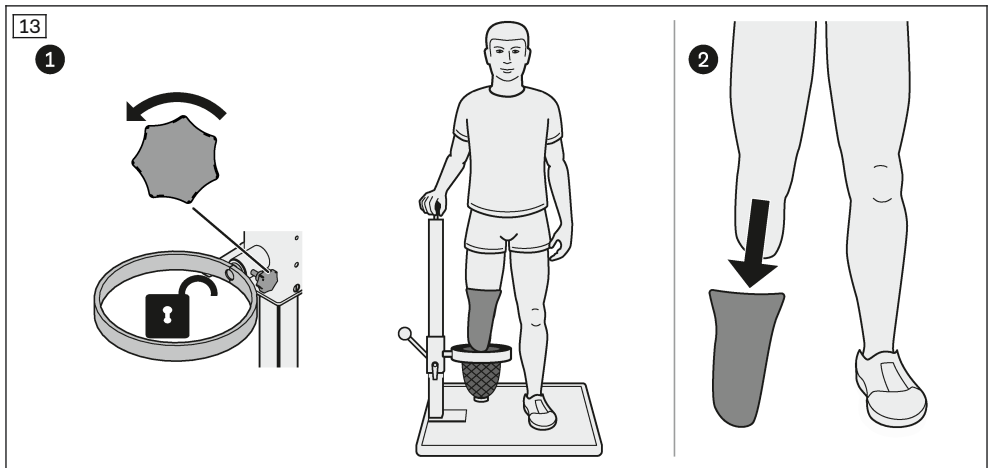Stepping out of the casting tube and pulling off the plaster cast (13)

► After pulling off the plaster cast, continue with prosthesis fabrication in the usual way.

6 Cleaning

CAUTION

Failure to follow the instructions for disinfection or cleaning

Health hazard due to contact with pathogens and damage to the product

- ▶ Observe all instructions in this document.

Cleaning the casting tube

- ▶ Rinse the casting tube thoroughly with water immediately after use to remove plaster residues.
- ▶ Then dry the metal ring with a clean cloth and allow the casting tube to air dry (do not use a heater or other heat sources!).

Cleaning light soiling

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

Cleaning heavier soiling

- > **Required materials:** cleaning cloth, 634A58 isopropyl alcohol, soft cloth
- ▶ **NOTICE!** Verify material compatibility.
 - Clean the product with a cleaning cloth and isopropyl alcohol.
- ▶ Dry the product with a cloth.

Cleaning with Disinfectants

- > **Required materials:** colourless, alcohol-free disinfectant (verify material compatibility!), soft cloth
- ▶ Disinfect the product with the disinfectant.
- ▶ Dry the product with a cloth.
- ▶ Allow to air dry in order to remove residual moisture.

7 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

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Otto Bock Equipment B.V.