
1B1-2=* Meridium

DE	Gebrauchsanweisung [kurz] (Fachpersonal)	3
EN	Instructions for use [short] (qualified personnel)	12

Herunterladen der Gebrauchsanweisung (Fachpersonal)
Download the Instructions for use (qualified personnel)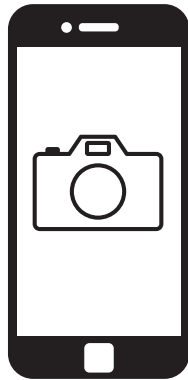

oder / or

http://new.transfer.ottobock.de/media/IFU/Meridium_1B1-2/647G1441-0=all_INT-05-2105.pdf

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INFORMATION

Date of last update: 2021-05-27

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

Note the instructions for use

This document contains the sections "Intended use" and "Safety", and does not replace the instructions for use. The instructions for use are available in digital form (PDF format) (see page 2 of this document). To view PDF files, you need an Internet browser or other program capable of opening this file format.

You may request a printed copy of the instructions for use under the link on page 2. This will be delivered to the specified address within 7 days after the order is received.

The product "1B1-2=* Meridium" is referred to as the product/component/prosthesis/prosthetic foot below.


Basic UDI-DI: 4064411000000001B1-2G4

2 Intended use

2.1 Indications for use

The product is to be used **solely** for lower limb exoprosthesis fittings.

2.2 Conditions of use

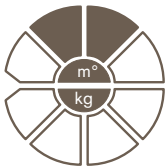The product is recommended for mobility grade 2 (restricted outdoor walker) and mobility grade 3 (unrestricted outdoor walker).

The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports with excessive impact load (tennis, basketball, running, ...) or extreme sports (free climbing, paragliding, etc.).

Permissible ambient conditions are described in the technical data (Technical data).

The product is intended **exclusively** for use on **one** patient. Use of the product by another person is not approved by the manufacturer.

Our components perform optimally when paired with appropriate components based upon weight and mobility grades identifiable by our MOBIS classification information and which have appropriate modular connectors.

Foot size [cm]	24 to 25	26 to 29
Max. body weight [kg]	100	125

2.3 Indications

- For users with unilateral knee disarticulation and unilateral transfemoral amputation
- For users with unilateral or bilateral transtibial amputation
- The user must fulfil the physical and mental requirements for perceiving optical/acoustic signals and/or mechanical vibrations

2.4 Contraindications

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

2.5 Qualification

The product may be fitted only by qualified personnel authorised by Ottobock after completing the corresponding training.

3 Safety

3.1 Explanation of warning symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

3.2 Structure of the safety instructions

<p> WARNING</p> <p>The heading describes the source and/or the type of hazard</p> <p>The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:</p> <ul style="list-style-type: none"> > E.g.: Consequence 1 in the event of failure to observe the hazard > E.g.: Consequence 2 in the event of failure to observe the hazard <p>▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.</p>
--

3.3 General safety instructions

<p> WARNING</p> <p>Non-observance of safety notices</p> <p>Personal injury/damage to the product due to using the product in certain situations.</p> <p>▶ Observe the safety notices and the stated precautions in this accompanying document.</p>

<p> WARNING</p> <p>Use of damaged power supply unit, adapter plug or battery charger</p> <p>Risk of electric shock due to contact with exposed, live components.</p> <ul style="list-style-type: none"> ▶ Do not open the power supply unit, adapter plug or battery charger. ▶ Do not expose the power supply unit, adapter plug or battery charger to extreme loading conditions. ▶ Immediately replace damaged power supply units, adapter plugs or battery chargers.
--

<p> CAUTION</p> <p>Failure to observe warnings/error signals</p> <p>Falling due to unexpected behaviour of the prosthesis because of changed damping behaviour.</p> <p>▶ Observe the warnings/error signals and corresponding change in damping settings.</p>
--

<p> CAUTION</p> <p>Independent manipulation of the product and the components</p> <p>Falling due to breakage of load-bearing components or malfunction of the product.</p> <ul style="list-style-type: none"> ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted. ▶ The battery may only be handled by authorised, qualified Ottobock personnel (no replacement by the user). ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.
--

⚠ CAUTION

Mechanical stress on the product

- > Falling due to unexpected product behaviour as the result of a malfunction.
- > Falling due to breakage of load-bearing components.
- > Skin irritation due to defects on the hydraulic unit with leakage of liquid.
- ▶ Do not subject the product to mechanical vibrations or impacts.
- ▶ Check the product for visible damage before each use.

⚠ CAUTION

Use of the product when battery charge level is too low

Falling due to unexpected behaviour of the prosthesis because of changed damping behaviour.

- ▶ Check the current charge level before use and charge the prosthesis if required.
- ▶ Note that the operating time of the product may be reduced at low ambient temperatures or due to ageing of the battery.

⚠ CAUTION

Using the product while switched off

Falling due to unexpected behaviour of the product because of changed damping behaviour.

- ▶ Before using the product, switch it on by connecting the power supply and battery charger.

⚠ CAUTION

Penetration of liquids into the product

Falling due to unexpected product behaviour as the result of a malfunction.

- ▶ When the footshell is intact, the product is protected against splashed water from all directions. However, it is not protected against submersion, jets of water and steam.
- ▶ If water has penetrated the product, remove the footshell and allow both components to dry. The product must be inspected by an authorised Ottobock Service Center.
- ▶ If salt water has penetrated the prosthesis, the footshell must be removed immediately. The product must be inspected by an authorised Ottobock Service Center.
- ▶ Do not use the product for bathing prostheses.

⚠ CAUTION

Overloading due to unusual activities

- > Falling due to unexpected product behaviour as the result of malfunction.
- > Falling due to breakage of load-bearing components.
- > Skin irritation due to defects on the hydraulic unit with leakage of liquid.
- ▶ The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports with excessive impact load (tennis, basketball, running, ...) or extreme sports (free climbing, paragliding, etc.).
- ▶ Careful handling of the product and its components not only increases their service life but, above all, ensures the patient's personal safety!
- ▶ If the product and its components have been subjected to extreme loads (e.g. due to a fall, etc.), then the product must be inspected for damage immediately. If necessary, forward the product to an authorised Ottobock Service Centre.

⚠ CAUTION

Mechanical stress during transport

- > Falling due to unexpected product behaviour as a result of a malfunction.
- > Falling due to breakage of load-bearing components.
- > Skin irritation due to defects on the hydraulic unit with leakage of liquid.
- ▶ Only use the transport packaging for transportation.

CAUTION**Signs of wear and tear on the product components**

Falling due to damage or malfunction of the product.

- ▶ Regular service inspections (maintenance) are mandatory in the interest of patient safety and in order to maintain operating reliability and protect the warranty.

CAUTION**Use of unapproved accessories**

> Falling due to product malfunction as a result of reduced interference resistance.

> Interference of other electronic devices due to increased emissions.

- ▶ Use the product only in combination with the accessories, signal converters and cables listed in the sections "Scope of delivery" (Scope of delivery) and "Accessories" (Accessories).

NOTICE**Improper product care**

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Only clean the product with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock DermaClean).

NOTICE**Mechanical damage to the product**

Change in or loss of functionality due to damage.

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

INFORMATION

When exoprosthetic components are used, the use of control functions which are performed hydraulically or movement of the component in the footshell can cause movement noise. This kind of noise is normal and unavoidable. It generally does not cause any problems. If movement noise increases noticeably during the lifecycle of the component, the component should be inspected by an authorised Ottobock Service Centre immediately.

Signs of changes in or loss of functionality during use

Decreased forefoot resistance or changes in roll-over behaviour are noticeable indications of loss of functionality.

INFORMATION**Cybersecurity**

- ▶ Keep your operating system up to date and always install any available security updates.
- ▶ Protect your computer from unauthorised access (e.g., by using virus scans, password protection etc.).
- ▶ Do not use unsecured networks.
- ▶ Please contact the manufacturer if you suspect cybersecurity problems.

INFORMATION

Before mounting the component on the tube adapter, make a note of the serial number of the component. It is found next to the pyramid on the spherical cap (Design).

The serial number is required to connect to the adjustment software and the Cockpit app and must be entered in the prosthesis passport.

INFORMATION

The charge level cannot be displayed during the charging process.

INFORMATION
 If the parameter is set to '0' in the Cockpit app, there are no beep signals (Changing prosthesis settings).

3.4 Information on the Power Supply/Battery Charging

CAUTION
Charging the prosthesis without taking it off
 > Falling due to walking and getting caught on the connected battery charger.
 > Falling due to unexpected behaviour of the prosthesis because of changed damping behaviour.
 ► Instruct the patient that the prosthesis must be taken off before it is charged.

CAUTION
Charging the product with damaged power supply unit/charger/charger cable
 Falling due to unexpected behaviour of the product caused by insufficient charging.
 ► Check the power supply unit, charger and charger cable for damage before use.
 ► Replace any damaged power supply unit, charger or charger cable.

NOTICE
Use of incorrect power supply unit/battery charger
 Damage to product due to incorrect voltage, current or polarity.
 ► Use only power supply units/battery chargers approved for this product by Ottobock (see instructions for use and catalogues).

3.5 Battery charger information

NOTICE
Penetration of dirt and humidity into the product
 Lack of proper charging functionality due to malfunction.
 ► Ensure that neither solid particles nor liquids can penetrate into the product.

NOTICE
Mechanical stress on the power supply/battery charger
 Lack of proper charging functionality due to malfunction.
 ► Do not subject the power supply/battery charger to mechanical vibrations or impacts.
 ► Check the power supply/battery charger for visible damage before each use.

NOTICE
Operating the power supply unit/charger outside of the permissible temperature range
 Lack of proper charging functionality due to malfunction.
 ► Only use the power supply unit/charger for charging within the allowable temperature range. The section "Technical data" contains information on the allowable temperature range (Technical data).

NOTICE
Independent changes or modifications carried out to the battery charger
 Lack of proper charging functionality due to malfunction.
 ► Have any changes or modifications carried out only by Ottobock authorised, qualified personnel.

3.6 Information on Alignment/Adjustment

CAUTION
Use of unsuitable prosthesis components
 Falling due to unexpected behaviour of the product or breakage of load-bearing components.
 ► Use the product only in combination with components listed in the section "Combination possibilities" (Combination possibilities).

⚠ CAUTION**Use of unapproved accessories**

- > Falling due to product malfunction as a result of reduced resistance to interference.
- > Interference of other electronic devices due to increased emissions.
- ▶ Use the product only in combination with the accessories, signal converters and cables listed in the sections "Combination possibilities" (Combination possibilities), "Scope of delivery" (Scope of delivery) and "Accessories" (Accessories).

⚠ CAUTION**Incorrect alignment or assembly**

Falling due to damage to the prosthesis components.

- ▶ Observe the alignment and assembly instructions.

⚠ CAUTION**Disconnecting/establishing the connection during the adjustment process with the adjustment software**

Falling due to unexpected behaviour of the product.

- ▶ When wearing the product, the patient must not remain unattended during the configuration process while connected to the adjustment software.
- ▶ Observe the maximum range of the Bluetooth connection.
- ▶ During the data transfer (PC to product), the patient must stand or sit without moving.
- ▶ If the connection fails while making adjustments, the prosthetist must immediately warn and secure the patient.
- ▶ The connection to the product must always be disconnected after adjustments have been completed.

⚠ CAUTION**Operator errors during the adjustment process with the adjustment software**

Falling due to unexpected behaviour of the product.

- ▶ Participation in an Ottobock product training course is mandatory prior to using the product. Additional product training courses may become necessary to qualify for software updates.
- ▶ Correctly entering the patient data (e.g. body weight) is an important criterion for the quality of the fitting.
- ▶ Always enter the weight in the specified unit.
- ▶ Ensure that the required dimensions are selected and entered correctly.
- ▶ If the patient uses walking aids (e.g. crutches or walking canes) during the adjustment process, readjustment is required as soon as the patient stops using these walking aids.

3.7 Information on Proximity to Certain Areas**⚠ CAUTION****Insufficient distance to HF communication devices (e.g. mobile phones, Bluetooth devices, WiFi devices)**

Falling due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Therefore, keeping a minimum distance of 30 cm to HF communication devices is recommended.

⚠ CAUTION**Operating the product in very close proximity to other electronic devices**

Falling due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Do not operate the product in the immediate vicinity of other electronic devices.
- ▶ Do not stack the product with other electronic devices during operation.
- ▶ If simultaneous operation cannot be avoided, monitor the product and verify proper use in the existing setup.

⚠ CAUTION

Proximity to sources of strong magnetic or electrical interference (e.g. theft prevention systems, metal detectors)

Falling due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Ensure that the patient is not in the vicinity of sources of strong magnetic and electrical interference during trial fitting (such as theft prevention systems, metal detectors...).
- If this cannot be avoided, ensure at least that the patient has a safeguard when walking or standing (e.g. a handrail or the support of another person).
- ▶ In general, monitor the product for unexpected changes in the damping behaviour when electronic or magnetic devices are in the immediate vicinity.

⚠ CAUTION

Entering a room or area with strong magnetic fields (e.g. magnetic resonance tomographs, MRT (MRI) equipment...)

- > Falling due to unexpected restriction of the product's range of motion caused by metallic objects adhering to the magnetised components.
- > Irreparable damage to the product due to the effect of strong magnetic fields.
- ▶ Make sure that the patient takes off the product before entering the room or area and stores the product outside this room or area.
- ▶ Damage to the product caused by exposure to strong magnetic fields cannot be repaired.

⚠ CAUTION

Remaining in areas outside the allowable temperature range

Falling due to malfunction or the breakage of load-bearing product components.

- ▶ Ensure that the patient is not in areas outside the permissible temperature range (Technical data) during trial fitting.

3.8 Information on Use

⚠ CAUTION

Walking up stairs

Falling due to foot placed incorrectly on stair.

- ▶ Ensure that the patient uses the handrail when walking up stairs and places most of the sole of the foot on the stair surface. The toe plate may fold away if only the front part of the foot is placed on the edge of the stair.
- ▶ Inform the patient that particular caution is required when carrying children while walking up stairs.

⚠ CAUTION

Walking down stairs

Falling due to foot being placed incorrectly on stair as a result of changed damping behaviour.

- ▶ Ensure that the patient uses the handrail when walking down stairs and places most of the sole of the foot on the stair surface. Rolling the foot over the edge of the stair is not necessary.
- ▶ Inform the patient that particular caution is required when carrying children while walking down stairs.

⚠ CAUTION

High temperature of the hydraulic unit due to uninterrupted, increased activity (e.g. extended walking downhill)

Falling due to unexpected behaviour of the product caused by use in overheating mode.

- ▶ Be sure to pay attention when pulsating vibration signals start. They indicate the risk of overheating.
- ▶ As soon as these pulsating vibration signals begin, you must reduce your level of activity so the hydraulic unit can cool down.
- ▶ Please note that the range of motion in the ankle joint is reduced as the temperature increases, and this may ultimately result in the ankle joint being locked completely. Special caution is therefore required, especially when walking down stairs.
- ▶ Full activity may be resumed after the pulsating vibration signals stop.

⚠ CAUTION**Improper mode switching**

Falling due to unexpected behaviour of the product because of changed damping behaviour.

- ▶ Ensure that the patient stands securely during all switching processes.
- ▶ Inform the patient that the changed damping characteristics have to be verified after switching and feedback from the acoustic signal emitter must be observed.
- ▶ Switching back to basic mode is mandatory once the activities in MyMode have been completed.
- ▶ If required, take the weight off the product and correct the switching.

⚠ CAUTION**Incorrect assembly/disassembly and improper use of footshell**

> Falling due to unexpected product behaviour as the result of malfunction.

> Falling due to breakage of load-bearing components.

- ▶ Use the prosthetic foot only in combination with the footshell.
- ▶ Remove the footshell from the prosthetic foot only when necessary.
- ▶ Only use the 2C101 changing device and the 704G30 clamping device for mounting/removing.
- ▶ Replace worn footshells. Do not use the prosthetic foot when the footshell is damaged.

⚠ CAUTION**Using prosthetic foot without footshell**

Falling due to slipping when walking on smooth floor surfaces (tiles).

- ▶ Do not use the prosthetic foot without the accompanying footshell.

⚠ CAUTION**Using prosthetic foot with a damaged foot shell**

> Falling due to unexpected product behaviour as the result of malfunction.

> Falling due to breakage of load-bearing components.

- ▶ Do not use the prosthetic foot when the foot shell is damaged. Replace damaged foot shells immediately, before using the product again.

3.9 Notes on the safety modes**⚠ CAUTION****Using the product in safety mode**

Falling due to unexpected product behaviour because of changed damping behaviour.

- ▶ The warnings/error signals (Warnings/error signals) have to be observed.

⚠ CAUTION**Safety mode cannot be activated due to malfunction caused by water penetration or mechanical damage**

Falling due to unexpected product behaviour because of changed damping behaviour.

- ▶ Using the product when it is defective is prohibited.
- ▶ The product must be inspected by an authorised Ottobock Service Centre.

⚠ CAUTION**Safety mode cannot be deactivated**

Falling due to unexpected product behaviour because of changed damping behaviour.

- ▶ If safety mode cannot be deactivated by recharging the battery, a permanent error has occurred.
- ▶ Using the product when it is defective is prohibited.
- ▶ The product must be inspected by an authorised Ottobock Service Centre.

⚠ CAUTION

Safety signal occurs (ongoing vibration)

Falling due to unexpected product behaviour because of changed damping behaviour.

- ▶ The warnings/error signals (Warnings/error signals) have to be observed.
- ▶ After the safety signal has been emitted, further use of the product is prohibited.
- ▶ The product must be inspected by an authorised Ottobock Service Centre.

3.10 Information on the use of a mobile device with the cockpit app

⚠ CAUTION

Improper use of the mobile device

Falling due to altered damping behaviour as a result of unexpected switching into a MyMode.

- ▶ Use the instructions for use (user) to instruct the patient on the proper handling of the mobile device with the Cockpit app.

⚠ CAUTION

Independently applied changes or modifications made to the mobile device

Falling due to altered damping behaviour as a result of unexpected switching to a MyMode.

- ▶ Do not make any independent changes to the hardware of the mobile device on which the app is installed.
- ▶ Do not make any independent changes to the software/firmware of the mobile device that are not included in the update function of the software/firmware.

⚠ CAUTION

Improper mode switching with the mobile device

Falling due to unexpected product behaviour because of changed damping behaviour.

- ▶ Ensure that the patient stands securely during all switching processes.
- ▶ Inform the patient that the changed damping characteristics have to be verified after switching, and feedback from the acoustic signal emitter and the mobile device display must be observed.
- ▶ Switching back to basic mode is mandatory once the activities in the MyMode have been completed.

NOTICE

Failure to observe the system requirements for the installation of the Cockpit app

Mobile device malfunction.

- ▶ The Cockpit App should only be installed on mobile devices and versions which comply with the specifications in the respective online stores (e.g. Apple App Store, Google Play Store, ...)

A series of horizontal lines for writing, spanning the width of the page.

A series of horizontal lines for writing.

Otto Bock Healthcare Products GmbH
Brehmstraße 16 · 1110 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com