HR2233=0000_K

DE Gebrauchsanweisung	3
EN Instructions for use	17
IT Istruzioni per l'uso	31
ES Instrucciones de uso	45
PL Instrukcja użytkowania	61
SK Návod na používanie	75
RU Руководство по применению	89

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DE

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Table of contents

EN

1	Foreword	19
2	Product description	19
2.1	Function	19
3	Intended use	19
3.1	Indications for use	19
3.2	Indications	19
3.3	Contraindications	20
3.3.1	Absolute Contraindications	20
3.3.2	Relative Contraindications	20
3.4	Qualification	20
4	Safety	20
4.1	Explanation of warning symbols	20
4.2	Safety instructions for assembly and adjustments	20
4.3	Safety instructions for use	20
4.4	Side effects	22
4.5	Further information	22
4.6	Nameplate and warning labels	22
4.6.1	Nameplate	22
4.6.2	Warning labels	23
5	Delivery	23
5.1	Scope of delivery	23
5.2	Storage	23
5.2.1	Storage during daily use	23
5.2.2	Storage during extended disuse	23
6	Preparing the product for use	23
6.1	Assembly	23
7	Delivery	24
7.1	Transport to the customer	24
7.2	Handing over the product	24
8	Use	25
8.1	Instructions for use	25
8.2	Basic functions	25
8.2.1	Attaching/detaching the seating unit	25
8.2.2	Adjusting the seat height	25
8.2.3	Adjusting the seat inclination	26
8.2.4	Adjusting the anti-tipper	26
8.2.5	Engaging the wheel lock	27
8.3	Additional options	27
8.3.1	Push handle	27
8.4	Care	28
8.4.1	Cleaning	28
8.4.2	Disinfection	28
9	Maintenance and repair	28
9.1	Maintenance	28
9.1.1	Maintenance intervals	28
9.2	Repair	29
10	Disposal	29
10.1	Disposal information	29
10.2	Information on re-use	29
11	Legal information	29
11.1	Liability	29
11.2	Warranty	29
11.3	Lifetime	29

12 Technical data29

1 Foreword

INFORMATION

Date of last update: 2021-07-06

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

INFORMATION

- ▶ New information regarding product safety and product recalls as well as the declaration of conformity can be obtained at ccc@ottobock.com or from the manufacturer's service department (see inside or outside of back cover for addresses).
- ▶ You can request this document as a PDF file at ccc@ottobock.com or from the manufacturer's service department (see inside or outside of back cover for addresses). The PDF file can also be displayed in a larger size.

You have purchased a medical device for indoor use.

In order to exclude injuries of any type, familiarise yourself with the handling, functions and intended use of the product before using it. These instructions for use provide you with the related necessary information.

Please note the following in particular:

- All users and/or their attendants must be trained by qualified personnel in the use of the product. In particular, users and/or attendants must be informed of the residual risks with the aid of the safety notices in the instructions for use (user).
- The product was adapted to the needs of the user. Subsequent changes may be made only by qualified personnel. We recommend checking the product settings **once per year** to ensure optimal treatment over the long term. Especially for users with a changing anatomy (for example body dimensions, weight), an adjustment at least **once every six months** is recommended.
- Your product may differ from the models shown. In particular, not all the options described in these instructions for use will be installed on your product.
- The manufacturer reserves the right to make technical changes to the model described in these instructions for use.

2 Product description

2.1 Function

The product is intended exclusively for holding seating systems to transport one child on the seat.

The product can be used indoors on solid surfaces.

3 Intended use

The safe use of the product can only be ensured in case of intended use in accordance with the information contained in these instructions for use. The user is ultimately responsible for accident-free operation.

3.1 Indications for use

The product is intended for everyday indoor use, by an attendant, for children with temporary or permanent limitations of the ability to walk, inability to walk or difficulty standing up.

The product is suitable for users with intact skin whose anatomy (such as body dimensions and weight) permits the intended use of the product. It is operated by an attendant.

The product may only be used with the options that are listed on the product order form. Ottobock assumes no liability for combinations with medical devices and/or accessories from other manufacturers outside the modular system.

Combinations based on a combination agreement that have been evaluated for effectiveness and safety are an exception to this.

3.2 Indications

- Minor to pronounced or complete restrictions of mobility

3.3 Contraindications

3.3.1 Absolute Contraindications

- None known

3.3.2 Relative Contraindications

- Failure to meet physical or mental requirements

3.4 Qualification

Installation and adjustment tasks may only be carried out by qualified personnel. Compliance with all manufacturer specifications and all applicable legal provisions is required. Please contact the manufacturer's service department for further information (see inside or outside of rear cover for addresses).

4 Safety

4.1 Explanation of warning symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

4.2 Safety instructions for assembly and adjustments

 WARNING Making incorrect adjustments Risk of falling, tipping over or malposition of the user due to incorrect settings ▶ Settings may only be changed by qualified personnel unless otherwise prescribed. ▶ Before testing setting changes with the user seated, all screw connections must be firmly tightened. ▶ Check the product for safety and functionality after changing the settings.
 WARNING Use of alternative seating systems from other manufacturers Serious injuries to the user due to improperly dimensioned seating shells ▶ If the product is fitted with an alternative seating system or seating shells from other manufacturers, make sure that the seating shell is the right size. ▶ It is the duty of the qualified personnel to ensure the static stability of a combination of this kind. ▶ Maintain the maximum load (see page 29).
 CAUTION Unsecured screw connections Pinching, crushing, tipping over, falling of user due to assembly errors ▶ After all adjusting/readjusting work, retighten the attachment screws/nuts firmly. ▶ Please note that all adjustment levers are tightened by hand without the help of tools.

4.3 Safety instructions for use

Hazards during preparation for use

 WARNING Independent modification of settings Serious injuries to the user due to improper changes to the product ▶ Do not modify the settings established by the qualified personnel. Only the settings described in the section "Use" in these instructions for use may be adjusted independently. ▶ In case of problems with the settings, please contact the qualified personnel who adjusted your product.

⚠ WARNING**Improper handling of packaging materials**

Risk of suffocation due to neglect of the duty to supervise

- Packaging materials must be kept out of the reach of children.

Hazards while driving and parking**⚠ CAUTION****Improper use of the product**

Falling of the user due to neglect of the duty to supervise

- The product is a therapy aid and may only be used under supervision. Never leave your child unattended.
- Pay particular attention to potentially hazardous situations, e.g. while setting up and positioning the product.
- In the interest of the user's safety, the correct function of the therapy aid must be checked before every use.

⚠ CAUTION**Improper use**

User falling, tipping due to user error

- The user must not lean too far out of the seat when reaching for objects. There is also a potential risk of tipping when the user makes sudden movements.
- The product may be used only on solid, level surfaces indoors.
- The product may be parked only with the wheel locks engaged.
- Driving on steep slopes is not permitted. Do not park the product on steep inclines.
- Always provide support to the rear when briefly negotiating ramps.
- Pay particular attention to potentially hazardous situations, e.g. overcoming steps.

⚠ CAUTION**Hanging heavy bags on the product may change the centre of gravity**

Tipping over, rolling over of the user due to user error

- Do not hang heavy bags or the like on the therapy device.
- Observe the maximum load for each model (see section "Technical data").

Hazards due to fire, heat and cold**⚠ CAUTION****Extreme temperatures**

Hypothermia or burns due to contact with components, failure of components

- Do not expose the product to any extreme temperatures (e.g. direct sunlight, sauna, extreme cold).
- Do not leave the product in the immediate vicinity of heaters.

Hazards due to improper use of the product**⚠ WARNING****Overloading**

Severe injuries if the product tips over due to overloading, damage to the product

- Do not exceed the maximum load capacity (see the nameplate and section "Technical data").
- Please note that certain accessories and add-on components will reduce the remaining load capacity.

⚠ WARNING**Exceeding the service life**

Serious injuries due to failure to observe the manufacturer's requirements

- Using the product beyond the specified expected service life leads to increased residual risk.
- Observe the specified service life.

⚠ CAUTION**Uncontrolled driving behaviour, unexpected sounds or odours**

Falling, tipping, collision with persons or nearby objects due to defects

- ▶ If any faults, defects or other hazards that can lead to personal injury are detected, the product must be taken out of service immediately. This includes uncontrolled movements as well as sounds that are unexpected or previously not noted or odours that deviate significantly from the state of the product at the time of delivery.
- ▶ Contact the qualified personnel.

⚠ CAUTION**Lack of maintenance**

User falling or tipping over due to lack of maintenance

- ▶ Check that the product is working correctly before each use.
- ▶ Observe the specified maintenance intervals (see page 28, "Maintenance" section).

4.4 Side effects

The following side effects may occur during use of the product:

- Neck, muscle and joint pain
- Circulatory disorders, risk of pressure sores

Contact a doctor or therapist in case of problems.

4.5 Further information**INFORMATION**

Please note that all options and add-on components will reduce the remaining load capacity.

INFORMATION

The serial number required for enquiries and ordering spare parts and accessories is found on the nameplate. For explanations of the nameplate, see the section "Nameplate" (see page 22).

4.6 Nameplate and warning labels**4.6.1 Nameplate**

Label	Meaning
	A Manufacturer's product name
	B CE marking
	C Maximum load (see section "Technical data")
	D Manufacturer information/address
	E Serial number ¹⁾
	F Manufacturing date ²⁾
	G Symbol for medical device
	H WARNING! Read the instructions for use before using the product. Observe important safety-related information (e.g. warnings, precautions).
	I Manufacturer's reference number for the product variant
	J Serial number (PI) ^{3),1)}
	K Global Trade Item Number (DI) ⁴⁾



¹⁾ YYYY = year of manufacture; WW = week of manufacture; PP = production site; XXXX = sequential production number

²⁾ YYYY = year of manufacture; MM = month of manufacture; DD = day of manufacture

³⁾ UDI-PI to GS1 standard; UDI = Unique Device Identifier, PI = Product Identifier

⁴⁾ UDI-DI to GS1 standard; UDI = Unique Device Identifier, DI = Device Identifier

4.6.2 Warning labels

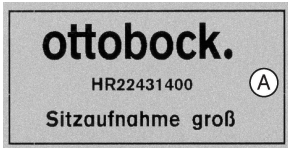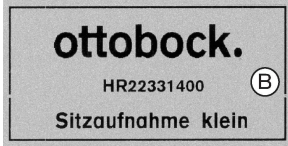Label	Meaning
	A Base frame with large seat adapter
	B Base frame with small seat adapter

5 Delivery

5.1 Scope of delivery

- Chassis
- Slide-in module with tilt adjustment mechanism
- Instructions for use and required tools
- Push handle (optional accessory)

5.2 Storage

5.2.1 Storage during daily use

Store the product in a dry, enclosed room with sufficient air circulation and protection from external influences.

5.2.2 Storage during extended disuse

The product must be stored in a dry place. Specific information for extended storage: see page 29.

It is not necessary to disassemble the product.

Maintain sufficient clearance from sources of heat.

Direct exposure to sunlight or UV radiation causes materials to harden and the product to age more quickly. When storing the product, do not expose it to direct sunlight or UV light.

6 Preparing the product for use

6.1 Assembly

⚠ CAUTION

Exposed pinch points

Crushing, pinching due to incorrect handling

- Only grip the specified components during assembly.



ℹ INFORMATION

- This section describes only the simple assembly of the indoor mobility base. Please note the specifications regarding the seat height adjustment (see page 25), setting the seat inclination (see page 26) and setting the anti-tipper (see page 26). Carry out these settings if needed at the same time as the assembly.
- To install the push handle option: see page 27.

Installing the base frame

- 1) Remove the transport locks and packaging material.
- 2) Loosen and remove the screws on the base frame (see fig. 1, item 1).
- 3) Slide the adapter into the base frame and use the screws to attach it to the base frame (see fig. 2, item 1; see fig. 1, item 1).


INFORMATION: In doing so, please note that the bore hole for the tripod spring of the tilt adjustment mechanism is in the left tube of the adapter (see fig. 2, item 2) with respect to the direction of travel (see fig. 2, item 3).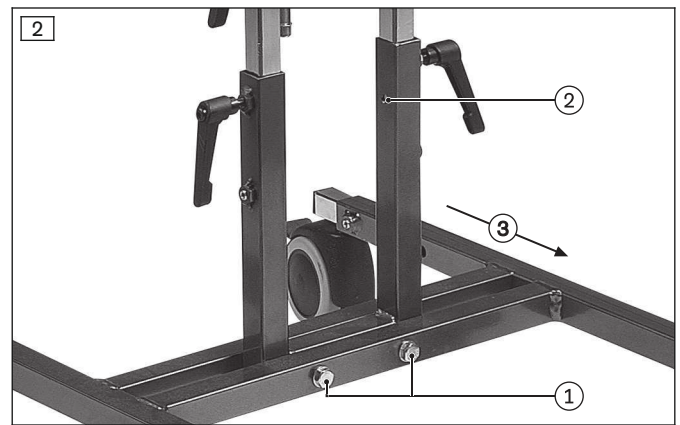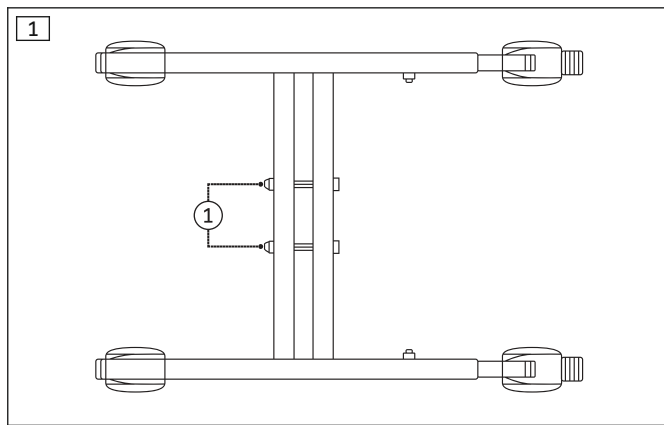

Installing the tilt adjustment mechanism

- While the tripod spring is depressed, slide the slide-in module into the adapter until the tripod spring engages in the hole (see fig. 3, item 1).

As you do so, pay attention to the alignment of the tilt adjustment mechanism: Align the tilt adjustment mechanism towards the rear with respect to the direction of travel (see fig. 4, item 1).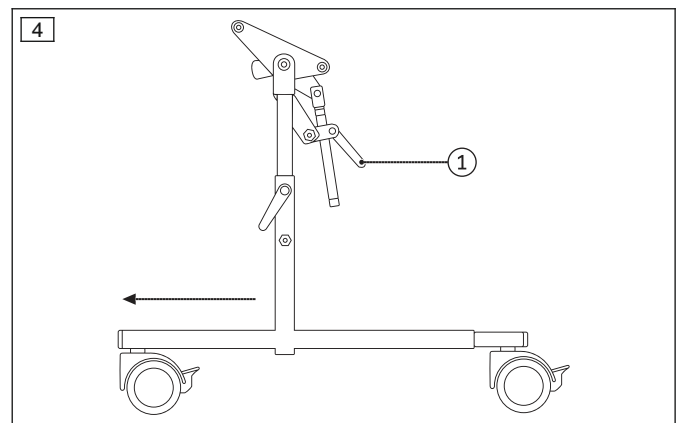

7 Delivery

7.1 Transport to the customer

NOTICE

Use of unsuitable packaging

Damage to the product caused by transportation using incorrect packaging

- Use only the original packaging for delivery of the product.

7.2 Handing over the product

⚠ WARNING

Lack of instruction

Tipping over, falling of the user due to lack of knowledge

- Instruct the user or the attendant in the proper use of the product when handing it over.

The following steps must be performed for the safe delivery of the product:

- The existing seating unit has to be adapted to the indoor mobility base for test purposes. In doing so, pay special attention to the firm attachment of the seating unit.
- Instruct the user and attendants in the safe use of the product. In particular, the enclosed instructions for use are to be used.
- The instructions for use must be given to the attendant upon delivery of the product.

8 Use

8.1 Instructions for use

- Attaching loads (e.g. backpacks) can adversely affect stability. Therefore, suspending additional loads on the product is not permitted.

8.2 Basic functions

8.2.1 Attaching/detaching the seating unit

CAUTION

Failure to secure the seating unit when attaching it

User falling, tipping or falling out of the seating unit due to it not being secured or parts coming loose

- ▶ Before attaching and removing the seating unit and before the user gets in or out, always engage the wheel lock on the outdoor mobility base/seating shell mobility base. Check the braking function.
- ▶ When attaching the seating unit, always make sure it engages audibly in the outdoor mobility base/seating shell mobility base.
- ▶ Secure the seating unit using the seat lock.

INFORMATION

- ▶ A brief general description of removing and mounting the seating unit is provided below.
 - **For a device with a Kimba seating unit:** Please refer to the instructions for use for the Kimba seating unit for further information.
 - **For a device with a custom seating shell or another seating system:** Please contact the qualified personnel who adapted, selected or fabricated the seating unit for further information.

The seating unit can be easily removed from the indoor mobility base for transferring the seating unit to a rehab buggy. This is particularly important if the seating unit is used as an indoor device on an indoor mobility base and as an outdoor device on an outdoor mobility base and therefore needs to be easily transferable.

Removing the seating unit (not illustrated)

- 1) Stand next to the seating unit and hold the back support with one hand.
- 2) Pull the handle of the release lever located under the seat towards the leg support. The retaining claw will now release the seat adapter.
- 3) Tilt the seat back by about **45°** and then lift it out of the unit.

Mounting the seating unit (not illustrated)

- 1) Hold the seat by the back support and under the seat surface.
- 2) First place the locking unit under the seat surface onto the rear tube of the seat adapter at an angle of approx. **45°**.
- 3) Press the front edge of the seat onto the mobility base until the retaining claw audibly engages on the front tube of the seat adapter.
- 4) Give the seating unit a few short tugs to make sure it is secure.

8.2.2 Adjusting the seat height

CAUTION



Unintentional loosening of the seat height adjustment

Falling of the user due to incorrect adjustment

- ▶ Tighten the clamping levers after carrying out any adjustment of the seat.

- 1) Loosen the clamping levers (see fig. 5, item 1).
- 2) Loosen the set screws (see fig. 5, item 2).
- 3) Move the seat adapter to the desired height (height adjustment is restricted when the tripod spring engages, see section "Assembly": see page 23).
- 4) Close the clamping levers tightly. Tighten the set screws again.

INFORMATION: The clamping levers can be moved freely. Pull the clamping levers away from the set screws slightly and turn them to a more convenient operating position.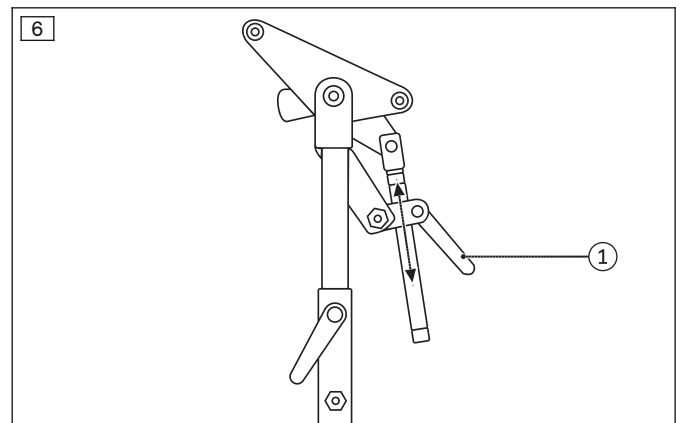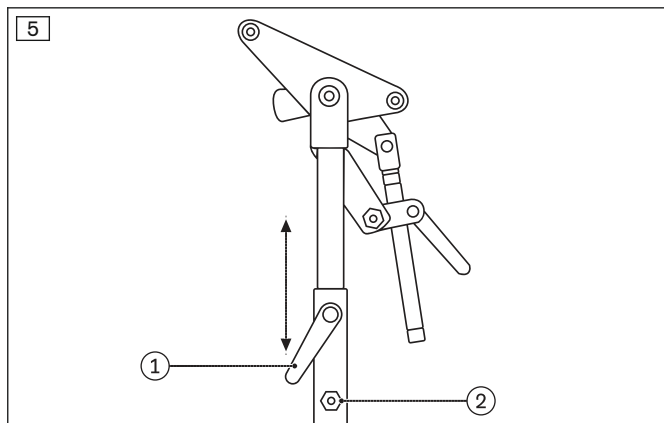

8.2.3 Adjusting the seat inclination

⚠ CAUTION

Independent adjustment of the seat inclination

Falling of the user due to incorrect adjustment

► Tighten the clamping levers after carrying out any adjustment of the inclination.

- 1) Loosen the clamping levers (see fig. 6, item 1).
- 2) Adjust the angle of the seat adapter.
- 3) Retighten the clamping levers.

CAUTION! The base frame must be lowered due to the increased risk of tipping when adjusting the seat inclination. Observe the specifications in the following section.

8.2.4 Adjusting the anti-tipper



⚠ CAUTION

Unstable base frame

Tipping over, falling due to incorrectly mounted anti-tipper

- The base frame must be lowered due to the increased risk of tipping when adjusting the seat inclination.
- Firmly retighten the set screws after any adjustment of the anti-tipper (see fig. 8, item 1).
- In case of extended use without adjustment, check that the set screws are firmly secured. Retighten them if necessary.

- 1) Loosen the set screws (see fig. 7, item 1; see fig. 8, item 1).
- 2) Pull the telescoping tubes on the wheels to a maximum of when the tripod springs engage (see fig. 7, item 2; see fig. 8, item 2).
- 3) Tighten the set screws again.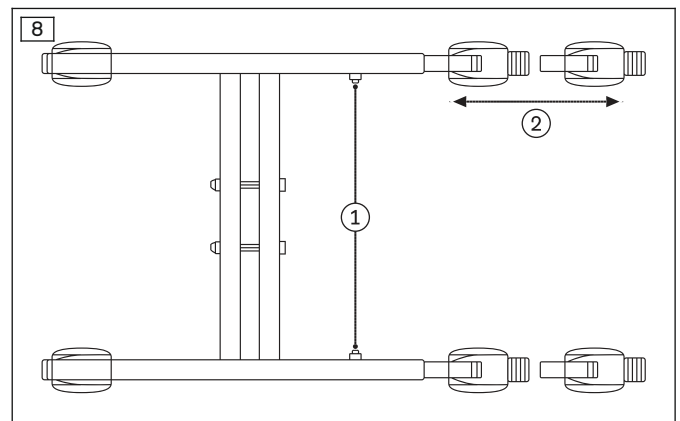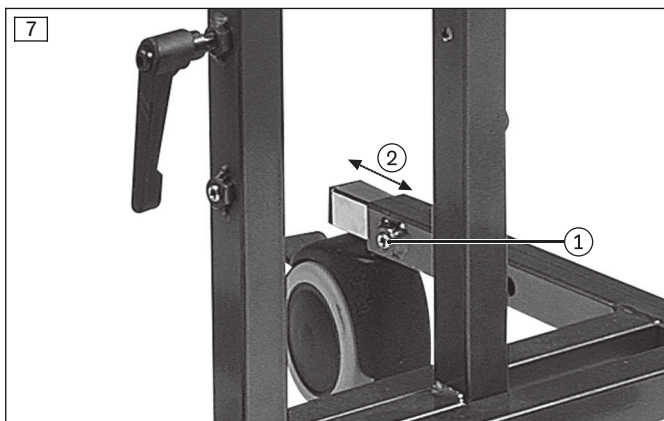

8.2.5 Engaging the wheel lock


CAUTION

Wheel lock not engaged

Falling, tipping over of the user due to rolling away

- Before getting into the mobility base or out of it, always engage the wheel lock.
- Verify the secure function of the wheel lock from time to time.

- 1) **Engage the wheel locks:** Press the brake levers down with the tip of your foot (see fig. 9, item 1).
- 2) **Release the wheel locks:** Pull the brake levers up with the tip of your foot.

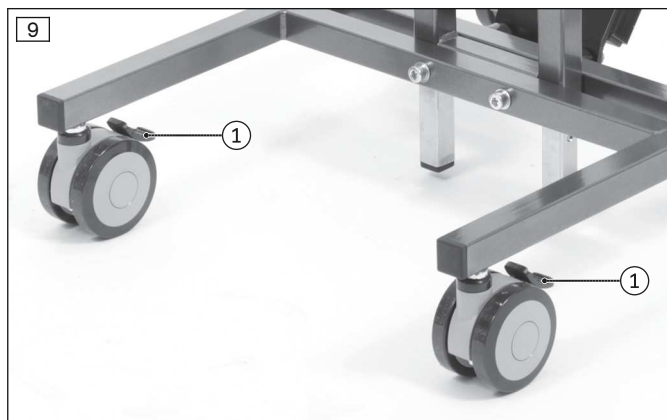8.3 Additional options

8.3.1 Push handle

- 1) Loosen the screws on the basic frame and replace them with the supplied long screws (see fig. 11, item 1).
- 2) Mount the adapter plate for the push handle on the side of the basic frame on which the telescoping tubes of the wheels are installed (see fig. 11, item 2).
- 3) Firmly tighten the long screws (see fig. 11, item 1).
- 4) Insert the push handle into the adapter plate and let it engage (see fig. 10).

INFORMATION: The push handle can be removed at any time if required. To do so, pull the catch behind the adapter plate towards you and lift the push handle up and out of the adapter plate (see fig. 11, item 3).

8.4 Care

CAUTION

Lack of or improper cleaning

Health hazard due to infections, damage to the product due to user error

- ▶ Clean the product at regular intervals.
- ▶ Do not clean the product with a jet of water or a pressure washer. The penetration of water can cause corrosion and subsequent malfunctions.
- ▶ Check the driving behaviour of the product after cleaning it.

8.4.1 Cleaning

Clean the product regularly depending on the degree of soiling and frequency of use, **at least 1x per month**:

Important information on cleaning

- Use warm soapy water for cleaning. Add some disinfectant if needed. Carefully dab the product with a damp cloth or sponge and wipe with a dry, clean cloth.
- Do not use any aggressive cleaners, solvents or hard brushes etc.
- Do not expose the product to direct heat (e.g. sunlight, stove or radiator heat).

8.4.2 Disinfection

- Thoroughly clean the product before disinfecting.
- Wipe all parts of the product with a disinfectant.
- Only use colourless water-based disinfectants. Observe the instructions for use provided by the manufacturer.

9 Maintenance and repair

9.1 Maintenance

WARNING

Insufficient maintenance

Severe user injuries, damage to the product due to failure to observe maintenance intervals

- ▶ Only carry out the maintenance tasks described in this section. All other maintenance and service tasks may only be carried out by qualified personnel.
- ▶ The functionality and operating safety of the product must be verified and a service performed at least **once per year**.
- ▶ For users with a changing anatomy (for example body dimensions, weight) or users with a changing clinical picture, have the product inspected, adjusted and serviced at least **once every six months**.

- The function of the product should be checked **before each use**.
- The product may not be used if defects are noted. This applies in particular in case of instability of the product or altered driving characteristics as well as problems with the user's seating position or the stability of the seat. Inform the qualified personnel promptly for the rectification of defects.
- This also applies if loose, worn, bent or damaged components, cracks in the frame or broken frame components are identified.
- Some maintenance tasks can be carried out to a certain extent by the user at home (see the section "Maintenance intervals").
- Failure to maintain the product can lead to injuries for the user of the product.

9.1.1 Maintenance intervals

The functions described below must be checked by the user or an attendant at the specified intervals:

To be checked before use	Daily	Weekly	Monthly
Functional test of the seat adapter and the anti-tipper	X		
Functional test of the wheel lock		X	
Check screw connections		X	
Visual inspection of wear and tear parts, e.g. wheels			X
Check the legibility of all labels and markings on the product			X

9.2 Repair

WARNING

Prohibited repairs

Severe user injuries, damage to the product due to adjustment and installation errors

- All repairs may only be carried out by qualified personnel.

10 Disposal

10.1 Disposal information

Return the product to the qualified personnel for disposal.

All components of the product must be disposed of properly in accordance with the respective national environmental regulations.

10.2 Information on re-use

The product is suitable for re-use.

Similar to second-hand machines or vehicles, products that are being re-used are subject to increased strain. Features and performance must not change in a way that could impair the safety of users or third parties during the period of use.

The relevant product must be thoroughly cleaned and disinfected before re-use. Then have the product inspected by qualified personnel with respect to its condition, wear and tear, and damage. Worn and damaged parts as well as components that do not fit or are unsuitable for the user must be replaced.

Detailed information on replacing components as well as information on the required tools can be found in the service manual.

11 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

11.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

11.2 Warranty

Further information on the warranty terms and conditions can be obtained from the qualified personnel that has fitted this product or the manufacturer's service (see inside back cover for addresses).

11.3 Lifetime

Expected lifetime: **4 years**

The design, manufacturing and requirements for the intended use of the product are based on the expected lifetime. These also include the requirements for maintenance, ensuring effectiveness and the safety of the product.

12 Technical data

INFORMATION

- Much of the technical data below is given in mm. Please note that product settings – unless otherwise specified – cannot be adjusted in the mm range but only in increments of approx. **0.5 cm** or **1 cm**.
- Note that the values achieved during adjustment may deviate from the values specified below. The deviation can be **±10 mm** and **±2°**.

Dimensions and weights	Model 2231 / 2241	Model 2233 / 2243
Length [mm]	620–780	620–780
Width [mm]	450	450
Height to upper edge of seat adapter* [mm]	380–580	410–620
Seat adapter tilt [°]	-10 to +35	-10 to +35
Max. load [kg]	50	50

Dimensions and weights	Model 2231 / 2241	Model 2233 / 2243
Weight [kg]	6.4 / 6.6	6.4 / 6.6

* for 125 mm wheel diameter: +70 mm

Ambient conditions

Temperatures and relative humidity	
Temperature during use [°C (°F)]	-10 to +40 (14 to 104)
Transport and storage temperature [°C (°F)]	-10 to +40 (14 to 104)
Relative humidity [%]	10 to 85; non-condensing

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
A series of horizontal lines for writing, spanning the width of the page.

Blank lined area for writing or editing.

Kundenservice/Customer Service

Europe

Otto Bock HealthCare Deutschland GmbH
Max-Näder-Str. 15 · 37115 Duderstadt · Germany
T +49 5527 848-3433 · F +49 5527 848-1460
healthcare@ottobock.de · www.ottobock.de

Otto Bock Healthcare Products GmbH
Brehmstraße 16 · 1110 Wien · Austria
F +43 1 5267985
service-admin.vienna@ottobock.com · www.ottobock.at

Otto Bock Adria d.o.o. Sarajevo
Ramiza Salčina 85
71000 Sarajevo · Bosnia-Herzegovina
T +387 33 255-405 · F +387 33 255-401
obadria@bih.net.ba · www.ottobockadria.com.ba

Otto Bock Bulgaria Ltd.
41 Tzar Boris III' Blvd. · 1612 Sofia · Bulgaria
T +359 2 80 57 980 · F +359 2 80 57 982
info@ottobock.bg · www.ottobock.bg

Otto Bock Suisse AG
Luzerner Kantonsspital 10 · 6000 Luzern 16 · Suisse
T +41 41 455 61 71 · F +41 41 455 61 70
suisse@ottobock.com · www.ottobock.ch

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Otto Bock South Africa (Pty) Ltd
Building 3 Thornhill Office Park · 94 Bekker Road
Midrand · Johannesburg · South Africa
T +27 11 564 9360
info-southafrica@ottobock.co.za
www.ottobock.co.za

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Otto Bock HealthCare Canada
5470 Harvester Road
Burlington, Ontario, L7L 5N5, Canada
T +1 800 665 3327 · F +1 800 463 3659
CACustomerService@ottobock.com
www.ottobock.ca

Sucursal Otto Bock Cuba
Centro de Negocios Miramar
Edificio Jerusalem Oficina 112
Calle 3ra e/ 78 y 80.
Playa La Habana, Cuba
T +53 720 430 69 · +53 720 430 81
hector.corcho@ottobock.com.br
www.ottobock.com.br

Otto Bock HealthCare Andina Ltda.
Calle 138 No 53-38 · Bogotá · Colombia
T +57 1 8619988 · F +57 1 8619977
info@ottobock.com.co · www.ottobock.com.co

Otto Bock de Mexico S.A. de C.V.
Prolongación Calle 18 No. 178-A
Col. San Pedro de los Pinos
C.P. 01180 México, D.F. · Mexico
T +52 55 5575 0290 · F +52 55 5575 0234
info@ottobock.com.mx · www.ottobock.com.mx

Otto Bock HealthCare LP
11501 Alterra Parkway Suite 600
Austin, TX 78758 · USA
T +1 800 328 4058 · F +1 800 962 2549
USCustomerService@ottobock.com
www.ottobockus.com

Asia/Pacific

Otto Bock Australia Pty. Ltd.
Suite 1.01, Century Corporate Centre
62 Norwest Boulevard
Baulkham Hills NSW 2153 · Australia
T +61 2 8818 2800 · F +61 2 8814 4500
healthcare@ottobock.com.au · www.ottobock.com.au

Beijing Otto Bock Orthopaedic Industries Co., Ltd.
B12E, Universal Business Park
10 Jiuxianqiao Road, Chao Yang District
Beijing, 100015, P.R. China
T +8610 8598 6880 · F +8610 8598 0040
news-service@ottobock.com.cn
www.ottobock.com.cn

Otto Bock Asia Pacific Ltd.
Unit 1004, 10/F, Greenfield Tower, Concordia Plaza
1 Science Museum Road, Tsim Sha Tsui
Kowloon, Hong Kong · China
T +852 2598 9772 · F +852 2598 7886
info@ottobock.com.hk · www.ottobock.com

Otto Bock HealthCare India Pvt. Ltd.
20th Floor, Express Towers
Nariman Point, Mumbai 400 021 · India
T +91 22 2274 5500 / 5501 / 5502
information@indiaottobock.com · www.ottobock.in

Otto Bock Japan K. K.
Yokogawa Building 8F, 4-4-44 Shibaura
Minato-ku, Tokyo, 108-0023 · Japan
T +81 3 3798-2111 · F +81 3 3798-2112
ottobock@ottobock.co.jp · www.ottobock.co.jp

Otto Bock Korea HealthCare Inc.
4F Agaworld Building · 1357-74, Seocho-dong
Seocho-ku, 137-070 Seoul · Korea
T +82 2 577-3831 · F +82 2 577-3828
info@ottobockkorea.com · www.ottobockkorea.com

Otto Bock South East Asia Co., Ltd.
1741 Paholyothin Road
Kwaeng Chatuchark · Khet Chatuchark
Bangkok 10900 · Thailand
T +66 2 930 3030 · F +66 2 930 3311
obsea@ottobock.co.th · www.ottobock.co.th

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Otto Bock Mobility Solutions GmbH