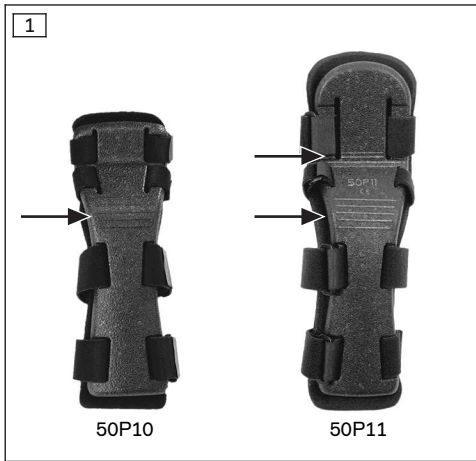
50P10 Manu Immobil

50P11 Manu Immobil Long

DE Gebrauchsanweisung	3
EN Instructions for use	7
FR Instructions d'utilisation	10
IT Istruzioni per l'uso	14
ES Instrucciones de uso	18
PT Manual de utilização	22
NL Gebruiksaanwijzing	26
SV Bruksanvisning	30
DA Brugsanvisning	34
PL Instrukcja użytkowania	38
HU Használati utasítás	42
CS Návod k použití	46
BG Инструкция за употреба	50
TR Kullanma talimatı	54
EL Οδηγίες χρήσης	58
RU Руководство по применению	62
ZH 使用说明书	66

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INFORMATION

Date of last update: 2021-08-20

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the 50P10 Manu Immobil and 50P11 Manu Immobil Long wrist supports.

2 Intended use

2.1 Indications for use

The support is intended **exclusively** for treating the upper limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

2.2 Indications

50P10 Manu Immobil

- Preoperative/postoperative/posttraumatic conditions that require immobilization of the wrist
- Sprains
- Osteoarthritis of the wrist
- Acute and chronic irritation in the wrist (e.g. carpal tunnel syndrome)
- Subluxation/dislocation

50P11 Manu Immobil Long

- Preoperative/postoperative/posttraumatic conditions that require immobilization of the wrist
- Posttraumatic and postoperative conditions that require a high degree of finger joint immobilization
- Sprains
- Acute and chronic irritation in the wrist (e.g. carpal tunnel syndrome)
- Subluxation/dislocation

The indication must be determined by the physician.

2.3 Contraindications

2.3.1 Absolute Contraindications

None known.

2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent, swollen scars, reddening and hyperthermia in the relevant area; pronounced varicose veins, especially with impaired blood return, lymph drainage disorders and unclear soft tissue swelling distal to the device; sensory and circulatory disorders in the area of the forearm and hand, e.g. associated with diabetic neuropathy.

2.4 Mechanism of Action

50P10 Manu Immobil

The brace provides a high degree of immobilization of the wrist joint.

50P11 Manu Immobil Long

The brace provides a high degree of immobilization of the wrist joint and finger joints.

2.5 Lifetime

The product is designed for a maximum lifetime of **1 year**.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage

- ▶ Keep the product away from open flames, embers and other sources of heat.

CAUTION

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

NOTICE

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

CAUTION

Incorrect or excessively tight application

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

CAUTION

Use with heat-sensitive skin

Skin irritation due to overheating

- ▶ Do not use the product in case of a known heat allergy.
- ▶ Do not continue to use the product if skin irritation occurs.

NOTICE

Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

4 Handling

⚠ CAUTION

Improper molding or application

Injuries or damage to the brace due to overloading of the material and improper fit of the brace due to breakage of load-bearing components

- ▶ Do not make any improper changes to the brace.
- ▶ Always apply the brace according to the information in the instructions.

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

4.1 Selecting the Size

50P10 Manu Immobil

The product is available in a universal size.

50P11 Manu Immobil Long

- 1) Measure the hand length from the tip of the middle finger to the carpus.
- 2) Determine the brace size (see size chart).

4.2 Adaptation

Optional: Cold shaping

Cold shaping of the brace up to a maximum of $\pm 20^\circ$ is possible in the area of the wrist joint and finger joints. The areas that can be shaped on the underside of the brace are marked with embossed lines (see fig. 1).

- 1) Position the brace on a plane, solid surface.
- 2) Cold shape the brace by up to $\pm 20^\circ$. Avoid shaping multiple times.

Optional: Changing the strap closing direction

The brace is pre-configured for application on the right hand. When the brace is being used on the left hand, the closing direction of the straps has to be switched.

- 1) Open all hook-and-loop closures on the straps.
- 2) Remove the pad. Loosen the pad on the hook-and-loop patches and pull the pad out of the brace (see fig. 2).
- 3) Pull the straps out of the brace.
- 4) Turn the straps by 180° and reinsert them.
- 5) Reinstall the pad. In order to do so, slide the pad into the brace and fasten it on the hook-and-loop patches.

Optional with 50P11 Manu Immobil Long: Positioning the finger strap

- 1) Remove the pad. Loosen the pad on the hook-and-loop patches and pull the pad out of the brace (see fig. 2).
- 2) Slide the finger strap to the desired position on the underside of the brace.

- 3) Reinstall the pad. In order to do so, slide the pad into the brace and fasten it on the hook-and-loop patches. This secures the finger strap.

4.3 Application

- > All hook-and-loop closures on the straps are open.
- 1) Insert the hand into the brace from behind until the hand is lying fully on the hand support.
 - 2) Fasten the straps with hook-and-loop. Note the following sequence: wrist strap, back of hand strap, finger strap, forearm strap (see fig. 3).
 - 3) Make sure that the straps fit closely without constriction (see fig. 4).
 - 4) Shorten the straps as needed.

4.4 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Remove the straps and pads from the brace.
- 2) Fasten all hook-and-loop closures.
- 3) Hand-wash the straps and pads in warm water at 30 °C (86 °F) using neutral detergent. Rinse well.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).
- 5) Clean contamination off the brace frame with a moist cloth and allow to air-dry.

5 Disposal

Dispose of the product in accordance with national regulations.

6 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

6.1 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

6.2 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorized modification of the product.

6.3 CE Conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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5 UTILIZATION

UTILIZATION of the device is carried out in accordance with the provisions of the regulations in force in the country of use of the device.

6 LEGAL INSTRUCTIONS

All legal instructions apply to the law of the country in which the device is used, therefore these instructions may vary.

6.1 Local legal instructions

Legal instructions, which find their application **exclusively** in individual countries, are listed under this chapter in the official language of the corresponding country, in which the device is used.

6.2 Liability

The manufacturer bears responsibility in the case where the device is used in accordance with the descriptions and instructions given in this document. The manufacturer is not responsible for damage caused by negligence of the provisions of this document, in particular in the case of improper use or unauthorized modification of the device.

6.3 Compliance with CE standards

This device complies with the requirements of the Regulation (EC) 2017/745 on medical devices. Declaration of CE compliance can be downloaded on the manufacturer's website.

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