

17B23, 17B45, 17B92, 17B33, 17B91, 17B23=K

DE	Gebrauchsanweisung	4
EN	Instructions for use	11
FR	Instructions d'utilisation	17
IT	Istruzioni per l'uso	25
ES	Instrucciones de uso	32
PT	Manual de utilização	40
NL	Gebruiksaanwijzing	47
SV	Bruksanvisning	54
DA	Brugsanvisning	61
NO	Bruksanvisning	67
FI	Käyttöohje	74
PL	Instrukcja użytkowania	80
HU	Használati utasítás	87
CS	Návod k použití	94
RO	Instrucțiuni de utilizare	101
HR	Upute za uporabu	108
SL	Navodila za uporabo	115
SK	Návod na používanie	121
BG	Инструкция за употреба	128
TR	Kullanma talimatı	135
EL	Οδηγίες χρήσης	142
RU	Руководство по применению	150
JA	取扱説明書	158
ZH	使用说明书	164
KO	사용 설명서	170


1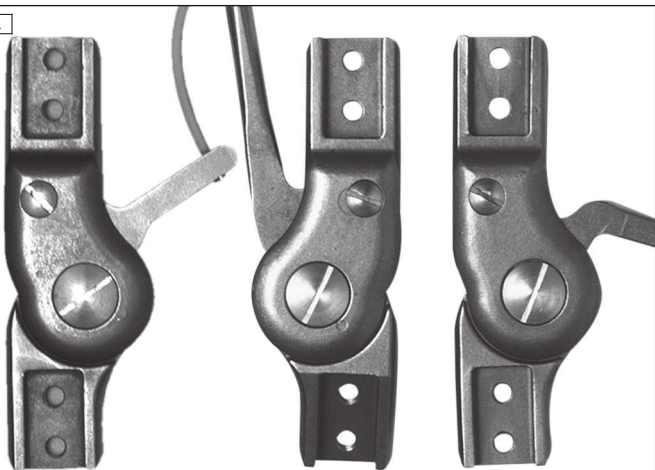


2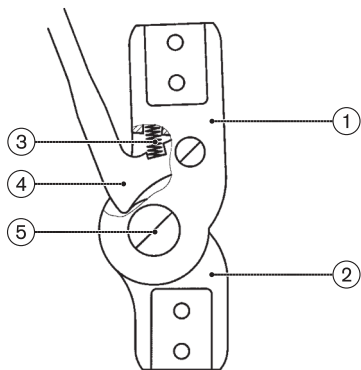

3

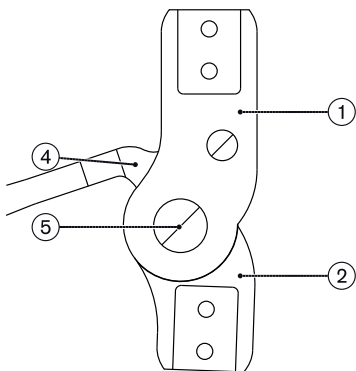4

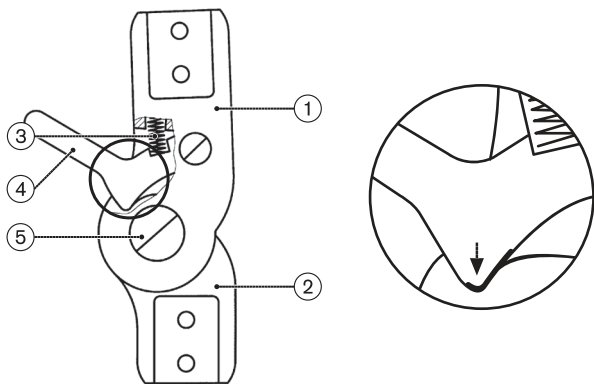INFORMATION

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INFORMATION

Date of last update: 2021-08-05

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the processing of the 17B23*, 17B33*, 17B45*, 17B91* and 17B92* orthotic knee joints with Swiss locking mechanism.

2 Product description

2.1 Components/design

	Component	17B23*, 17B45*, 17B92* (see fig. 2)	17B33*, 17B91* (see fig. 3)	17B23*K (see fig. 4)
		Reference number		
1	Upper joint section	30X59	30X59	30X59
2	Lower joint section	30U88=Z	30U88=Z	30U88
3	Compression spring	513D19=3.8x16	-	513D19=3.8x16
4	Lock lever	17Y20	17Y34	17Y104
5	Bearing nut/ Joint screw	17Y93=9x9.2xM6/ 501S32=M6x14x14	17Y93=9x9.2xM6/ 501S32=M6x14x14	17Y93=9x9.2xM6/ 501S32=M6x14x14

2.2 Function

17B23*, 17B33*, 17B45*, 17B91*, 17B92*

The locking mechanism consists of a centric lock lever and a centric lower joint section. The lock lever of the orthotic knee joint with a concealed locking mechanism and compression spring (17B23*, 17B45*, 17B92*) points up. The lock lever of the orthotic knee joint without a compression spring (17B33*, 17B91*) points down.

17B23=LR16K, 17B23=L/R20K

The locking mechanism consists of an eccentric lock lever and an eccentric lower joint section. The short lock lever has a stepped bore for the cable pull release. The locking mechanism fits tightly and is self-readjusting.

3 Intended use

3.1 Indications for use

The orthotic knee joints are to be used **exclusively** in pairs to provide orthotic treatment for a patient's lower limbs.

3.2 Indications

- Partial or total paralysis of the leg muscles
 - Orthopaedic diseases of the lower limbs
- Indications must be determined by the physician.

3.3 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

4 Safety

4.1 Explanation of warning symbols

 CAUTION	Warning regarding possible risks of accident or injury.
---	---

 NOTICE	Warning regarding possible technical damage.
--	--

4.2 Safety instructions

 CAUTION

Excessive strain on load-bearing components

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.

- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

⚠ CAUTION

Incorrect alignment or assembly

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

⚠ CAUTION

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

⚠ CAUTION

Improper handling

Risk of falling due to accidentally opening the lock

- ▶ Prior to handing over the orthosis, ensure that the locks are engaged securely and in parallel.
- ▶ Avoid accidentally opening the locks.
- ▶ Instruct the patient in the correct usage of the system joints.

⚠ CAUTION

Risk of pinching in the knee joint flexion area

Injury due to pinching of body parts and articles of clothing.

- ▶ Ensure that no body parts or garments are in this area when flexing the product.

⚠ CAUTION

Excessive strain due to use on more than one patient

Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

NOTICE**Thermal overloading of the product**

Damage due to improper thermal treatment

- ▶ Do not carry out any heat treatment at temperatures above **300 °C (570 °F)**.
- ▶ Prior to thermal treatment, remove all temperature-critical components (such as plastic parts).

NOTICE**Exposure of the product to unsuitable environmental conditions**

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C (14 °F) or above +60 °C (140 °F) (e.g. sauna, excessive sunlight, drying on a radiator).

5 Preparing the product for use

Gluing in bars

INFORMATION**Corrosion protection**

The side bars and orthotic joints are resistant to corrosion. Fine grinding and polishing increase corrosion protection. Use sintering powder (e. g. 618T40) for coating.

Combine the orthotic joints with:

Side bars	17B38*, 17B39*, 17B40*, 17B41*, 17B84*, 17B85*
Lamination bars	17Y128*, 17Y129*

Required materials: 636W18 special glue and 636W19 hardener or 636W28 special glue, degreasing cleaner

The bars must be glued in the insertion zones to ensure stability.

- 1) Clean the connecting surfaces with a degreasing cleaner.
- 2) Mix the special glue and the hardener.
- 3) Apply the mixture to the insertion zones.
- 4) Insert the bars.
- 5) Secure with the enclosed screws.

- 6) Allow to cure for at least **4 hours**.

INFORMATION: Final bonding strength will be reached after 16 hours.

Inserting the compression spring

- Only for 17B23*, 17B45*, 17B92*: Insert the compression spring into the upper joint section.

Converting 17B23=L/R16-K, 17B23=L/K20-K to long lock lever

Replace the **centric** lower joint section and the **centric** lock lever together.

INFORMATION: The lower joint section and the long lock lever are marked with a Z (centric).

- 1) Loosen the screws and remove the bearing nuts.
- 2) Disassemble the lock lever and the orthotic joint.
- 3) Replace the lower joint section with the 30U88* centric lower joint section.
- 4) Replace the lock lever with the 17Y34/17Y20 centric lock lever.
- 5) Assemble the lock lever and the orthotic joint with the screws and bearing nuts.
- 6) Secure the screws with Loctite 241.

Converting the 17B23*, 17B33*, 17B45*, 17B91*, 17B92* to cable pull release

Replace the **eccentric** lower joint section and the short lock lever together.

- 1) Loosen the screws and remove the bearing nuts.
- 2) Disassemble the lock lever and the orthotic joint.
- 3) Replace the centric lower joint section with the 30U88* eccentric lower joint section.
- 4) Replace the centric lock lever with the 17Y104 eccentric lock lever.
- 5) Assemble the lock lever and the orthotic joint with the screws and bearing nuts.
- 6) Secure the screws with Loctite 241.

Replacing the short lock lever

Inspect the 17Y104 lock lever and replace it in case of signs of wear.

- 1) Loosen the lock screw and remove the bearing nut from the upper joint section.
- 2) Remove the lock lever.
- 3) Insert the replaced lock lever into the upper joint section.

- 4) **Optional** Rework the lower side of the short lock lever until the engagement reserve reaches 5 mm (see fig. 4).

INFORMATION: Do not rework the locking surfaces of the lower joint section and the lock lever.

- 5) Assemble the lock lever and the orthotic joint with the screws and bearing nuts.
- 6) Secure the screws with Loctite 241.

Connecting the lock lever

For simultaneous triggering of the two 17Y57 lock levers, the lock levers can be interconnected.

- 1) Weld or solder the 17Y53 connection tube to the lock lever.
- 2) Verify that the two locks release and engage simultaneously.
- 3) Connect the lock levers with a traction element (e.g. elastic strap) to enable engaging.

Using protective devices

- ▶ Protect the product against dirt: Use suitable protective devices (e.g. joint protectors).

6 Cleaning

Promptly clean the product after contact with water containing salt, chlorine or soap, or if it gets dirty.

- 1) Rinse system joints with clean fresh water.
- 2) Dry system joints with a cloth or allow to air dry. Avoid exposure to direct heat (e.g. from an oven or radiator).

7 Maintenance

⚠ CAUTION

Use of an incorrect lock

Injuries from falling due to an incorrect lock being installed

- ▶ Only use locks of the same size when reinstalling or replacing.

INFORMATION

The product may be exposed to increased loads by the patient.

- ▶ Shorten the maintenance intervals according to the expected loads.

INFORMATION

Increased wear and tear

The orthotic knee joints are intended only for use in combination with free-motion or foot lifting orthotic ankle joints. Wear occurs more rapidly in combinations involving orthotic ankle joints with a dorsal stop.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Spare parts are listed in the section "Components/design".

Only use 633F7 special lubricant.

Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241.

If you have any doubts, please contact the manufacturer.

8 Disposal

Dispose of the product in accordance with national regulations.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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