Varos Testset

DE Gebrauchsanweisung (Fachpersonal)	2
EN Instructions for use (qualified personnel)	8
FR Instructions d'utilisation (Personnel spécialisé)	15
IT Istruzioni per l'uso (Personale tecnico specializzato)	21
ES Instrucciones de uso (Personal técnico especializado)	28
PT Manual de utilização (Pessoal técnico)	34
NL Gebruiksaanwijzing (Vakmensen)	41
SV Bruksanvisning (Fackpersonal)	48
DA Brugsanvisning (Faguddannet personale)	54
NO Bruksanvisning (Fagpersonell)	60
FI Käyttöohje (Ammattihenkilöstö)	66
PL Instrukcja użytkowania (Personel fachowy)	72
HU Használati utasítás (szakszemélyzet)	79
JA 取扱説明書 (有資格担当者)	85

INFORMATION

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INFORMATION

Date of last update: 2022-01-20

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

This additional documentation contains all information required to fabricate a check socket with the Varos test set. The 647G1099=all_INT instructions for use must be observed.

This additional documentation is intended for O&P professionals and assumes that these persons are trained in the handling of the recommended materials, machines and tools.

INFORMATION

This document does not replace the detailed information on the handling and safety of your product provided in the 5A60=* Varos instructions for use. Please read the instructions for use carefully!

2 Product description

2.1 Function

The Varos test set is used to determine the suitability of a patient for a fitting with the 5A60=* Varos.

The Varos test set is a collection of various shell and side rail assemblies of different sizes.

3 Safety

3.1 Explanation of warning symbols

⚠ WARNING Warning regarding possible serious risks of accident or injury.

⚠ CAUTION Warning regarding possible risks of accident or injury.

ℹ NOTICE Warning regarding possible technical damage.

3.2 Structure of the safety instructions

⚠ WARNING

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in the event of failure to observe the hazard
- > E.g.: Consequence 2 in the event of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

3.3 General safety instructions

⚠ WARNING

Non-observance of safety notices

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

⚠ WARNING

Use of the product near active implanted systems

The product's magnetic field can interfere with active implanted systems (e.g. pacemakers, defibrillators etc.).

- ▶ When using the product in the immediate vicinity of active implanted systems, ensure that the minimum distances stipulated by the manufacturer of the implant are observed.
- ▶ Make sure to observe any operating conditions and safety instructions stipulated by the manufacturer of the implant.

⚠ CAUTION

Independent manipulation of system components

Falling due to breakage of load-bearing components or malfunction of the product.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The product and any damaged components may only be opened and repaired or replaced by qualified personnel.

⚠ CAUTION

Mechanical overload during transportation

Falling due to breakage of load-bearing components or malfunction of the product.

- ▶ Only use the transport packaging for transportation.

⚠ CAUTION

Mechanical overloading of the product

Falling due to breakage of load-bearing components or malfunction of the product.

- ▶ Do not subject the product to excessively strong mechanical vibrations or impacts.
- ▶ Check the product for visible damage before each use.

⚠ CAUTION

Signs of wear on system components

Falling due to breakage of load-bearing components or malfunction of the product.

- ▶ Annual safety inspections have to be carried out in the interest of the patient's safety and in order to maintain operational reliability.

⚠ CAUTION

Risk of pinching when inserting the liner into the socket

Injury due to powerful magnetic attraction between the magnetic cap of the matching 6Y200=* Varos liner and the magnetic bottom shell of the product.

- ▶ When inserting the liner into the socket, ensure that no fingers/body parts are between the liner and socket.

⚠ CAUTION

Storage of the product in direct sunlight

> Falling due to the breakage of load-bearing components caused by plastic parts of the product becoming brittle due to UV exposure.

> Discolouration of plastic parts.

- ▶ Avoid storing the product in direct sunlight.

⚠ CAUTION

Contact of the product with embers or fire

Injuries due to contact with molten material.

- ▶ Keep the product away from embers and open flames.

⚠ CAUTION

Incorrectly secured screws

Falling due to breakage of load-bearing components caused by screw connections coming loose.

▶ Observe the specified torque values.

- ▶ Observe the instructions for securing the screw connections.

⚠ CAUTION

Use of a damaged product

Falling due to breakage of load-bearing components or malfunction of the product.

- ▶ Regularly check the socket, Boa cable, Boa dials, socket brim pad and fabric or plastic adhesive strips for wear and tear or damage.
- ▶ If components are damaged or a Boa cable is torn, the socket has to be repaired by qualified personnel.

⚠ CAUTION

Reuse on another patient and improper cleaning

Skin irritation, development of eczema or infections due to contamination with germs.

- ▶ The product should be used only on the patient for whom it was fitted.
- ▶ If the product is to be used on another patient, it must first be cleaned with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock DermaClean) and disinfected.
- ▶ If a surface disinfection process using an alcohol-based disinfectant without additional additives, for example "Terallin liquid", is used for disinfection, the application instructions and safety notices of the disinfectant manufacturer must be observed.

4 Fabricating a check socket with the Varos test set

Required tools

For the required tools to fabricate the check socket, see the instructions for use for the 5A60=* Varos.

A trial fitting with a Varos check socket has to be carried out to determine the suitability of the potential user.

The check socket can be fabricated in a few steps using the Varos test set. With the check socket, the fit, pressure distribution in the socket, handling and gait pattern can be observed. The application of a check socket with various socket lengths is recommended.

None of the components are permanently modified (e.g. shortened) at the time of applying the check socket.

4.1 Design

The test set consists of the following components:

- | | |
|--|---|
| <ol style="list-style-type: none"> 1. Shell assembly, left, 200 mm 2. Shell assembly, left, 240 mm 3. Clamping piece set, art. no. 5E8=1 4. Side rail assembly, 300 mm 5. Side rail assembly, 270 mm 6. Side rail assembly, 240 mm 7. Side rail assembly, 220 mm 8. Side rail assembly, 200 mm | <ol style="list-style-type: none"> 9. Connection adapter, art. no. 5E6=1 with screw set, connection adapter, art. no. 5E18=1 10. Magnetic cap, threaded version, art. no. 5E10=* 11. Magnetic bottom shell, art. no. 5E5=* 12. Shell assembly, right, 240 mm 13. Shell assembly, right, 200 mm |
|--|---|

4.2 Fabricating the check socket

Parts required

To prepare the Varos test set, the following parts must be ordered under the article no. below:

- | | |
|--|---|
| <ul style="list-style-type: none"> • 2 pc. medial socket shell, art. no. 5E15=L-* • 2 pc. medial socket shell, art. no. 5E15=R-* • 2 pc. lateral socket shell, art. no. 5E16=L-* Contains: 2 pc. riveted BOA housings, 2 pc. BOA dials, 2 pc. BOA reels • 2 pc. lateral socket shell, art. no. 5E16=R-* Contains: 2 pc. riveted BOA housings, 2 pc. BOA dials, 2 pc. BOA reels • 2 pc. socket brim pad, art. no. 5E17=L-* • 2 pc. socket brim pad, art. no. 5E17=R-* • 4 pc. fabric adhesive strip set, art. no. 5E7=* • 4 pc. BOA lace set, art. no. 5E12=1 • 5 pc. medial side rail, art. no. 5E1=* | <ul style="list-style-type: none"> • 5 pc. lateral side rail, straight, art. no. 5E2=* • 5 pc. axle, art. no. 5E4=1 • 1 pc. connection adapter, art. no. 5E6=1 • 1 pc. magnetic bottom shell, art. no. 5E5=* • Connection adapter screw set, art. no. 5E18=1 Contains: 6 pc. M6x10 cap screws • 1 pc. clamping piece set, art. no. 5E8=1 Contains: 3 pc. clamping pieces, 7 pc. slot nuts, 7 pc. M4x8 countersunk screws • 1 pc. magnetic cap, threaded version, art. no. 5E10=* Contains: magnetic cap and M10 countersunk screw |
|--|---|

- Rubber nub set, art. no. 5E13=1
Contains: 14 pc. rubber nubs

Check socket alignment

⚠ CAUTION

Reduced strength of the check socket

Falling due to breakage of load-bearing components.

- ▶ When a check socket is combined with a longer side rail assembly and shorter shell assembly, it may only be used subject to complying with safety measures, e.g. walking between parallel bars.

INFORMATION

For the alignment of the check socket with a desired socket length, the longest possible shell assembly and longest possible side rail assembly always have to be used.

The check socket is fabricated with the components named above, similar to the fabrication of the definitive socket.

- ▶ Follow the steps in the sections "Cutting off and gluing in the fabric adhesive strips" up to and including "Screwing the bottom plate to the connection adapter".

Inserting the liner

A liner from a previous device or the 6Y200=* Varos liner may be used.

To use a 6Y200=* Varos liner, follow the steps in the instructions for use.

INFORMATION: The Varos liner trimmed for the check socket cannot be reused.

Complete the following steps to insert the liner (from a previous fitting):

- ▶ Screw the magnetic cap, threaded version (see fig. 1, item 2) to the M10 connection thread of the liner (see fig. 1, item 1) with the corresponding countersunk screw.

INFORMATION: The liner must have a connection cover with a central M10 connection thread.

INFORMATION

The optimum socket length may change depending on the chosen liner. This socket length must be observed for the final fitting with the product.

4.3 Information on using the check socket

Reinforcing the liner

When longer side rail assemblies are combined with shorter shell assemblies, a window is formed in the distal area of the socket, respectively on the frontal and dorsal sides, and soft tissue of the residual limb (see fig. 2, item 1) may be squeezed out here when a load is put on the socket. This leads to increased axial pistoning between loading and unloading the socket, and an increased pressure load on the medial socket brim. Therefore, excessive squeezing out of soft tissue has to be prevented by reinforcing the liner (e.g. by applying a circular hook-and-loop strap) in the area of the windows (see fig. 2, item 1).

Suspension of the liner on the check socket

When a liner from a previous fitting is used, note that the suspension between the liner and check socket may be less than the suspension between the definitive socket and definitive liner. This means it may only be possible to verify the fit, pressure distribution and handling, but not the gait pattern.

5 Cleaning

CAUTION

Reuse on another patient and improper cleaning

Skin irritation, development of eczema or infections due to contamination with germs.

- ▶ The product should be used only on the patient for whom it was fitted.
- ▶ If the product is to be used on another patient, it must first be cleaned with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock DermaClean) and disinfected.
- ▶ If a surface disinfection process using an alcohol-based disinfectant without additional additives, for example "Terallin liquid", is used for disinfection, the application instructions and safety notices of the disinfectant manufacturer must be observed.

> **Required materials:** colourless, alcohol-free disinfectant, soft cloth

- 1) Disinfect the product with the disinfectant.
- 2) Dry the product with a cloth.
- 3) Allow to air dry in order to remove residual moisture.

6 Legal information

Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

7 Symbols Used

Please note the instructions for use

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Otto Bock Healthcare Products GmbH