DynamicArm Plus 12K110N

DE	Gebrauchsanweisung [kurz] (Fachpersonal)	3
EN	Instructions for use [short] (qualified personnel)	7

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INFORMATION

Date of last update: 2022-02-17

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.

- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

Note the instructions for use

This document contains the sections "Intended use" and "Safety", and does not replace the instructions for use. The instructions for use are available in digital form (PDF format) (see page 2 of this document). To view PDF files, you need an Internet browser or other program capable of opening this file format.

You may request a printed copy of the instructions for use under the link on page 2. This will be delivered to the specified address within 7 days after the order is received.

The product "12K110N=* DynamicArm Plus" is referred to simply as the product/elbow joint below.

Basic UDI-DI: 406441100000012K110NQH

2 Intended use

2.1 Indications for use

The product is to be used **exclusively** for exoprosthetic fittings of the upper limbs and serves as an anatomical replacement for the elbow joint and forearm.

2.2 Conditions of use

The product is suitable for TMR patients with unilateral or bilateral amputations.

The product is intended **exclusively** for use on adults.

The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports which involve excessive strain and/or shocks to the wrist joint (pushups, downhill racing, mountain biking, ...) or extreme sports (free climbing, paragliding, etc.).

The product is intended **exclusively** for the fitting of a patient. Use of the product by a further person is prohibited by the manufacturer.

2.3 Contraindications

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

2.4 Qualification

The fitting of a patient with the product may only be carried out by O&P professionals who have been authorised with the corresponding Ottobock training.

3 Safety

3.1 Explanation of warning symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
--	---

 CAUTION	Warning regarding possible risks of accident or injury.
--	---

3.2 Structure of the safety instructions

WARNING

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in the event of failure to observe the hazard
- > E.g.: Consequence 2 in the event of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

3.3 General safety instructions

WARNING

Non-observance of safety notices

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

WARNING

Operating the product near active implanted systems

Interference with active implantable systems (e.g., pacemaker, defibrillator, etc.) due to electromagnetic interference of the product.

- ▶ When operating the product in the immediate vicinity of active implantable systems, ensure that the minimum distances stipulated by the manufacturer of the implant are observed.
- ▶ Make sure to observe any operating conditions and safety instructions stipulated by the manufacturer of the implant.

WARNING

Use of the product while operating a vehicle or machinery

- > Risk of accident due to unexpected behaviour of the product.
- > Risk of injury due to faulty control or malfunction of the product.
- ▶ Applicable national legal regulations must be complied with when operating vehicles of any kind or when operating machinery with the product.
- ▶ Have your driving ability examined and verified by an authorised test centre.
- ▶ Switch off the product before driving.

WARNING

Charging the prosthesis without taking it off

Risk of electric shock due to defects in the power supply unit or in the battery charger.

- ▶ For safety reasons, remove the prosthesis prior to charging it.

CAUTION

Overloading due to unusual activities

Injury due to unexpected product behaviour as a result of a malfunction.

- ▶ The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports with excessive strain and/or shocks to the wrist joint (pushups, downhill mountain biking, ...) or extreme sports (free climbing, paragliding, etc.).

- ▶ Careful handling of the product and its components not only increases their service life but, above all, ensures the patient's personal safety!
- ▶ If the product and its components have been subjected to extreme loads (e.g. due to a fall, etc.), then the product must be inspected for damage immediately. If necessary, forward the product to an authorised Ottobock Service Centre.

⚠ CAUTION

Independent user manipulation of system components

Injury due to faulty control or malfunction of the product.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The battery may only be handled by Ottobock authorised, qualified personnel (no replacement by the user).
- ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.

⚠ CAUTION

Use of unsuitable prosthetic components

Injury due to unexpected product behaviour.

- ▶ Use the product only in combination with components listed in the section "Combination Possibilities" (Combination possibilities).

⚠ CAUTION

Improper use

Injury due to loss of product functionality.

- ▶ Instruct the patient in the proper use of the product.

⚠ CAUTION

Manual unlocking of elbow lock under load

Injury by release of elbow lock under load.

- ▶ Particular caution should be exercised when unlocking the elbow lock while lifting heavy loads.
- ▶ Be careful when unlocking the lock under such conditions due to the possibility of injury.

⚠ CAUTION

Proximity to sources of strong magnetic or electrical interference (e.g. theft prevention systems, metal detectors)

Injury due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Avoid remaining in the vicinity of visible or concealed theft prevention systems at the entrance/exit of stores, metal detectors/body scanners for persons (e.g. in airports) or other sources of strong magnetic and electrical interference (e.g. high-voltage lines, transmitters, transformer stations, computer tomographs, magnetic resonance tomographs, etc.).
- ▶ When walking through theft prevention systems, body scanners or metal detectors, watch for unexpected behaviour of the product.

⚠ CAUTION

Mechanical stress on the prosthesis

Injury due to faulty control or malfunction of the prosthesis.

- ▶ Do not subject the prosthesis to mechanical vibrations or impacts.
- ▶ Check the prosthesis for visible damage before each use.

⚠ CAUTION

Penetration of dirt and humidity into the product

Risk of injury due to unexpected product behaviour or malfunction.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.
- ▶ Do not expose the product, and especially the elbow joint, to dripping or splashing water.
- ▶ Wear the product, and especially the elbow joint, under suitably resistant clothing in the rain.

⚠ CAUTION

Changing prosthetic components when switched on

Injury due to faulty control or malfunction of the prosthetic components.

- ▶ Before changing prosthetic components (e.g. terminal device) switch the product off.

⚠ CAUTION

Signs of wear and tear on the product components

Injury due to faulty control or malfunction of the product

- ▶ In the interest of patient safety and in order to maintain operating reliability and protect the warranty, the specified service intervals must be observed (see the section "Maintenance").

⚠ CAUTION

Injury due to incorrect adjustment of the product.

Loosening of the components.

- ▶ In case of a product with a quick-disconnect wrist, position the terminal device before use so that a slight turn cannot disconnect the terminal device from the prosthesis.

⚠ CAUTION

Incorrect donning and doffing of the prosthesis

Risk of injury due to sudden flexion of the prosthesis.

- ▶ Switch off the prosthesis before donning or doffing.
- ▶ The prosthesis must always be flexed for donning and doffing.

⚠ CAUTION

Unintentional unlocking of the terminal device

Risk of injury due to releasing the terminal device from the forearm (e.g. while carrying objects).

- ▶ When using a quick-disconnect wrist unit, position the terminal device so that slight twisting does not release the terminal device from the forearm.

⚠ CAUTION

Charging the product with damaged power supply unit/charger/charger cable

Injury due to unexpected behaviour of the product caused by insufficient charging.

- ▶ Check the power supply unit, charger and charger cable for damage before use.
- ▶ Replace any damaged power supply unit, charger or charger cable.

⚠ CAUTION

Risk of pinching in the joint flexion area

Injuries due to pinching of body parts.

- ▶ Ensure that fingers and other body parts are not in this area when bending the joint.

⚠ CAUTION

Electrode lifting off

Injury due to unexpected product behaviour as a result of uncontrolled activation of the prosthesis component.

- ▶ Inform the patient that uncontrolled movement of the prosthesis component may occur if the electrode lifts off.

⚠ CAUTION

Overheating of drive unit due to continuous activity

Injury due to touching the overheated drive unit.

- ▶ Avoid touching the housing of the drive unit in case of overheating.

⚠ CAUTION

Removing the pull cable

Injury due to malfunction of the product.

- ▶ The pull cable is to be used for emergency operation only. For safety reasons, it must not be removed. When the product is switched off or defective and when the battery is empty, the elbow joint cannot be locked or unlocked without the pull cable.

A series of 21 horizontal lines spaced evenly down the page, intended for writing or drawing.

Otto Bock Healthcare Products GmbH
Brehmstraße 16 · 1110 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com