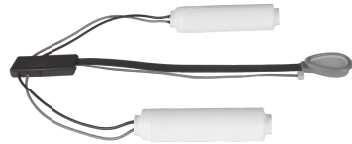
757B35=0

757B35=1/757B35=3

757B35=4










757B35=5

## 757B35=0, 757B35=1, 757B35=3, 757B35=4, 757B35=5
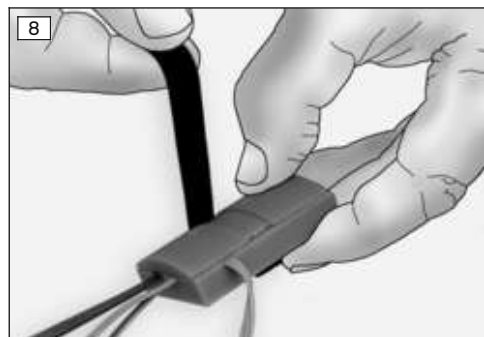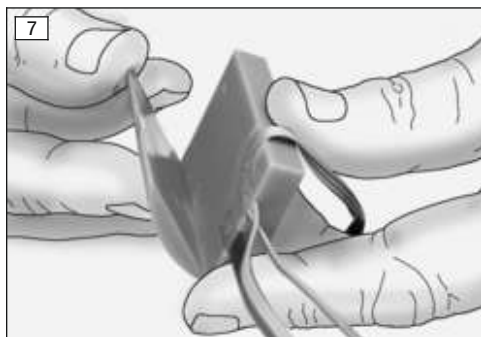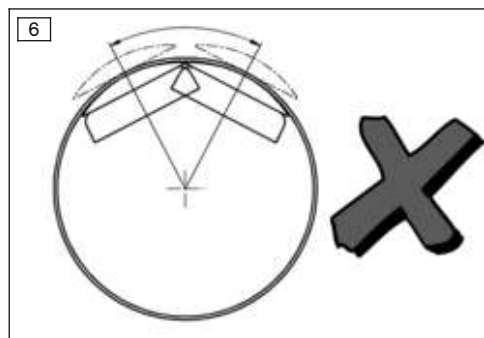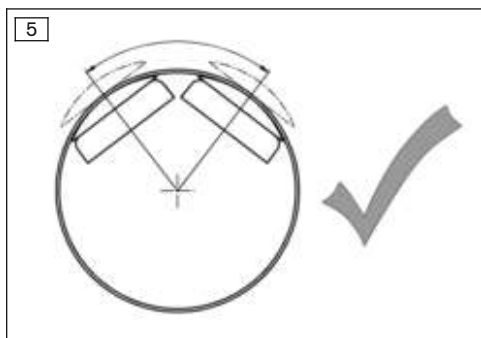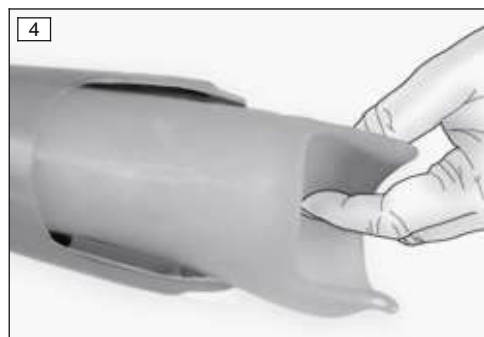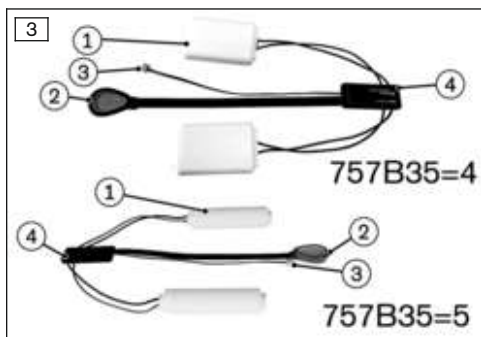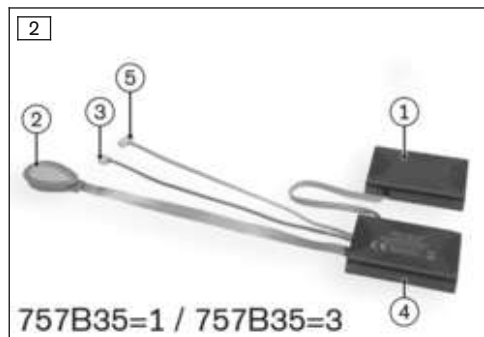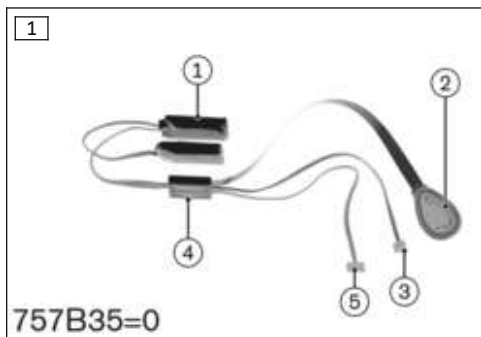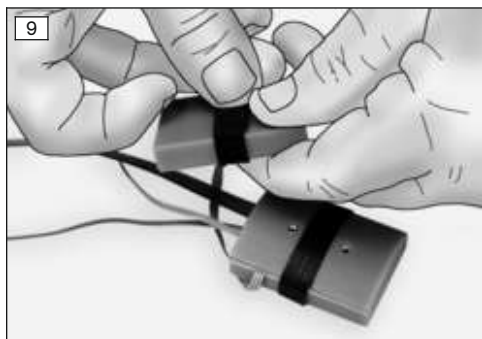
<b>DE</b> Gebrauchsanweisung (Fachpersonal) .....	4
<b>EN</b> Instructions for use (qualified personnel) .....	14
<b>FR</b> Instructions d'utilisation (Personnel spécialisé) .....	24
<b>IT</b> Istruzioni per l'uso (Personale tecnico specializzato) .....	35
<b>ES</b> Instrucciones de uso (Personal técnico especializado) .....	46
<b>PT</b> Manual de utilização (Pessoal técnico) .....	56
<b>NL</b> Gebruiksaanwijzing (Vakmensen) .....	67
<b>SV</b> Bruksanvisning (Fackpersonal) .....	77
<b>DA</b> Brugsanvisning (Faguddannet personale) .....	87
<b>NO</b> Bruksanvisning (Fagpersonell) .....	97
<b>FI</b> Käyttöohje (Ammattihenkilöstö) .....	107
<b>SK</b> Návod na používanie (Odborný personál) .....	117
<b>TR</b> Kullanma talimatı (Uzman personel) .....	127
<b>EL</b> Οδηγίες χρήσης (Τεχνικό προσωπικό) .....	136
<b>RU</b> Руководство по применению (Квалифицированный персонал) .....	147
<b>JA</b> 取扱説明書 (有資格担当者) .....	159

## INFORMATION

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### INFORMATION

Date of last update: 2022-02-14

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The 757B35=\* MyoEnergy Integral is referred to simply as the product below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

The product may not be transferred to the patient without prior instruction.

## 2 Product description

### 2.1 Construction and Function

The product serves to provide power to a prosthesis.

The product consists of several components. These components are inseparable and ready for mounting.

#### 2.1.1 Components

The product consists of the following components:

##### Rechargeable battery

The battery (see fig. 1, see fig. 2 and see fig. 3 , item 1) consists of two cells.

##### Charging receptacle

The charging receptacle (see fig. 1, see fig. 2 and see fig. 3, item 2) has the following functions:

- Contact for charging the battery
- LED to indicate the current charge level

- LED for providing feedback on operating states
- Button for turning the prosthesis on and off
- Beeper for providing feedback on operating states

**Supply cable**

The supply cable (see fig. 1, see fig. 2 and see fig. 3, item 3) establishes the connection between the product and the respective prosthetic component.

**Electronics**

The electronics (see fig. 1, see fig. 2 and see fig. 3, item 4) protect the battery against short circuits, overvoltage, undervoltage and charging outside the allowable temperature range.

**Communication cable**

The communication cable (see fig. 1 and see fig. 2, item 5) is used to exchange data and connects the respective prosthetic components to the product.

**INFORMATION**

The communication cable is included only with the products 757B35=0, 757B35=1 and 757B35=3.

**3 Intended use**

**3.1 Indications for use**

The product to be used **exclusively** to supply energy to myoelectric upper limb prosthesis components.

**3.2 Conditions of use**

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

The product is intended **exclusively** for use on **one** patient. Use of the product by another person is not approved by the manufacturer.

Permissible ambient conditions are described in the technical data (see page 22).

**3.3 Contraindications**

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

**3.4 Qualification**

The fitting of a patient with the product may only be carried out by O&P professionals who have been authorised with the corresponding Ottobock training.

**4 Safety**

**4.1 Explanation of warning symbols**

 <b>WARNING</b>	Warning regarding possible serious risks of accident or injury.
 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

## 4.2 Structure of the safety instructions

### WARNING

#### **The heading describes the source and/or the type of hazard**

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in the event of failure to observe the hazard
- > E.g.: Consequence 2 in the event of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

## 4.3 General safety instructions

### WARNING

#### **Non-observance of safety notices**

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

### WARNING

#### **Use of the incorrect power supply unit, adapter plug or battery charger**

Injury due to leakage currents when using a third-party, non-medical power supply unit/adapter plug or battery charger.

- ▶ Only use power supply units/battery chargers approved for this product by Ottobock (see instructions for use and catalogues).

### CAUTION

#### **Changing prosthetic components when switched on**

Injury due to faulty control or malfunction of the prosthetic components.

- ▶ Before changing prosthetic components (e.g. terminal device) switch the product off.

### CAUTION

#### **Use of unsuitable prosthetic components**

Injury due to unexpected product behaviour.

- ▶ Use the product only in combination with components listed in the section "Combination possibilities" (Combination possibilities).

### CAUTION

#### **Use of unapproved accessories**

> Injury due to product malfunction as a result of reduced interference resistance.

> Interference of other electronic devices due to increased emissions.

- ▶ Use the product only in combination with the accessories, signal converters and cables listed in the section "Combination possibilities" (Combination possibilities).

### CAUTION

#### **Overloading due to unusual activities**

Injury due to unexpected product behaviour as a result of a malfunction.

- ▶ The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, sports with excessive strain and/or shocks to the wrist joint (pushups, downhill mountain biking, ...) or extreme sports (free climbing, paragliding, etc.).
- ▶ Careful handling of the product and its components not only increases their service life but, above all, ensures the patient's personal safety!
- ▶ If the product and its components have been subjected to extreme loads (e.g. due to a fall, etc.), then the product must be inspected for damage immediately. If necessary, forward the product to an authorised Ottobock Service Centre.

**⚠ CAUTION**

**Penetration of dirt and humidity into the product**

Injury due to unexpected prosthesis behaviour.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.
- ▶ Do not use the product for bathing prostheses.

**⚠ CAUTION**

**Penetration of perspiration into the product**

Injury and chemical burns due to leakage of battery fluids.

- ▶ Wear protective clothing when removing or replacing the battery (disposable gloves and protective goggles at a minimum).
- ▶ Avoid direct skin contact!

**⚠ CAUTION**

**Independent manipulation of system components**

Injury due to faulty control or malfunction of the product.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.

**⚠ CAUTION**

**Using the product with pointed or sharp objects**

Destruction or damage to the product during installation.

- ▶ Do not use any pointed or sharp objects when installing the product.

**⚠ CAUTION**

**Damage to the product due to improper installation**

Injury due to product damage.

- ▶ Install the battery according to the steps in the "Laminating instructions" section (see fig. 5).

**NOTICE**

**Improper care of the housing**

Damage to the casing through the use of acetone, white spirit or similar solvents.

- ▶ Only clean the housing with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).

## 4.4 Patient information

### **WARNING**

#### **Use of the incorrect power supply unit, adapter plug or battery charger**

Injury due to leakage currents when using a third-party, non-medical power supply unit/adapter plug or battery charger.

- ▶ Only use power supply units/battery chargers approved for this product by Ottobock (see instructions for use and catalogues).

### **WARNING**

#### **Operating the product near active implanted systems**

Interference with active implantable systems (e.g., pacemaker, defibrillator, etc.) due to electromagnetic interference of the product.

- ▶ When operating the product in the immediate vicinity of active implantable systems, ensure that the minimum distances stipulated by the manufacturer of the implant are observed.
- ▶ Make sure to observe any operating conditions and safety instructions stipulated by the manufacturer of the implant.

### **CAUTION**

#### **Mechanical stress on the product**

Injury due to faulty control or malfunction of the product components.

- ▶ Do not subject the product to mechanical vibrations or impacts.
- ▶ Check the product for visible damage before each use.

### **CAUTION**

#### **Remaining, operating and charging in areas outside the allowable temperature range**

Injury due to faulty control or malfunction of the product.

- ▶ Avoid remaining, operating and charging in areas outside the allowable temperature range (see page 22).

### **CAUTION**

#### **Proximity to sources of strong magnetic or electrical interference (e.g. theft prevention systems, metal detectors)**

Injury due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Avoid remaining in the vicinity of visible or concealed theft prevention systems at the entrance/exit of stores, metal detectors/body scanners for persons (e.g. in airports) or other sources of strong magnetic and electrical interference (e.g. high-voltage lines, transmitters, transformer stations, computer tomographs, magnetic resonance tomographs, etc.).
- ▶ When walking through theft prevention systems, body scanners or metal detectors, watch for unexpected behaviour of the product.

### **CAUTION**

#### **Signs of wear and tear on the product components**

Injury due to faulty control or malfunction of the product.

- ▶ Excessive pressure on the charging receptacle can lead to increased wear and tear on the button.

### CAUTION

#### **Independent manipulation of the product**

Injury due to malfunction and resulting unexpected prosthesis actions.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.

### CAUTION

#### **Charging the product with soiled or damaged contacts**

Injury due to unexpected behaviour of the product caused by insufficient charging.

- ▶ Make sure that the contacts are always clean and free of grease.
- ▶ Clean the electrical contacts of the charging plug and charging receptacle regularly using cotton swabs and a mild soap solution.
- ▶ Take care to avoid damaging the contacts with pointed or sharp objects.

### CAUTION

#### **Distance to HF communication devices is too small (e.g. mobile phones, Bluetooth devices, WiFi devices)**

Injury due to unexpected behaviour of the product caused by interference with internal data communication.

- ▶ Therefore, keeping the following minimum distances to these HF communication devices is recommended:
  - Mobile phone GSM 850/GSM 900: 0.99 m
  - Mobile phone GSM 1800/GSM 1900/UMTS: 0.7 m
  - DECT cordless phones incl. base station: 0.35 m
  - WiFi (routers, access points,...): 0.22 m
  - Bluetooth devices (third-party products not approved by Ottobock): 0.22 m

## **5 Scope of Delivery and Accessories**

### **5.1 Scope of delivery**

- 1 pc. 757B35=\* MyoEnergy Integral
- 1 pc. Instructions for use (qualified personnel)

### **5.2 Accessories**

- 2 pc. lamination dummy
- 1 pc. lamination dummy – charging receptacle
- 1 pc. drilling template for charging receptacle

#### **Only in scope of delivery of the 757B35=0, 757B35=4 and 757B35=5 MyoEnergy Integral**

- 1 pc. electronics lamination dummy

#### **Only included with 757B35=1 and 757B35=3 MyoEnergy Integral**

- 2 pc. battery installation aid
- 2 pc. Hook and loop closure strap (only included in the scope of delivery for the 757B35=3 MyoEnergy Integral)
- 4 pc. countersunk head screws

#### **Required accessories for operation**

- 757L35 MyoCharge Integral

## 6 Preparing the product for use

### 6.1 Laminating instructions

The product must be properly installed in the socket. Please note the corresponding technical information.

### 6.2 Fastening the MyoEnergy Integral in the outer socket

#### 6.2.1 Battery assembly for 757B35=0 / 757B35=4 / 757B35=5

- 1) Fasten the batteries and electronics in the outer socket with double-sided adhesive tape.
- 2) Slide the inner socket into the outer socket and screw them together (see fig. 4).

#### 6.2.2 Installing the 757B35=1 / 757B35=3 battery

##### Assembly of the separate battery units

- 1) Determine the battery positions using the installation aids.

**INFORMATION: Maintain sufficient distance between the installation aids/batteries (see fig. 5).**

- 2) Mark the hole positions on the socket and drill them with the drill bit.
- 3) Pull the double-sided adhesive tape off the installation aids and apply it to the battery (see fig. 7).
- 4) Secure the installation aid and battery with the hook-and-loop closure strap (757B35=3) or adhesive tape (757B35=1).

**INFORMATION: Lay the hook-and-loop closure strap into the groove on the installation aid (see fig. 8). The overlapping ends of the hook-and-loop closure strap have to be up against the battery.**

- 5) Use the countersunk head screws to screw the batteries to the socket.

##### Installation of the assembled battery units

- 1) Fasten the batteries and installation aid with the double-sided adhesive tape (see fig. 10).
- 2) Wrap the installation aid and batteries with the hook and loop closure strap (757B35=3) or adhesive tape (757B35=1).

**INFORMATION: Lay the hook and loop closure strap into the groove on the installation aid.**

## 7 Handling

### 7.1 Switching the terminal device on and off

#### CAUTION

##### Signs of wear and tear on the product components

Injury due to faulty control or malfunction of the product.

- ▶ Excessive pressure on the charging receptacle can lead to increased wear and tear on the button.

#### INFORMATION

Prior to initial operation of the product, charge the supplied battery.

- ▶ 1) Push and hold the charging receptacle button until a confirmation sound is heard (at least 1 second).
- The prosthesis and prosthetic components are switched on.
- Repeat this process to switch the prosthesis and prosthetic components off.

## 7.2 Charging the battery

We recommend charging on a daily basis. If the device is not used for an extended period, the battery charge level should be checked regularly and the battery recharged.

- 1) Connect the charging plug to the charging receptacle of the product.
  - A correct connection between the battery charger and the product is indicated by feedback (see page 24).
  - The prosthesis is switched off automatically.
  - The charging process starts.
- 2) Disconnect the product after the charging process is complete.

## 7.3 Display of the current charge level

The battery charge level can be queried at any time.

- 1) With the prosthesis switched on, press the charging receptacle button and hold for less than one second.
- 2) The LED display on the charging receptacle provides information on the current battery charge level (Status signals).

## 7.4 Safety shutoff

The purpose of the safety shutoff is to protect the product; it is triggered in case of:

- Excessively high or low temperature in the battery's environment during the charging process
- Short circuit
- Overvoltage or undervoltage

### INFORMATION

After a safety shutdown, the battery charger must be connected to reactivate the electronics.

## 8 Cleaning and Care

### NOTICE

#### Improper care of the housing

Damage to the casing through the use of acetone, white spirit or similar solvents.

- ▶ Only clean the housing with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

## 9 Legal information

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for

damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

## 9.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

## 9.3 CE conformity

Otto Bock Healthcare Products GmbH hereby declares that the product is in compliance with applicable European requirements for medical devices.

The product meets the requirements of the RoHS Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic devices.

The full text of the regulations and requirements is available at the following Internet address: <http://www.ottobock.com/conformity>

## 10 Technical data

Ambient conditions	
Storage (with and without packaging)	+5 °C/+41 °F to +40 °C/+104 °F Max. 85% relative humidity, non-condensing For reasons of quality (service life), it is recommended that the product be stored at low temperatures.
Transport (with and without packaging)	-20 °C/-4 °F to +60 °C/+140 °F Max. 90% relative humidity, non-condensing
Operation	-5 °C/+23 °F to +45 °C/+113 °F Max. 95% relative humidity, non-condensing
Charging the battery	+5 °C/+41 °F to +40 °C/+104 °F Max. 85% relative humidity, non-condensing

Rechargeable battery of the product	757B35=0	757B35=1	757B35=3	757B35=4	757B35=5
Battery type	Li-poly	Li-poly	Li-ion	Li-ion	Li-ion
Output voltage	Approx. 7.4 V	Approx. 7.4 V	Approx. 7.4 V	Approx. 7.4 V	Approx. 7.4 V
Charging voltage	Approx. 8.2 V	Approx. 8.2 V	Approx. 8.2 V	Approx. 8.2 V	Approx. 8.2 V
Capacity	300 mAh	600 mAh	1150 mAh	2350 mAh	3450 mAh
Dimensions	35 x 20 x 6.5 mm/battery cell 22 x 17 x 5 mm/electronics	52 x 25 x 9 mm/battery cell	52 x 36 x 9 mm/battery cell	59.8 x 34.8 x 11.3 mm/battery cell 43 x 19 x 7 mm/electronics	79.5 x 20 mm/battery cell 43 x 19 x 7 mm/electronics
Weight	25 g	45 g	70 g	100 g	120 g

Rechargeable battery of the product	757B35=0	757B35=1	757B35=3	757B35=4	757B35=5
Operating time of prosthesis	Approx. 1000 grip cycles for MyoBock adult prostheses Approx. 2500 grip cycles for MyoBock prostheses for children	Approx. 1500–2000 grip cycles for MyoBock adult prostheses Approx. 4000 grip cycles for MyoBock prostheses for children	Approx. 4000 grip cycles for MyoBock adult prostheses Approx. 8 hours for the bebionic hand	Approx. 16 hours for MyoBock adult prostheses Approx. 16 hours for the bebionic hand	Approx. 24 hours for MyoBock adult prostheses Approx. 24 hours for the bebionic hand
Battery life-time	2 years	2 years	2 years	2 years	2 years
Charging time until battery is fully charged	2.0 h	2.5 h	2.5 h	3.0 h	3.0 h

## 11 Appendices

### 11.1 Symbols Used

Declaration of conformity according to the applicable European directives

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

Lot number (PPPP YYYY WW)  
PPPP – plant  
YYYY – year of manufacture  
WW – week of manufacture

Manufacturer

#### Only included with 757B35=0 MyoEnergy Integral:

# IP33

Protection against penetration of solid foreign objects with a diameter > 2.5 mm, protection against water spray

#### Only included with 757B35=0 MyoEnergy Integral:

Global Trade Item Number

## 11.2 Operating states/error signals

### 11.2.1 Status signals

Charging receptacle	Event
LED shows green light	Battery fully charged
LED shows yellow light	Battery 50% charged
LED shows orange light	Battery empty

### 11.2.2 Beep signals

The following beep signals are set by default:

Beep signal	Additional display	Event
1 x long	-	<ul style="list-style-type: none"><li>• Turn off prosthesis on charging receptacle</li><li>• Charging begins (charging plug connected to charging receptacle)</li><li>• Charging ends (charging plug disconnected from charging receptacle)</li></ul>
2 x short	LED on charging receptacle lights up briefly	Switch on prosthesis
3 x short	-	Battery voltage too low, prosthesis shuts off automatically

The following beep signals exist only in conjunction with the 7in1 Controller (9E420=\*):

Beep signal	Status
1 x long	Disconnect the MyoLino Link: Prosthesis was previously switched off
2 x short	Connect the MyoLino Link
2 x short	Disconnect the MyoLino Link: Prosthesis was previously switched on
Continuous pulses (as long as the button is being pressed)	Parental access switch

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