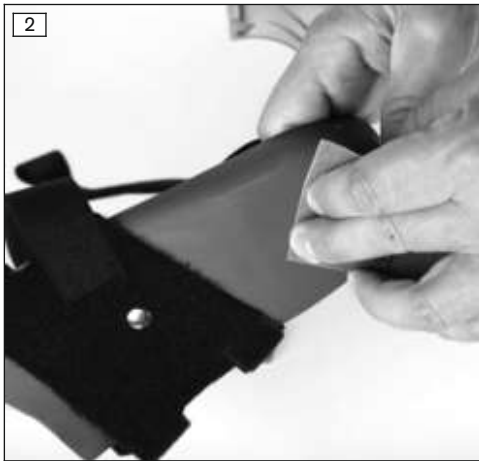
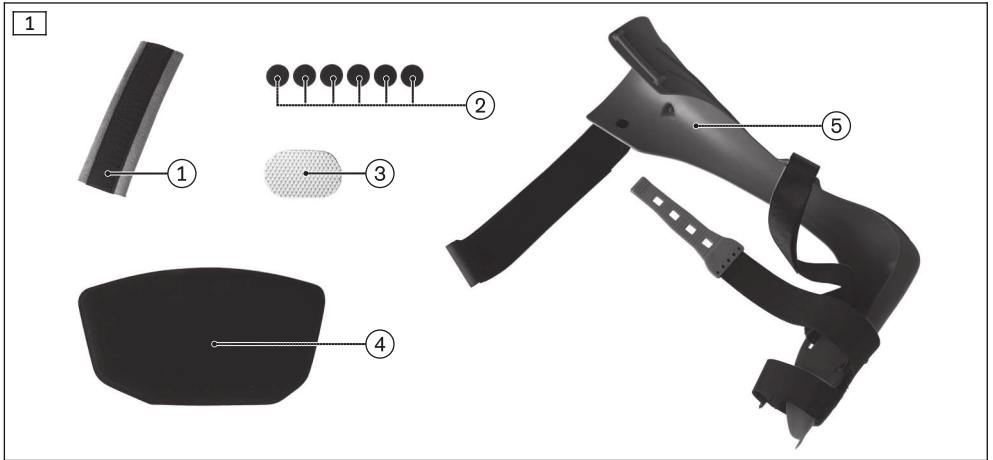
## 28U50 Malleo Neurexa Pro

<b>DE</b> Gebrauchsanweisung .....	4
<b>EN</b> Instructions for use .....	8
<b>FR</b> Instructions d'utilisation .....	11
<b>IT</b> Istruzioni per l'uso .....	16
<b>ES</b> Instrucciones de uso .....	20
<b>PT</b> Manual de utilização .....	24
<b>NL</b> Gebruiksaanwijzing .....	28
<b>SV</b> Bruksanvisning .....	32
<b>DA</b> Brugsanvisning .....	35
<b>PL</b> Instrukcja użytkowania .....	39
<b>HU</b> Használati utasítás .....	43
<b>CS</b> Návod k použití .....	47
<b>RO</b> Instrucțiuni de utilizare .....	51
<b>BG</b> Инструкция за употреба .....	55
<b>EL</b> Οδηγίες χρήσης .....	59
<b>RU</b> Руководство по применению .....	64

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## INFORMATION

Date of last update: 2022-05-18

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use contain important information for fitting and setting up your 28U50 Mal-  
leo Neurexa Pro ankle foot orthosis.

## 2 Intended use

### 2.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the lower limbs and **exclusively** for contact with intact skin.

The orthosis must be used in accordance with the indications.

### 2.2 Indications

- Dorsal flexor weakness, (e.g. after stroke, traumatic brain injury, in multiple sclerosis, neuromuscular atrophy, or peroneal paralysis)

Indications must be determined by the physician.

### 2.3 Contraindications

#### 2.3.1 Absolute Contraindications

- Moderate to severe spasticity in the lower leg
- Moderate to severe oedema
- Leg ulcers
- Moderate to severe foot deformities

#### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb/body area; lymphatic flow disorders, including unclear soft tissue swelling distal to the body area to which the medical device will be applied; sensory and circulatory disorders in the lower leg and the foot area.

### 2.4 Restrictions for use

The orthosis must not be used for the prevention of contractures (shortening) etc.

The orthosis must not be used for sports activities that involve jumping, sudden movements or rapid step sequences (e.g. basketball, badminton, riding sports). Sports activities should generally be discussed with the patient.

### 2.5 Lifetime

The product is designed for a maximum lifetime of **1 year**.

### 2.6 Effects

The ankle foot orthosis provides lifting of the foot and limits plantar flexion.

## 3 Safety

### 3.1 Explanation of warning symbols

**⚠ WARNING** Warning regarding possible serious risks of accident or injury.

**⚠ CAUTION** Warning regarding possible risks of accident or injury.

**NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

#### **⚠ CAUTION**

##### **Reuse on other persons and improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

#### **⚠ CAUTION**

##### **Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

#### **⚠ CAUTION**

##### **Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

#### **NOTICE**

##### **Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

## 4 Use

#### **⚠ WARNING**

##### **Operation of motor vehicles**

Risk of accident due to limited body function

- ▶ The product has not been approved by the manufacturer for the operation of motor vehicles.
- ▶ Observe the applicable legal regulations for the operation of motor vehicles and have your driving ability examined by an authorised test centre.

#### **INFORMATION**

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.

- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

#### 4.1 Size Selection

- ▶ Select the orthosis size based on the shoe size (see sizing table).

#### 4.2 Adaptation

##### **⚠ CAUTION**

##### **Improper moulding or application**

Injuries or damage to the orthosis due to overloading of the material and improper fit of the orthosis due to breakage of load-bearing components

- ▶ Do not make any improper changes to the orthosis.
- ▶ Always apply the orthosis according to the information in the instructions.

##### **⚠ CAUTION**

##### **Use of the product without shoes**

Risk of injury due to slipping on slippery surfaces

- ▶ Do not use the product on slippery surfaces.

##### **NOTICE**

##### **Use without suitable footwear**

Restriction of functionality due to insufficient stabilisation

- ▶ Only use the orthosis with closed footwear, taking the allowable effective heel height into account.

- 1) **If necessary:** Thermoplastically re-form the orthosis by applying gentle heat to a limited extent.  
**NOTICE! Re-forming the orthosis in cold conditions or at temperatures above 80 °C is not permitted.**
- 2) **If necessary:** Lightly sand the orthosis in the heel area (see fig. 2) and affix the supplied rubber pad (see fig. 1, item 3) with double-sided adhesive tape (see fig. 3).
- 3) Attach the calf pad (see fig. 1, item 4) using the supplied hook-and-loop tabs (see fig. 1, item 2) (see fig. 4).
- 4) Attach the pad (see fig. 1, item 1) to the shin strap.

#### 4.3 Application

##### **⚠ CAUTION**

##### **Incorrect or excessively tight application**

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

##### **NOTICE**

##### **Mechanical overload**

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

- 1) Position the foot in the orthosis; note the position of the heel and forefoot (see fig. 5).
- 2) Put the upper hook and loop strap through the loop and fasten it (see fig. 6).
- 3) Fasten the lower hook and loop strap on the forefoot (see fig. 7).
- 4) Pull the middle hook and loop strap from the inside of the foot across the top of the foot, through the outer strap guide loop, and tighten the strap till the heel fits snugly in the shell (see fig. 8).
- 5) Stretch the elastic functional strap and fasten it (see fig. 9).

## 5 Cleaning

### NOTICE

#### Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Clean the orthosis regularly:

- ▶ **Allowable cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)

#### Orthosis frame

- 1) Remove soiling on the orthosis frame with a damp cloth.
- 2) Allow to air dry. Do not expose to direct heat (e.g. from the sun, an oven or radiator).

#### Pads

- 1) Remove the padding from the orthosis.
- 2) Recommendation: Use a laundry bag or net.
- 3) Hand wash the padding in warm water at **30°C** with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).
- 5) Reattach the padding to the orthosis.

## 6 Disposal

Dispose of the product in accordance with national regulations.

## 7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 7.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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