

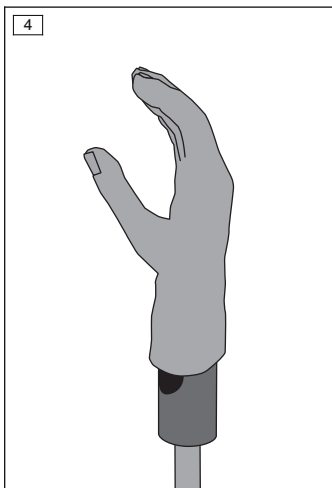
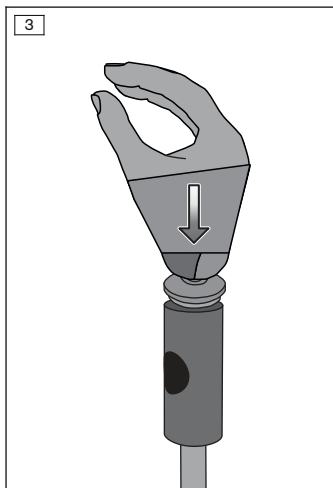
ottobock.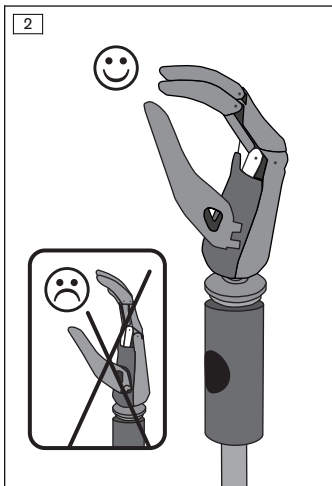

88A16=* MySkin Move

DE Gebrauchsanweisung	3
EN Instructions for use	8

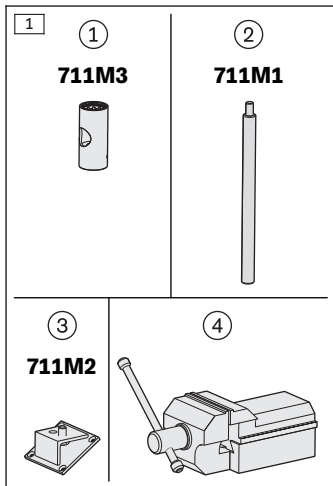INFORMATION

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INFORMATION

Date of last update: 2022-06-14

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.

- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The 88A16=* MySkin Move is a silicone cover for prosthetic hands. The cover is pulled directly over the prosthetic hand (without inner hand) and protects the components underneath against moisture and dirt. The appearance is based on the contralateral side in order to achieve a high degree of naturalness.

The material is coloured by hand, giving rise to possible variations in appearance, which are not a product defect.

The fingernails **cannot** be painted.

2 Intended use

2.1 Indications for use

The product is to be used solely for upper limb exoprosthetic fittings.

2.2 Area of application

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

2.3 Combination possibilities

Limited combination options

Compatible prosthetic hands: 8K35*

2.4 Environmental conditions

Allowable environmental conditions
Allowable operating temperature: -10 °C to +60 °C (+14 °F to +140 °F)
Allowable storage temperature: -20 °C to +40 °C (-4 °F to +104 °F)
Allowable transport temperature: -20 °C to +60 °C (-4 °F to +140 °F)
Cleaning agents: warm water and soap

Unallowable environmental conditions
Mechanical vibrations or impacts, touching of sharp-edged or pointed objects
Contact with acids, unallowable cleaning agents, abrasive media (e.g. sand), damp soil, dust, dirt, highly staining materials (e.g. printing ink, pen ink, toner, new textiles), solvents (e.g. thinner, acetone, petrol), fire, embers and unallowable heat (> +60 °C / > +140 °F), silicone spray

2.5 Lifetime

The product is a wear part, which means it is subject to natural wear and tear. The lifetime diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximal lifetime, in dependence of the level of use.

3 Safety

3.1 Explanation of warning symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

3.2 General safety instructions

Contact with cold, heat, embers or fire

Injuries (e.g. burns) and damage to the product due to unallowable environmental conditions

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

Unallowable combination of prosthetic components

Product damage due to malfunctions

- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination Possibilities".

- ▶ Verify that the prosthetic components may be combined with each other by consulting their instructions for use.

NOTICE**Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

NOTICE**Careless use of the product**

Damage to the product due to contact with pointed or sharp-edged objects

- ▶ Do not allow the product to come into contact with the aforementioned objects.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	647G1763=all_INT
1	MySkin Move prosthesis cover	88A16=*

5 Preparing the product for use

NOTICE**Incorrect donning and removal of the prosthesis cover**

Mechanical damage to the prosthetic hand and prosthesis cover

- ▶ Observe all instructions for donning and removal. The prosthetic hand must always be mounted on the donning tool.
- ▶ Only use the materials listed.
- ▶ Avoid damage that can be caused to the prosthetic hand and the prosthesis cover, e.g. from tools and excessive force (in particular on the thumb).

5.1 Preparations for donning and removal

Prosthetic hand with quick-disconnect wrist unit

- > **Required materials (see fig. 1):** ①: 711M3 mounting tool, ②: 711M1 mounting tool (alignment rod), ③: 711M2 mounting plate or ④: vice
- 1) Install the prosthetic hand on the mounting tool.
 - 2) Install the mounting tool on the alignment rod on the side with the interior thread.
 - 3) Attach the alignment rod to the mounting plate or clamp it in a vice.

5.2 Pulling on the prosthesis cover

- 1) **NOTICE! Do not use silicone spray for pulling on the prosthesis cover. Optionally, 640F18 donning spray can be used. Do not glue the prosthesis cover to the prosthesis.**
- 2) Make sure that the prosthetic hand is in the half-open state and that the thumb is in opposed position (see fig. 2).
- 3) Turn the cover inside out up to the middle joints, position the cover on the prosthesis and pull it over the prosthetic hand (see fig. 3). In doing so, ensure that the fingers of the prosthetic hand are positioned in the openings provided in the prosthesis cover.
- 4) Pull the cover fully over the prosthesis and align it (see fig. 4).

5.3 Pulling off the prosthesis cover

- 1) **NOTICE! Avoid all excessive force on the fingers during the removal process. Do not fold over the sleeve end of the the prosthesis cover for removal.**
- 2) Make sure that the prosthetic hand is in the open state and that the thumb is in opposed position.
- 3) Carefully hold the prosthesis cover and pull it off.

6 Use

INFORMATION

Minor damage to the product can be repaired. Contact the manufacturer to discuss the further procedure.

6.1 Cleaning

NOTICE

Non-observance of cleaning instructions

Damage to product and permanent soiling due to use of improper cleaning agents

- ▶ Only clean the product according to the cleaning instructions provided in the instructions for use.

NOTICE

Penetration of moisture

Loss of or limited functionality of the prosthesis

- ▶ Make sure that no water gets into the prosthesis while cleaning the prosthesis cover. The prosthesis cover is not firmly secured to the socket, so that water can get into the prosthesis.

INFORMATION

The product must be cleaned and inspected for damage daily.

- > **Recommended cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)
- 1) Clean the product with clear fresh water and a pH neutral soap.
- 2) Rinse the soap away with clear fresh water.
- 3) Dry the product with a soft cloth.
- 4) Allow to air dry in order to remove residual moisture.

7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

8 Legal information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the

information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

Ottobock SE & Co. KGaA
Max-Näder-Straße 15 · 37115 Duderstadt · Germany
T +49 5527 848-0 · F +49 5527 848-3360
healthcare@ottobock.de · www.ottobock.com