**USER MANUAL/GEBRAUCHSANLEITUNG/
MANUALE D'USO/MANUEL D'EMPLOI**

(EN - DE - IT - FR)

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KLAXON Mobility GmbH - EN

Dear customer,

Thank you for choosing a KLAXON® product.

We sincerely hope that the KLAXON MOBILITY products can improve your day and your activities in many ways, and we remain at your disposal for any information or clarification you may require. We also encourage you to keep up to date on the latest news of our product range, on the accessories available and on our company's events and initiatives.

You can contact us by writing an e-mail to info@klaxon-klick.com, through our web site at www.klaxon-klick.com, on our LinkedIn page www.linkedin.com/company/klaxon-mobility or on our Facebook page www.facebook.com/klaxonklickworld.

	The linking system described in this manual is considered to be an integral part of the KLICK devices with which it is associated.
	The linking system described in this manual cannot be used separately from the KLICK devices for which it was developed or with devices other than the KLICK devices for which it was developed.
	For details about the compatibility of the linking system described in this manual with the KLAXON products, please contact an authorised KLAXON dealer.
	For compliance with the regulatory and normative requirements, please refer to the END USER MANUAL supplied with the KLICK device.
	EU WARNING: In the event of any serious accident related to the KLICK device, the user must immediately notify the manufacturer and the competent authority of the European country where the user has bought the product.
NOTE	For visually impaired people, a pdf format of this manual is available on our website www.klaxon-klick.com .
NOTE	You can find the addresses of Klaxon Dealers on our website www.klaxon-klick.com in the "Klick Stores" area.

Thank you again for having chosen our products. We wish you a good day and enjoyable use.

Best regards,

The Staff of KLAXON MOBILITY GMBH

KLAXON Mobility GmbH - EN

CONTENTS

PRELIMINARY INFORMATION	PAGE 3
1. WARNINGS	PAGE 3
2. SAFETY NOTICES	PAGE 4
3. INTENDED USE OF THE DEVICE	PAGE 4
4. USAGE ENVIRONMENT	PAGE 4
5. OPERATING LIMITS	PAGE 5
6. USER REQUIREMENTS	PAGE 5
7. WARNINGS AND PRECAUTIONS	PAGE 5
8. DESCRIPTION OF THE LINKING SYSTEM	PAGE 8
8.1 LATERAL LINKING SYSTEM KLAXON®	PAGE 8
8.2 DOCUMENTATION	PAGE 8
9. INSTALLATION AND COUPLING	PAGE 8
9.1 INSTALLATION	PAGE 8
9.2 WHEELCHAIR	PAGE 8
9.3 COUPLING STEPS - STANDARD HANDLEBAR	PAGE 9
9.4 COUPLING STEPS - TETRA HANDLEBAR	PAGE 9
10. KLICK GENERAL OPERATION AND HANDLEBAR SETUP	PAGE 10
11. UNLINKING THE KLICK DEVICE	PAGE 10
11.1 TURNING OFF AND RELEASING FROM THE WHEELCHAIR - STANDARD HANDLEBAR	PAGE 10
11.2 TURNING OFF AND RELEASING FROM THE WHEELCHAIR - TETRA HANDLEBAR	PAGE 11
12. CLEANING	PAGE 11
12.1 DISINFECTION	PAGE 12
13. SERVICING	PAGE 12
13.1 PERIODICAL MAINTENANCE	PAGE 12
13.2 MALFUNCTIONS	PAGE 12
13.3 SERVICE LIFE OF THE PRODUCT	PAGE 13
14. DISPOSAL OF THE DEVICE	PAGE 13
15. PRODUCT WARRANTY	PAGE 13
16. PROPERTY TRANSFER	PAGE 13
17. ORIGINAL TEXT	PAGE 13

PRELIMINARY INFORMATION

INTRODUCTION

This User's Manual explains how to use the lateral Linking System Klaxon®. The KLICK product involved in the explanation of this manual are:

- POWER (P) that can be equipped with the STANDARD or TETRA handlebar
- RACE (R) that can be equipped with the STANDARD or TETRA handlebar
- MONSTER (M) that can be equipped with the STANDARD handlebar

MANUFACTURER'S IDENTIFICATION DATA

KLAXON MOBILITY GMBH
 Industriestrasse, 1
 9601 - Arnoldstein (Austria)
 Tel.: +43 (0)664 4681294
 e-mail: info@klaxon-klick.com
 Hereinafter: **KLAXON**.

REVISION INFORMATION

REV.	Description of the change	DATE
01	Initial release	2021.02.09

All information, illustrations and specifications in this document are based on the latest product information available at the time of publication. This document is subject to change due to technical innovations, without prior notice.

1. WARNINGS

- Read these instructions carefully before using the device.
- KLAXON is not liable for any failure of the end user to comply with the contents of this document.
- The use of KLICK devices is specifically dedicated to persons with disabilities who are using a manual wheelchair.
- The use of KLICK devices in public spaces (streets, sidewalks, squares, cycle paths, etc.) must comply with the specific regulations of the country in which they are used. KLAXON is not liable for use that is not in accordance with specific regulations in force.
- If any anomalies that may endanger the user are found during the usage: do not use the device and promptly contact KLAXON's technical support.
- Non-original spare parts and accessories have not been tested by the manufacturer. Therefore we cannot certify that these components are compliant with the needed performance and safety requirements.
- KLAXON is not responsible for any damage caused by the use of non-original spare parts or accessories.
- The installation of the product and its maintenance must be carried out exclusively by authorised KLAXON technicians, who will issue a declaration of compliance. The user must request the declaration of compliance during the first activation and after every maintenance. If the device does not comply with the declaration, the warranty will expire with immediate effect and KLAXON will not be liable for any damage resulting from the use of the device.
- Not all the provisions included in this manual are valid for all the countries and for all the KLICK models. Please always refer to the local regulations, settings and models that are provided and applied in the country of purchase and of use.

The points of the manual marked with the following symbols should be read carefully:

DANGER	The word DANGER indicates a potentially hazardous situation, which, if not avoided, can result in death, serious injury and/or property damage.
CAUTION	The word CAUTION indicates a potentially hazardous situation, which, if not avoided, may cause minor personal injuries or property damage.
NOTE	The word NOTE indicates important instructions or special information.
	Generic warning sign: indicates a danger due to various causes.
	High voltage warning sign: indicates the presence of high voltages.

2. SAFETY NOTICE

NOTE	KLICK devices meet all the requirements of ISO 7176-14.
	The user must read this document entirely and have become familiar with its contents before using the device.
	If the device is not used respecting the specifications described herein, the security level specified by the manufacturer may be reduced.
	During the initial period of use the user must familiarize with operation to avoid potential risks caused by incorrect use.
	EMC - The device is compliant with ISO 7176-21:2009 CLAUSE 5.4 on electromagnetic compatibility, but to ensure the user's safety we recommend not using this device in the presence of strong environmental electromagnetic fields.
	EMC - During its operation, the device can affect other devices with electromagnetic field.
DANGER	Children, incapacitated persons, or those that are insufficiently familiar with the device or physically unsuited, must not use it.
NOTE	For technical and safety info you can contact a Klaxon Dealer. You can find the addresses of Klaxon dealers on our website www.klaxon-klick.com in the "Klick Stores" area.

- The device may skid on wet pavement, on gravel or on uneven terrain: always adjust speed and driving style to the conditions (weather, surface, individual capacity, etc.); never drive straight towards an object, and drive carefully towards narrow passages.
- Do not attempt to go over steps higher than 50 mm, both uphill and downhill, along the direction travelled. Do not drive the KLICK-equipped wheelchair on roads with slopes that exceed 6° / 10%. These obstacles must always be overcome frontally and slowly. The proper way to go over steps and to go up slopes is shown in Figure 1.
- If the pavement is wet or uneven, the maximum rated slope and the maximum height of the steps allowed must be considered lower than those previously indicated in order to ensure the safety of the user.
- In the event of a collision, the user could suffer serious injuries to its body parts.
- At high speed the user may lose control of the wheelchair and it may tip over. Never drive faster than 15 km/h (both uphill and downhill), and avoid impacts in general.
- Always use the KLICK-equipped wheelchair suitably. Always brake in order to avoid obstacles (steps, edges, door frames, etc.), during descents, curves and/or dangerous slopes.
- Driving performance of the KLICK-equipped wheelchair may be affected by electromagnetic fields. If a malfunction occurs in these conditions please contact KLAXON or one of its authorised resellers.
- Avoid contact with any liquid coming out of the battery if it is damaged.
- While using the device, some system components can get very hot. A contact of these components with the skin may cause scalding and burns: be extremely careful to avoid contact.
- Always observe the operating limits for the manual wheelchair on which the KLICK system is installed.

3. INTENDED USE OF THE DEVICE

KLICK devices can be used to add an auxiliary power assist system to most manual wheelchairs, quickly and easily.

Coupling the KLICK device to the wheelchair raises the front castors off the ground. The single wheel of the "traction unit" is then used for steering.

The system consisting of the Klick device coupled to the manual wheelchair falls within Class B according to CEI EN 12184. The Klick devices meet the requirements of Class B as they provide a maximum autonomy of approximately 50 km (in the version equipped with a fully charged 11.6 Ah battery, driven on a flat course by an hypothetical user weighing 75 kg).

In accordance with the definition of class B of CEI EN 12184, the device is for indoor use and can handle outdoor obstacles.

	Any other use or misuse could result in hazardous situations.
---	---

4. USAGE ENVIRONMENT

Operating Environment

KLICK series devices are made for use coupled to most manual wheelchairs on the market. KLICK solutions can be used both indoors and outdoors, with the limits stated in this document, and in general with the same limitations set for the use of the manual wheelchair.

Medical use and types of users

KLICK is not intended for specific clinical use, but as a support to the mobility of persons who use manual wheelchairs. Therefore, bearing in mind the operating precautions described in this document, there is no need for professional, technical or aptitude requirements to operate or use a KLICK device.

5. OPERATING LIMITS

The device must be used consistently with the intended use described in the preceding paragraphs, any other use is considered as improper.

In particular do not use KLICK devices:

- If not coupled with a manual wheelchair.
- After intake of alcohol or if the user's psycho-physical conditions are unsuitable.
- On roads with unsuitable pavement.
- In poor visibility (at night, in fog, etc.).
- If the user weighs over 120 kg or is heavier than allowed for the wheelchair used.
- In adverse weather (rain, strong wind, etc.).
- When going uphill or downhill on slopes that are exceeding 6° / 10%. (see Figure 2).
- To go over steps higher than 50 mm, both uphill and downhill. (see Figure 3).

6. USER REQUIREMENTS

KLICK series devices and its accessories can be used independently by users that:

- That are familiar and have practised proper use of the device sufficiently.
- That are aware of the risks related to the use of the device.
- That have sufficient physical and mental capacity to ensure the appropriate safe use of the device.

7. WARNINGS AND PRECAUTIONS

	This User's Manual is an integral part of the device and must always be available to the user. If it is lost or damaged, contact KLAXON or download the manual from the specific product webpage on www.klaxon-klick.com .
	KLICK series devices and accessories can be used by users who are aware of the risks associated with the use thereof.
	KLICK series devices and accessories must be used only and exclusively by users that are familiar and have practised proper use of the device sufficiently.
	Absolutely never remove, modify or replace components of the device. If necessary, these operations must be carried out only by technical personnel authorised to do so by KLAXON.
	Never use the device after having drunk alcohol, taken drugs and in general if the user's psycho-physical conditions are unsuitable.
	The use of KLICK devices is prohibited for users weighing more than 120 kg; the total weight of the user must not exceed the quantity specified by the manufacturer of the wheelchair in use.
	The user is responsible for monitoring and maintaining proper operation of the devices and of their accessories as indicated herein.
	Absolutely never use the device outside of the stated environmental operating conditions.
	Periodic maintenance must be performed to ensure proper operation and the safety of the device.
	Do not use a KLICK device to carry more than one person.

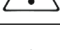
KLAXON Mobility GmbH - EN

	Do not use a KLICK device unless it is coupled with a manual wheelchair.
	Do not use the supplied battery pack to power systems other than the KLICK device for which it was designed
	Do not leave the battery in the sun or adverse weather when not in use.
	Do not short-circuit the battery.
	Do not throw the battery in water.
	Do not attempt to open the battery pack.
DANGER	Do not allow children to play with the battery.
	Do not allow children to use the KLICK device.
	If the battery has reached its end of life, it must be disposed according to the local regulations applied in the Country.
	While using the device, some system components can get very hot and contact of these components with the skin may cause scalding and burns: be extremely careful to avoid contact.
	The components can become very hot if exposed to the sunlight. Skin contact may cause scalding and burns: be extremely careful to avoid contact.

	DANGER: The voltages inside the device can be lethal.
	DANGER: Do not connect/disconnect power cables when the device is on.
	DANGER: Maintenance or cleaning the device must always be carried out when the device is off and disconnected from the mains.
	DANGER: Do not remove any of the unit's cover panel, contact the Technical Support Service.

	The use of KLICK devices in public spaces (streets, sidewalks, squares, cycle paths, etc.) must comply with the specific regulations of the country in which they are used. KLAXON is not liable for use that is not in accordance with specific regulations in force.
	Adjust speed in relation to the characteristics of the wheelchair, the road conditions, the lateral inclination of the road and the radius of the curves to take.
	Do not operate the device in poor lighting conditions. The device can only be used in low-light conditions if an optional light is mounted (according to the local country regulations). Please contact your Klaxon dealer to purchase the light accessory available as an optional. You can find the addresses of Klaxon dealers on our website www.klaxon-klick.com in the "Klick Stores" area.
	Pay attention to side-skidding due to strong wind.

KLAXON Mobility GmbH - EN

	Pay attention to side-skidding due to steering while moving.
	Pay attention to side-skidding due to the slope of the road.
	Do not use KLICK devices on roads with unsuitable pavement.
	Do not use KLICK devices with adverse weather (rain, strong wind, etc.).
	Do not use the KLICK devices to go up or down steps higher than 50 mm (see Figure 3).
	Do not use KLICK devices on slopes greater than 6° / 10% . The rated slope's limit shall be reduced in accordance with the pavement and the environmental condition (see Figure 2).
	Consider that traction and stability might be affected by the pavement and environmental condition even within the maximum rated slope value.
	When driving downhill, do not exceed two-thirds of the maximum speed; avoid sudden braking or accelerations on slopes.
	Never go over obstacles when driving uphill or downhill.
	Do not go up or down staircases.
	Do not attempt to go over obstacles diagonally: make sure that the rear wheels always go over the obstacle together. Never stop halfway. Do not attempt to go over obstacles over 50 mm high.
	When the device is in motion, avoid moving the centre of gravity or abruptly moving your body.
	It is advisable to drive through narrow passages at minimum speed and with utmost caution.
	When using KLICK devices, the risk of tipping-over during cornering increases: reduce speed before turning. Accelerate again only after completing the curve.
	When using KLICK devices, do not make U-turns or turn from a stationary start on slopes greater than 3° (or lower according to pavement condition) and at a speed higher than level 1 and 3 km/h (or lower according to pavement condition).
	It can be difficult to manoeuvre in front of a lift or at the entrance of a building since the wheelchair's turning radius increases when coupled with a KLICK device; this could conflict with the standards of the buildings or the actual size of the entrances.
	When entering a building and/or an elevator, always take into account the turning radius of your wheelchair equipped with the KLICK device. Avoid driving into situations from which you would not be capable of exiting because you would be unable to turn your wheelchair.
	Absolutely never use the Klick device along tramway, rail and light railway tracks. Pay the utmost attention at level crossings, keeping the Klick's and wheelchair wheels at due distance. Failure to comply with this precaution could cause serious damages to the device and the user. The same precautions apply to water drains, manholes and all kinds of hollows along the way.
DANGER	Consider that the stopping distance on slopes can be significantly greater than on a flat ground.

KLAXON Mobility GmbH - EN

	Switch off the device before entering and before leaving an indoor ambient.
	Potential finger traps are shown in Figure 4. Potential squeezing points are shown in Figures 5 and 6.
DANGER	Do not start or use the Intelligent Cruise Control function on an uphill slope that is rated above 6°/10% (or lower according to pavement condition) and a downhill slope that is rated above 3°/5% (or lower according to pavement condition).
CAUTION	Handling points for carry the device are shown in Figure 7.

8. DESCRIPTION OF THE LINKING SYSTEM

8.1 LATERAL LINKING SYSTEM KLAXON®

See Figure 8.

A. CLAMPING RINGS

The clamping rings are the parts of the KLAXON® connection that always remain attached to the wheelchair frame (see Figure 9). They must be installed on the wheelchair by a technician authorised by KLAXON and must never be removed. Various clamping ring sizes and shapes are available to suit the different tubing diameters and types of wheelchair frames. The clamping rings can be removed and/or moved from a wheelchair to another only if the operation is performed by a Klaxon Authorized Technician

Figure 10 illustrates a CLAMPING RING and identifies the elements that make it up.

1. BRACKET
2. CLAMP
3. Clamp's lever screw

B. LATERAL LINKING SYSTEM KIT

Figure 11 illustrates the LATERAL LINKING SYSTEM KIT and identifies the elements that make it up.

1. CROSSBEAM
2. FORKS
3. FRONT HOOK 2.0

C. FRONT HOOK 2.0

Figure 12 illustrates the FRONT HOOK 2.0 in details and identifies the elements that make it up.

1. Linking/unlinking LEVER
2. SAFETY BUTTON
3. Seat of upper pin
4. Seat of lower pin.

8.2 DOCUMENTATION

This END USER'S MANUAL is provided with the chosen linking system in printed format and must be stored with the utmost care. If you lose the present document you can ask for a copy by contacting KLAXON or download the manual from the specific product webpage on www.klaxon-klick.com.

9. INSTALLATION AND COUPLING

9.1 INSTALLATION

KLICK devices are provided by a technician authorised by KLAXON only after having pre-installed the coupling set on the wheelchair.

	The linking systems can only be installed on compatible wheelchairs. KLAXON provides to the distributors the Installation Manual which contains the coupling criteria. The linking systems must be installed only by authorised KLAXON technicians who may decide, under their own responsibility, if the specific wheelchair can be properly used with that linking system (according to both wheelchair features and general condition).
---	--

9.2 WHEELCHAIR

	KLICK devices can be used only with compatible wheelchairs. KLAXON provides to the distributors the Installation Manual which contains the coupling criteria. Only authorised KLAXON technicians can decide, under their own responsibility, if the specific wheelchair can be properly used with the KLICK device (according to both wheelchair features and general condition).
---	---

9.3 COUPLING STEPS - STANDARD HANDLEBAR

	During the entire coupling operation, stop in a flat area, as free as possible from uneven ground and away from hazards that could cause injury to yourself and/or to others.
DANGER	Be careful not to squeeze or trap your fingers during the coupling operation. Potential finger traps are shown in Figure 4 Potential squeezing points are shown in Figures 5 and 6.

- Loose both the CLAMPS' lever screws enough to let the FORKS' tubes slide in easily and without interferences (Figure 13).
- Insert the FORKS' tubes into the CLAMPS according to the distance indicated by the installation technician. It is important to ensure that the FORKS protrude from the rear of the CLAMPS for at least 3 cm and to the same extent on both sides (Figure 14).
- Verify that the forks are correctly and evenly inserted. In case something is wrong correct the insertion and then proceed as follows.
- Tight both the CLAMPS' lever screws until the forks are safely and strongly fixed (Figure 15).
- Verify that the FORKS are correctly secured so that they cannot move and so that they cannot slide out of the CLAMPS during use. In case the FORKS are not fixed enough, tighten more the CLAMPS' lever screws until they meet the requirement then proceed as follows.
- After having inserted the FORKS, the device will be as shown in Figure 16.
- To secure the KLICK device to the connection, move the wheelchair into position as shown (Figure 17 and 18). Make sure that the upper pin is placed on the FRONT HOOK's upper pin seat (Figure 19).
- Holding it in place, brake the wheelchair with its brakes and lock the connection to the upper pin by pushing the linking LEVER upward (Figure 20).
- Proceed according to Option 1 or Option 2.

OPTION 1: STANDARD LINKING OPERATION

- At this point, grab the handlebar and raise it upward and forward until the lower pin is fully engaged; at that point the SAFETY BUTTON is locked (Figure 21).
- The KLICK is connected.

OPTION 2: REVERSE GEAR OPERATION

- Turn on the KLICK device and activate the REVERSE GEAR (as explained in the End User's Manual of the KLICK devices).
- At this point, use the reverse gear gently to lift the KLICK until the lower pin is fully engaged; at that point the SAFETY BUTTON is locked (Figure 22).
- The Klick is connected.

DANGER!

Once the device is connected to the wheelchair, it is important to ensure that:

- The ground clearance of the front wheels of the wheelchair should be between 25mm and 40 mm (Figure 23).
- The kickstand is located above the line of the front wheels and placed below the footrest (Figure 24).

DANGER	If these requirements are not met, never use the device for any reason and immediately contact an authorised KLAXON service centre.
--------	---

9.4 COUPLING STEPS - TETRA HANDLEBAR

	During the entire coupling operation, stop in a flat area, as free as possible from uneven ground and away from hazards that could cause injury to yourself and/or to others..
DANGER	Be careful not to squeeze or trap your fingers during the coupling operation. Potential finger traps are shown in Figure 4. Potential squeezing points are shown in Figures 5 and 6.

- Loose both the CLAMPS' lever screws enough to let the FORKS' tubes slide in easily and without interferences (Figure 13).
- Insert the FORKS' tubes into the CLAMPS according to the distance indicated by the installation technician. It is important to ensure that the FORKS protrude from the rear of the CLAMPS for at least 3 cm and to the same extent on both sides (Figure 14).
- Verify that the FORKS are correctly and evenly inserted. In case something is wrong correct the insertion and then proceed as follows.
- Tight both the CLAMPS' lever screws until the forks are safely and strongly fixed. (Figure 15).
- Verify that the FORKS are correctly secured so that they cannot move and so that they cannot slide out of the CLAMPS during use. In case the FORKS are not fixed enough, tighten more the CLAMPS' lever screws until they meet the requirement then proceed as follows.

- After having inserted the FORKS, the device will be as shown in Figure 16.
- To secure the KLICK device to the connection, move the wheelchair into position as shown (Figure 17 and 18). Make sure that the upper pin is placed on the FRONT HOOK's upper pin seat (Figure 19).
- Holding it in place, brake the wheelchair with its brakes and lock the connection to the upper pin by pushing the linking LEVER upward (Figure 20).
- Proceed according to Option 1 or Option 2.

OPTION 1: STANDARD LINKING OPERATION

- Release the wheelchair's brake
- At this point, grab firmly the TETRA HANDLEBAR and push it forward until the lower pin is fully engaged; at that point the SAFETY BUTTON is locked (Figure 25).
- The KLICK is connected.

OPTION 2: REVERSE GEAR OPERATION

- Turn on the KLICK device and activate the REVERSE GEAR (as explained in the End User's Manual of the KLICK devices).
- At this point, use the reverse gear gently to lift the KLICK until the lower pin is fully engaged; at that point the SAFETY BUTTON is locked (Figure 26).
- The Klick is connected.

DANGER!

Once the device is connected to the wheelchair, it is important to ensure that:

- The ground clearance of the front wheels of the wheelchair should be between 25mm and 40 mm (Figure 23).
- The kickstand is located above the line of the front wheels and placed below the footrest (Figure 24).

DANGER	If these requirements are not met, never use the device for any reason and immediately contact an authorised KLAXON service centre.
--------	---

10. KLICK GENERAL OPERATION AND HANDLEBAR SETUP

For information related to the user interface, the display, turning on/off the device and setting up the device and all the handlebar set up please refer to the END USER'S MANUAL provided together with the KLICK device. If you have lost this document, you can retrieve a copy by contacting KLAXON or by downloading them from our website at www.klaxon-klick.com.

11. UNLINKING THE KLICK DEVICE

11.1 TURNING OFF AND RELEASING FROM THE WHEELCHAIR - STANDARD HANDLEBAR

	During the entire releasing operation, stop in a flat area, as free as possible from uneven ground and away from hazards that could cause injury to yourself and/or to others.
DANGER	Be careful not to squeeze or trap your fingers during the releasing operation. Potential finger traps are shown in Figure 4. Potential squeezing points are shown in Figures 5 and 6.

To perform the operation properly, carefully follow the following instructions.

- Stop the wheelchair on a flat surface.
- Apply the brake on the wheelchair.
- Turn off the KLICK by pressing and holding the "POWER" button.
- Push the handlebar forward with one hand grasping it in the middle and simultaneously move the unlinking LEVER upward with the other hand. This frees the lower pin of the KLICK coupling (Figure 27).
- Reduce forward pressure on the handlebar, thus lowering the wheelchair so that its front wheels make contact with the ground (Figure 28).
- The lower pin is now unlinked from the system.
- Proceed according to Option 1 or Option 2.

OPTION 1: FULL UNLINKING

- Push the SAFETY BUTTON in order to free the upper pin (Figure 29).
- Release the wheelchair's brakes and push yourself back to move away from the KLICK device (which in the meantime will remain upright thanks to its KICKSTAND - Figure 30).
- Loosen the CLAMPS' lever screws until the forks can slide out easily and without interferences (Figure 15).

- Remove the LATERAL LINKING SYSTEM KIT by sliding the FORKS out of the CLAMPS (Figure 13).
- At this point the lateral linking system is disconnected both from the KLICK and the wheelchair as shown in Figure 31.
- Store the LATERAL LINKING SYSTEM KIT in a safe place.

OPTION 2: PARTIAL UNLINKING

- Loose the CLAMPS' lever screws until the forks can slide out easily and without interferences (Figure 15).
- Release the wheelchair's brakes and push yourself back to move away from the LATERAL LINKING SYSTEM KIT, which will slide out of the CLAMPS together with the FORKS (Figure 32). Make sure that the FORKS descend gently.
- The KLICK device will remain upright thanks to its KICKSTAND with the LATERAL LINKING SYSTEM KIT is still connected to it (Figure 33).

DANGER	During the KLICK release operation, pay attention to not endanger the top parts of your body (fingers, hands, arms) as they could be affected by the sudden movement of the handlebar (or other parts of the device) if it is released too quickly. To avoid this danger, after pressing the release button, gently release the handlebar to lower the wheelchair gradually.
--------	--

11.2 TURNING OFF AND RELEASING FROM THE WHEELCHAIR - TETRA HANDLEBAR

	During the entire releasing operation, stop in a flat area, as free as possible from uneven ground and away from hazards that could cause injury to yourself and/or to others.
DANGER	Be careful not to squeeze or trap your fingers during the releasing operation. Potential finger traps are shown in Figure 4. Potential squeezing points are shown in Figures 5 and 6.

To perform the operation properly, carefully follow the following instructions.

- Stop the wheelchair on a flat surface.
- Turn off the KLICK by pressing and holding the **"POWER"** button.
- Push the TETRA handlebar forward with one hand grasping it in the middle and simultaneously move the unlinking LEVER upward with the other hand. This frees the lower pin of the KLICK coupling (Figure 34).
- Reduce forward pressure on the TETRA handlebar, thus lowering the wheelchair so that its front wheels make contact with the ground (Figure 35).
- The lower pin is now unlinked from the system.
- Proceed according to Option 1 or Option 2.

OPTION 1: FULL UNLINKING

- Push the SAFETY BUTTON in order to free the upper pin (Figure 29).
- Push yourself back to move away from the KLICK device (which in the meantime will remain upright thanks to its KICKSTAND - Figure 36).
- Loose the CLAMPS' lever screws until the forks can slide out easily and without interferences (Figure 15).
- Remove the LATERAL LINKING SYSTEM KIT by sliding the FORKS out of the CLAMPS (Figure 13).
- At this point the lateral linking system is disconnected both from the KLICK and the wheelchair as shown in Figure 37.
- Store the LATERAL LINKING SYSTEM KIT in a safe place.

OPTION 2: PARTIAL UNLINKING

- Loose the CLAMPS' lever screws until the forks can slide out easily and without interferences (Figure 15).
- Push yourself back to move away from the LATERAL LINKING SYSTEM KIT, which will slide out of the CLAMPS together with the FORKS (Figure 38). Make sure that the FORKS descend gently.
- The KLICK device will remain upright thanks to its KICKSTAND with the LATERAL LINKING SYSTEM KIT is still connected to it (Figure 39).

DANGER	During the KLICK release operation, pay attention to not endanger the top parts of your body (fingers, hands, arms) as they could be affected by the sudden movement of the handlebar (or other parts of the device) if it is released too quickly. To avoid this danger, after pressing the release button, gently release the handlebar to lower the wheelchair gradually.
--------	--

12. CLEANING

Regularly cleaning the linking system makes it last longer and work better.

We recommend:

- To always switch the device off and to disconnect the battery charger before cleaning.
- To not use abrasive cleaning agents, detergents and high pressure cleaners.
- To clean the linking system with a soft, damp cloth.

- To dry the linking system thoroughly after using it out in the rain.
- In case the linking system is dirty, to soften the dirt and to remove it as soon as possible; then to dry the system carefully.
- To clean the linking system at least once a month and after each ride on muddy terrains.
- To not wash the parts under running water.
- To dry the parts immediately after washing by using a soft cloth.

12.1 DISINFECTION

Disinfecting the KLICK device and the linking system is as important as the regular cleaning operation described in this chapter. In order to do so please proceed by using a detergent that complies with the disinfection procedures that respects the materials and components of the device. If in doubt, please refer to the instructions on the packaging of the chosen disinfectant. The disinfection must be done with certified, tested and of proven effectiveness disinfection products that are complying with your national hygiene standards. A list of the currently approved and certified disinfectants can be found on the websites of renowned institutions such as, for example, The European Centre for Disease Prevention and Control (www.ecdc.europa.eu), Centers for Disease Control and Prevention (www.cdc.gov), Robert Koch Institute's website (www.rki.de) and similar.

13. SERVICING

The KLICK device and the linking system must be serviced regularly, to ensure safe, reliable and efficient operation.

	KLAXON is not responsible for any damage caused by non-maintenance or by insufficient maintenance. In such cases the warranty will be voided. The use of the linking system without carrying out the maintenance procedures is considered as improper and thus voids the warranty. The user is fully and solely responsible for any injury or damage resulting from a malfunction of the linking system due to not performing the mandatory maintenance.
	KLICK device and the linking system must be serviced by KLAXON's authorized technicians only. Unscheduled maintenance can be carried out only by KLAXON's authorized technicians.
	Do not remove any component from the device. Only KLAXON's authorized technicians can remove components from Klick devices. The only component that can be removed by the user is the battery.
	It is mandatory to verify the correct installation after one month. In order to do so please contact an authorised KLAXON technician. This operation will check the system settlement.
NOTE	Please contact a Klaxon Dealer to know whether a spare part is available or not.

13.1 PERIODICAL MAINTENANCE

The following periodical maintenance activities are mandatory:

	At least once every two months, check the tightness of all screws and fasteners.
	Have a KLAXON authorized technician overhaul the entire device at least once a year, and/or whenever a malfunction occurs.

13.2 MALFUNCTIONS

If you encounter a malfunction during use of the KLICK device, contact KLAXON or one of its authorised resellers.

Malfunction found	Possible cause	Corrective action
It vibrates excessively during use	Loose screws	Do not use the device. Contact an authorised KLAXON service centre
You have problems coupling the traction unit to the front hook 2.0	The front hook 2.0 is not adjusted properly	Do not use the device. Contact an authorised KLAXON service centre
	The safety button is not working correctly	
You have problems inserting the forks	Something obstructs the clamp	Remove the obstruction from the clamp
	The clamp's lever screw is not unscrewed sufficiently	Loose the clamp's lever screw until the forks can easily be inserted

KLAXON Mobility GmbH - EN

Wheelchair front wheels touch the ground	The linking system is mounted too low	Do not use the device. Contact an authorised KLAXON service centre
--	---------------------------------------	--

	Never use the Klick device if the traction unit and/or the linking system are behaving abnormally. Immediately contact a Klaxon Dealer.
---	---

13.3 SERVICE LIFE OF THE PRODUCT

After careful analysis of the market, the technologies used, the state of the art, the intended use and the maintenance specifications KLAXON has estimated that the product has a service life of approximately 5 years. KLAXON would like to emphasize, in a very clear way, that this period is still an estimation and that it can be extended beyond this date if the user follows the correct maintenance procedures, uses the product properly and with the appropriate care.

If the product reaches the end of this period please contact your health insurance, local health service, KLAXON dealer or the shop where the product was purchased.

14. DISPOSAL

	The disposal of the linking system must absolutely comply with regulations of the country in which the devices are used. KLAXON is not responsible for disposal that does not comply with the specific local regulations in force.
---	--

15. PRODUCT WARRANTY

The warranty period is 24 months for the traction device and the linking system.

Natural wear and tear of parts is not covered by the warranty, except for improper wear due to a manufacturing defect.

If a defect occurs during the warranty period, KLAXON will, at its discretion, repair or replace the defective component.

The manufacturer assumes no liability for damages caused by negligence, careless use and maintenance, tampering or incorrect maintenance performed by unauthorised persons, or for failure to follow the instructions for use set out in this Manual.

16. PROPERTY TRANSFER

If the KLICK device and the linking system have been provided by a medical insurance or national sanitary system and is not needed anymore, please contact the involved insurance company, local sanitary system or your local Klaxon Dealer. This action will lead to the assignment of the KLICK device, together with the linking system, to a new owner.

Everytime the KLICK device and/or the linking system changes owner, even in the case of transfer of property through private negotiations, it must undergo maintenance and disinfection prior its first use. Please contact an authorized Klaxon Dealer to perform the maintenance. Please follow the correct cleaning and disinfection procedures as explained on the chapter 12 of the present manual.

17. ORIGINAL TEXT

This document is a translation of the original version written in Italian. Please note that in case of discrepancies between the two versions, the Italian version will be the one considered as official and valid.

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KLAXON MOBILITY GmbH