50S12-1


50S14-1

Malleo Immobil Air Walker High/Low

DE Gebrauchsanweisung	4
EN Instructions for use	8
FR Instructions d'utilisation	12
IT Istruzioni per l'uso	17
ES Instrucciones de uso	21
PT Manual de utilização	25
NL Gebruiksaanwijzing	30
SV Bruksanvisning	34
DA Brugsanvisning	38
NO Bruksanvisning	43
FI Käyttöohje	47
PL Instrukcja użytkowania	51
HU Használati utasítás	55
CS Návod k použití	60
RO Instrucțiuni de utilizare	64
HR Upute za uporabu	68
SL Navodila za uporabo	72
SK Návod na používanie	76
BG Инструкция за употреба	81
TR Kullanma talimatı	85
EL Οδηγίες χρήσης	89
RU Руководство по применению	94
ZH 使用说明书	99

Malleo Immobil	Art-Nr.	Max. Körpergewicht / User weight limit	Schuhgröße / Shoe Size		
			EU	US	
				Männer / Men's	Frauen / Women's
Air Walker High	50S12-1=S	200 kg 440 lbs	35-37	3½-5	½-6
	50S12-1=M		38-42	5½-8½	6½-9½
	50S12-1=L		43-45	9-11	10-12
	50S12-1=XL		46-50	11½-15	12½-13
Air Walker Low	50S14-1=S	200 kg 440 lbs	35-37	3½-5	½-6
	50S14-1=M		38-42	5½-8½	6½-9½
	50S14-1=L		43-45	9-11	10-12
	50S14-1=XL		46-50	11½-15	12½-13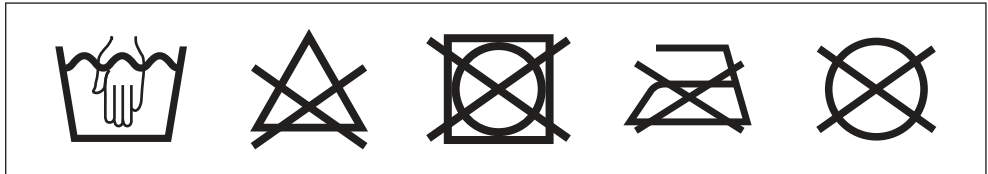

Material	Liner: PA, PU Schaumstoff/PU Foam Walker: Glasfaserverstärkter Kunststoff/glass-fibre reinforced plastic
-----------------	---

INFORMATION

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INFORMATION

Date of last update: 2022-07-20

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the 50S12-1 Malleo Immobil Air Walker High and 50S14-1 Malleo Immobil Air Walker Low immobilisation orthoses.

2 Intended use

2.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the lower limbs and **exclusively** for contact with intact skin.

The orthosis must be used in accordance with the indications.

2.2 Indications

50S12-1 Malleo Immobil Air Walker High

- Postoperative, conservative treatment of stable forefoot, metatarsal and/or ankle fractures
- Postoperative, conservative treatment of distal fibula fractures
- Soft tissue injuries of the foot
- Severe ankle sprains

50S12-1 Malleo Immobil Air Walker High with 29S35 heel wedge

- Postoperative, conservative treatment of Achilles tendon ruptures

50S14-1 Malleo Immobil Air Walker Low

- Postoperative, conservative treatment of stable forefoot, metatarsal and/or ankle fractures
- Soft tissue injuries of the foot
- Exostosis treatment for hallux valgus

Indications must be determined by the physician.

2.3 Contraindications

2.3.1 Absolute Contraindications

Not known.

2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb; pronounced varicose veins, especially with return flow impairment, lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the support will be applied; sensory and circulatory disorders in the area of the lower leg and the foot, e.g. in case of diabetic neuropathy.

2.4 Lifetime

The product is designed for a maximum lifetime of **6 months**.

2.5 Effects

The orthosis is used to immobilise the foot/ankle joint area.

2.6 Options/spare parts (not included in the scope of delivery)

50S12-1 Malleo Immobil Air Walker High

- 29S35 Heel wedge 15°/30°
- 29S45 Night cover
- 29S38 Rain cover
- 29S142 Replacement liner

50S14-1 Malleo Immobil Air Walker Low

- 29S45 Night cover
- 29S39 Rain cover
- 29S143 Replacement liner

3 Safety

3.1 Explanation of warning symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

3.2 General safety instructions

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

⚠ CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

NOTICE

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

NOTICE

Incorrect setting

Damage to the orthosis due to overloading of the material and improper fit of the orthosis due to breakage of load-bearing components

- ▶ The orthosis may only be adjusted by qualified personnel.
- ▶ Do not make any improper changes to the settings.

NOTICE

Improper use and changes

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

NOTICE

Exposure of the product to unsuitable environmental conditions

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below $-10\text{ }^{\circ}\text{C}$ ($14\text{ }^{\circ}\text{F}$) or above $+60\text{ }^{\circ}\text{C}$ ($140\text{ }^{\circ}\text{F}$) (e.g. sauna, excessive sunlight, drying on a radiator).

4 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel according to the instructions of the treating physician.
- ▶ Consult a physician if any exceptional changes are noted (such as worsening of the complaint).

4.1 Size Selection

- ▶ Select the orthosis size based on the shoe size (see sizing table).

4.2 Adaptation

> Open all hook-and-loop closures on the orthosis.

- 1) The 50S12-1 Malleo Immobil Air Walker High can be used with the **optional** 29S35 heel wedge. To do so, position the heel wedge in the liner.

- 2) Position the foot in the liner (see fig. 1). Check the position of the heel whilst doing this!
INFORMATION: The foot angle should be 90° (except model with 29S35 heel wedge).
- 3) Close all hook-and-loop closures on the liner (see fig. 2).
- 4) Pull out the hook-and-loop pad cover by the label (see fig. 3).
- 5) Press on the lateral guides at the desired medial and lateral positions (see fig. 4).
→ The liner is secured in the Walker.
- 6) Close all hook-and-loop closures on the Walker one after the other, starting at the top (see fig. 5).

4.3 Application

⚠ CAUTION

Incorrect or excessively tight application

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

⚠ CAUTION

Use with heat-sensitive skin

Skin irritation due to overheating

- ▶ Do not use the product in case of a known heat allergy.
- ▶ Do not continue to use the product if skin irritation occurs.

NOTICE

Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

> Open all hook-and-loop closures on the orthosis.

- 1) Position the foot in the orthosis. Check the position of the heel whilst doing this (see fig. 1).
- 2) Close all hook-and-loop closures on the liner (see fig. 2).
- 3) Close all hook-and-loop closures on the Walker one after the other, starting at the top (see fig. 5).

Increasing compression

⚠ CAUTION

Pumping up the air cushions too much

Risk of local pressure and constriction of adjacent blood vessels and nerves due to excessively tight application

- ▶ Only fill the air cushions with enough air to impede relative movements between the body part and the orthosis.

- 1) Push the pump (see fig. 6, item 1) as often as required to achieve the desired compression.
- 2) **Optional:** to deflate, engage the pressure valve (see fig. 6, item 2).

5 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

► Only clean the product with the approved cleaning agents.

Clean the orthosis regularly:

Liner

- 1) Remove the liner from the orthosis frame.
- 2) Fasten all hook-and-loop closures.
- 3) Hand wash the liner in warm water at **30 °C** with standard mild detergent. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

Orthosis frame

- 1) Wipe with a damp cloth as needed.
- 2) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

6 Disposal

Dispose of the product in accordance with national regulations.

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

7.2 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

7.3 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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