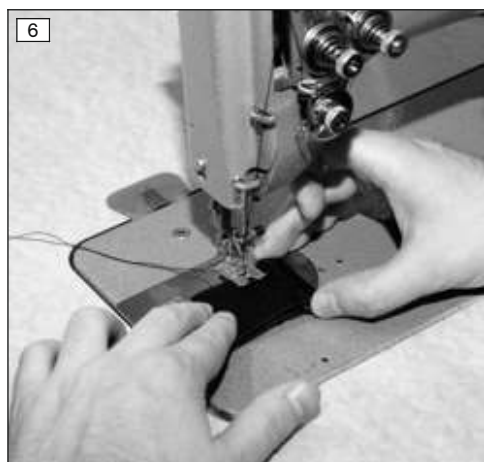
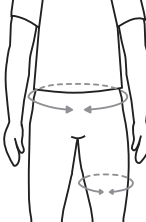
28L100 Cosa Junior, 28L101 Cosa Active

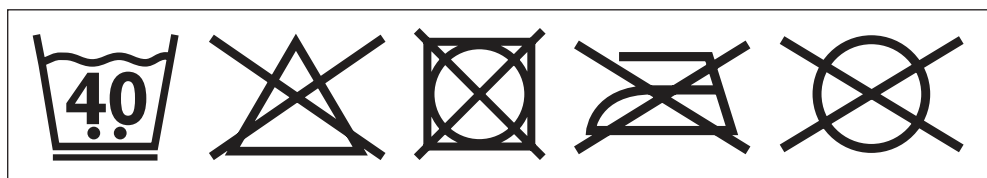
DE Gebrauchsanweisung	7
EN Instructions for use	11
FR Instructions d'utilisation	16
IT Istruzioni per l'uso	21
ES Instrucciones de uso	25
PT Manual de utilização	30
NL Gebruiksaanwijzing	35
SV Bruksanvisning	39
DA Brugsanvisning	44
NO Bruksanvisning	48
FI Käyttöohje	53
PL Instrukcja użytkowania	57
HU Használati utasítás	62
CS Návod k použití	67
RO Instrucțiuni de utilizare	71
SK Návod na používanie	76
BG Инструкция за употреба	80
EL Οδηγίες χρήσης	85
ET Kasutusjuhhis	90
RU Руководство по применению	94
ZH 使用说明书	99

	Artikelnummer / Article number	Konfektions- größe / Size	Hüftumfang / Hip circumference [cm]	Polsternummer / Pad number	Oberschenkel- umfang / Thigh circumference [cm]
	28L100=80	74-86	57-59	1*	26-32
	28L100=92	86-98	59-62		
	28L100=104	98-110	62-64	2*	32-36
	28L100=116	110-122	64-68		
	28L101=128	122-134	68-72	3*	36-44
	28L101=140	134-146	72-78		
	28L101=152	146-158	78-86	4*	44-54
	28L101=S	158-170	86-96		
		♀ 36-38			
		♂ 44-46			
28L101=M	♀ 40-42	96-105	5*	54-62	
	♂ 48-50				
	♀ 44-46	105-109			
♂ 52-54					

Material	PA, EA, PU Schaum/PU foam
-----------------	---------------------------

INFORMATION

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INFORMATION

Date of last update: 2022-10-06

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the Cosa Active and Cosa Junior abduction shorts.

2 Product description

The orthosis is supplied as a starter set in two versions:

- 28L100 Cosa Junior (shorts without zip)
- 28L101 Cosa Active (shorts with zip)

Each starter set contains:

- 2 shorts (including strap set)
- 1 abduction pad set (including instructions for use)

Article	Article no. Shorts	Quantity	Article no. Abduction pad set	Quantity	Article no. Strap set	Quantity
Cosa Junior	29L100=80	2	29L102=1	1	33L110=1	2
	29L100=92		29L102=2			
	29L100=104					
	29L100=116					
Cosa Active	29L101=128	2	29L102=3	1	33L110=2	2
	29L101=140		29L102=4			
	29L101=152					
	29L101=S					
	29L101=M		29L102=5			
	29L101=L					

3 Intended use

3.1 Indications for use

The orthosis is intended exclusively for orthotic fittings of the hip and exclusively for contact with intact skin.

The orthosis must be used in accordance with the indications.

3.2 Indications

- Diplegia, spastic
- Hip dysplasia
- Hip joint subluxation, congenital
- Hypotonicity
- Tetraplegia, spastic
- Spina bifida

3.3 Contraindications

3.3.1 Absolute Contraindications

- Hip joint luxation

3.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb/body area; lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the medical device will be applied; sensory and circulatory disorders in the leg area.

Consulting a doctor is also required in case of contractures and shortening in the area of the lumbar spine and the lower limbs.

3.4 Lifetime

The product is designed for a maximum lifetime of **2 years**.

3.5 Effects

In case of neuromuscular weakness, the orthosis prevents excessive adduction while lying down, sitting, standing and walking through the use of pads between the legs. The orthosis enables a firm basis for sitting and standing, thereby improving the torso position and leading to better balance control.

In patients who are able to walk, the orthosis improves pelvic symmetry and increases knee freedom. The scissor gait is reduced and more physiological walking is promoted.

By using the orthosis, patients who are unable to walk do not have to use separate abductors in the wheelchair. Hip abduction with the orthosis improves balance while sitting by establishing a broader seat basis. Using the hands for support is no longer necessary.

When lying down, e.g. while sleeping, the orthosis reduces muscle shortening due to spasticity.

4 Safety

4.1 Explanation of warning symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

4.2 General safety instructions

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

5 Handling

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

5.1 Size Selection

Selection according to clothes size

- ▶ Select the orthosis size based on the clothing size (see sizing table).

Selection according to circumferences

- 1) Measure the hip circumference.
- 2) Determine the size of the shorts (see size chart).
- 3) Measure the thigh circumference at the height of the pads.
- 4) Determine the size of the pads (see size chart).

5.2 Adaptation

CAUTION

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

Adapting the pads

- 1) Apply the pads to the thigh so that the guide loops face in the proximal direction (see fig. 1).
- 2) Determine the required abduction angle.
- 3) **If necessary:** Use sandpaper to roughen the surfaces of the pads and glue on additional layers (see fig. 2).
- 4) Grind the surfaces of the pads (see fig. 3).

NOTICE! Do not grind the pad surfaces further than the mark.

Adapting the belt length

- 1) Trial fit the shorts on the patient.
- 2) Mark the required length of the belt ends.
- 3) Mark the position of the guide loop (see fig. 4) and the hook-and-loop closure (see fig. 5) for sewing.
- 4) Sew the guide loop and hook-and-loop closure for the belt (see fig. 6).
- 5) **If necessary:** Adapt the fit of the shorts.

Adjusting the pads

- 1) Insert the pads into the pad pockets on the shorts so that the guide loops face in the proximal direction (see fig. 7).
- 2) Pull the guide loops of the pads out through the openings in the shorts (see fig. 8).
- 3) Close the hook-and-loop closures on the pockets.
- 4) Attach the cross straps on the guide loops of the pads (see fig. 9).
- 5) Trial fit the shorts on the patient (see fig. 10).
- 6) Determine the position of the cross straps and temporarily secure the cross straps with double hook-and-loop (see fig. 11).
- 7) Check the tension of the cross straps (see fig. 13).
- 8) Mark the position of the cross straps for later sewing (see fig. 12).

INFORMATION: Make sure that the routing of the straps is symmetrical in case of symmetrical hips (see fig. 14).

- 9) Sew the cross straps (see fig. 15).

Functional test

- 1) Verify the fit and functionality of the orthosis.
- 2) **If necessary:** Fine-tune the pads using the straps.

5.3 Application

CAUTION

Incorrect or excessively tight application

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

CAUTION

Use with heat-sensitive skin

Skin irritation due to overheating

- ▶ Do not use the product in case of a known heat allergy.
- ▶ Do not continue to use the product if skin irritation occurs.

NOTICE

Improper use and changes

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

- > The belt is open.
- > **Cosa Active only:** The zip is open.
- 1) Put on the shorts.
- 2) Fasten the belt and close the zip if applicable.
- 3) Check the positioning of the pads and readjust them with the cross straps as needed.

6 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Clean the orthosis regularly:

- 1) Loosen the cross straps.
- 2) Open the hook-and-loop closures of the pockets.
- 3) Remove the pads.
- 4) Fasten all hook-and-loop closures.
- 5) **Recommendation:** Use a laundry bag or net.
- 6) Wash the shorts in warm water at **40 °C** with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 7) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).
- 8) Put the pads back in the shorts.

7 Disposal

Dispose of the product in accordance with national regulations.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

8.2 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.3 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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