8S710=* , 8S711=* bebionic Skin Silicone

DE Gebrauchsanweisung (Fachpersonal)	2
EN Instructions for use (qualified personnel)	10
FR Instructions d'utilisation (Personnel spécialisé)	19
IT Istruzioni per l'uso (Personale tecnico specializzato)	27
ES Instrucciones de uso (Personal técnico especializado)	36
PT Manual de utilização (Pessoal técnico)	44
NL Gebruiksaanwijzing (Vakmensen)	53
SV Bruksanvisning (Fackpersonal)	61
DA Brugsanvisning (Faguddannet personale)	69
NO Bruksanvisning (Fagpersonell)	78
FI Käyttöohje (Ammattihenkilöstö)	86
HU Használati utasítás (szakszemélyzet)	94
TR Kullanma talimatı (Uzman personel)	103
JA 取扱説明書 (有資格担当者)	111

INFORMATION

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INFORMATION

Date of last update: 2022-11-02

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Construction and Function

The bebionic Skin Silicone prosthetic glove is used as a protective glove for a prosthetic hand. The multi-layered silicone material maintains its high elasticity even at low temperatures. The fingernails **cannot** be painted.

8S710 bebionic Skin Silicone Black

The 8S710 bebionic Skin Silicone is a prosthetic glove with no skin structure.

8S711 bebionic Skin Silicone

The 8S711 bebionic Skin Silicone is a prosthetic glove with skin structure.

The appearance is based on a real hand in order to achieve a high degree of naturalness.

1.2 Combination possibilities

Designation	Reference number
Prosthetic hand	8E70=*, 8E71=*, 8E72=*, BBH* bebionic Hand

2 Intended use

2.1 Indications for use

The product may only be used as part of an upper limb prosthesis with the bebionic hand.

2.2 Area of application

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

2.3 Environmental conditions

Allowable environmental conditions
Allowable operating temperature: -10 °C to +60 °C (+14 °F to +140 °F)
Allowable storage temperature: -20 °C to +40 °C (-4 °F to +104 °F)
Allowable transport temperature: -20 °C to +60 °C (-4 °F to +140 °F)
Allowable relative humidity: max. 80%
Cleaning agents: warm water and soap

Unallowable environmental conditions
Mechanical vibrations or impacts, touching of sharp-edged or pointed objects
Contact with acids, unallowable cleaning agents, abrasive media (e.g. sand), damp soil, dust, dirt, highly staining materials (e.g. printing ink, new textiles), solvents (e.g. thinner, acetone, petrol), fire, embers and unallowable heat (> +60 °C / > +140 °F)

2.4 Lifetime

The product is a wear part, which means it is subject to natural wear and tear. The lifetime diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximal lifetime, in dependence of the level of use.

3 Safety

3.1 Explanation of warning symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

3.2 General safety instructions

 CAUTION
Contact with cold, heat, embers or fire
Injuries (e.g. burns) and damage to the product due to unallowable environmental conditions
▶ Keep the product away from open flames, embers and other sources of heat.
▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").

NOTICE

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

NOTICE

Careless use of the product

Damage to the product due to contact with pointed or sharp-edged objects

- ▶ Do not allow the product to come into contact with the aforementioned objects.

NOTICE


Unallowable combination of prosthetic components

Product damage due to malfunctions


- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination Possibilities".
- ▶ Verify that the prosthetic components may be combined with each other by consulting their instructions for use.

4 Scope of Delivery and Accessories

The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering as single components (■), single components with minimum order quantities (▲) or single-component packs (●):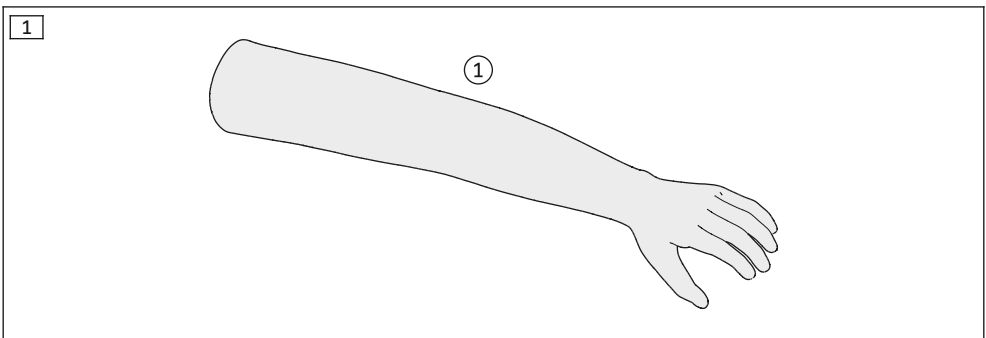

bebionic Skin Silicone scope of delivery					
Fig.	Item no.		Quantity	Designation	Reference number
-	-	■	1	Instructions for use (qualified personnel)	647G1362=all_INT
-	-	■	1	Instructions for use (patient)	647H897=all_INT
-	-	■	1	Sleeve	647G1367=all_INT
1	①	-	1	Prosthetic glove	-

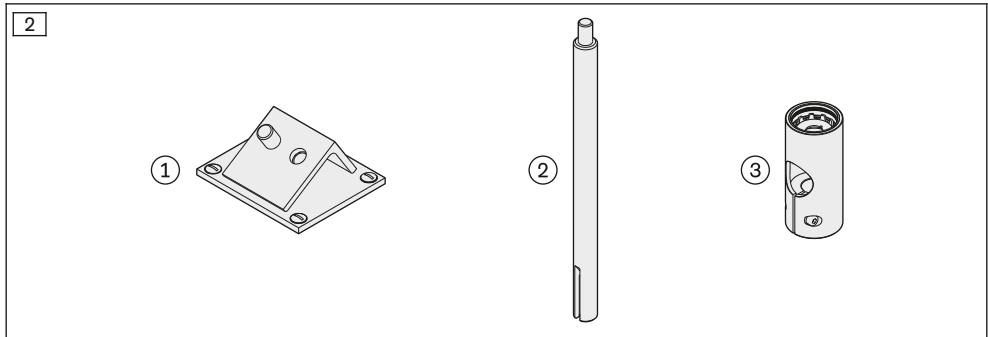Accessories					
Fig.	Item no.		Quantity	Designation	Reference number
2	①	■	0	Mounting plate	711M2
2	②	■	0	Mounting tool (alignment rod)	711M1
2	③	■	0	Mounting tool	711M3

5 Preparing the product for use

NOTICE

Incorrect donning and removal of the prosthetic glove

Damage to the prosthetic hand and the prosthetic glove due to non-observance of instructions for use

- ▶ Observe all instructions for donning and removal. The prosthetic hand must always be mounted on the donning tool.
- ▶ Only use the components listed (equipment, tools, materials) and have them close to hand.
- ▶ Avoid damage that can be caused to the prosthetic hand and the prosthetic glove, e.g. from tools and excessive force (in particular on the thumb).


5.1 Preparations for donning and removal

NOTICE

Non-observance of instructions for the donning tool

Damage to the product due to unsuitable donning tool


- ▶ Only use the donning tool specified and follow the donning instructions.

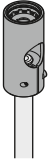- > **Tools required:**
711M3 mounting tool, 711M1 mounting tool (alignment rod) and either 711M2 mounting plate or vice
- ▶ Install the mounting tool on the alignment rod.
INFORMATION: The alignment rod must be clamped in a vice, vertically for donning and horizontally for removal.


- ▶ **When using the mounting plate:**
INFORMATION: Install the mounting plate on a suitable surface and at optimal working height.
Attach the alignment rod to the mounting plate.

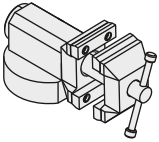
- ▶ **When using a vice:**
Clamp the alignment rod into a vice at optimal working height.


- ▶ Lock the wrist joint, if any.
- ▶ Set the prosthetic hand to hand position "Donning Mode" as follows: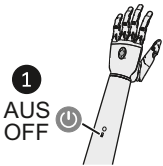


- ▶ ① Turn the prosthesis off using the switch on the prosthetic arm.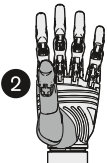


- ▶ ② Move the thumb to the opposition position.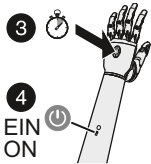

- ▶ 3 Press and hold the program switch.
- ▶ 4 Turn the prosthesis on using the switch on the prosthetic arm.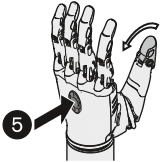

- ▶ 5 Continue to hold the program switch.
→ The thumb moves to the donning mode position.

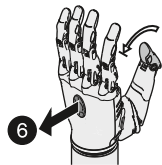- ▶ 6 Release the switch when the thumb is pointing fully inwards (donning mode position).


- ▶ Slide the prosthetic hand onto the mounting tool.

- ▶ Engage the balls by turning to the left and to the right.

5.2 Don the prosthetic glove on the prosthetic hand


- ▶ **NOTICE!** Never heat the prosthetic glove (e.g. with a blow drier or oven).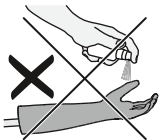

- ▶ **NOTICE!** Do not use 640F18 donning spray for application and removal.

- ▶ **NOTICE!** Do not fold over the sleeve end for application and removal.

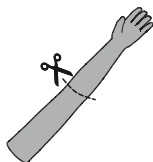- ▶ **NOTICE!** To prevent malfunctions, the prosthetic glove must not be twisted in the area of the fingers and especially the thumb.

Only shorten the prosthetic glove to the final length once it has been applied and aligned.

- ▶ Shorten the prosthetic glove to one length if necessary.

- ▶ **NOTICE!** Avoid all excessive force on the fingers (especially the thumb) during the donning process.

Position the prosthetic glove on the prosthetic hand for donning.


- ▶ Hold the prosthetic glove at the sleeve end and pull it slowly and carefully over the prosthetic hand.

- ▶ **NOTICE!** To prevent malfunctions, the prosthetic glove must not be twisted in the area of the fingers and especially the thumb.

Adjust the prosthetic glove to fit the prosthetic hand properly.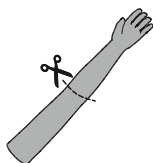


- ▶ **INFORMATION:** Shorten the prosthetic glove so that it does not lie on the skin when wearing the prosthesis.

Having no contact between the prosthetic glove and prosthetic socket after shortening will ensure the best rotation for the prosthetic wrist joint.

Shorten the prosthetic glove to the desired length.

5.3 Removal from the prosthetic hand

- ▶ **NOTICE!** Never heat the prosthetic glove (e.g. with a blow drier or oven).

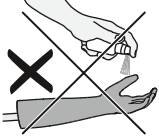- ▶ **NOTICE! Do not use 640F18 donning spray for application and removal.**


- ▶ **NOTICE! Do not fold over the sleeve end for application and removal.**

Removal with re-use of the prosthetic glove


- ▶ **NOTICE! Avoid all excessive force on the fingers (especially the thumb) during the removal process.**
Carefully pull the prosthetic glove off the prosthetic hand.

Removal without re-use of the prosthetic glove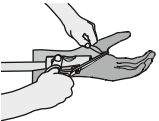

- ▶ **NOTICE! Avoid damaging the prosthetic hand during cutting.**
Cut open the prosthetic glove on the inside.

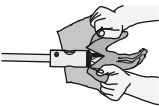- ▶ **NOTICE! Avoid all excessive force on the fingers (especially the thumb) during the removal process.**
Pull the sleeve apart and carefully pull off the prosthetic glove.

6 Use

6.1 Instructing the patient in the use of the product

INFORMATION

- ▶ Inform the patient about all the important points relevant to them that are contained in these instructions for use.
- ▶ Instruct the patient in the correct use of the product in accordance with the **Instructions for use (patient)** included in the scope of delivery.
- ▶ Hand over the Instructions for use (patient) to the patient when the briefing has been completed.

6.2 Cleaning

NOTICE

Non-observance of cleaning instructions

Damage to product and permanent soiling due to use of improper cleaning agents

- ▶ Only clean the product according to the cleaning instructions provided in the instructions for use.

NOTICE

Penetration of moisture

Loss of or limited functionality of the prosthesis

- ▶ Make sure that no water gets into the prosthesis while cleaning the prosthesis cover. The prosthesis cover is not firmly secured to the socket, so that water can get into the prosthesis.

> **Materials required:**

warm water, soap, cloth, lint-free cloth

- 1) Clean the product with warm water, soap and a cloth.
- 2) Rinse off soap residues with water.
- 3) Dry the product with a lint-free cloth and allow it to air dry fully.

7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

8 Legal information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

9 Technical data

Reference number	8S710=*
Sides	right (R), left (L)
Sizes	S, M, L
Colours	20

Reference number	8S711=*
Sides	right (R), left (L)
Sizes	S, M, L
Colours	2, 3, 4, 6, 9, 11, 14, 16

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