8S710=* , 8S711=* bebionic Skin Silicone

DE Gebrauchsanweisung (Patient)	2
EN Instructions for use (patient)	5
FR Instructions d'utilisation (Patient)	8
IT Istruzioni per l'uso (Paziente)	11
ES Instrucciones de uso (Paciente)	15
PT Manual de utilização (Paciente)	18
NL Gebruiksaanwijzing (Patiënt)	21
SV Bruksanvisning (Brukare)	24
DA Brugsanvisning (Patient)	27
NO Bruksanvisning (Pasient)	30
FI Käyttöohje (Potilas)	33
HU Használati utasítás (páciens)	36
TR Kullanma talimatı (Hasta)	39
JA 取扱説明書(患者用)	42

INFORMATION

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INFORMATION

Date of last update: 2022-11-02

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The bebionic Skin Silicone prosthetic glove is used as a protective glove for a prosthetic hand. The fingernails **cannot** be painted.

8S710 bebionic Skin Silicone Black

The 8S710 bebionic Skin Silicone is a prosthetic glove with no skin structure.

8S711 bebionic Skin Silicone

The 8S711 bebionic Skin Silicone is a prosthetic glove with skin structure.

The appearance is based on a real hand in order to achieve a high degree of naturalness.

Combination possibilities

Designation	Reference number
Prosthetic hand	8E70=*, 8E71=*, 8E72=*, BBH* bebionic Hand

2 Intended use

2.1 Indications for use

The product may only be used as part of an upper limb prosthesis with the bebionic hand.

2.2 Area of application

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

2.3 Environmental conditions

Allowable environmental conditions

Allowable operating temperature: **-10 °C to +60 °C (+14 °F to +140 °F)**

Allowable storage temperature: **-20 °C to +40 °C (-4 °F to +104 °F)**

Allowable transport temperature: **-20 °C to +60 °C (-4 °F to +140 °F)**

Allowable relative humidity: **max. 80%**

Cleaning agents: warm water and soap

Unallowable environmental conditions

Mechanical vibrations or impacts, touching of sharp-edged or pointed objects

Contact with acids, unallowable cleaning agents, abrasive media (e.g. sand), damp soil, dust, dirt, highly staining materials (e.g. printing ink, new textiles), solvents (e.g. thinner, acetone, petrol), fire, embers and unallowable heat (**> +60 °C / > +140 °F**)

2.4 Lifetime

The product is a wear part, which means it is subject to natural wear and tear. The lifetime diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximal lifetime, in dependence of the level of use.

3 Safety

3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Contact with cold, heat, embers or fire

Injuries (e.g. burns) and damage to the product due to unallowable environmental conditions

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").

NOTICE

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

4 Use

4.1 Information for everyday use

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.

- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

NOTICE

Careless use of the product

Damage to the product due to contact with pointed or sharp-edged objects

- ▶ Do not allow the product to come into contact with the aforementioned objects.

INFORMATION

The product must be cleaned and inspected for damage daily.

The sections "Area of application" and "Environmental conditions", as well as the safety information, contain important information regarding the safe use of the product and point out how damage to the product can be avoided.

The following points provide suggestions how problems that occur in daily use can be addressed:

- **Safe handling of the product**

A prosthetic glove enables the use of the prosthetic hand in everyday life. Practical exercises are required within a course of therapy to become familiar with the product, which makes it possible to use the product safely in a wide range of situations.

- **Prevention of soiling and damage**

A range of activities (e.g. in the house and garden) can cause soiling or damage to the prosthetic glove due to unallowable environmental conditions. It is nevertheless possible to perform these activities by taking corresponding protective measures (e.g. putting on working or safety gloves) and exhibiting due care and attention. Because it is not always possible to avoid light soiling, regular and immediate cleaning is required to keep the prosthetic glove clean. In order to reduce bleeding, strongly coloured clothing should be washed according to the manufacturer's recommendations before being worn for the first time. Using caution when handling coloured pens, printers and copiers as well as printed matter (e.g. newspapers, photocopies) is important in order to avoid pigment particles from permanently staining the material.

4.2 Cleaning

NOTICE

Non-observance of cleaning instructions

Damage to product and permanent soiling due to use of improper cleaning agents

- ▶ Only clean the product according to the cleaning instructions provided in the instructions for use.

NOTICE

Penetration of moisture

Loss of or limited functionality of the prosthesis

- ▶ Make sure that no water gets into the prosthesis while cleaning the prosthesis cover. The prosthesis cover is not firmly secured to the socket, so that water can get into the prosthesis.

- > **Materials required:**

warm water, soap, cloth, lint-free cloth

- 1) Clean the product with warm water, soap and a cloth.
- 2) Rinse off soap residues with water.
- 3) Dry the product with a lint-free cloth and allow it to air dry fully.

5 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

6 Legal information

6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

6.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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