moveable

DE

EN

IT

mHand Adapt

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Explanation of Symbols

CAUTION

Warnings regarding possible risks of accident or injury.

NOTICE

Warnings regarding possible technical damage.

Intended Use

The mHand Adapt is a medical device intended for use as a hand component in a prosthetic fitting for individuals with congenital defect or upper limb loss including:

- Wrist disarticulation
- Transradial amputation
- Elbow disarticulation
- Transhumeral amputation

Indication

The mHand Adapt is appropriate for prosthetic users with sufficient shoulder and/or elbow control to control the hand.

The mHand Adapt is suitable for intensive use with normal loads.

Contraindication

Users who are not able to control the movement or force needed to use the hand (very short stump or no option to lock the elbow component)

Users who require a hand that can handle high loads (lifting more than 10 kg)

Function

The mHand Adapt can grasp and hold a variety of objects, either with or without the aid of the contralateral hand. The thumb can be rotated to adjust to the desired grasping pattern.

Safety Instructions

CAUTION:

This product is developed for everyday use. Do not use this device for sports which involve excessive strain and/or shocks to the wrist joint (pushups, downhill racing, mountain biking, ...) or extreme sports or similar high load or impact activities. Furthermore, the product should not be used to operate motor vehicles, heavy equipment (e.g. construction machines), industrial machines or motor-driven equipment. Do not carry heavy loads (above 10kg) with this hand model.

NOTICE:

The end user should be properly instructed in the use of the mHand Adapt. Proper care is required to prevent dirt, sand, dust and moisture from entering the hand mechanism causing function loss. In case the product gets wet, please make sure it is completely dried in a warm place for sufficient amount of time.

Qualification of the Prosthetist

The fitting with the mHand Adapt may only be carried out by certified prosthetists experienced in upper limb prosthetic fitting.

Installation

Installing the wrist unit

Attach the hand to the chosen wrist unit, f.e. 11S47=40 Ottobock Quick Disconnect, Secure the thread with Loctite 242 (blue).

Setting the thumb friction

To adjust the thumb friction, use a long 3 mm hex key to adjust the screw on top of the thumb. This screw is located between the pointing finger and the middle finger. The screw is easier to reach with the hand in the closed position.

Installing the cosmetic glove

NOTICE: Always use a cosmetic glove in the setup of the mHand Adapt!



Without a cosmetic glove, friction of the hand surface is too low to effectively grasp objects. Furthermore, damage to the hand and malfunction can be expected when not using a cosmetic glove.

Hand Model	PH01-275 (XS)	PH01-300 (S)	PH01-325 (M)	PH01-375 (L)
Recommended glove	Otto Bock 88A16 MySkin Move Individual Cosmetic Glove			


- In combination with the MySkin Move, the use of donning spray is not necessary and usually complicates the donning process.
- Slightly close the hand.
- Pull the cosmetic glove over the fingers and the hand.
- Check the correct fit

NOTICE: After installation of the glove carefully check that closing and releasing / resetting of the hand functions well!

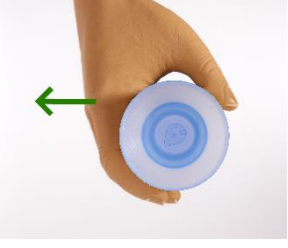User Instructions

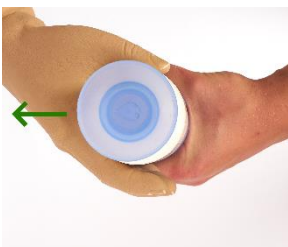Direct Grasp	 <p>Place the thumb in front of the object. Using the stump, put pressure on the thumb.</p>	 <p>The wrist extension causes the fingers to close, thus grasping the object.</p>	 <p>NOTICE: Do not put pressure on the finger tips as this will not lead to hand closure!</p>
Indirect Grasp	 <p>Push the object in the palm of the hand, this will cause the wrist to extend backwards.</p>	 <p>The wrist extension will cause the fingers to close, thus grasping the object.</p>	 <p>NOTICE: Do not put pressure on the finger tips as this will not lead to hand closure!</p>
Release	 <p>Open the hand by pressing slightly on the thumb. The hand will then open by itself.</p>	 <p>Or open the hand by pressing slightly on the object/palm of the hand.</p>	

Grip patterns

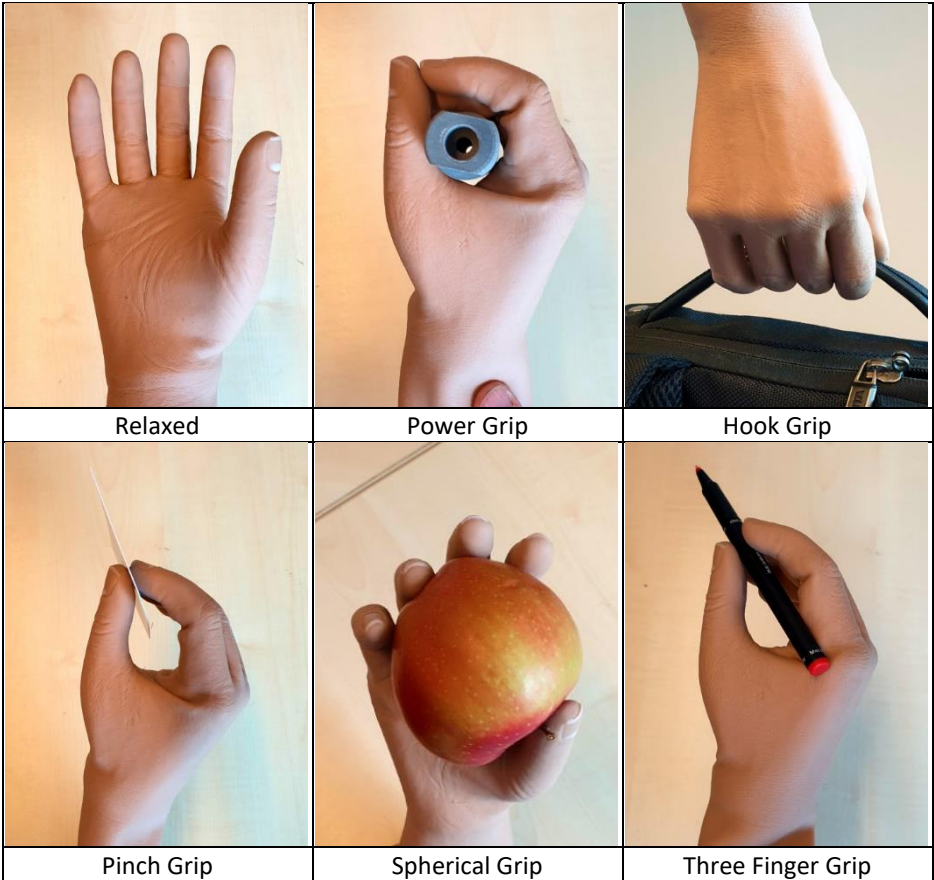Examples of use

Troubleshooting

The hand does not close when trying to grasp an object

Hand closure is directly dependent on the backward extension of the wrist. In order to extend the wrist, the direction of the load on the hand must be perpendicular to the stump. Be sure to either press against the thumb or against the palm of the hand, but not against the fingers!

The hand does not remain closed when grasping an object

The ratchet mechanism of the hand resets after hand opening in order to be ready for the next grasping movement. If the hand does not open completely due to too much influence of the cosmetic glove, the hand will not reset.

The thumb rotates when trying to grasp an object

Increase the thumb friction as described in the installation chapter

Maintenance

Check the hand regularly, at least every 6 months, for function and defects. The hand can be cleaned with water or a mild detergent. Dirt can be cleaned with compressed air.

CAUTION:

After an unplanned high impact on the prosthetic hand, carefully inspect the hand. If there is any doubt if the mechanism of the hand is fully intact, please contact the manufacturer. In the unlikely event of a serious incident caused by the product please inform the manufacturer and the responsible authorities.


Service Lifetime

The hand is designed for 5 years or 100 000 cycles of service, whichever is first. Usage beyond these limits might lead to failure and dropping of the carried object.

Scope of Delivery

- mHand Adapt
- Instructions for Use

Technical data

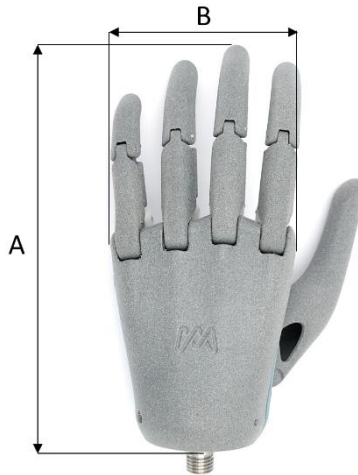Hand Model	PH01-275 (XS)	PH01-300 (S)	PH01-325 (M)	PH01-375 (L)
Weight	160 gram	170	200	230
Length (A)	166 mm	169	172	182
Width (B)	66mm	74	84	92
Normal use temperature range	-10 – 40°C 32-104°F			
Storage temperature range	-10 to 60°C 14 to 140°F			

Disposal

You can send the product back to the distributor or manufacturer of the hand. They will take care of environmental responsible disposal.

If you dispose this product otherwise, make sure you comply to the local laws and regulations regarding disposal

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