## 1S49, 1S66, 1S67, 1S90 SACH

<b>DE</b> Gebrauchsanweisung .....	4	<b>CS</b> Návod k použití.....	67
<b>EN</b> Instructions for use .....	8	<b>RO</b> Instrucțiuni de utilizare.....	71
<b>FR</b> Instructions d'utilisation.....	13	<b>HR</b> Upute za uporabu .....	76
<b>IT</b> Istruzioni per l'uso .....	18	<b>SL</b> Navodila za uporabo .....	81
<b>ES</b> Instrucciones de uso .....	24	<b>SK</b> Návod na používanie .....	86
<b>PT</b> Manual de utilização.....	29	<b>BG</b> Инструкция за употреба .....	90
<b>NL</b> Gebruiksaanwijzing.....	34	<b>TR</b> Kullanma talimatı .....	96
<b>SV</b> Bruksanvisning .....	38	<b>EL</b> Οδηγίες χρήσης .....	100
<b>DA</b> Brugsanvisning .....	43	<b>RU</b> Руководство по применению .....	105
<b>NO</b> Bruksanvisning .....	48	<b>JA</b> 取扱説明書.....	111
<b>FI</b> Käyttöohje.....	52	<b>ZH</b> 使用说明书.....	116
<b>PL</b> Instrukcja użytkowania.....	57	<b>KO</b> 사용 설명서.....	119
<b>HU</b> Használati utasítás .....	62		

1

2

2 ■

3

4

# 1 Produktbeschreibung

Deutsch

## INFORMATION

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### INFORMATION

Date of last update: 2023-08-17

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.

- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

### 1.1 Construction and Function

The 1S49, 1S66 and 1S90 SACH prosthetic feet (solid ankle cushion heel) are suitable for use in both modular and exoskeletal prostheses. The functional properties are achieved through the combination of a contoured core and functional foam.

### 1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

#### Allowable ankle blocks and screw connections

Foot size [cm]	Reference number (ankle block and screw connection)
24 to 30	2K34=30 and 2Z22=M10

#### Allowable foot adapters

Size [cm]	Body weight [kg]	Allowable foot adapter
21 to 30	≤100	2R8=M10, 2R31=M10, 2R54=M10
26 to 30	101 to 125	2R8=M10, 2R31=M10

## 2 Intended use

### 2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

### 2.2 Area of application

The MOBIS classification describes the mobility grade and body weight, and makes it easy to identify compatible components.

The product is recommended for mobility grade 1 (indoor walker) and mobility grade 2 (restricted outdoor walker).

- The maximum approved body weight is specified in the technical data (see page 13).

## 2.3 Environmental conditions

#### Storage and transport

Storage temperature: -20 °C to +60 °C, relative humidity: 20 % to 90 %, no mechanical vibrations or impacts

#### Allowable environmental conditions

**Temperature for use:** -10 °C to +45 °C

**Moisture:** relative humidity: 20 % to 90 %, non-condensing

#### Prohibited environmental conditions

**Chemicals/liquids:** fresh water, salt water, perspiration, urine, acids, soapsuds, chlorine water

**Solids:** highly liquid-binding particles (e.g. talcum), dust, sand

## 2.4 Lifetime

The product was tested by the manufacturer with 2 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 3 years.

## 3 General safety instructions

### CAUTION! Risk of injury and risk of product damage

- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ Note the combination possibilities/combination exclusions in the instructions for use of the products.
- ▶ Observe the maximum lifetime of the product.
- ▶ Check the product for damage and readiness for use prior to each use.

- ▶ Do not expose the product to excessive strain (section: "Area of application" see page 9) or to prohibited environmental conditions (see page 9).
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ The product may only be used by one person; reuse on another person is prohibited.

#### Signs of changes in or loss of functionality during use

Decreased forefoot resistance or changes in roll-over behaviour are noticeable indications of loss of functionality.

#### 4 Scope of delivery

Quantity	Designation
1	Instructions for use
1	Prosthetic foot

#### Spare parts/accessories (not included in the scope of delivery)

Fig.	Item	Designation	Reference number
2	1	Foot adapter with screw connection	2R8*, 2R31*, 2R54*
2	2	Connection plate	2R14
3	-	Ankle block (exoskeletal design)	2K34=*
4	-	Screw connection for ankle block (exoskeletal design)	2Z22=M10
-	-	Ottobock special adhesive with hardener	636W28 or 636W18 and 636W19

#### 5 Preparing the product for use

##### CAUTION

##### Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

##### INFORMATION

- ▶ Not all of the materials may be available in your country. In this case, please contact the local branch of the manufacturer to obtain information on alternative materials.

#### 5.1 Installing the foot adapter

- ▶ Install the foot adapter on the prosthetic foot according to its instructions for use.

#### 5.2 Installing the ankle block (exoskeletal design)

> **Required materials** : Screw connection (see page 10), 710D21 torque wrench, 40 grit sandpaper, 617H21 sealing resin, 617P37 hardener powder, suitable solvent (e.g. 634A28), only for 1WR95: 636W17 plastic adhesive

- 1) Insert the threaded socket of the screw connection in the proximal side of the hole in the ankle block.
- 2) Insert the screw on the foot sole side in the prosthetic foot and screw the ankle block to the prosthetic foot (mounting torque: **30 Nm**).
- 3) Conduct the fitting of the ankle block.
- 4) Loosen the screw connection between the prosthetic foot and the ankle block.
- 5) Sand the ankle block according to the contour of the prosthetic foot.
- 6) Roughen the connection surface of the prosthetic foot in an a – p direction using the sandpaper.
- 7) Remove the dust from the sanded surfaces.

- 8) Mix the sealing resin with **2% to 3%** hardener powder for the connection between the ankle block and the prosthetic foot.
- 9) **INFORMATION: Do not put the resin in the hole for the screw connection.**  
Apply the resin evenly to the connection surfaces of the ankle block and prosthetic foot. At least **2 applications** are required due to the absorbency of the wood.
- 10) When the resin is slightly cured (threads on the spatula), align the ankle block and prosthetic foot with each other.
- 11) Insert the screw on the foot sole side in the prosthetic foot and screw the ankle block to the prosthetic foot (mounting torque: **30 Nm**).
- 12) Wipe off resin that comes out of the joint using a cloth.
- 13) **1WR95 only:** Apply plastic adhesive to the plug for the screw connection and press it fully into the prosthetic foot.
- 14) Remove any remnants of the adhesive using a suitable solvent.
- 15) Allow the adhesive to cure.

## 5.3 Alignment

### NOTICE

#### Grinding the prosthetic foot

Premature wear resulting from damage to the prosthetic foot

- ▶ Do not grind the prosthetic foot.

### 5.3.1 Bench Alignment

#### Bench alignment, transtibial prosthesis

##### Bench alignment process

**Required materials:** 662M4 goniometer, 743S12 heel height measuring apparatus, 743A80 50:50 gauge, alignment apparatus (e.g. 743L200 L.A.S.A.R. Assembly or 743A200 PROS.A. Assembly)

##### Bench alignment process

Perform the assembly and alignment of the prosthetic components in the alignment apparatus according to the following specifications:

##### Sagittal plane

①	Heel height: <b>Effective heel height</b> (shoe heel height - sole thickness of forefoot) <b>+ 5 mm</b>
②	Exterior foot rotation: <b>approx. 5°</b>
③	Anterior placement, middle of the prosthetic foot to the alignment reference line: <b>30 mm</b>
④	Connect the prosthetic foot and prosthetic socket using the chosen adapters. The instructions for use of the adapters must be observed.
⑤	Determine the centre of the prosthetic socket with the 50:50 gauge. Align the prosthetic socket centrally to the alignment reference line. Socket flexion: <b>Individual residual limb flexion + 5°</b>

##### Frontal plane

⑥	Alignment reference line of prosthetic foot: <b>between big toe and second toe</b> Alignment reference line of prosthetic socket: <b>along the lateral patella edge</b>
⑦	Observe the abduction or adduction position.

#### Bench alignment, thigh/knee disarticulation prosthesis

- ▶ Observe the instructions for use for the prosthetic knee joint.

### 5.3.2 Static Alignment

- Ottobock recommends checking the alignment of the prosthesis using the L.A.S.A.R. Posture and adapting it as needed.

The alignment recommendations (TF modular leg prostheses: **646F219\***, TT modular leg prostheses: **646F336\***) can be requested from Ottobock as needed.

### 5.3.3 Dynamic Trial Fitting

The optimal gait pattern is developed during dynamic fitting. The prosthesis is aligned optimally in the frontal plane and the sagittal plane for this purpose.

► **TT fittings:** Make sure that physiological knee movement in the sagittal and frontal planes is achieved after the leg begins to bear weight following the heel strike. Avoid medial movement of the knee joint.

→ If the knee joint moves in the medial direction in the first half of the stance phase, move the prosthetic foot in the medial direction.

→ If the knee joint moves in the medial direction in the second half of the stance phase, reduce the exterior rotation of the prosthetic foot.

### 5.4 Optional: Installing the foam cover

The foam cover sits between the prosthetic socket and prosthetic foot. It is cut longer in order to compensate for the movements of the prosthetic foot and prosthetic knee joint. During flexion of the prosthetic knee joint, the foam cover undergoes posterior compression and anterior elongation. The foam cover should be stretched as little as possible in order to increase its service life. There is a connecting element (such as a connection plate, connection cap or connection cover) on the prosthetic foot.

> **Required materials:** degreasing cleaner (e.g. 634A58 isopropyl alcohol), 636N9 contact adhesive or 636W17 plastic adhesive

1) Measure the length of the foam cover on the prosthesis and add the length allowance.

**TT prostheses:** Distal allowance for movement of the prosthetic foot.

**TF prostheses:** Allowance proximal of the knee rotation point for flexion of the prosthetic knee joint and distal allowance for movement of the prosthetic foot.

2) Cut the pre-shaped foam cover to length and fit it in the proximal area on the prosthetic socket.

- 3) Pull the foam cover over the prosthesis.
- 4) Set the connecting element onto the footshell or prosthetic foot. Depending on the version, the connecting element engages in the edge or rests on the foot adapter.
- 5) Install the prosthetic foot on the prosthesis.
- 6) Mark the outer contour of the connecting element on the distal face of the foam cover.
- 7) Disassemble the prosthetic foot and remove the connecting element.
- 8) Clean the connecting element using a degreasing cleaner.
- 9) Glue the connecting element onto the distal face of the foam cover according to the marked outer contour.
- 10) Let the glue dry (approx. **10 minutes**).
- 11) Install the prosthetic foot and adapt the exterior cosmetic shape. Take into account compression caused by cosmetic stockings or SuperSkin.

## 6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

## 7 Maintenance

- A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- During regular inspection: Inspect the prosthesis for signs of wear and check its function.
- Conduct annual safety inspections.

## 8 Disposal

Do not dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

## 10 Technical data

Reference number	1S49	1S66	1S90
Heel height [mm]	10 ± 5	18 ± 5	10 ± 5
Mobility grade	1 + 2		
Colours	All: beige (4), 1S90 only: brown (16)		

Size [cm]	21	22	23	24	25	26	27	28	29	30
System height with adapter	52	55	58	61	64	67	70	72	74	76
Build height with adapter	70	73	76	79	82	85	88	90	92	94
Max. body weight [kg]	100					125				

Weight without foot adapter [g]										
Size [cm]	21	22	23	24	25	26	27	28	29	30
1S49	265	290	330	365	390	475	535	575	–	–
1S66	–	325	325	360	395	485	555	565	625	625

Weight without foot adapter [g]										
Size [cm]	21	22	23	24	25	26	27	28	29	30
1S67	–	330	365	400	440	530	–	–	–	–
1S90	–	300	330	380	420	460	545	635	–	–

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