1E90* Sprinter

DE	Gebrauchsanweisung	4
EN	Instructions for use	10
FR	Instructions d'utilisation	16
IT	Istruzioni per l'uso	22
ES	Instrucciones de uso	29
PT	Manual de utilização	35
NL	Gebruiksaanwijzing	41
SV	Bruksanvisning	48
DA	Brugsanvisning	53
NO	Bruksanvisning	59
FI	Käyttöohje	65
PL	Instrukcja użytkowania	71
HU	Használati utasítás	77
CS	Návod k použití	83
RO	Instrucțiuni de utilizare	89
HR	Upute za uporabu	96
SL	Navodila za uporabo	102
SK	Návod na používanie	107
BG	Инструкция за употреба	114
TR	Kullanma talimatı	120
EL	Οδηγίες χρήσης	126
RU	Руководство по применению	133
JA	取扱説明書	140
ZH	使用说明书	146
KO	사용 설명서	151

Bench alignment, transfemoral

Alignment reference line

Socket

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INFORMATION

Date of last update: 2024-03-13

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The 1E90* Sprinter prosthetic foot is intended for use in a sport prosthesis. It is distinguished by its low weight. The spring contour ensures a high driving force and a low resistance.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings. The product is not suitable for use in everyday prostheses.

2.2 Area of application

- The maximum approved body weight is specified in the technical data (see page 15).

2.3 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

INFORMATION

The prosthetic socket must be able to withstand the higher loading of the sport.

Limited combination options for Ottobock components

Outer soles and safeguard sticker	Designation	Reference number
	Outer sole	2Z500
	Outer sole with spikes	2Z501
	Safeguard sticker	2Z538
Combinations for transtibial prostheses	Designation	Reference number
	TT test sports foot adapter	4R206
	TT definitive sports foot adapter	4R204
	Posterior connection plate (set)	4R420
	T-adapter for TT	2R176=T
	5° L-adapter	2R177=5
18° L-adapter	2R177=18	
Combinations for transfemoral and knee disarticulation prostheses	Designation	Reference number
	Sports knee joint	3S80
	TF test sports foot adapter	4R206
	TF definitive sports foot adapter	4R204

2.4 Environmental conditions

Storage and transport
Storage temperature: -20 °C to +60 °C, relative humidity: 20 % to 90 %, no mechanical vibrations or impacts
Allowable environmental conditions
Temperature during use: -10 °C to +45 °C
Chemicals/liquids: Fresh water, salt water, perspiration, urine, soapsuds, chlorine water
Moisture: Submersion: max. 1 h in 3 m depth, relative humidity: no restrictions
Solids: Dust, occasional contact with sand
Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and tear and damage (see page 14).
Prohibited environmental conditions
Solids: Highly liquid-binding particles (e.g. talcum), continuous contact with sand
Chemicals/liquids: Acids, continuous use in liquid media

2.5 Lifetime

The lifetime of a sports prosthesis depends on the frequency and intensity of use. The expected lifetime ranges from 6 months for competitive sports to 2 years for recreational sports.

3 General safety instructions

CAUTION! Risk of injury and risk of product damage

- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ Note the combination possibilities/comboination exclusions in the instructions for use of the products.
- ▶ Observe the maximum lifetime of the product.
- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not expose the product to excessive strain (section: "Area of application" see page 10) or to prohibited environmental conditions (see page 11).
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ The product may only be used by one person; reuse on another person is prohibited.

Signs of changes in or loss of functionality during use

Decreased forefoot resistance or changes in roll-over behaviour are noticeable indications of loss of functionality.

4 Scope of delivery

Quantity	Designation
1	Instructions for use
1	Prosthetic foot

5 Preparing the product for use

⚠ CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

5.1 Selecting the stiffness

Otto Bock recommends that the stiffness version is selected according to body weight and running style.

Stiffness version		Short-distance running	Long-distance running
		Body weight	Body weight
SPR-1	[kg]	40 to 52	40 to 59
	[lbs]	90 to 115	90 to 130
SPR-2	[kg]	53 to 63	60 to 70
	[lbs]	116 to 140	131 to 155
SPR-3	[kg]	64 to 79	71 to 86
	[lbs]	141 to 175	156 to 190
SPR-4	[kg]	80 to 95	87 to 102
	[lbs]	176 to 210	191 to 225
SPR-5	[kg]	96 to 111	103 to 118
	[lbs]	211 to 246	226 to 260

5.2 Installing the outer sole

CAUTION

Incorrect use of prosthetic foot and outer sole

Risk of injury due to lack of surface grip and damage to the product

- ▶ Only use the prosthetic foot with an outer sole.
- ▶ Only use outer soles with spikes on sports floors suitable for this purpose.

- ▶ Fit the outer sole according to the information in its instructions for use.

Instructions for use for outer soles 2Z500 and 2Z501	647G848
---	---------

5.3 Alignment

Bench alignment

- ▶ **Sagittal plane:** Perform the bench alignment of the prosthesis (see illustrations at the beginning of the document).
- ▶ **Frontal plane:** The alignment reference line runs centrally through the prosthetic foot, centrally through the entrance plane of the prosthetic socket for transfemoral prostheses, and at the lateral edge of the patella for transtibial prostheses. Observe the abduction/adduction position. Align the prosthetic foot without external rotation.
- ▶ Observe the instructions for use for the selected prosthetic components.

Instructions for use for TF sports foot adapter	647G839
--	---------

Instructions for use for sports knee joint 3S80	647G813
--	---------

Instructions for use for TT sports foot adapters (4R208, 4R210)	647G840
--	---------

Instructions for use for TT sports foot adapters (4R212)	647G981
---	---------

The mounting instructions for the 4R420, 2R176=T, 2R177=5, 2R177=18 adapters are included with the products.

Static alignment and dynamic fitting

During the static alignment and the dynamic fitting, the settings of the prosthesis (length, angle, displacements) are checked and adapted on the user. Adjusting the prosthetic foot has a major influence on the behaviour of the prosthesis:

Sagittal plane

- **Moving in the anterior direction:** Increases the knee extending moment and makes the knee joint more secure (while standing and while walking). The prosthesis has diminished forward thrust. The user may feel like walking uphill.
- **Moving in the posterior direction:** Reduces the knee extending moment. The prosthesis has increased forward thrust. The user needs to use more muscle strength to keep the knee joint extended.
- **Adjusting the angle to be steeper:** Shortens the forefoot lever. This increases the forward thrust of the prosthesis and makes the prosthetic foot feel stiffer.
- **Adjusting the angle to be flatter:** Extends the forefoot lever. The prosthesis' forward thrust decreases and the prosthetic foot feels softer.

Frontal plane

- Adjust the prosthesis so the user steps down flat and rolls over the virtual big toe.

6 Cleaning

- > **Recommended cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)
- 1) Clean the product by hand with clean water and a pH neutral soap.
- 2) Rinse the soap away with clean water.
- 3) Dry the product with a soft cloth.
- 4) Allow to air dry in order to remove residual moisture.

7 Maintenance

⚠ CAUTION

Failure to follow the maintenance instructions

Risk of injuries due to changes in or loss of functionality and damage to the product

- ▶ Observe the following maintenance instructions.

- ▶ Following an individual period for the patient to get accustomed to the product, check the settings of the prosthesis and, if necessary, adapt them.
- ▶ Arrange regular maintenance intervals with the patient depending on the level of use.
- ▶ During regular inspection: Inspect the prosthesis for signs of wear and check its function.
- ▶ Conduct annual safety inspections.

8 Disposal

Do not dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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