
## 17KF110=\* , 17KF210=\*


<b>DE</b> Gebrauchsanweisung .....	7
<b>EN</b> Instructions for use .....	16
<b>FR</b> Instructions d'utilisation .....	25
<b>IT</b> Istruzioni per l'uso .....	35
<b>ES</b> Instrucciones de uso .....	45
<b>PT</b> Manual de utilização .....	55
<b>NL</b> Gebruiksaanwijzing .....	65
<b>SV</b> Bruksanvisning .....	75
<b>DA</b> Brugsanvisning .....	83
<b>NO</b> Bruksanvisning .....	92
<b>FI</b> Käyttöohje .....	100
<b>PL</b> Instrukcja użytkowania .....	109
<b>HU</b> Használati utasítás .....	119
<b>CS</b> Návod k použití .....	128
<b>HR</b> Upute za uporabu .....	137
<b>SK</b> Návod na používanie .....	146
<b>TR</b> Kullanma talimatı .....	155
<b>RU</b> Руководство по применению .....	164
<b>JA</b> 取扱説明書 .....	174
<b>ZH</b> 使用说明书 .....	183

1

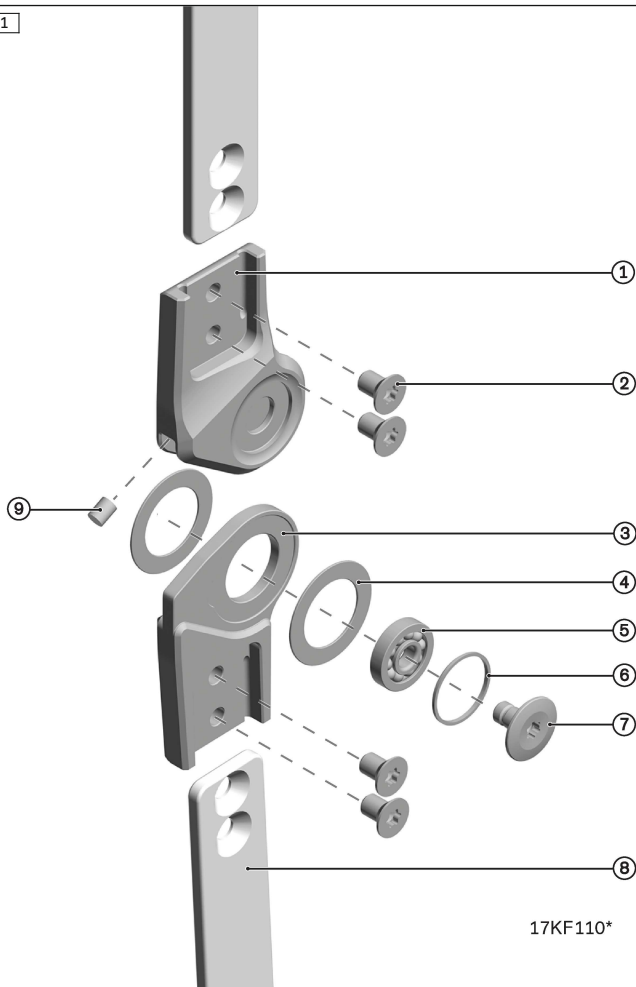
17KF110\*

2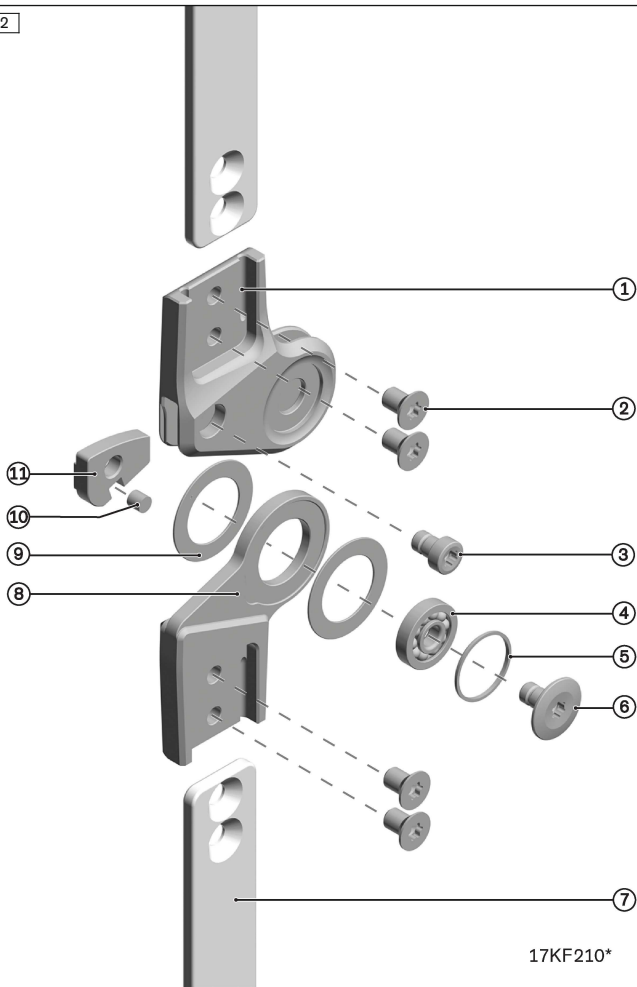


17KF210\*


3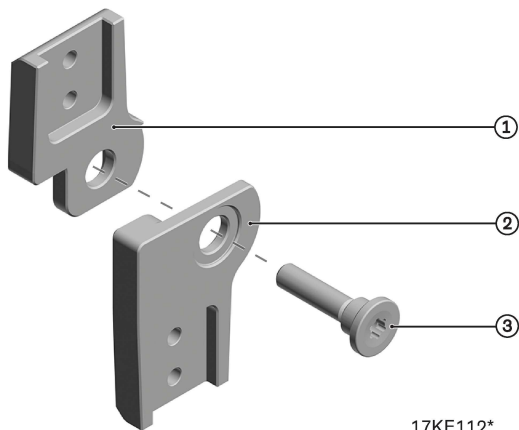


4

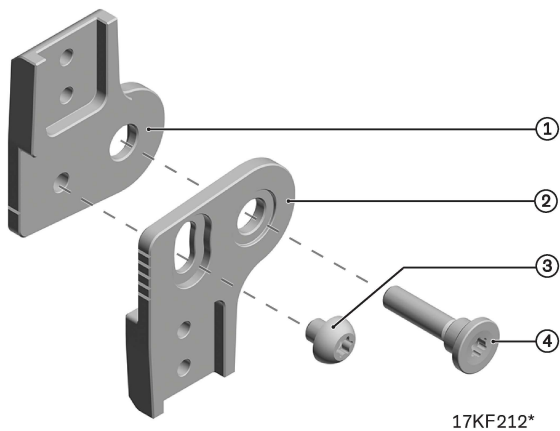
5

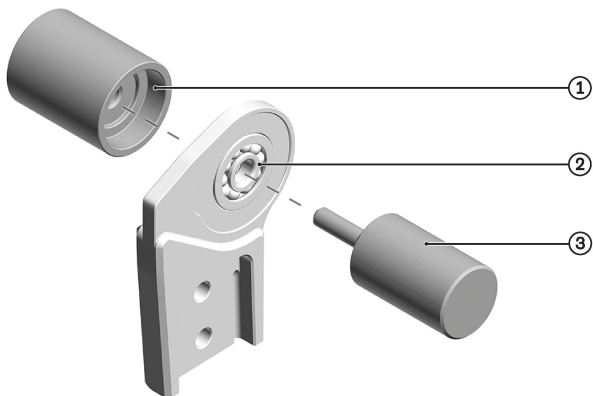
6

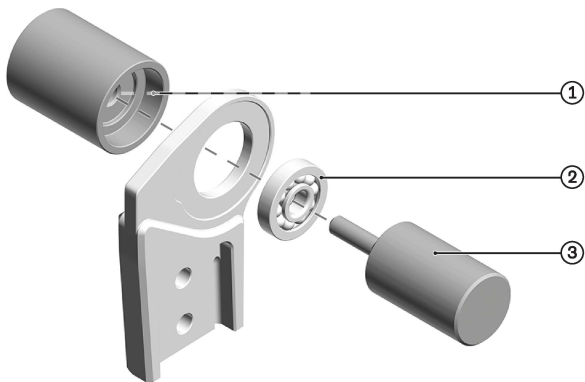7

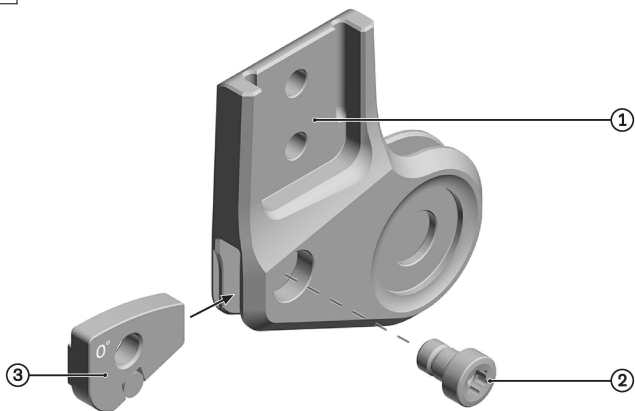8
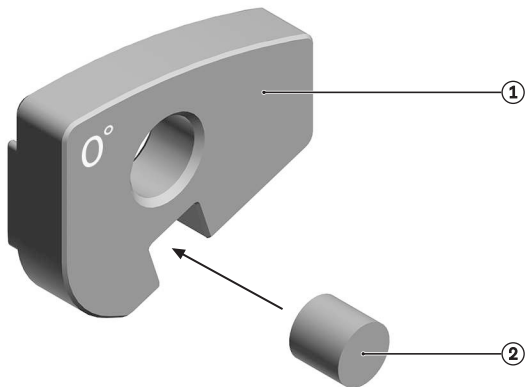
## INFORMATION

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### INFORMATION

Date of last update: 2024-06-21

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the processing of the 17KF110\* and 17KF210\* orthotic knee joints.

## 2 Product description

### 2.1 Available sizes

Article number	Size	Width in mm	Material	Max. body weight kg (lbs)
17KF110*, 17KF210*	10	10	Steel/titanium	15 (33)
17KF110*, 17KF210*	12	12	Steel/titanium	20 (44)

Article number	Size	Width in mm	Material	Max. body weight kg (lbs)
17KF110*, 17KF210*	14	14	Steel/titanium	50 (110)
17KF110*, 17KF210*	16	16	Steel/titanium	85 (187)
17KF110*, 17KF210*	20	20	Steel/titanium	110 (242.5)

## 2.2 Design

The 17KF110\* orthotic knee joint comprises a ball bearing and two axial washers. The 17KF210 orthotic knee joint has a posterior location and comprises a ball bearing, two bearing washers and a replaceable extension stop with stop bumper. For the 17KF210\*, there are replaceable extension stops in -5°, (0° in scope of delivery), 5°, 10° and 15° available.

17KF210* Nexgear posterior knee		
Reference number	Size	Posterior location in mm
17KF210=*	10	10
17KF210=*	12	12
17KF210=*	14	14
17KF210=*	16	16
17KF210=*	20	20

## 2.3 Components/design

Scope of delivery for 17KF110=* (see fig. 1)			
Item	Quantity [Piece (s)]	Designation	Reference number
1	1	Upper joint section	<b>Service set</b> 17KF111=* (comprising: stop bumper, axial washers, ball bearing)
2	4	Screw	
3	1	Lower joint section	
4	2	Axial washer	
5	1	Ball bearing	
6	1	Ornamental ring	
7	1	Joint screw	

<b>Scope of delivery for 17KF110=* (see fig. 1)</b>			
<b>Item</b>	<b>Quantity [Piece (s)]</b>	<b>Designation</b>	<b>Reference number</b>
9	1	Stop bumpers	<b>Service set</b> 17KF111=* (comprising: stop bumper, axial washers, ball bearing)

<b>Scope of delivery for 17KF210=* (see fig. 2)</b>			
<b>Item</b>	<b>Quantity [Piece (s)]</b>	<b>Designation</b>	<b>Reference number</b>
1	1	Upper joint section	<b>Service set</b> 17KF211=* (comprising: stop bumper, axial washers, ball bearing)
2	4	Screw	
3	1	Stop screw	
4	1	Ball bearing	
5	1	Ornamental ring	
6	1	Joint screw	
8	1	Lower joint section	
9	2	Axial washer	
10	1	Stop bumpers	
11	1	Extension stop 0°	

<b>Not included in scope of delivery</b>			
<b>Item</b>	<b>Quantity [Piece (s)]</b>	<b>Designation</b>	<b>Reference number</b>
Not illustrated	4	Extension stop (-5°, 5°, 10°, 15°)	<b>Stop set</b> 17KF213=*

<b>Not included in scope of delivery (see fig. 5, see fig. 6)</b>				
<b>Item</b>	<b>Quantity [Piece (s)]</b>	<b>Designation</b>	<b>Article number</b>	<b>Orthotic joint size</b>
Not illustrated	1	Ball bearing assembly aid	17KF113=1	*=10*/*=12*/*=1-4*
			17KF113=2	*=16*/*=20*

<b>Lamination dummy not in scope of delivery (see fig. 3)</b>				
<b>Item</b>	<b>Quantity [Piece (s)]</b>	<b>For orthotic joint</b>	<b>Designation</b>	<b>Reference number</b>
1	1	17KF110*	Lamination dummy upper section	<b>Dummy set</b> 17KF112=*
2	1	17KF110*	Slotted oval head screw	
3	1	17KF110*	Lamination dummy lower section	

<b>Lamination dummy not in scope of delivery (see fig. 4)</b>				
<b>Item</b>	<b>Quantity [Piece (s)]</b>	<b>For orthotic joint</b>	<b>Designation</b>	<b>Reference number</b>
1	1	17KF210*	Lamination dummy upper section	<b>Dummy set</b> 17KF212=*
2	1	17KF210*	Lamination dummy lower section	
3	1	17KF210*	Clamping screw	
4	1	17KF210*	Slotted oval head screw	

<b>Not included in scope of delivery</b>			
<b>Item</b>	<b>Quantity [Piece (s)]</b>	<b>Designation</b>	<b>Reference number</b>
(see fig. 1, item 8, see fig. 2, item 7)	2	Side bar	17SB100=*
Not illustrated	2	Lamination bar	17SL100=*

## 3 Intended use

### 3.1 Indications for use

The orthotic joint is intended **exclusively** for use as a pair for orthotic fittings of the lower limbs on one patient.

Ottobock recommends using the lamination and prepreg technique for the lamination bars, and the thermoplastic and bar-clamp technique for the side bars.

### 3.2 Indications

- Partial or total paralysis of the leg muscles
- Orthopaedic diseases of the lower limbs

Indications must be determined by the physician.

### 3.3 Lifetime

The product is designed for the following lifetime, given intended use and proper installation:

Joint sizes 10, 12: **1 year**

Joint sizes 14, 16, 20: **3 years**

### 3.4 Qualification

Users may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

### 3.5 Combination possibilities

The following orthotic ankle joints can be combined:

- 17AD200\*
- 17AD300\*
- 17LA3N\*

## 4 Safety

### 4.1 Explanation of warning symbols

 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

## 4.2 Safety instructions

### CAUTION

#### **Risk of pinching in the knee joint flexion area**

Injury due to pinching parts of the body and garments.

- ▶ Ensure that no body parts or garments are in this area when flexing the product.

### CAUTION

#### **Excessive strain on load-bearing components**

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

### CAUTION

#### **Incorrect alignment or assembly**

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

### CAUTION

#### **Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

### CAUTION

#### **Excessive strain due to use on more than one patient**

Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

#### NOTICE

##### **Thermal overloading of the product**

Damage due to improper thermal treatment

- ▶ Do not carry out any heat treatment at temperatures above **300 °C (570 °F)**.
- ▶ Prior to thermal treatment, remove all temperature-critical components (such as plastic parts).

#### NOTICE

##### **Exposure of the product to unsuitable environmental conditions**

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below **-10 °C (14 °F)** or above **+60 °C (140 °F)** (e.g. sauna, excessive sunlight, drying on a radiator).

## **5 Preparing the product for use**

#### INFORMATION

##### **Parallel alignment of the orthotic joints**

Use the 743R6 alignment fixture to position the orthotic joints on the plaster positive.

##### **Using the lamination dummy**

- 1) Use the screws to screw the lamination bars to the lower and upper sections of the lamination dummy.
- 2) Secure and align the lamination dummy on the model using the slotted oval head screw.
- 3) **17KF210\***: Transfer the joint angle of the orthotic knee joint to the lamination dummy and secure it with the clamping screw (see fig. 4, item 3).
- 4) Bend the bars.

##### **Mounting the bar**

- 1) Use the screws to screw the bar to the insertion zone.
- 2) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241 and the corresponding tightening torque.

Tightening torques in Nm					
Torx	T20	T20	T20	T25	T25
<b>17KF110=*, 17KF210=*</b>	<b>*10</b>	<b>*12</b>	<b>*14</b>	<b>*16</b>	<b>*20</b>
Joint screw	3.5 Nm	3.5 Nm	7 Nm	7 Nm	7 Nm
Stop screw	3.5 Nm	3.5 Nm	3.5 Nm	7 Nm	7 Nm
Screw	3.5 Nm	3.5 Nm	3.5 Nm	7 Nm	7 Nm

### Using protective devices

- ▶ Protect the product against dirt: Use suitable protective devices (e.g. joint protectors).

## 6 Cleaning

Promptly clean the product after contact with water containing salt, chlorine or soap, or if it gets dirty.

- 1) Rinse the product with clear fresh water.
- 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e. g. from an oven or radiator).

## 7 Maintenance

### INFORMATION

**The product may be exposed to increased loads by the patient.**

- ▶ Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Spare parts are listed in the section "Components/design".

Only use 633F7 special lubricant.

### Removing the ball bearing

- 1) Loosen the joint screw and disassemble the orthotic joint.
- 2) Remove the axial washers.
- 3) Guide the pin assembly aid (3) through the ball bearing (2) and position it with the orthotic joint in the receiver assembly aid (1) (see fig. 5).
- 4) Push out the ball bearing (e.g. in the vice).

## Inserting the ball bearing

### NOTICE

#### Canting the ball bearing

Mechanical damage

- ▶ Use only the appropriate 17KF113\* tool to replace the ball bearing.
- ▶ Do not cant or damage the ball bearing while inserting it into the orthotic joint.

- 1) Guide the pin assembly aid (3) through the ball bearing (2) and position it with the orthotic joint in the receiver assembly aid (1) (see fig. 6).
- 2) Press in the ball bearing (e.g. in the vice).
- 3) Assemble the orthotic joint with the axial washers and the joint screw.
- 4) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241, using the corresponding tightening torque (see tightening torque table).

#### Replacing the axial washers for 17KF110\*, 17KF210\*

- 1) Loosen the joint screw and remove it from the upper joint section.
- 2) Disassemble the orthotic joint.
- 3) Insert the replaced axial washers into the lower joint section.
- 4) Assemble the orthotic joint with the joint screw.
- 5) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241 and the corresponding tightening torque.

#### Replacing the extension stop for 17KF210\*

- 1) Loosen the stop screw (2) and remove it from the upper joint section (1).
- 2) Disassemble the extension stop (3).
- 3) Insert the replaced extension stop into the upper joint section, noting the position of the extension stop (see fig. 7)
- 4) Screw the stop screw into the upper joint section.
- 5) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241 and the corresponding tightening torque.

#### Replacing the stop bumper for 17KF110\*, 17KF210\*

- 1) **17KF110\***: Loosen the joint screw and remove it from the upper joint section.
- 2) Press the replaced stop bumper into the upper joint section.
- 3) **17KF210\***: Loosen the stop screw and remove it from the upper joint section.

- 4) **17KF210\***: Remove the extension stop.
- 5) **17KF210\***: Press the replaced stop bumper (2) into the extension stop (1) (see fig. 8).
- 6) **17KF110\***, **17KF210\***: Assemble the orthotic joint.
- 7) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241 and the corresponding tightening torque.

## 8 Disposal

Dispose of the product in accordance with national regulations.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medicinal devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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