DynamicArm 12K100N

DE Gebrauchsanweisung [kurz] (Fachpersonal)	3
EN Instructions for use [short] (qualified personnel)	6

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INFORMATION

Date of last update: 2024-09-25

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

Note the instructions for use

This document contains the sections "Intended use" and "Safety", and does not replace the instructions for use. The instructions for use are available in digital form (PDF format) (see page 2 of this document). To view PDF files, you need an Internet browser or other program capable of opening this file format.

You may request a printed copy of the instructions for use at the e-mail address on page 2. This will be delivered to the specified address within 7 days after the order is received.

The product "DynamicArm 12K100N=*" is referred to as the product/elbow joint below.
Basic UDI-DI: 406441100000012K100NQC

2 Intended use

2.1 Indications for use

The product is to be used solely for upper limb exoprosthetic fittings.

2.2 Conditions of use

The product is suitable for users with unilateral or bilateral upper arm amputations.

The product was developed for everyday use and must not be used for unusual activities. These activities include, for example, sports/extreme sports with high strain on the wrist and/or shocks such as push-ups, downhill racing, mountain biking, free climbing, paragliding, etc. The product is intended **exclusively** for use on one user. Use of the product by a further person is prohibited by the manufacturer.

2.3 Contraindications

- All conditions which contradict or go beyond the specifications listed in the sections on "Safety" (see page 7) and "Intended use" (see page 6).

2.4 Qualification

Treatment may only be carried out by trained, qualified personnel.

3 Safety

3.1 Explanation of warning symbols

 **WARNING** Warning regarding possible serious risks of accident or injury.

 **CAUTION** Warning regarding possible risks of accident or injury.

3.2 General safety instructions

 **WARNING**

Charging the prosthesis without taking it off

Risk of electric shock due to defects in the power supply unit or in the battery charger.

- ▶ For safety reasons, remove the prosthesis prior to charging it.

 **CAUTION**

Electrode lifting off

Injury due to unexpected product behaviour as a result of uncontrolled activation of the prosthesis component.

- ▶ Inform the user that uncontrolled movement of the prosthesis component may occur if the electrode is lifted off.

 **CAUTION**

Damage to connecting cable due to kinking or small radii

Risk of injury due to unexpected product behaviour as a result of a malfunction.

- ▶ Replace the connecting cable immediately if it is damaged.
- ▶ Make sure there are no tight angles or kinks when installing the connecting cables.

 **CAUTION**

Incorrect donning and doffing of the prosthesis

Risk of injury due to sudden flexion of the prosthesis.

- ▶ Switch off the prosthesis before donning or doffing.
- ▶ The prosthesis must always be flexed for donning and doffing.

 **CAUTION**

Removing the switch cable

Risk of injury due to malfunction of the product.

- ▶ The switch cable is to be used for emergency operation only. For safety reasons, it must not be removed! If the product is switched off or defective or the rechargeable battery is drained, the elbow joint cannot be locked or unlocked without the switch cable.

⚠ CAUTION

Incorrect electrode settings due to muscle fatigue

Injury due to faulty control or malfunction of the product.

- ▶ Make sure you take breaks during calibration.

⚠ CAUTION

Risk of pinching in the joint flexion area

Injuries due to pinching of body parts.

- ▶ Ensure that fingers and other body parts are not in this area when bending the joint.

⚠ CAUTION

Charging the product with damaged power supply unit/charger/charger cable

Injury due to unexpected behaviour of the product caused by insufficient charging.

- ▶ Check the power supply unit, charger and charger cable for damage before use.
- ▶ Replace any damaged power supply unit, charger or charger cable.

⚠ CAUTION

Independent user manipulation of system components

Injury due to faulty control or malfunction of the product.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The battery may only be handled by Ottobock authorised, qualified personnel (no replacement by the user).
- ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.

⚠ CAUTION

Unintentional unlocking of the terminal device

Risk of injury due to releasing the terminal device from the forearm (e.g. while carrying objects).

- ▶ When using a quick-disconnect wrist unit, position the terminal device so that slight twisting does not release the terminal device from the forearm.

⚠ CAUTION

Insufficient skin contact of the electrodes

Injury due to faulty control or malfunction of the product.

- ▶ The electrodes are to be placed on intact skin only and with as much electrode-skin contact as possible.
- ▶ In the case of strong interference from electronic devices, the position of the electrodes should be checked and changed if necessary.
- ▶ If the interference cannot be eliminated or if you do not achieve the expected results by adjustment or selection of the appropriate control programme, please contact the Ottobock branch responsible for your country.

⚠ CAUTION

Improper handling

Injury due to loss of product functionality.

- ▶ Instruct the user in the proper use of the product.

⚠ CAUTION

Injury due to incorrect adjustment of the product.

Loosening of the components.

- ▶ In case of a product with a quick-disconnect wrist, position the terminal device before use so that a slight turn cannot disconnect the terminal device from the prosthesis.

⚠ CAUTION

Signs of wear and tear on the product components

Injury due to faulty control or malfunction of the product

- ▶ In the interest of patient safety and in order to maintain operating reliability and protect the warranty, the specified service intervals must be observed (see the section "Maintenance").

⚠ CAUTION

Use of inappropriate components

Risk of injury due to malfunction of the product.

- ▶ Operate the product only with components recommended by the manufacturer. You can find the list of components in the "Combination possibilities" section (Combination possibilities).
- ▶ Operate the product only with accessories that have been recommended by the manufacturer. You can find the list of components in the "Scope of delivery" section (Scope of delivery).

Otto Bock Healthcare Products GmbH
Brehmstraße 16 · 1110 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com